

Softform Premier Active 2



Static Dynamic Mattress



The **Softform Premier Active 2** is a further development of the clinically trialled Active system, incorporating a range of new features (patents applied for) designed to further improve product performance and patient comfort.

The **Softform Premier Active 2** is appropriate for patients in Acute hospital environments, Nursing and Residential and Homecare settings.


The mattress features an alternating air insert beneath the castellated foam surface of the **Softform Premier** mattress. In static mode, the mattress retains its properties as a high risk static surface, but should a patient require “stepping up” to a dynamic surface, then a pump can be easily fitted to the air insert, transforming the mattress into an alternating surface. There is no requirement to move the patient, thereby minimising any discomfort.

Static Dynamic Mattress

The air insert contained within the **Softform Premier** mattress allows stepping up and down between static and dynamic alternating systems with minimum patient disruption.

Digital Pump

Powered by an innovative digital pump, the air cells alternate on a ten minute cycle, delivering a very low impact at the point of change. This significantly enhances patient comfort and makes a positive contribution to reducing the potential negative impact of vigorous reperfusion on blood capillary vessels.

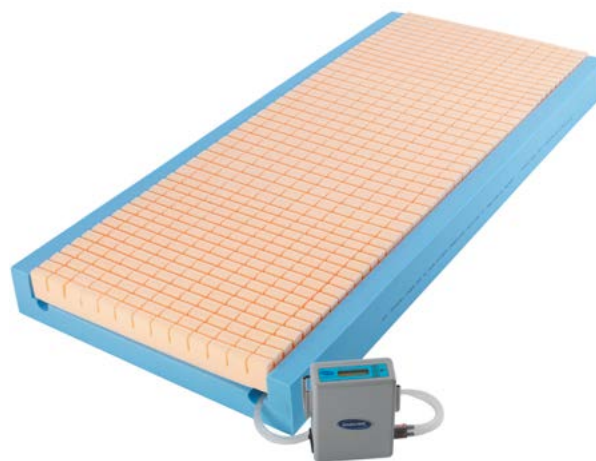

Multi-stretch vapour-permeable cover

The mattress cover is Multi-stretch and conforms to the deep contoured castellations on the upper surface of the mattress, minimising “rucking”, and further reducing the potential for skin tissue damage.

Features and options



To set up a patient, detach the pipes from the foot end of the mattress.



Fit pipes to the Active 2 digital pump.



To activate, press the “On” switch. (No additional adjustments are required).



The patient is now lying on an alternating surface with a 10 minute cycle.

Features and options



The innovative digital pump features a LCD screen, plus audio alarm.



"Intelligent" software in the pump will identify and advise pump status via the LCD screen.



When alternating, the unique* design and construction of the air cells ensures that the patient is effectively supported across the upper surface of the mattress. (*patents pending)



LED screen information is available in nine different language options.



In case of accidental puncturing, individual air inserts can be easily replaced.



Air insert within *Softform Premier* mattress foam construction



The pump hose is sheathed, promoting effective infection control.



The pump hangers are designed to accommodate a wide range of bed end sizes.

Allows stepping up and down between static and dynamic alternating systems with minimum patient disruption.



The *Invacare Softform Premier* is a leading pressure reducing mattress.



The *Softform Active 2* mattress features a two-way stretch vapour-permeable cover.









In the event of a power failure, the patient is still resting on a high risk pressure reducing surface, minimising the potential for pressure damage occurring.



Modular construction facilitates easy spare part replacement.

Technical data

For more comprehensive pre-sales information about this product, including the product's user manual, please see your local Invacare website.

	Width	Length	Size	Total product weight	Max. user weight	Max. cleaning temperature
Softform Premier Active 2	 830 - 900 mm (10 mm increments)	 1810 - 2100 mm (10mm increments)	 152 mm	 14 kg	 247.6 kg	 80°

Mattress is manufactured to:


European Standard: BS EN 597-1 Cig
European Standard: BS EN 597-2 Match
BS7177:1996 Crib 5.

Pump

Noise Level: 32Db
Cycle Time: 10 minutes (1 in 2 cycle)

Colour

Baltic Blue

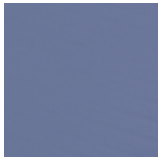

Please notice colors may vary slightly from those displayed above.

QR
Code

Tel: +41 61 487 70 70
hqeuropa@invacare.com
www.invacare.eu.com

Invacare International GmbH
Benkenstrasse 260
4108 Witterswil
Switzerland

