

# Slings




Optimal positioning and  
ease of application



Invacare's range of slings provide a safe and comfortable transfer solution for many types of physical disability. This selection of high quality slings has been developed with the collaboration of Occupational and Physiotherapists, to ensure optimal fit and support for the user and ease of application for the carer. The slings are available in five sizes and with the option of differing fabrics. A range of adjustments enable the slings to meet the specific needs of the individual. In order to help caregivers to position the sling correctly, a green label is to be found on the back of all Invacare slings illustrating how to position the sling according to spine of the patient.



*Invacare* **Universal High Sling**

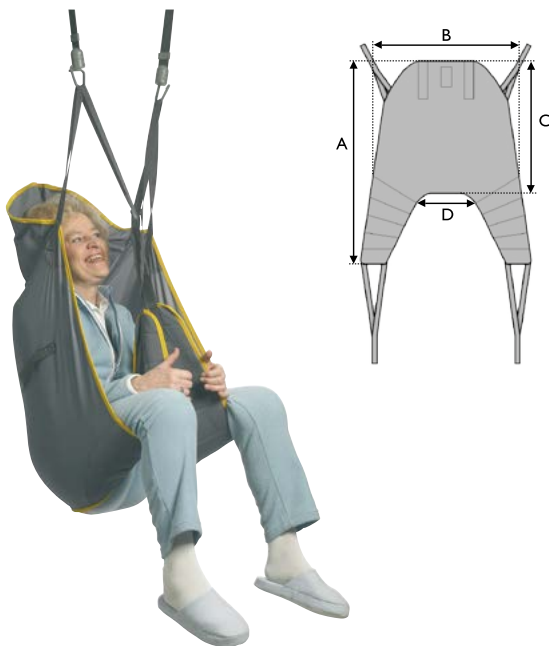


Supports the whole body and the head giving the user a comfortable and slightly reclined position.

The **Universal High Sling** is typically used for those who have limited control of the head, trunk or hips – or for transfers to and from a lying position. The **Universal High Sling** can also be used for seated to seated transfers. This sling can also be used without the head support, by simply removing the plastic rigidisers.

Size	A	B	C	D
XS	1040	740	610	340
S	1080	840	730	380
M	1420	1000	830	400
L	1550	1100	920	510
XL	1680	1150	920	550

*Invacare* **Universal High Plus Sling**

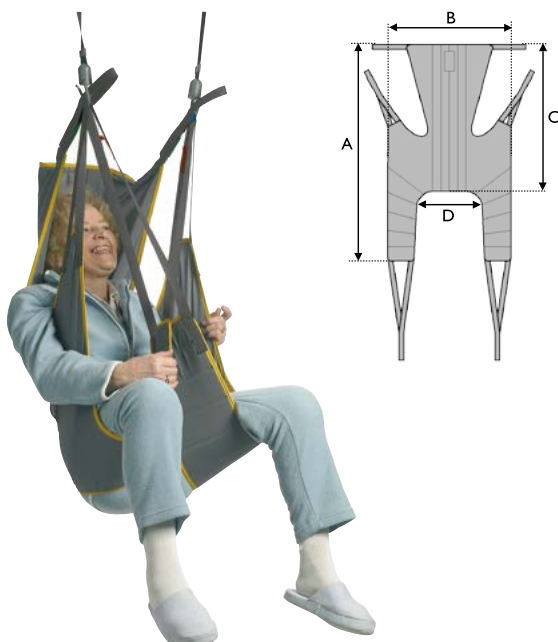

Supports the whole body including the head giving the user a comfortable and slightly reclined position. The **Universal High Plus sling** also allows users keep their arms outside the sling. It is typically used for those who have limited control of the head, trunk or hips – or for transfers to and from a lying position. The **Universal High sling** can also be used for seated to seated transfers. This model has a stiffened fabric in the back section, to ensure optimal weight distribution and comfort for the user.

Now also available in Net and Spacer material.


Size	A	B	C	D
S	1380	840	880	340
M	1510	950	930	450
L	1580	1100	970	560
XL	1590	1190	970	630

Invacare **Universal Standard Sling**



Supports the whole body; including the shoulders. This sling should only be used for those who have good head control. The **Universal Standard sling** is often preferred as an all-round sling for the elderly, as it supports the entire body and gives the user a feeling of comfort as well as an unobstructed view. This sling can be used for seated to seated transfers and is easy to apply. This model also has a stiffened fabric in the back section, to ensure optimal weight distribution and comfort for the user.

Size	A	B	C	D
S	1000	920	510	410
M	1090	1050	560	480
L	1180	1090	580	560
XL	1240	1250	585	710

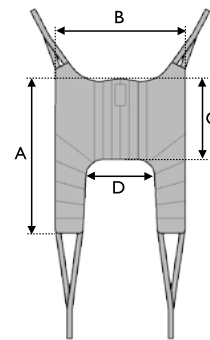


Invacare **Universal Low Sling**




Supports the hips and lower back. The **Universal Low sling** can be used for those who have some trunk control and full head control. It is used for seated to seated transfers and is easy to apply. This sling also has stiffened back section to ensure optimal weight distribution and comfort for the user.

Size	A	B	C	D
S	850	800	350	410
M	920	940	375	480
L	950	1075	405	575

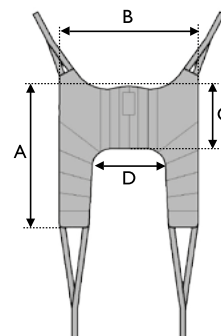


## Invacare **Dress Toileting Sling**

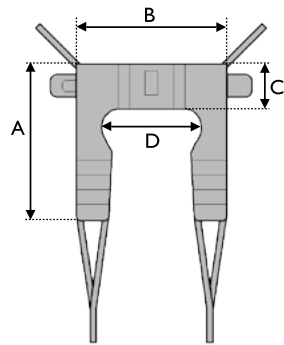

Within the Invacare sling range there are two toilet transfer slings available; with or without head support. Both models come with a belt for increased support around the body and for safe positioning. The **Dress Toileting Sling** is suitable for those who have limited mobility. The model with head support is designed for users who also have reduced head control. The **Dress Toileting Sling** is suitable for those who want to be able to dress and undress during the toilet transfer. It is used for seated to seated transfers and is very easy to apply.


Size	A	B	C	D
S	870	710	200	430
M	950	800	230	490
L	1020	900	230	600
XL	1100	1080	270	760

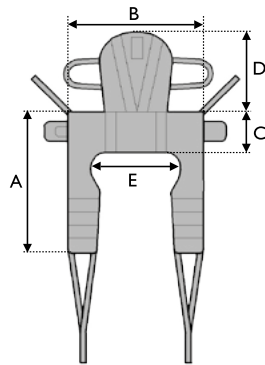

## Invacare **Dress Toileting High Sling**

Size	A	B	C	D	E
S	870	710	200	400	430
M	950	800	230	470	490
L	1020	900	230	470	600
XL	1100	1080	270	470	760


## Invacare **Easy-fit Sling**

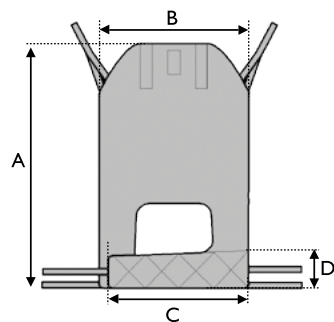

Supports the whole body and the head, and is therefore suited for users with the most challenging physical disabilities. The specially designed separate leg supports are easy to apply, and the sling affords a good hip position, which allows the user to sit relatively upright. The **Easy-fit sling** is also well-suited for amputees. It is easy to apply in both sitting and lying positions

Size	A	B	C	D
XS	1100	710	595	170
S	1130	760	640	180
M	1200	800	675	190
L	1270	860	720	210
XL	1390	940	780	220

Invacare **Comfort Sling, Comfort Sling with Hygiene Opening and Comfort Standard Sling**

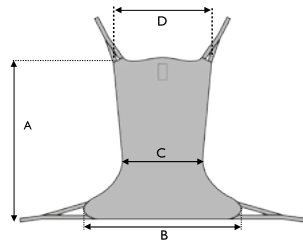


This sling model is available in two versions. The **Comfort Sling** supports the whole body - including the head. The **Comfort Standard Sling** supports the whole body including the shoulders, but not the head. The latter is suitable for those who have good head control.

The Invacare range of **Comfort Slings** is ideally suited to those with chronic pain. The sling is also useful for above-knee amputees or for those, who due to postural position or pain, prefer to remain in a seated position.

Both the **Comfort** and **Comfort Standard Sling** can be put on while the user is lying down and used to transfer the individual into a seated position.

The **Comfort Sling** with Hygiene Opening is worn the same way and used for toileting.

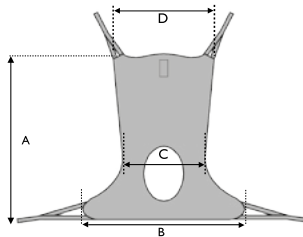


Invacare **Comfort Sling with and without Toileting Opening**

Size	A	B	C	D
XS	1040	970	510	960
S	1150	1050	550	1020
M	1395	1170	610	1160
L	1445	1290	675	1280
XL	1460	1330	745	1470

Invacare **Comfort Standard Sling**

Size	A	B	C	D
S	960	1050	550	1000
M	1020	1170	610	1050
L	1160	1290	675	1110

Invacare **Amputee Sling**

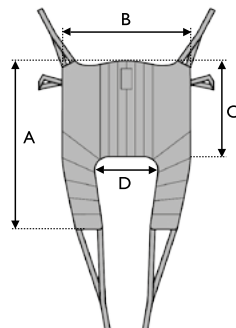

This model is specially designed for the safe transfer of users with single or double amputations. The sling affords the user comfortable support in a slightly reclined position. It is important that the user has good head control.

Size	A	B	C	D
S	1100	930	490	320
M	1160	970	520	360
L	1360	1230	570	430
XL	1400	1350	570	480

## Invacare **Standing Transfer Vest**




The **Standing Transfer Vest** is applied around the trunk. The user's feet must be on the floor and the user must also have some standing ability. The **Standing Transfer Vest** is used for those with reduced balance and/or walking ability. The vest is especially useful for toileting. It also reduces the risk of injury to the carer.

A groin support is available as an accessory to the **Standing Transfer Vest**. This helps keep the vest in place.

### Invacare **Standing Transfer Vest with Groin Band**

Size	A	B	C	D
XS	150	750	850	300
S	150	850	850	300
M	180	950	950	350
L	180	1050	950	350
XL	180	1200	950	350

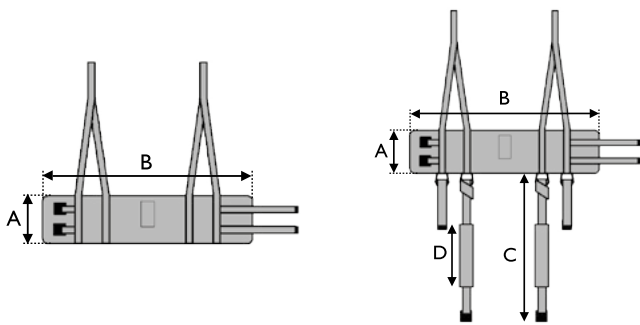

### Invacare **Standing Transfer Vest**


Size	A	B
XS	150	750
S	150	850
M	180	950
L	180	1050
XL	180	1200

## Invacare **Transfer Support**




This two-part sling is used for those with limited mobility. This sling can be chosen for those who require support but wish to move independently.

Size	A	B
Short	930	120
Long	1120	120

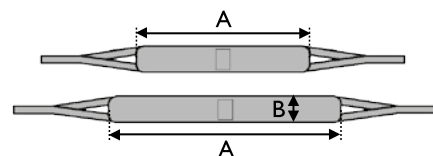


## Invacare **Stand Assist Sling**

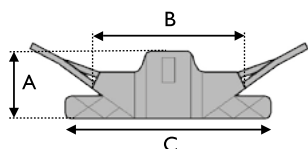

Designed for use with the **Reliant 350** lifter, but compatible with other types of Stand Assist lifter\*, the **Invacare Stand Assist Sling** offers a comfortable and safe transfer for users who have some weight bearing ability, but cannot stand independently. In addition, the user should be coherent, have good body and head control and the ability to grip the lift boom during the transfer. The sling is well padded and comes with non-slip material on the back to ensure a comfortable transfer position.

Size	A	B	C
S	260	840	950
M	330	895	1170
L	360	950	1270
XL	360	1020	1810

## Invacare **Transfer Stand Assist Sling**

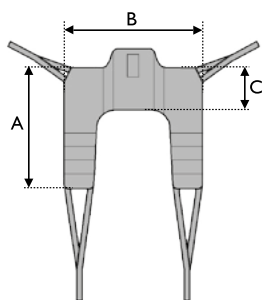

The **Invacare Transfer Stand Assist Sling\*** can be used with the **Reliant 350** lifter and enables a simple seated transfer. This sling is used for those with good body and head control who are able to grip the lift boom.

Size	A	B	C
S	860	840	240
M	1020	895	260
L	1180	950	330
XL	1230	1020	350

\* Please refer to the Lifter manufacturer's compatibility instructions

## Sling accessories



Fleece padding for extra comfort



Leg insert, Foam. For **Universal Standard**, **Universal High** and **High Plus** in Net Material



Extension loops - in order to give more positioning possibilities



Head support stiffeners - for Slings with head support. **Easy Fit**, **Universal High**

Sling Types	Fabric Matrix			Size Matrix				
	Solid	Net	Spacer	XS	S	M	L	XL
Universal Low	●				●	●	●	
Universal Standard	●	●	●		●	●	●	●
Universal High	●	●	●	●	●	●	●	
Universal High Plus	●	●	●		●	●	●	● <sup>1</sup>
Dress Toileting	●				●	●	●	
Dress Toileting High	●				●	●	●	●
Easy-fit	●		●	●	●	●	●	●
Amputee	●				●	●	●	●
Comfort		●	●	●	●	●	●	●
Comfort Toilet	●	●		●	●	●	●	●
Comfort Standard		●	●		●	●	●	
Standing Transfer Vest	●			●	●	●	●	●
Stand Assist	●				●	●	●	●
Transfer Stand Assist	●				●	●	●	●

1. Only available in solid material

## Fabrics



### Solid Polyester

A standard material that is easy to apply – slides smoothly and dries quickly after washing.



### Net\*

Used for bathing or if the user will stay seated on the sling for a period of time.




### Spacer\*

A slightly elastic fabric that adapts to the user's body shape. Well suited for those with skin integrity issues.


\* It is important to carry out a risk assessment with reference to skin integrity for those who be seated on a sling for long periods

## Size guide

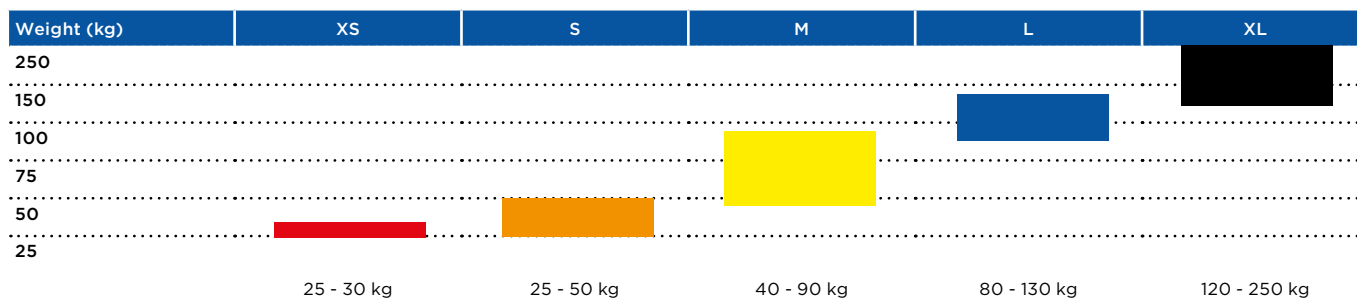

## Technical data

For more comprehensive pre-sales information about this product, including the product's user manual, please see your local Invacare website.

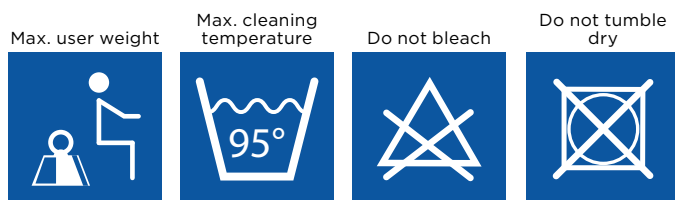

**Slings**  
200 kg<sup>1</sup>  
XL sizes: 250 kg

All slings should be tested and adjusted to the individual user to ensure optimal comfort and safety during transfers.

1. Except for Standing transfer vest in combination with groin band: the safe working load is 159 kg.

QR Code

Tel: +41 61 487 70 70  
hqeurope@invacare.com  
www.invacare.eu.com

Invacare International GmbH  
Benkenstrasse 260  
4108 Witterswil  
Switzerland

