

# NordBed™ Ultra

*Designed for Clients with Dementia*



ERGOMOVE™ TECHNOLOGY | SUPREME SAFETY | FALL PREVENTION



# NordBed Ultra

Designed for Clients  
with Dementia

The NordBed Ultra can be configured to meet the special requirements for clients suffering from dementia.

To ensure supreme safety for clients with dementia it is imperative that their care bed supports their independence and mobilisation.

With the low height of only 25 cm in combination with the under-bed light, this reduces the risk for clients to get injured while trying to get in/out of bed during the nights, to increase safety a crash mattress can be used to support the client if any falls occur. The use of side rails could also be used to prevent clients from falling out of bed.

To promote the working conditions of the care staff, the NordBed Ultra is able to reach a top height of 85 cm. The NordBed Ultra can evolve and adapt to the progress of the dementia stages with the client.

## Homely aesthetics

Equipped with the soft covers available on bed ends and side panels, gives the NordBed Ultra a homely, tranquil feel for the client. The soft cover provides a textured/smooth surface – a surface the client can feel. The slightly high/low fabric allows a tactile sensory stimulation.

People suffering from dementia can have difficulties to recognise new surroundings. To be able to adapt the NordBed Ultra to be as close to the client's normal environment is essential.

The wide range of bed ends, side panels and side rails in a variety of wood decors allows configuration to meet a recognisable aesthetical look.



## Perfect for clients suffering from dementia

- ▶ The low height allows safe environments for people who are mobile
- ▶ The highly configurable bed can be adapted:
  - To look like the more traditional bed
  - To fit in nicely with modern homely looking beds (box mattresses) for easy recognition
  - With a range of soft covers (high/low texture) for a cozy, confident and safe environment
  - Under bed light to reduce the risk of the client being injured while trying to get in/out of bed

4

### EXCLUSIVE FEATURES

1

#### ErgoMove technology

- ▶ Brings the client to a comfortable sitting position and back to horizontal without pushing the client downwards and without the need of repositioning.
- ▶ Reduces shear

3

#### Up-and-Out™

- ▶ Allows safe and smooth transfers
- ▶ Supports ergonomically correct movement

2

#### 25 cm lowest height

- ▶ With standard castors at 10 cm to allow safe transfers and increase fall prevention

4

#### Optimised dimensions of mattress support

- ▶ According to anthropometric measurement, for increased comfort

### Additional features

- ▶ **RememberMe™:**
  - Programmable LCD handset
  - ErgoMove and Up-and-Out can be adjusted to fit individual needs
- ▶ **Integrated extendable mattress platform**
  - Additional 10 cm at head and foot end to accommodate clients' requirements
- ▶ **Anti-Trendelenburg and *Optional* Trendelenburg**
  - Supports care staff when repositioning the client in bed
  - Easy to redistribute pressure
- ▶ **Under bed light**
  - Easy to turn on the light
  - Fall prevention/safety




## Hand control ▲

Easy recognisable colours for clients who are weak-sighted

- ▶ The key buttons, which will be used most frequently have easily identifiable colours. These colours have been chosen in collaboration with experts
- ▶ To assist the client when selecting a feature, all buttons will light up when any button is touched for easy recognition during the day or night
- ▶ For easy selection, the key buttons are positioned at the top of the handset and the adjustment features are placed below the up/down buttons
- ▶ The hand control is lockable as standard

## Technical data ▼

For more comprehensive pre-sales information about this product, including the product's user manual, please see your local Invacare website.

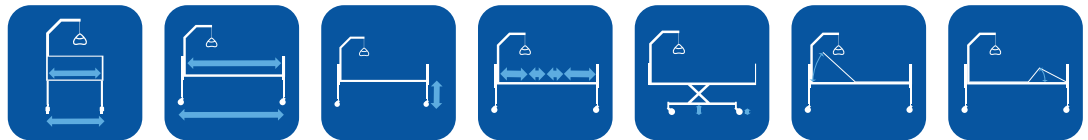

Width outside Width inside ▼	Length outside Length inside ▼	Height adjustment ▼	Mattress support dimensions ▼	Under bed clearance ▼	Backrest angle ▼	Thigh angle ▼
---------------------------------	-----------------------------------	---------------------	-------------------------------	-----------------------	------------------	---------------

<b>NordBed Ultra</b>	100 / 90 cm	213 / 200 cm	25 - 85 cm	80-17-30-65 cm	15 cm	0 - 70°	0 - 49°
----------------------	-------------	--------------	------------	-------------------	-------	---------	---------




Legrest lift ▼	Anti-Trendelenburg / Trendelenburg ▼	Max. user weight ▼	Total weight (without bed ends) ▼	Heaviest part of the product ▼	Washable in washing machine ▼
----------------	--------------------------------------	--------------------	-----------------------------------	--------------------------------	-------------------------------

<b>NordBed Ultra</b>	0 - 19°	+/- 15°	200 kg	105 kg	55 kg	Yes
----------------------	---------	---------	--------	--------	-------	-----

**NB:** The bed must not be used by patients under 12 years of age, or by patients with body size equivalent to an average 12 year old or smaller.

## Frame colour ▼ Woodwork ▼

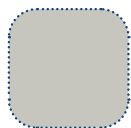

Aluminium grey



Beech



Oak



Umber



Beech/  
pale blue



Pale blue



White

Tel: +41 61 487 70 70  
hqeurope@invacare.com  
www.invacare.eu.com

Invacare International GmbH  
Benkenstrasse 260  
4108 Witterswil  
Switzerland



Yes, you can.®

© 2019 Invacare International GmbH. All rights reserved. All information quoted is believed to be correct at time of print.

Invacare reserves the right to alter product specifications without prior consultation. NordBed Ultra dementia - EU - 02/2019

