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| | | |
|-----|-----|---------------------|
| F | 1. | Guide utilisateur |
| GB | 2. | User's manual |
| D | 3. | Gebrauchsanweisung |
| E | 4. | Manual del usuario |
| I | 5. | Manuale dell'utente |
| NL | 6. | Gebruiksaanwijzing |
| P | 7. | Guia do utilizador |
| S | 8. | Bruksanvisning |
| DK | 9. | Brugsanvisning |
| FIN | 10. | Käyttäjän käsikirja |
| N | 11. | Bruksanvisning |

I. FRANCAIS

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2. ENGLISH

You have just purchased an INVACARE® product. Our products are designed and manufactured to the most rigorous quality and safety standards and conform to the European directives (MDD 93/42/EEC). They are guaranteed for one year. Certain products may have a longer guarantee; please check with your dealer.

The following operating and maintenance instructions must be observed:

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the restrictions on use, the adjustment or maintenance of the product, please contact your dealer.
- Do not exceed the maximum operating load  or the maximum load  shown on the product label.
- Always ensure that the product is suitable for your requirements; in case of doubt, check with a health professional.
- Worn, damaged or missing components must be replaced immediately by an approved repairer.
- Do not leave or store the product out of doors.
- Periodically check the stability and rigidity of the product and, where applicable, that the screws are correctly tightened and the rubber end caps in good condition.

C426AB KITCHEN HIGH CHAIR

- To install the back, remove the clips from the rear tubes. Insert the backrest tubing and position the clips in the holes.
- To install the feet, remove the clips, insert the four extension pieces and position the clips in the holes at the desired height. All four feet should be set to the same height.
- Ensure that all the clips are correctly positioned and that the chair is stable before use.

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- The grippers close by operating a lever on the handle.
- The end of the grippers has a magnet to facilitate picking up small metallic items.
- The "Plus" versions are covered in non-slip material and also have a clip to hang the grippers on one of the tubes on a chair and a hook on the end for picking up a handbag for example.

IN01 SOCK PULLER

- Knot the ends of the belt to improve the efficiency of the sock puller.
- Slide the sock over the uncurved plastic part with the heel of the sock downwards. The sock should not be higher than the belt fixing.
- Sit down and place the sock puller on the ground while holding on to the strap.
- Place the strap behind your leg, bend your knee, place your heel on the ground and slide your foot into the plastic part.
- Lift your heel, straighten your knee, point your toe and then pull on the belt to pull the sock up to the height of your knee. Remove the sock puller.

IN02 TIGHTS PULLER

- Proceed in the same way as for the sock puller but with both feet at the same time.

MAINTENANCE

Clean the C426AB kitchen chair regularly with a normal household cleaner. Use a soft, damp cloth to clean dirt and dust from the other products. Use soap and water to remove stubborn stains. Dry with a soft cloth. For disinfection in hospital environments, please follow the hygiene specialist's recommendations.

USER'S MAXIMUM WEIGHT

Please see the table at the end of this document.

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