DOLOMITE® Legacy™
Rollator

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DE

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FR

User Manual

EN

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IT

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NL

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
LT

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ES

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PT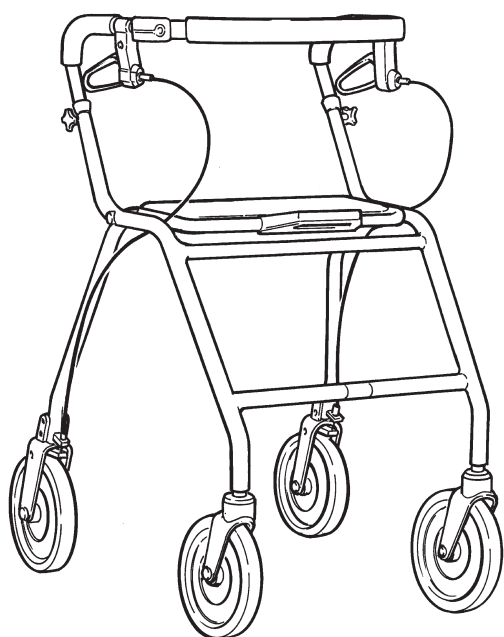

CE

DOLOMITE

Yes, you can.®

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User Manual

| | |
|-----------------------------------|-----------|
| 1 GENERAL | 21 |
| General Information..... | 21 |
| Symbols | 21 |
| Product Life Cycle..... | 21 |
| Product Labeling..... | 21 |
| User Weight Limitation..... | 22 |
| 2 SAFETY | 22 |
| Intended Use..... | 22 |
| General Warnings..... | 22 |
| 3 USAGE | 24 |
| Operating Position..... | 24 |
| Transport Position..... | 24 |
| Folding the Rollator..... | 24 |
| Adjusting the Handle Height | 24 |
| Adjustment of Brake Effect..... | 25 |
| Using the Hand Brake..... | 25 |
| Using the Parking Brake | 25 |
| Mounting of Basket..... | 26 |
| Mounting of Tray | 26 |
| Using the Kerb Climber | 26 |
| 4 CARE AND MAINTENANCE | 27 |
| 5 AFTER USE | 27 |
| Disposal..... | 27 |
| Reuse | 27 |
| 6 WARRANTY | 28 |

I GENERAL

I.1 General Information

Study the operating instructions carefully before use. The person who prescribes the product or who is otherwise responsible for the user must ensure that the product is suitable for the user.

I.2 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

- ① Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the EC declaration of conformity.


I.3 Product Life Cycle

The expected product life cycle of this product is 5 years when used in accordance with safety instructions, maintenance intervals and correct use, stated in this manual. The effective product life cycle can vary according to frequency and intensity of use.

I.4 Product Labeling

The label on the **DOLOMITE LEGACY** provides all important technical information.

- 1: Article number
- 2: Product name
- 3: Serial number
- 4: Date of manufacture
- 5: Max. user weight
- 6: NB! See user manual.
- 7: In- and outdoor rollator.
- 8: Max. width of rollator.
- 9: Manufacturer's name.

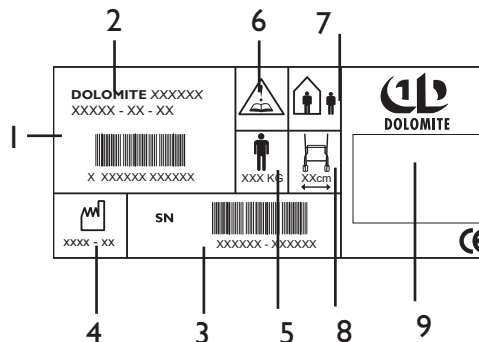1.5 User Weight Limitation

Your walker should only be used as a walking aid! User's max. weight: **LEGACY** 125 kg (275 lbs., 20 stones).

INVACARE will not accept responsibility for adaptations, faulty repairs, etc, that are outside their control. *

* The weight limitation includes the contents of the basket or tray and the weight of the user whether standing or sitting.

2 SAFETY

- ⓘ Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

2.1 Intended Use

The rollator is intended to support a person during walking and may also serve as a resting seat. The rollator is also intended to be able to carry a load either in the basket or on the tray.

The rollator is suitable for use both indoors and outdoors. The rollator must not be used under water, for example in a pool or open air bath, neither in hot areas, for example in saunas.

The user manual shall always be handed over to the user.

2.2 General Warnings

**WARNING**

Exercise caution when walking on sloping ground.

**WARNING**

Never lift the rollator in the folding handle when it is in use.

**WARNING**

Not to be used for standing on or as a wheelchair.

WARNING

Do not go up or down stairs with the rollator.

Do not use the rollator on escalators.

Do not under any circumstances stand on the walker.

If the push handles or the seat are exposed to extreme temperature (above 41°C) there might be risk for first degree burns.

The walker must be fully unfolded when used as walking support

Use seat for stationary seating only, both brakes must be in parking position when the rollator is used as a seat.

Never sit and push with your feet.

The handlebar must not be angled outwards as this reduces the rollator's lateral stability.

The parking brakes must be in braked position when the rollator is used as a support when getting up, for example from a chair.

Exercise caution when there is a heavy load in the basket. Loads should solely be transported in the basket or on the tray.

Max load for the 520 and 600 models:

In the basket: 9 kg (20 lbs) and on the tray: 5 kg (11 lbs).


Max load for the 450 model:

In the basket: 7 kg (15 lbs) and on the tray: 5 kg (11 lbs).

3 USAGE

3.1 Operating Position

To reset to walking position, press firmly downwards on the seat until a clicking sound is heard. In order to increase accessibility, the seat can be folded up to provide a larger walking space.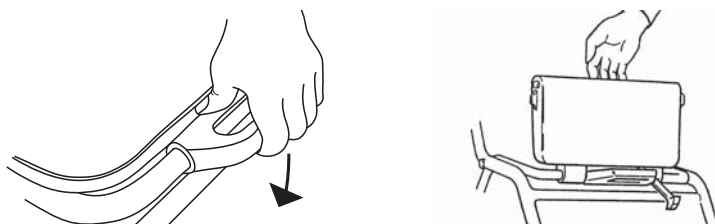


WARNING

Always check that the safety latch is in locked position.

3.2 Folding the Rollator

To fold the rollator, lift up the handle. When folded, the rollator is easy to lift and carry.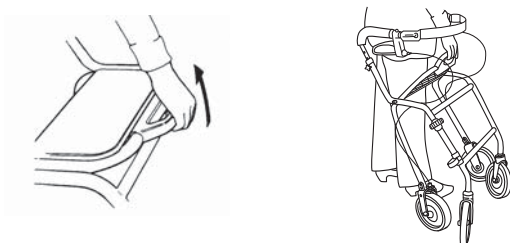

3.3 Adjusting the Handle Height


To adjust the handle to a suitable height, stand behind the walker with the shoulders level. The handles should be at the same height as the user's wrist. A therapist may suggest different handle heights to accommodate the special needs of a user.

Loosen the knob completely, remove the screw. Adjust the handle to suitable height. Fix the screw into the hexagonal hole and fasten the knob properly.

3.4 Adjustment of Brake Effect

For desirable braking strength, adjust the brake wire. Suitable tools 1x8 mm and 1x10 mm spanner (not included).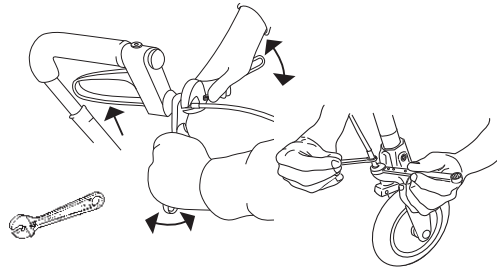


WARNING

Make sure to tighten the lock nuts after adjustment.

3.5 Using the Handbrake

Apply the brakes by pulling both brake handles upwards.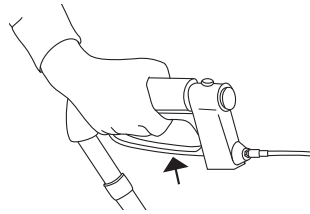


3.6 Using the Parking Brake

Press down carefully for parking brake. A clear sound is heard when the brake is placed in correct position.

- ① It is important that both brakes are put into parking position.

To release the brake pull the brake handle upwards.

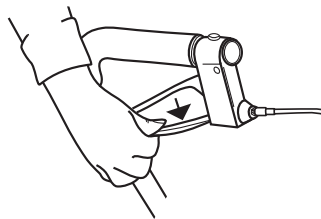
3.7 Mounting of Basket

Mounting of basket.


3.8 Mounting of Tray

Secure the tray using the plastic plugs.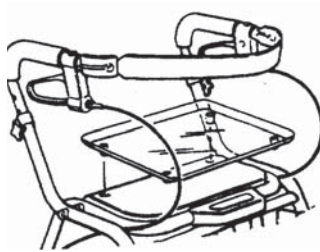

3.9 Using the Kerb Climber

Kerb climber.

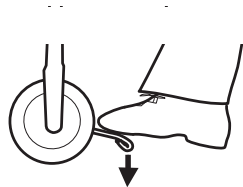4 CARE AND MAINTENANCE

- i** Maintenance should be done at least once a year. Do not use an abrasive detergent, cleaner or cloth to clean or dry the walker.

To ensure good performance and safety you should regularly check that the following is carried out:

- Clean the rollator with solvent-free household cleaner. Keep the wheels clean.
- Ensure that screws and adjustments are properly tightened.
- Immediately contact the nearest dealer if a fault should appear.
- Never use a defective walker. Do not attempt repairs by yourself!
- Verify brake operation. If brake adjustment is required, see section 3.4 in the user manual.
- If hand grips are loose Do not use the rollator. Change hand grips according to the service instructions which can be obtained from your Invacare representative.
- Periodically inspect the castors tightness, wear and damage.
- Make sure that the wheels are free of hair, lint and other debris.

5 AFTER USE

5.1 Disposal

When the product can no longer be used, it and its accessories as well as the packaging material should be sorted in the correct way. The individual parts can be separated and handled according to the list specification.

Frame: Polyester powder coated steel tubes.

Handle bars: Chromated steel.

Handgrips: TPE.

Wheels: Polypropene with treads of TPE

Plastic parts: Material according to marking (PA, PP, PE)

Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

5.2 After Use

WARNING: Risk of Injury

Always inspect the product according to the service instructions before transferring the product to a different user. If any damage is found Do not use the product. Otherwise injury may occur.

The service instructions can be obtained from your Invacare representative.

6 WARRANTY

We supply a 2 year guarantee ** from the date of delivery. Damage caused by wear to items such as wheels, handles and brakes is not covered by the guarantee. Damage that arises due to abnormal use is excluded. Damage caused by users who weigh more than 125 kg is excluded. Maintenance and operating instructions must be observed for the guarantee to be valid.

 ** Deviations may occur due to local agreements.

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Invacare® AG,
Invacare Rea AB, Box 200