Aquatec® Ocean / Ocean VIP / Ocean Dual VIP

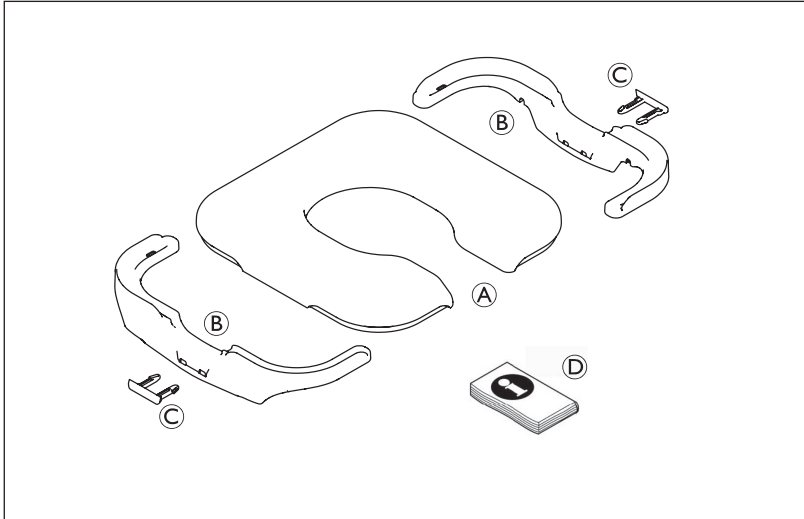
Fig. 1

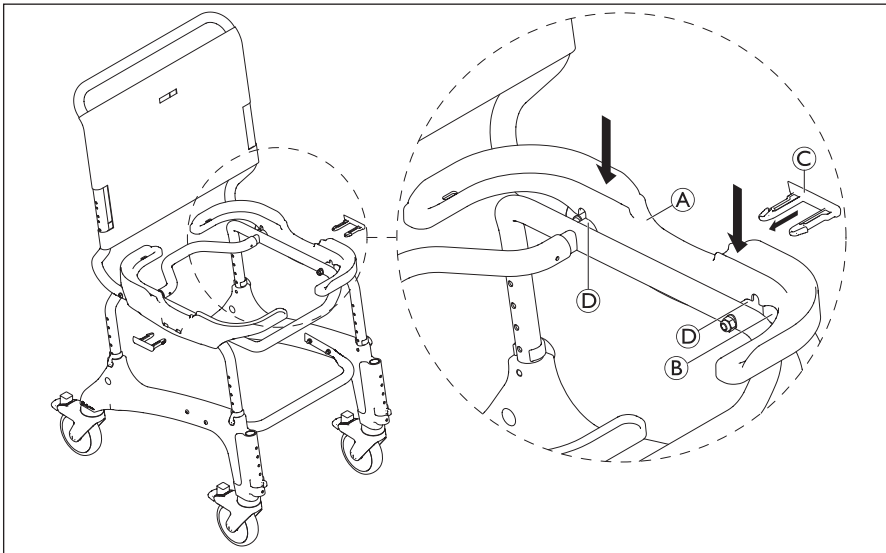
Fig. 2

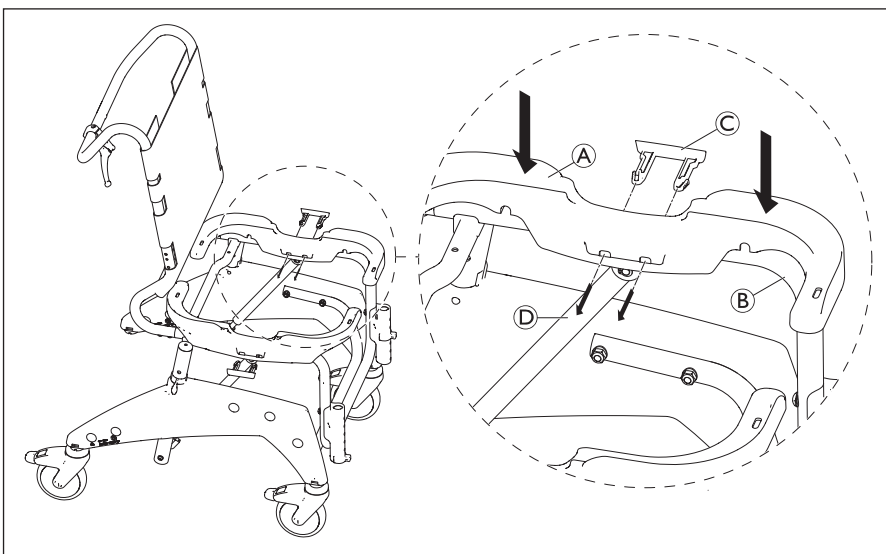
Fig. 3

- EN** Variable soft seat
Assembly instruction
- DE** Variabler Softsitz
Montageanweisung
- FR** Coussin d'assise variable
Instruction de montage
- IT** Sedile morbido variabile
Istruzioni di montaggio
- ES** Asiento blando variable
Instrucciones de montaje
- NL** Variabele zachte zitting
Montageaanwijzing
- DA** Variabelt blødt sæde
Monteringsanvisning
- PL** Miękkie siedzisko ze
zmiennym ustawieniem
Instrukcja montażu
- FI** Muunneltava pehmeä istuin
Kokoamisohjeet
- NO** Varierbart myksete
Monteringsanvisning
- SV** Justerbar mjuk sits
Monteringsanvisningar
- PT** Assento flexível variável
Instruções de montagem
- CS** Variabilní měkké sedátko
Pokyny k sestavení
- LV** Maināms mīkstsais sēdekļis
Uzstādīšanas norādījumi
- ET** Muudetav pehme iste
Koostejuhend

This manual must be given to the user of the product.
Before using the product, read this manual and save it
for future reference!

AQUATEC

Yes, you can.®

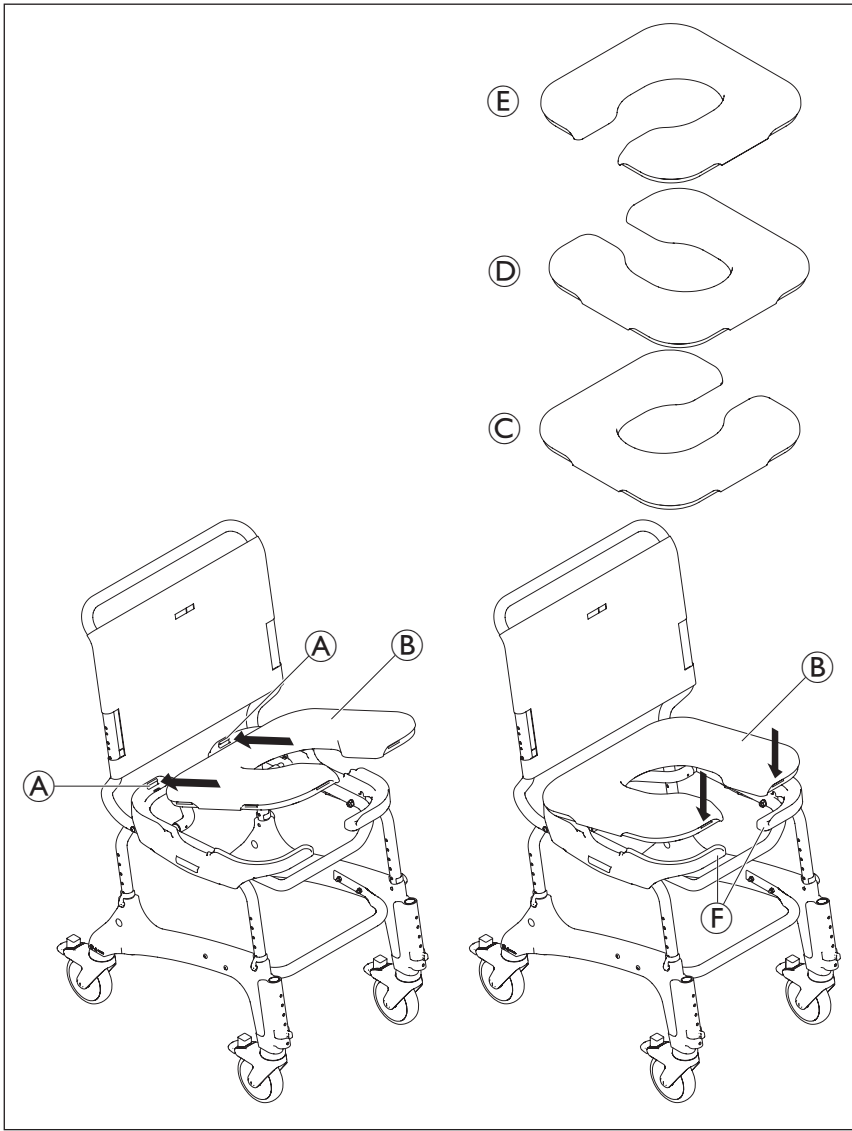
Fig. 4

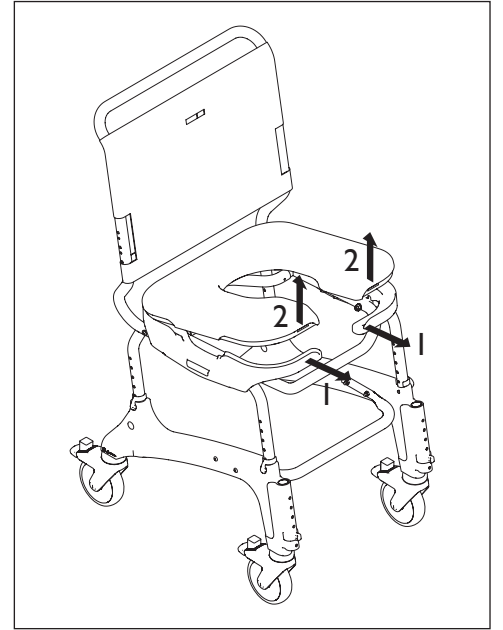
Fig. 5

Intended use

The variable soft seat is intended solely for use in conjunction with Aquatec Ocean / Ocean VIP / Ocean Dual VIP for variable usage of the hygiene recess. Any other use is prohibited.

Maximum user weight: 150 kg

Warranty / Service Life

See user manual for the Ocean shower and commode chair.

Scope of delivery (Fig. 1)

The following items are included in the scope of delivery:

A	Soft seat
B	Positioning adapters (2x)
C	Clips (2x)
D	Assembly instruction

The variable soft seat can be used on the Aquatec Ocean from serial number 14403200.

Safety**WARNING** Risk of injury!

- ▶ Do not use if defective.

**CAUTION**

- ▶ Check product regularly for damage and to ensure that it is securely assembled.
- ▶ Be careful during assembly.

**IMPORTANT**

- ▶ If necessary, the toilet pan holder must be fitted before the variable soft seat.
- ▶ The variable soft seat attachment increases the seat height by around 70 mm. The seat height, arm rest and footrest settings of the shower and commode chair may need to be adjusted.

Fitting the variable soft seat

1. Apply brakes.
2. Lift the seat plate of the Ocean shower and commode chair up and out.
3. Take clips C out of positioning adapter B (engaged when delivered).

For Aquatec Ocean (Fig. 2):

4. Place positioning adapter A on the outer frame tube B between the two saddles D.

For Aquatec Ocean VIP / Ocean Dual VIP (Fig. 3):

4. Place positioning adapter A with the holes for the right and left clips next to gas spring holder D on frame tube B.
5. Insert clip C from the outside with the curved section at the bottom into positioning adapter A.

**IMPORTANT**

- ▶ Ensure that the clips fully engage with a clearly audible "click" and that they are seated firmly and correctly.

6. Fit the second positioning adapter to the other side, mirror-inverted in the same way.

▶ Fig. 4

7. Insert the variable soft seat B with the commode seat aligned as required at a slight angle into the rear retainers A of the positioning adapters.
8. Press down the front of the variable soft seat B so that it engages in the retainers of the positioning adapters. Hold back the lugs F of the positioning adapters.

**IMPORTANT**

- ▶ Ensure that the variable soft seat is positioned correctly in the positioning adapters and engages at all four recesses.

The commode seat of the variable soft seat can be removed and reinserted so it positioned as shown in C, D or E.

Removing the variable soft seat (Fig. 5)**IMPORTANT**

- ▶ Before removing the variable soft seat, make sure that it is released from the front retainers of the positioning adapters.

1. Pull the lugs of the positioning adapters forwards slightly.
2. Remove the variable soft seat upwards.

Cleaning

Please note the cleaning instructions in the user manual for the Ocean shower and commode chair.

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3. Haal de clips © uit de plaatsingsadapter ② (zitten bij levering vastgeklipd).
Algol-Trehab Oy
Aquatec Operations GmbH