Aquatec® Accessories

OCEAN / OCEAN VIP / OCEAN DUAL VIP / OCEAN E-VIP

| | | |
|----|--------------------------------|----|
| en | Backrest | |
| | User Manual | 1 |
| de | Rückenlehne | |
| | Gebrauchsanweisung | 3 |
| fr | Dossier | |
| | Manuel d'utilisation | 4 |
| it | Schienale | |
| | Manuale d'uso | 5 |
| es | Respaldo | |
| | Manual del usuario | 6 |
| pt | Encosto | |
| | Manual de utilização | 6 |
| nl | Rugleuning | |
| | Gebruiksaanwijzing | 7 |
| da | Ryglæn | |
| | Brugsanvisning | 8 |
| no | Rygg | |
| | Bruksanvisning | 9 |
| sv | Ryggstöd | |
| | Bruksanvisning | 9 |
| fi | Selkänöja | |
| | Käyttöohje | 10 |

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

I General

I.1 Images

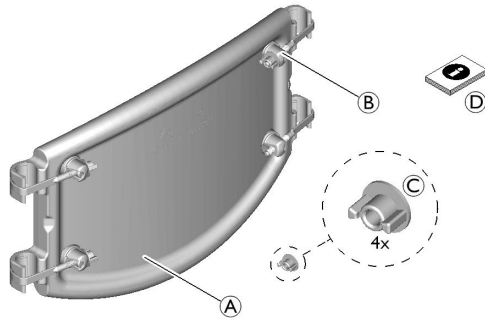
Fig. 1

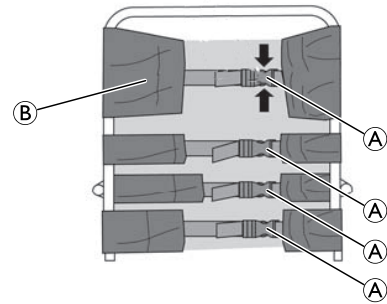
Fig. 2

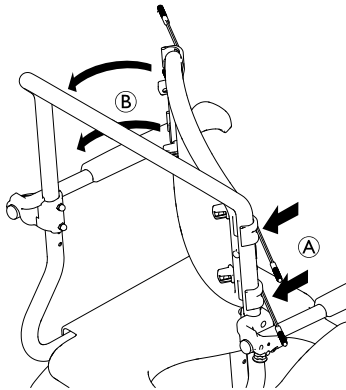
Fig. 3

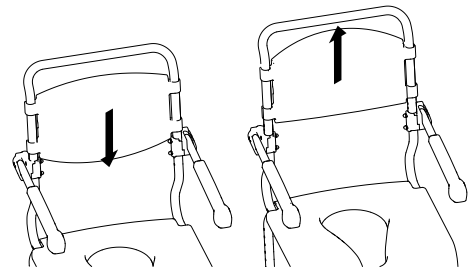
Fig. 4

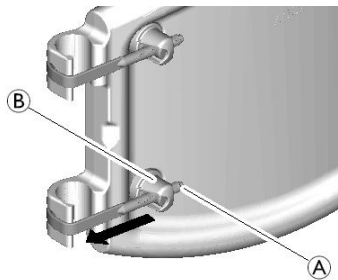
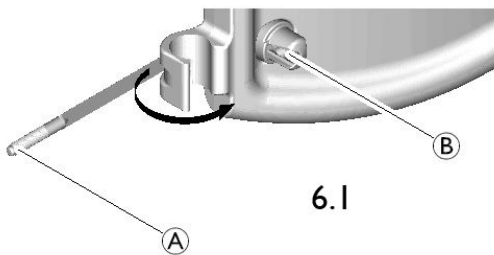
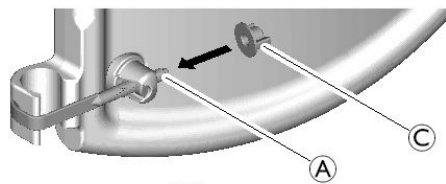
Fig. 5

6.1

6.2

6.3

Fig. 6

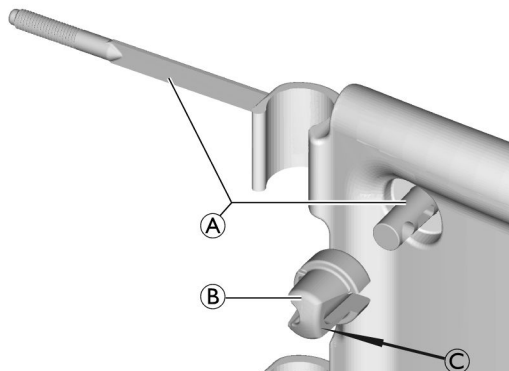
Fig. 7

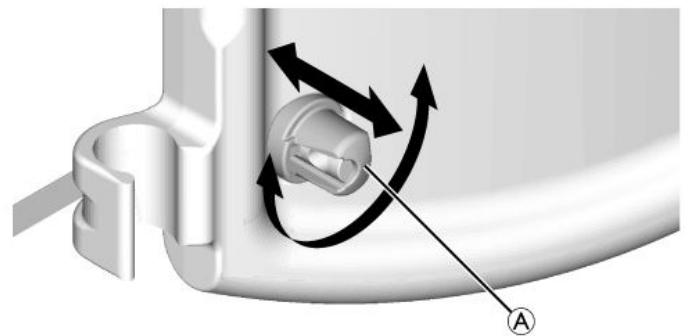
Fig. 8

1.2 Intended use

! IMPORTANT!

- Read sections General, Safety, Maintenance and After use contained in the Aquatec® OCEAN, OCEAN VIP, OCEAN DUAL VIP or OCEAN E-VIP user manual in conjunction with this document.

This product is to be used solely in combination with the mobile shower and toilet commode Aquatec® OCEAN, OCEAN VIP, OCEAN DUAL VIP or OCEAN E-VIP, to provide a stable seating. The combined product is intended to be used as an aid for sitting in the bathroom or in the shower for people with sufficient torso stability. Any other use is prohibited.

Operating temperature: 10 – 40 °C

1.3 Scope of delivery (Fig. 1)

The following components are included in the scope of delivery:

| | |
|---|--|
| Ⓐ | Backrest (PE HD) |
| Ⓑ | Tension straps (PA6.6) with puller sleeves (PA6.6), pre-mounted (4x) |
| Ⓒ | Wing nuts (PA6.6) (4x) |
| Ⓓ | User manual |

1.4 Service life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

2 Safety information

WARNING!

Risk of injury

- Do not use if defective.
- Check product regularly for damage and to ensure that it is securely assembled.
- Before each use, check the backrest to ensure it is securely fastened.
- Observe the nominal load of the chair.
- Fit the product carefully.
- Only fit the backrest with the curve to the back.
- In addition, observe all safety information stated in the Aquatec® OCEAN, OCEAN VIP, OCEAN DUAL VIP or OCEAN E-VIP user manual.

3 Assembling the backrest

CAUTION!

Risk of pinching

- Do not insert fingers between the backrest and the frame tubes when attaching the backrest.

1. Open buckles Ⓐ and remove the backrest cover Ⓑ (Fig. 2).
2. Pull the threaded bolts Ⓐ of the tension straps out of the four puller sleeves Ⓑ (Fig. 5).
3. Attach the backrest first on one side Ⓐ, and then on the other side Ⓑ (Fig. 3).
4. Re-insert the threaded bolts of the tension straps Ⓐ into the four puller sleeves Ⓑ (Fig. 6, 1).
5. Place the wing nuts Ⓒ onto the threaded bolts Ⓐ and screw them in by a few turns (Fig. 6, 2).
6. Finger tighten the four wing nuts Ⓒ crosswise (max. torque 1 Nm) (Fig. 6, 3).

The backrest can be attached with the armrests in the lower and upper positions (Fig. 4). To this end, the backrest can be turned 180° in order to adjust the distance between the seat plate and the backrest accordingly.

The product is disassembled in the reverse order.

4 Cleaning and disinfection

Instructions for cleaning and disinfection are given in the user manual Aquatec® OCEAN, OCEAN VIP, OCEAN DUAL VIP or OCEAN E-VIP shower chair in section “Cleaning and disinfection”.

For cleaning purposes the tension straps with puller sleeves can be dismantled:

1. Turn the puller sleeves Ⓐ slightly to the left or right and take them out of the backrest (Fig. 8).

WARNING!

Risk of injury due to wrong assembly

- When re-attaching the puller sleeves Ⓑ to the backrest make sure that the flat surfaces Ⓒ of the puller sleeves are positioned vertically and face to the center of the backrest (Fig. 7).

2. Re-attach the puller sleeves by clicking it with soft pressure on the bolt of tension strap Ⓐ (Fig. 7).

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- Placez les écrous papillon © sur les boulons filetés A et vissez-les de quelques tours (Fig. 6.2).
Invacare AB
Algol-Trehab Oy
Aquatec Operations GmbH