Aquatec® Ocean VIP / Dual VIP

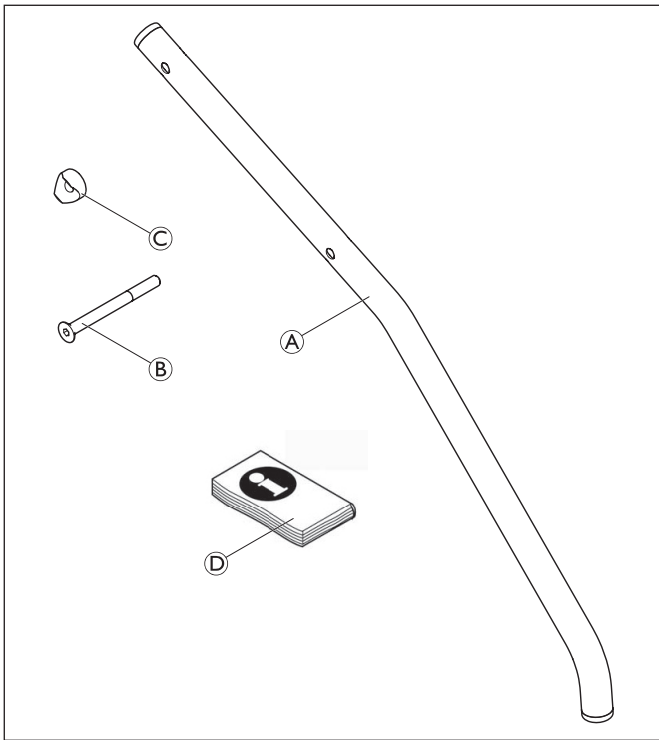
Fig. 1

- | | | | |
|-----------|--|-----------|--|
| EN | Front antitipper
Assembly instruction | FI | Etujalkojen keikahduksen estin
Kokoamisohje |
| DE | Vordere Kippstützen
Montageanweisung | NO | Tippsikring foran
Monteringsanvisning |
| FR | Dispositif anti-bascule avant
Instruction de montage | SV | Främre tipskydd
Monteringsanvisning |
| IT | Dispositivo antiribaltamento anteriore
Istruzioni di montaggio | PT | Mecanismo anti-basculante frontal
Instruções de montagem |
| ES | Dispositivo antivuelco frontal
Instrucciones de montaje | CS | Přední ochranný nástavec
Pokyny k sestavení |
| NL | Antikantelsteun voorzijde
Montageaanwijzing | LV | Priekšējie pretapgāšanās ritentiņi
Uzstādīšanas norādījumi |
| DA | Fronttipsikring
Monteringsanvisning | | |

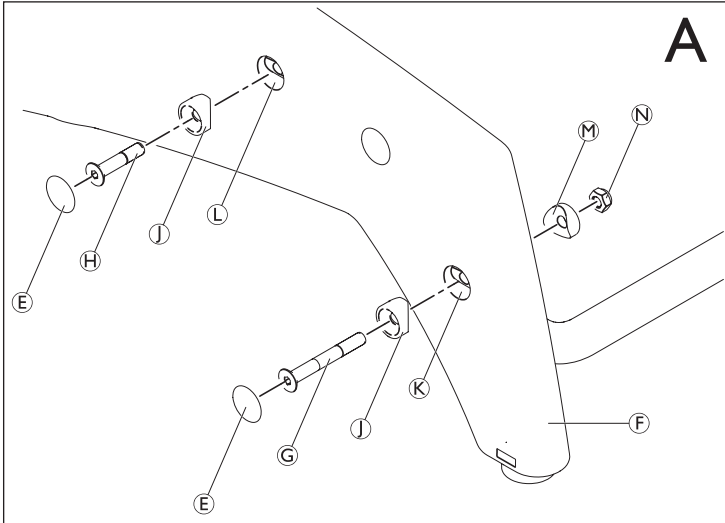
Fig. 2

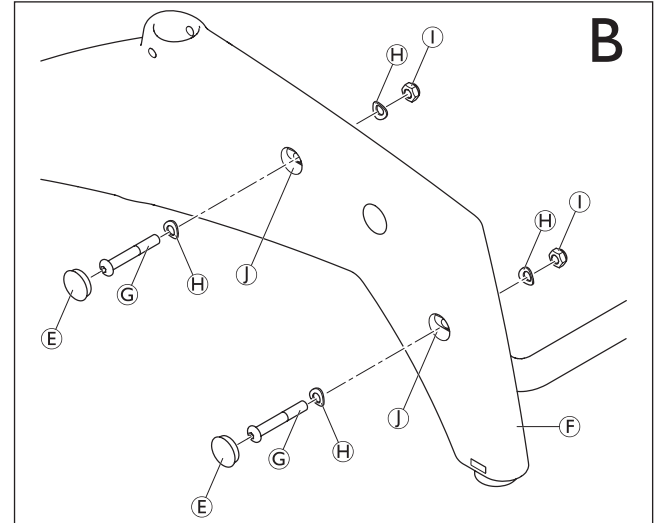
Fig. 4

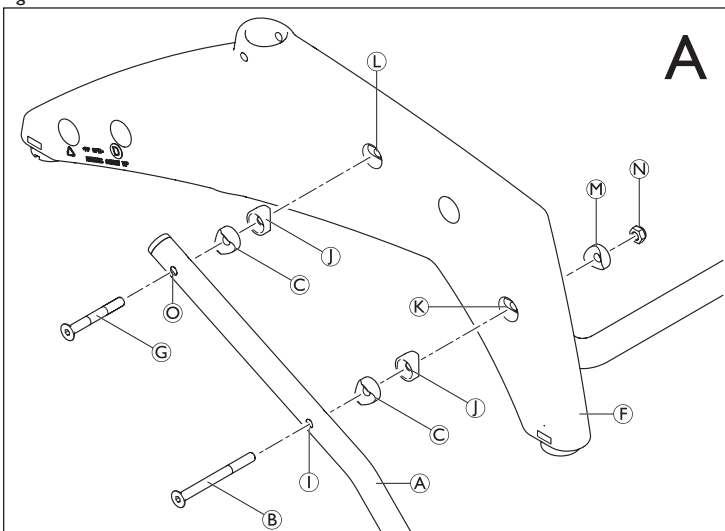
Fig. 3

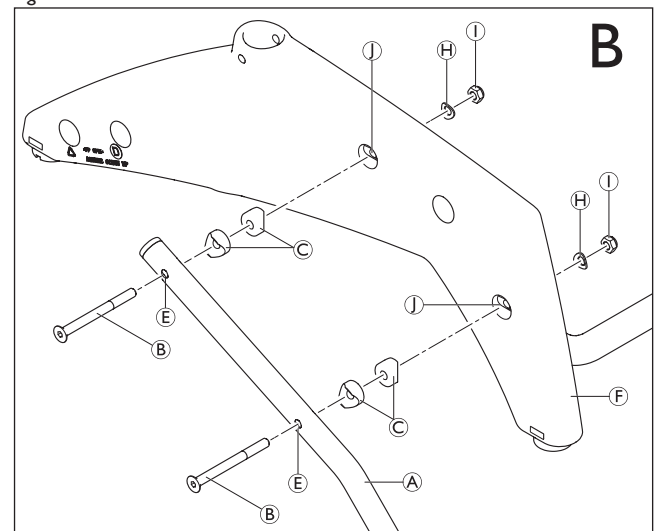
Fig. 5

This manual must be given to the user of the product. Before using the product, read this manual and save it for future reference!

AQUATEC®

Yes, you can.®

Intended use

The front antitippers are exclusively specified for use with Aquatec® Ocean VIP/ Ocean Dual VIP as protection against the shower stool tipping to the front. Any other use is prohibited.

Warranty

See user manual for the Ocean shower chair.

Scope of delivery (Fig. 1)

The following items are included in the scope of delivery:

A	Front antitipper tube (2x)
B	Screw M8 x 90 (4x)
C	Washer (8x)
D	Assembly instruction

! IMPORTANT

- Due to technical changes, there are two versions of the frame. The assembly of the different versions A and B is described below.

Version A: Only one fastening nut per side visible on the inside of the frame. For Ocean VIP to SNo. 14393905/For Ocean Dual VIP to SNo. 14394664.

Version B: Three fastening nuts per side visible on the inside of the frame. For Ocean VIP from SNo. 14393906/For Ocean Dual VIP ab SNo. 14394665.

Safety

WARNING Risk of injury!

- Do not use if defective.

CAUTION

- Check product regularly for damage and to ensure that it is securely assembled.
- Tighten screws and regularly check that they are secure.
- Be careful during assembly.

Mounting the front antitipper (Version A)

! IMPORTANT

- We strongly recommend that the assembly is carried out by a specialist dealer.

► Fig. 2

- Remove both cover caps E from the frame section F.
- Loosen screws G and H and remove them.
- Lay the washer M and nut N aside for re-use.

► Fig. 3

- Guide the supplied screw B through the hole I of the antitipper tube.
- Push the supplied washer C into position on screw B.
- Leave the washers J in the holes K and L of the frame section.
- Guide the screw B with the attached antitippertube A and washer C through the hole K of the frame section F.
- Push the existing washer M into position on screw B.

! IMPORTANT

- Make sure that the flat sides of the washers C and J are in contact with each other and that the radii are flush with the antitipper tube and the holes of the frame section.

- Mount the existing nut N on the screw B.
- Guide the existing screw G through the hole O of the antitipper tube A.
- Push the supplied washer C into position on screw G.
- Swivel the antitipper tube into the position of the hole L of the frame section and guide the screw G through the hole.
- Counter the nut N with the wrench SW13 and tighten the screw B with Allen key SW5 with a torque of 8 Nm.
- Tighten screw G with Allen key SW5 and a torque of 8 Nm.
- Mount the facing antitipper mirror-inverted in the same way.

Mounting the front antitipper (Version B)

► Fig. 4

- Remove both cover caps E from the frame section F.
- Unscrew screws G and remove them completely. Lay the four washers H and four nuts I aside for re-use.

► Fig. 5

- Guide the supplied screws B through the holes E of the antitipper tube A.
- Push the supplied washers C into position on screws B.

IMPORTANT

- Make sure that the flat sides of the washers C are in contact with each other and that the radii are flush with the antitipper tube and the holes of the frame section.

- Guide the screws B with the attached antitippertube and washers through the holes I of the frame section F.
- Push each of the existing washers H onto the inserted screws B.
- Mount the existing nuts I onto the screws B.
- Counter the nuts with wrench SW13 and tighten the screws with Allen key SW5 with a torque of 8 Nm.
- Mount the facing antitipper mirror-inverted in the same way.

Cleaning

Please note the cleaning instructions in the user manual for the Ocean shower chair.

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Algol-Trehab Oy
AQUATEC Operations GmbH