### SAVE THESE INSTRUCTIONS

*NOTE: Check all parts for shipping damage. If damage is found, DO NOT use. Contact Carrier/Invacare for further instructions.*

## Safety Summary

To ensure the safe and proper use of the XPO<sub>2</sub> supplemental battery, these instructions **MUST** be followed:

### **⚠ WARNING**

**DO NOT** use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

**Use only with Invacare XPO<sub>2</sub> portable oxygen concentrator.**

## Connecting the Supplemental Battery to the XPO<sub>2</sub>

*NOTE: For this procedure, refer to FIGURE 1.*

1. Connect the supplemental battery pack to the XPO<sub>2</sub> as shown in FIGURE 1. Ensure the supplemental battery is fully charged prior to initial use.

*NOTE: . This supplemental battery pack will provide 2.5 hours of additional run time at setting 2. This supplemental battery pack has its own battery status button and battery gauge to identify the level of battery charge.*

*NOTE: The supplemental battery cannot be used to charge the XPO<sub>2</sub> internal battery.*

**FIGURE 1** Connecting the Supplemental Battery to the XPO<sub>2</sub>

For more information regarding Invacare products, parts, and services, please visit [www.invacare.com](http://www.invacare.com)

---

## Charging the Supplemental Battery

*NOTE: For this procedure, refer to FIGURE 2.*

1. Connect the AC or DC power adapter to the supplemental battery pack.
2. Connect the other end of the AC or DC power adapter into the appropriate power source.

*NOTE: If completely discharged, the supplemental battery will fully charge in approximately 4 hours.*

**FIGURE 2** Charging the Supplemental Battery

## Charging the Internal and Supplemental Battery Pack Simultaneously

*NOTE: For this procedure, refer to FIGURE 3.*

1. To charge the supplemental battery pack and internal battery together, perform the following:
  - A. Connect the AC or DC power adapter to the supplemental battery pack as shown.
  - B. Connect the supplemental battery pack to the XPO<sub>2</sub> as shown.
  - C. Plug the other end of the adapter into the power source.

*NOTE: The internal battery charge status will be indicated by the XPO<sub>2</sub> battery gauge and the supplemental battery charge will be indicated by the illuminated lights (bars) located on the supplemental battery pack assembly.*

*NOTE: You may operate your XPO<sub>2</sub> Portable Concentrator while the internal and/or the supplemental battery is being charged. Use of the XPO<sub>2</sub> during charging will lengthen the amount of time required to reach full charge status of the batteries.*

**FIGURE 3** Charging the Internal and Supplemental Battery Pack Simultaneously

---


## Prolonging Battery Life

### Do's

- When you first receive your XPO<sub>2</sub>, fully charge the internal battery (and supplemental external battery if provided) overnight.
- Keep your batteries fully charged when using the XPO<sub>2</sub> regularly.
- Your XPO<sub>2</sub> and supplemental external battery can be recharged at any time. These batteries can have their charge “topped off” at any time.
- Always ensure the XPO<sub>2</sub> internal and supplemental external batteries are recharged as soon as possible after they become fully discharged. The batteries may be permanently degraded if left fully discharged for an extended length of time.
- Check the status of your XPO<sub>2</sub> internal and supplemental external battery once a month if you are not using your XPO<sub>2</sub> regularly. Both batteries should be maintained at 2 illuminated bars worth of charge.
- Heat is the worst enemy of a battery. Allow plenty of air to circulate around the XPO<sub>2</sub> so that the battery is kept as cool as possible when charging and also when in use.

### Don'ts

- DO NOT use or leave the XPO<sub>2</sub> or supplemental external battery in excessive heat or cold.
- DO NOT store or leave the XPO<sub>2</sub> or supplemental external battery in car trunks, etc. for extended periods of time.
- DO NOT store either battery fully charged (4 bars illuminated on the unit's battery gauge) if you are going to store your XPO<sub>2</sub> for any time greater than 2 weeks. Recharge or discharge the battery to 2 bars (50% charge) only. Storing a battery with a full charge may degrade its useful life.
- DO NOT leave your supplemental external battery plugged into the XPO<sub>2</sub> when the XPO<sub>2</sub> is not in use. The supplemental external battery will lose charge while plugged into the XPO<sub>2</sub> even with the XPO<sub>2</sub> turned off.

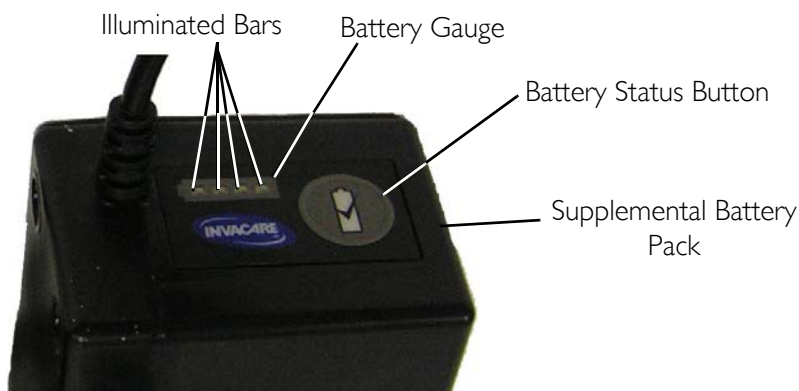**FIGURE 4** Prolonging Battery Life

# TYPICAL PRODUCT PARAMETERS

Dimensions:	6.8 in high x 2.5 in wide x 1.8 in deep (17.1 cm high x 6.4 cm wide x 4.4 cm deep) Without molded cord
Weight:	1.3 lbs (0.68 kg)
Battery Recharge:	4 Hours (Recharge time increases if battery is charging while unit is running. Times are approximate)
Battery Duration: (When used with discharged internal battery, times are approximate)	Setting 1 = 3.5 hrs
	Setting 2 = 2.5 hrs
	Setting 3 = 2.0 hrs
	Setting 4 = 1.5 hrs
	Setting 5 = 1.0 hrs
Battery Duration: (When used with fully charged internal battery, times are approximate)	Setting 1 = 7.0 hrs
	Setting 2 = 5.0 hrs
	Setting 3 = 4.0 hrs
	Setting 4 = 3.0 hrs
	Setting 5 = 2.0 hrs
Humidity:	Operating humidity: 15% to 60% non condensing
	Storage humidity: up to 95% non condensing
Temperature Range:	Operating Temperature: 41° F to 95° F (5° C to 35° C)
	Storage Temperature: -2° F to 140° F (-20° C to 60° C)
Extended Temperature Range: (Used with fully charged internal battery)	Operating Temperature: 95° F to 104° F (35° C to 40° C) Continuous use - all settings
Extended Temperature Range: (When used with discharged internal battery)	Operating Temperature: 95° F to 104° F (35° C to 40° C) Continuous use - settings 1,2,3 30 minutes (max) - setting 4 20 minutes (max) - setting 5

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Invacare AB, Fagerstagatan 9, S-163 91
Invacare Corporation
Invacare Deutschland GmbH