# Invacare® XPO<sub>2</sub>™ Portable Oxygen Concentrator

Kit No. 1157695

EN	<b>Battery Replacement</b> Service Manual .....	1
FR	<b>Remplacement de la batterie</b> @TahX_WXageXgXa ~...!	3
DE	<b>Batterieaustausch</b> Servicehandbuch .....	6

## I General EN

### I.1 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

Danger indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.

Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.  
Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.

Information gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with Directive 93/42/EEC concerning medical devices.

The launch date of this product is stated in the CE declaration of conformity.

### I.2 Kit Overview

This kit contains components to replace the XPO<sub>2</sub> internal battery.

The proper installation of this kit involves performing the procedures listed below:

- Access the XPO<sub>2</sub> battery.
- Replace the battery.
- Reassemble the XPO<sub>2</sub>.

### I.3 Kit Information

PART NUMBER	DESCRIPTION
n/a	Internal Battery Pack
n/a	Seal/Notice Label

PART NUMBER	DESCRIPTION
n/a	Ferrite
1154285	Battery Replacement Service Manual

### I.4 Customer Service and Warranty Information

For warranty information, please refer to the original owner's manual which came with this product, or contact Invacare for more information.

## 2 Safety

### 2.1 General Guidelines

The Safety section contains important information for the safe operation and use of this product

#### WARNING!

– DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

#### WARNING! ACCESSORIES

– Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

DEALER: Keep this manual. The procedures in this manual MUST be performed by a qualified technician.

Yes, you can.®

## 2.2 Installation

**WARNING!**  
**Risk of injury.**

These procedures **MUST** be performed by a qualified technician.

When accessing the XPO<sub>2</sub> battery, remember that battery power is still being applied to the internal (PCB) printed circuit board. Battery power remains on the PCB as long as the battery connection is made to the PCB.

- Care **MUST** be taken to minimize contact with the PCB. **DO NOT** use any kind of tool (screwdriver, pliers, etc.) when working on the internal structure. Otherwise, injury and/or damage may occur.

**CAUTION!**  
**Risk of damage.**

- **DO NOT** damage ribbon cable when removing or installing front cover.
- **DO NOT** use Methyl Ethyl Ketone (MEK). This will damage the plastic.

Check all parts for shipping damage. If damage is found, **DO NOT** use. Contact Carrier/Invacare for further instructions.


To ensure the safe and proper replacement of the XPO<sub>2</sub> battery, these instructions **MUST** be followed.

## 3 Service

### 3.1 Accessing the XPO<sub>2</sub> Battery

1. Remove the label **(A)** from the top of the XPO<sub>2</sub> **(B)** (Figure 1).

This label adheres to the front and rear sections of the XPO<sub>2</sub>. This label **MUST** be removed to separate the two sections.

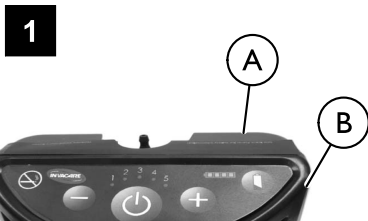2. Position the XPO<sub>2</sub> so that the back panel label **(C)** is facing up (Figure 2).

3. Use a T15 Torx® screwdriver to remove the four mounting screws **(D)** from the back of the XPO<sub>2</sub>.


4. While holding the unit together, rotate the XPO<sub>2</sub> and position it so that the back panel label is facing down.


The ribbon cable **(E)** is connected to the front cover **(F)** and the internal printed circuit board (PCB). Only open the XPO<sub>2</sub> as shown in Figure 3 and Figure 4.

5. Slowly, open the front cover **(F)**, ensuring not to damage the ribbon cable **(E)** (Figure 3 and Figure 4).

**3**

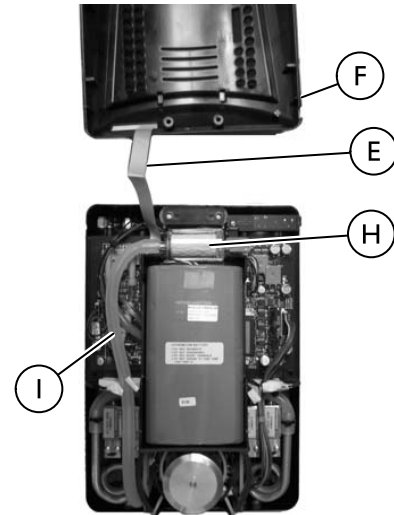**4**
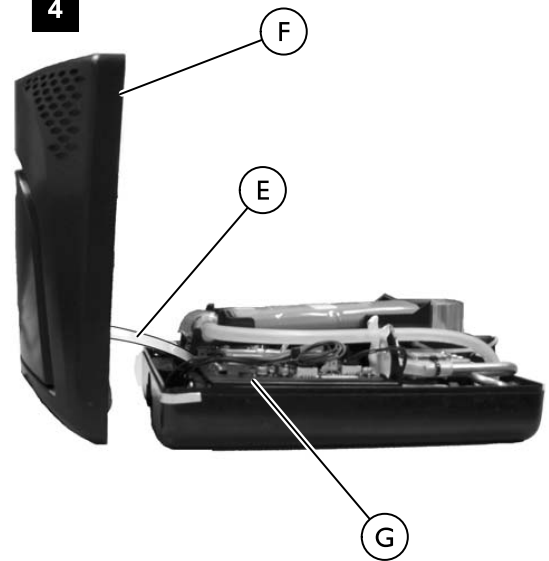
### 3.2 Replacing the XPO<sub>2</sub> Battery


**CAUTION!**

- Before handling the internal printed circuit board (PCB), ensure you are properly grounded to prevent static damage to the components of the board.
- A static cuff **MUST** be worn and properly grounded using an alligator clip.
- Electrical conduit or water pipe is normally sufficient when a known good ground is unavailable.
- Care should be taken to ensure that the alligator clip contacts a bare metal surface.

1. Perform the steps in 3.1 Accessing the XPO<sub>2</sub> Battery, page 2.
2. Remove the inlet filter **(H)** with tubing **(I)** attached from the holder (Figure 5).
3. Disconnect the battery wiring harness **(J)** from the internal printed circuit board (PCB) **(G)**.
4. Remove the battery **(K)** from the thermal pad and battery tray.

**i** The thermal pad is between the battery and the bottom of the tray. It is a soft “dough-like” material that adheres to the tray. If thermal pad is damaged or pulled up during battery removal it **MUST** be replaced.

**5**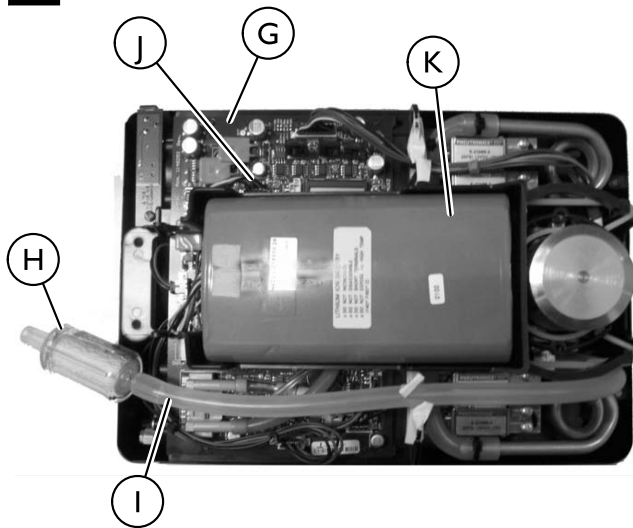


5. Locate the black wire **L** and the red wire **M** on the new battery pack (Figure 6).
6. Move the black and red wires away from the rest of the wires (yellow, white and green).

**i** The red and black wires **CAN NOT** be twisted. See Figure 6.


7. Open the ferrite **N**.
8. Place the ferrite around the black and red wires.
9. Close and lock the ferrite around the black and red wires.

**6**

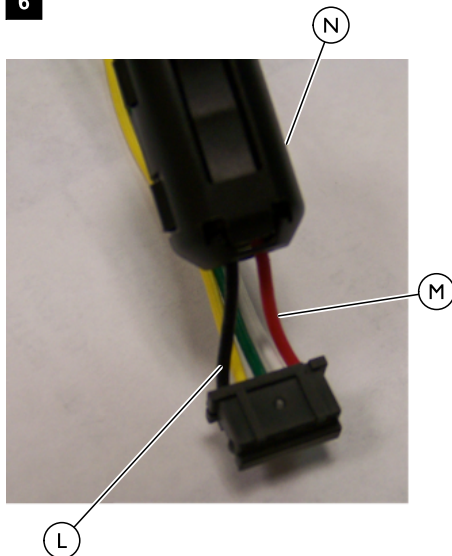10. Slide the ferrite to ensure it is not pinching the wires.

**7**

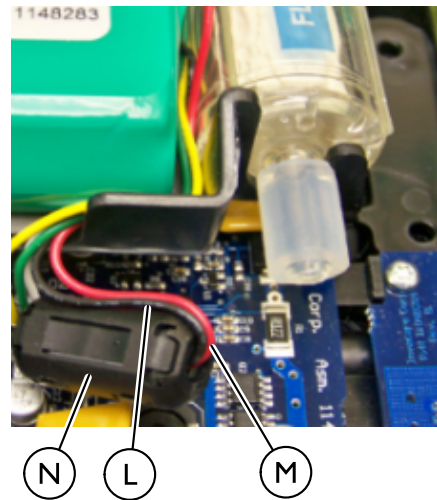11. Position the new battery on the thermal pad as shown in Figure 5.

**i** When installing the new battery, ensure the battery label faces up.

12. Connect the battery wiring harness **L** to the internal printed circuit board (PCB) **G**.
13. Re-assemble the XPO2. Refer to 3.3 Reassembling the XPO2, page 3.

### 3.3 Reassembling the XPO<sub>2</sub>

1. Carefully, re-install the front cover **F** onto the XPO<sub>2</sub> **B** (Figure 4).
2. Turn the XPO<sub>2</sub> On to ensure proper operation.
3. Turn the XPO<sub>2</sub> Off.
4. While holding the unit together, position the XPO<sub>2</sub> so that the back panel label **C** is facing up.
5. Install four mounting screws **D** and secure the two sections of the XPO<sub>2</sub> together (Figure 2). Securely tighten.
6. Remove the glue residue from the top of the XPO<sub>2</sub>, where the label was removed. Apply 91% isopropyl alcohol, and wipe with a clean cloth. Make sure the surface is dry and free of any residue. This will ensure a good seal for the new label.
7. Install replacement label **A** on the top of the XPO<sub>2</sub> so that it adheres to both the front and rear sections (Figure 1).

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
© 2013 Invacare® Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

Torx is a registered trademark of Acument Intellectual Properties, LLC.

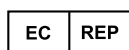**EU Representative**

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D-32457 Porta Westfalica

Tel: +49 (0)57 31 754 0

Tel: +49 (0)57 31 754 52191

**Manufacturer**

Invacare Corporation

2101 E. Lake Mary Blvd.

Sanford, FL 32773

USA

Tel: 407-321-5630

**Invacare Corporation**

**USA**

One Invacare Way

Elyria, Ohio USA

44036-2125

440-329-6000

800-333-6900

Technical Services

440-329-6593

800-832-4707

[www.invacare.com](http://www.invacare.com)

1154285-B 2013-01-29

**Making Life's Experiences Possible™**

**Yes, you can.®**

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Invacare Deutschland GmbH