

Emotion Programming

Remote Control for ECS

Capacity indication for e-motion-battery left/right (or indication of parameter during programming)

Indication of assistance level (0, 1, 2) or number of the parameter being programmed (0 – 9)

Switches

both wheels ON in level 2
both wheels ON in level 1
both wheels OFF



Button for activating the display manually

(Display is switched on, when e-motion is on, but switches off after a few minutes to increase battery lifetime ⇒ normal use: 5 years)

Symbols for

- programming mode
- service/error code
- capacity of battery in remote control

Reset button: Sets all parameters back to factory setting

Emotion Programming

Remote Control for ECS

How to program e-motion:

1. Insert programming key (like e-fix or ADVENTURE)
2. Programming of 10 parameters using the keys of the remote control, e. g.
 - Sensitivity left wheel
 - Sensitivity right wheel
 - Starting time in assistance level 1
 - Power assistance in assistance level 1
 - After-running time in assistance level 1
 - Starting time in assistance level 2
 - Power assistance in assistance level 2
 - After-running time in assistance level 2

⇒ **Perfectly adjustable to user's needs**

