## H243 H253

F	1.	Guide utilisateur
GB	2.	User's manual
D	3.	Gebrauchsanweisung
E	4.	Manual del usuario
I	5.	Manuale dell'utente
NL	6.	Gebruiksaanwijzing
P	7.	Guia do utilizador
S	8.	Bruksanvisning
DK	9.	Bruksanvisning
FIN	10.	Käyttäjän käsikirja
N	11.	Bruksanvisning

### I. FRANCAIS

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### 2. ENGLISH

You have just purchased an INVACARE® product. Our products are designed and manufactured to the most rigorous quality and safety standards and conform to the European directives (MDD 93/42/EEC). They are guaranteed for one year. Certain products may have a longer guarantee; please check with your dealer.

**The following operating and maintenance instructions must be observed:**

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the restrictions on use, the adjustment or maintenance of the product, please contact your dealer.
- Do not exceed the maximum operating load or the maximum load shown on the product label.
- Always ensure that the product is suitable for your requirements; in case of doubt, check with a health professional.
- Worn, damaged or missing components must be replaced immediately by an approved repairer.
- Do not leave or store the product out of doors.
- Periodically check the stability and rigidity of the product and, where applicable, that the screws are correctly tightened and the rubber end caps in good condition.
- Take particular care when using the product on damp surfaces.

#### ASSEMBLY

##### Model H253 (self-propelled chair) and H243 (push chair)

- Proceed to assemble the structure by screwing the footrest on each of the two sides using an Allen key.
- Fit the backrest over the tubes on the back of the seat.
- Screw down the two star knobs tightly to fasten the backrest.
- Clip the perforated seat onto the chair's frame. Ensure that the perforated seat is firmly in place before use.
- Slide the pail into its housing.

#### USE

##### Model H253 (self-propelled chair) and H243 (push chair)

- The armrests are height-adjustable. To adjust them, unscrew the base of the armrests using an Allen key. Position the base opposite the hole desired (4 positions possible) and then retighten the screws.
- The footrests are height-adjustable. To adjust them, unscrew the base of the footrests using an Allen key. Position the base opposite the hole desired (3 positions possible) and then retighten the screws.
- The footrests can be folded up towards the back of the chair.

#### Immobilisation / transfer

- Before transferring the chair, ensure that at least 2 brakes are engaged, that the wheels are pointing towards the front of the chair and that the footrests are raised.
- Do not press down on the armrests during transfer.
- To avoid any risk of being pinched, take care not to place your hands on the armrests' tilt system.
- Do not lean on the footrests to sit down or stand up.

##### Model H243 (push chair)

- To immobilise the chair and proceed to transfer it, both handbrakes must be engaged.

##### Model H253 (self-propelled chair)

- To immobilise the chair and proceed to transfer it, both handbrakes must be engaged in the locked position. Warning: use in damp areas may sometimes affect the brakes' efficiency.

#### Moving

##### Model H253 (self-propelled chair) and H243 (push chair)

- Do not run the chair over sloping or broken surfaces. DO not cross over obstacles more than 5mm in height.
- Do not lean forwards beyond the end of the armrests.
- Do not lean backwards over the backrest.
- Check that no part of the body is outside the chair when going through doors (particularly near the armrests).
- Commode chairs are strictly reserved for indoor use.

#### MAINTENANCE

Clean regularly with a normal household cleaner. The toilet bowl can be put through an autoclave (1 hour at 80°). For disinfection in hospital environments, please follow the hygiene specialist's recommendations.

#### USER'S MAXIMUM WEIGHT

Please see the table at the end of this document.

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