## Wartung

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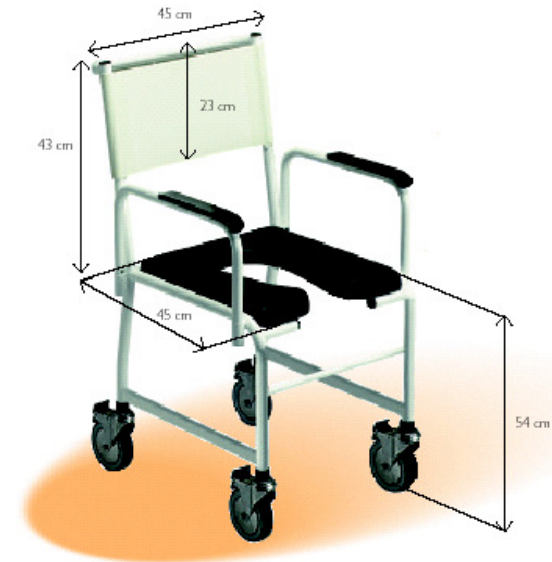
7709 / 7710 / 7711 / 7712 / 7719 / 7720 / 7721 / 7722

plus the option

SEPARATELY FOLDING STAINLESS STEEL ARMRESTS  
(-73)

Production / sales / logistics/ administration

Invacare Limited  
Pencoed Technology Park, Pencoed  
Bridgend CF35 5AQ  
Tel: (44) (0) 1656 776200  
Fax: (44) (0) 1656 776201  
[UK@invacare.com](mailto:UK@invacare.com) / [www.invacare.co.uk](http://www.invacare.co.uk)

## General

### MOBILE SHOWER SEAT

The mobile shower seat provides an aid to washing and the care of patients and residents. The mobility of the seat optimises user comfort and convenience.

The following mobile shower seat models are available:

- Mobile shower seat with fixed armrests; type 7709
- Mobile shower seat with detachable armrests; type 7710
- Mobile shower (toilet) seat with detachable armrests, polyurethane seat, rail for bedpan or bucket; type 7711
- Mobile shower (toilet) seat with detachable armrests, polyurethane seat, rail for bedpan or bucket, folding/detachable/height adjustable legrests; type 7712
- Mobile shower (toilet) seat, self-propelled with two 24" wheels, detachable armrests, polyurethane seat, rail for bedpan or bucket, folding/detachable/height adjustable legrests; type 7719
- Mobile shower seat, self-propelled with two 24" wheels, detachable armrests; type 7720
- Mobile shower seat, self-propelled with two 24" wheels, detachable armrests and retractable foot plate; type 7721
- Mobile shower seat, self-propelled with two 24" wheels, detachable armrests and folding/detachable/height adjustable legrests; type 7722
- All models are also available with separately folding stainless steel armrests (option 73)

### Introduction

We at Invacare are delighted that you have chosen to purchase a quality aid. A long history of developing and manufacturing bathroom, shower and toilet aids, has enabled Invacare to become a leading company in this field. These mobile shower seats satisfy the CE safety standards. These standards are in compliance with the medical devices directive of 14 June 1993 no. 93/42 EEC.

### Warranty

All Invacare shower and toilet aids come with a 2-year manufacturer's warranty. Normal wear and tear and improper use do not fall under the terms of this warranty. Invacare does not accept any liability for personal injury which is caused by the improper use of the mobile shower seats.

### Safety

The mobile shower seat is a safe and comfortable product designed primarily for use in wet cells

Here are a few safety tips:

- **Never leave the patient alone in the seat (except in emergencies)**
- **Apply the brake when using the seat**
- **Never push the seat too close to walls or doorframes, as this may cause patients to trap their fingers between the seat and the wall or doorframe.**

In the event of queries or defects, please contact Invacare.

### Maintenance

The mobile shower seat and optional armrests (73) can be rinsed down after use with water and normal household detergents; do not use a scourer or aggressive solvents for this purpose.

### Folding the armrests (option 73)

The separately folding armrests can easily be lowered and raised. When in their raised position, the armrests should remain firmly in position. If not consult your dealer.

**Please note: When raising / lowering the armrests, take care not to trap your fingers in the folding mechanism.**

## Technical specifications

- Steel frame with shockproof and impact-resistant Rilsan coating.
- Polyurethane folding armrests (option 73).
- Maximum capacity: 135 kg.
- Dead weight (basic model) 7709: 11.4 kg

7710:	11.4 kg
7710,073:	12.0 kg
7711:	13.0 kg
7711,073:	13.6 kg
7712:	16.0 kg
7712,073:	16.6 kg
7719:	22.0 kg
7719,073:	22.6 kg
7720:	18.4 kg
7720,073:	19.0 kg
7721:	19.6 kg
7721,073:	20.2 kg
7722:	18.8 kg
7722,073:	19.4 kg