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GB INSTRUCTIONS FOR DESTRUCTION OF PRODUCT AND PACKING:
When the product can no longer be used, it and its accessories as well as the packaging material should be assorted in the correct way. The individual parts can be separated and handled according to the list specification.

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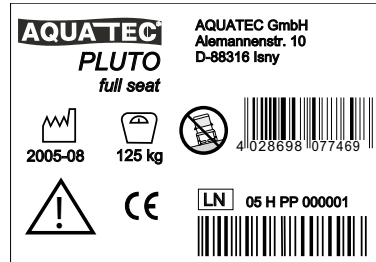
GB OPERATING INSTRUCTIONS

Study the operating instructions carefully before use. The person who prescribes the product or who is otherwise responsible for the user must ensure that the product is suitable for the user.

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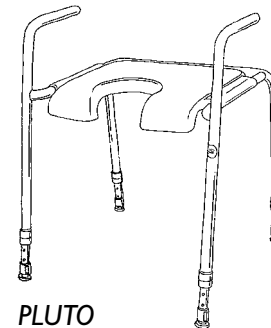
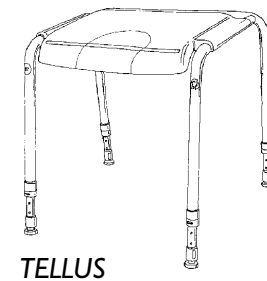
GB CARE

Before cleaning the frame without seat, make sure the pin holes are covered.
Wash the chair with a mild detergent and wipe dry.
N.B. User's max. weight: 125 kg.

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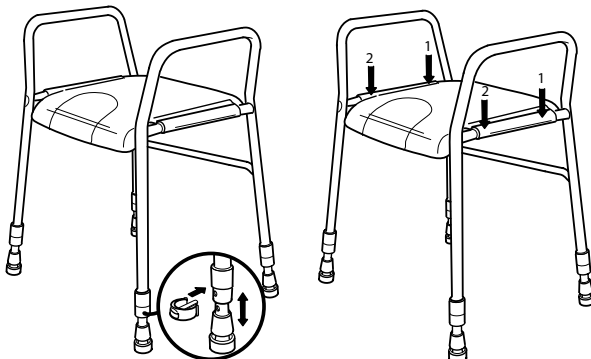
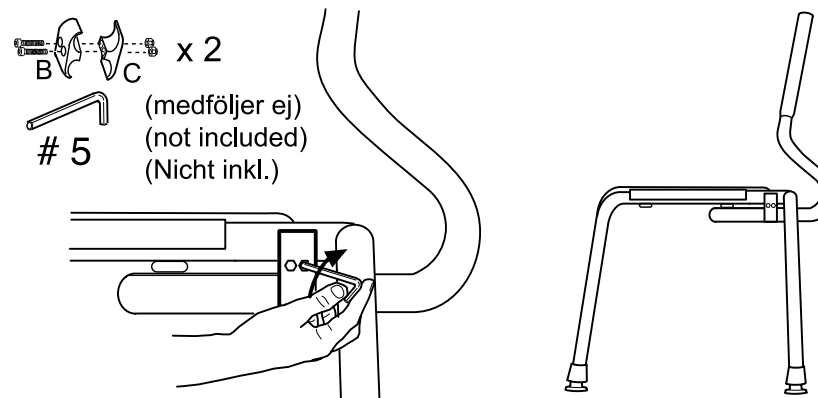
GB

1. The tube fastener (B and C) should be placed together by using the enclosed screws and locking nuts. Screws – place at the side with rounded holes, locking nuts – place at the side with edged holes. N.B. The blue ring at the locking nut must be placed outwards.
2. As illustrated the back support tube should be positioned on the underside of the seat. Use tube fastener to secure.
3. The backrest shall be placed at its most forward position, and the screws in the fastening clamps must be securely tightened. **N.B. Ensure that the screws are tightened periodically.**

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