## Aquatec® 90000

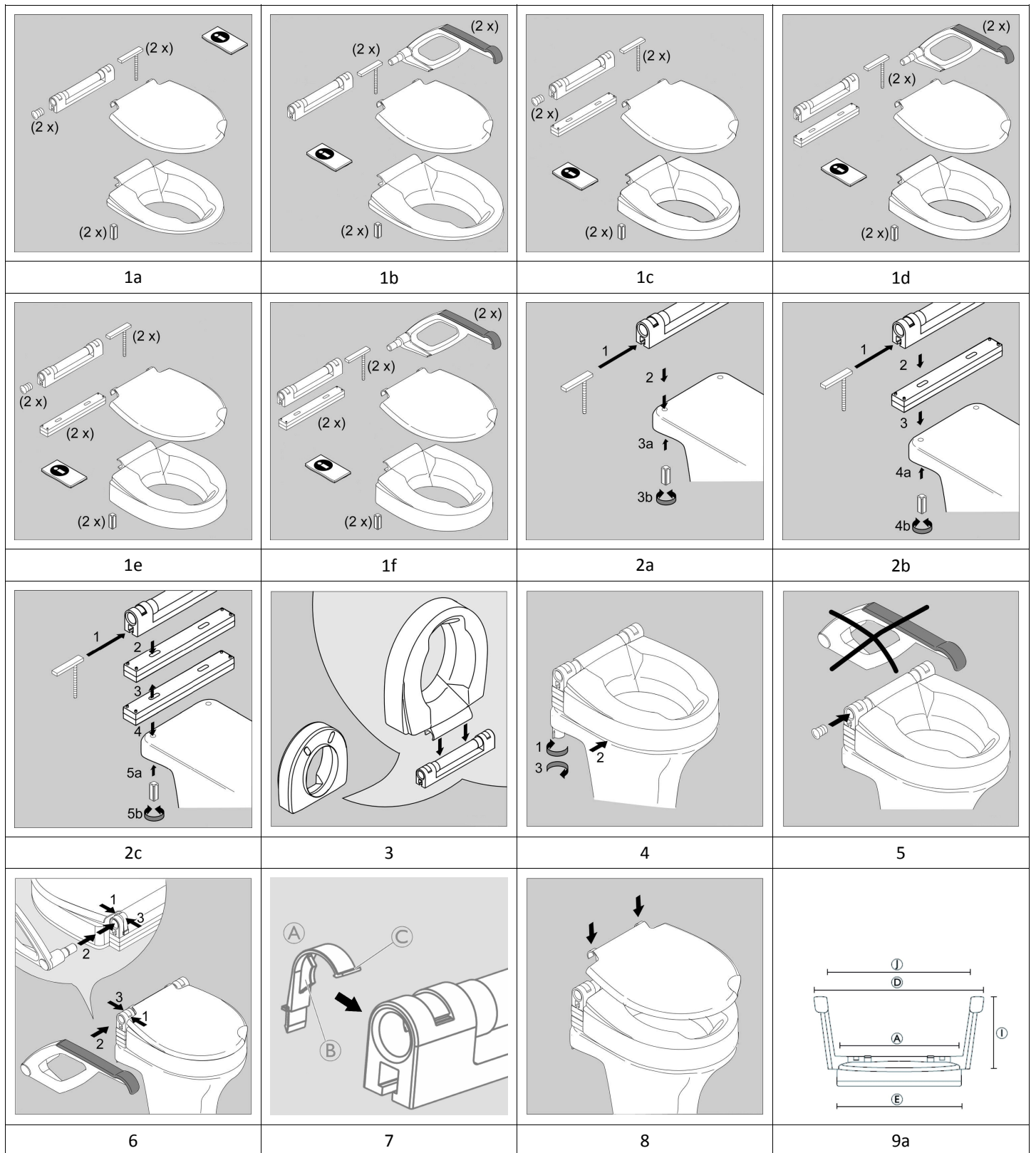
en	Raised Toilet Seat User Manual . . . . .	3
de	Toilettensitzerhöhung Gebrauchsanweisung . . . . .	6
fr	Réhausse WC Manuel d'utilisation . . . . .	9
it	Rialzo WC Manuale d'uso . . . . .	13
es	Asiento elevador de inodoro Manual del usuario . . . . .	16
pt	Elevador de sanita Manual de utilização . . . . .	19
nl	Verhoogde toiletzitting Gebruiksaanwijzing . . . . .	22
da	Opslået toiletsæde Brugsanvisning . . . . .	26
sv	Upphöjd toalettsits Bruksanvisning . . . . .	29
no	Toalettforhøyer Bruksanvisning . . . . .	32
fi	Kohotettu wc-istuin Käyttöohje . . . . .	35
hr	Povišena daska za WC Upute za uporabu . . . . .	38

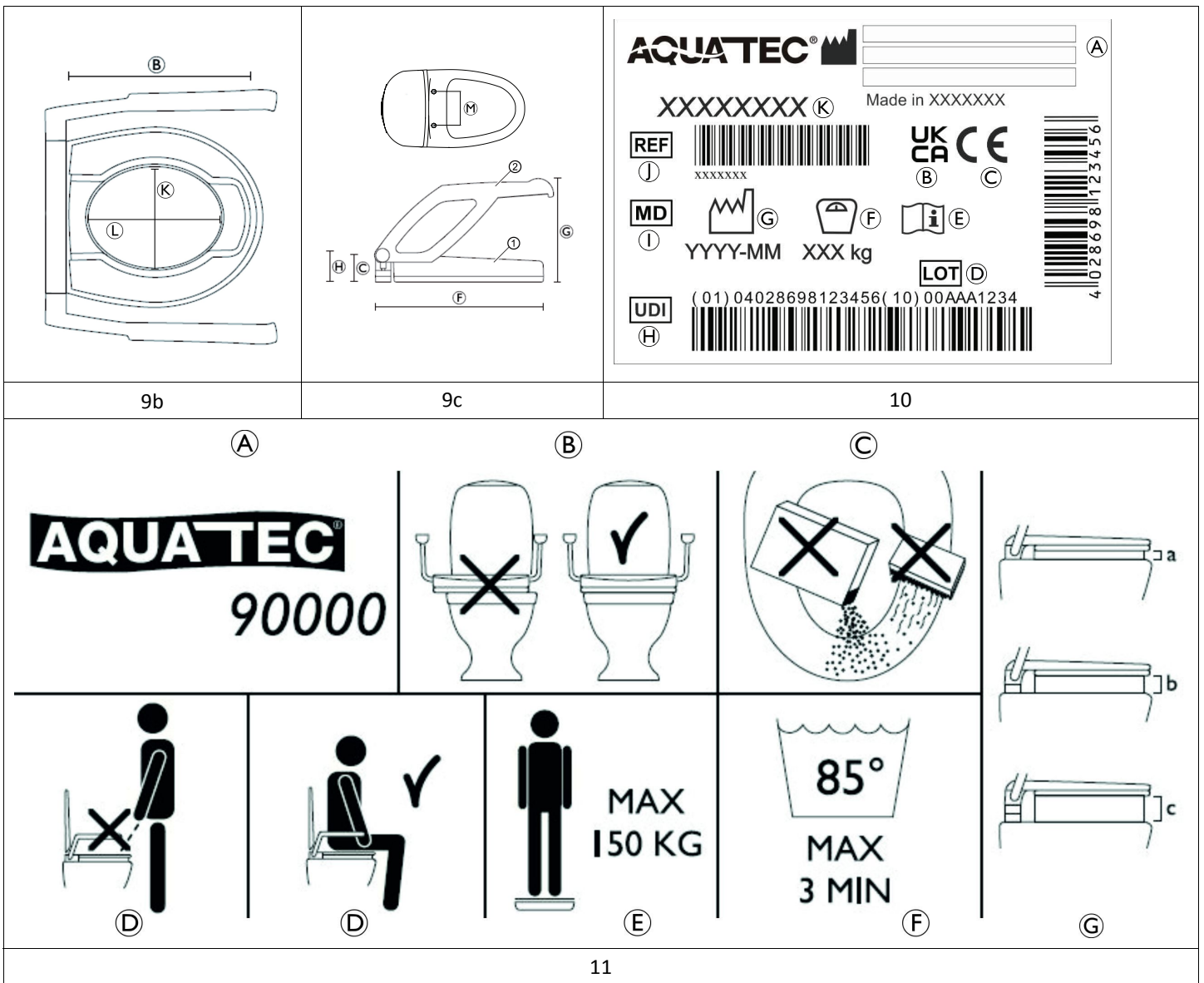
**AQUATEC®**

This manual MUST be given to the user of the product.  
BEFORE using this product, this manual MUST be read and saved for future reference.

**Yes, you can.®**

11

## 1 General

### 1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

### 1.2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1470052-F

### 1.3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I.

This product features the UKCA mark, in compliance with Part II UK MDR 2002 (as amended) Class I.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

#### Product-specific Standards

The product has been tested and conforms to the standard ISO 17966 (Assistive products for personal hygiene that support users) and related standards. For further information about local standards and regulations, contact your local Invacare representative.

### 1.4 Service Life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

### 1.5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

## 2 Safety

### 2.1 Safety Information

#### WARNING!

##### Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

#### WARNING!

##### Risk of Falling

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- The armrests are intended to be used on both sides evenly. If pressure is applied on one side only, use only half of the body weight. At overload, there is a risk that the armrest will break.
- If the product is used with armrest, the toilet bowl must be securely fixed.
- Do not stand on this product.

#### CAUTION!

##### Risk of Trapping your Fingers

- When lowering the armrests, ensure that your fingers are away from the hinges to avoid catching them.

#### WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

#### CAUTION!

##### Risk of Injury

- Non-original or wrong parts may affect the function and safety of the product.
- Only use original parts for the product in use.
  - Due to regional differences, refer to your local Invacare catalog or website for available options or contact your Invacare distributor. See addresses at the end of this document.

## 2.2 Labels and Symbols on the Product

### Identification Label (Fig. 10)

(A)	Manufacturer	(G)	Production date
(B)	UK Conformity Assessed	(H)	Unique Device Identifier
(C)	European Conformity	(I)	Medical Device
(D)	Lot number	(J)	Reference number
(E)	Read user manual	(K)	Product designation
(F)	Maximum user weight		

The identification label is attached to underside of the product

### Additional Label (Fig. 11)

(A)	Product designation
(B)	Product must be positioned centrally on the toilet bowl, see 4 Setup (Fig. 2a – 8), page 5 .
(C)	Never use corrosive fluids or abrasive agents for cleaning, see 5.2 Cleaning and Disinfection, page 5 .
(D)	Please sit down on the toilet.
(E)	Maximum user weight
(F)	Maximum cleaning time / temperature, see 5.2 Cleaning and Disinfection, page 5 .
(G)	Sitting height a, b, c. (Depending on purchased model)

The additional label is attached to underside of the lid

## 3 Product Overview

### 3.1 Intended Use

Aquatec 90000 is a raised toilet seat with foldable armrests that elevates the seating height on toilets (available in: 2 cm, 6 cm or 10 cm) for a better seating position. It is fixed with a screw system at the rear of the toilet bowl.

This product is intended to be used as an aid for self-hygiene care and toilet purposes. The product provides support for sitting down or standing up from a toilet.

#### Intended Users

Users are any adults and adolescents with restricted or limited mobility. The user must be able to keep own balance or ensures the availability of additional support, i.e. wall mounted grab bar.

#### Indications

- Restricted or limited mobility (e.g. legs and/or hips) or
- having balance issues to stand up or walk due to transitional or permanent disabilities.

There are no contraindications known when using the product as intended. For the maximum user weight of the product, see 7.1 Technical Data (Fig. 9a – c), page 6 .

### 3.2 Scope of Delivery (Fig. 1a – 1f)

#### 2 cm, with and without Armrest (Fig. 1a, 1b)

- Corpus, 2 cm
- Lid
- Base incl. 2x clips
- Screw (2x)
- Plastic nut (2x)
- Plug (2x) (for version without armrest)

- Armrest (2x), (for version with armrest)
- User manual

#### 6 cm, with and without Armrest (Fig. 1c, 1d)

- Corpus, 6 cm
- Lid
- Base incl. 2x clips
- Screw (2x)
- Plastic nut (2x)
- Height adapter
- Plug (2x) (for version without armrest)
- Armrest (2x), (for version with armrest)
- User manual

#### 10 cm, with and without Armrest (Fig. 1e, 1f)

- Corpus, 10 cm
- Lid
- Base incl. 2x clips
- Screw (2x)
- Plastic nut (2x)
- Height adapter (2x)
- Plug (2x) (for version without armrest)
- Armrest (2x), (for version with armrest)
- User manual

## 4 Setup (Fig. 2a – 8)

### CAUTION!

#### Risk of Pinching

– Take care during assembly.

### Installing the Raised Toilet Seat (Fig. 2a – 8)

1. Slide the screws in the designated slot in the base on both sides. Place the base and the height adapter(s) (if applicable) on your toilet as shown on the image. (See table) Tighten the plastic nut.

Height 2 cm	Fig. 2a
Height 6 cm	Fig. 2b
Height 10 cm	Fig. 2c

2. Place the corpus by holding it upright and fixing it to the base. (Fig. 3)
3. Loosen the plastic nuts, align the raised toilet seat centrally to your toilet. Close the plastic nuts. (Fig. 4)
4. If your product is without armrests install the plugs on both sides. (Fig. 5)
5. Clip on the lid. (Fig. 8)

To dismount the raised toilet seat reverse step 2 and 1.

### Installing the Armrests (if applicable) (Fig. 6, 7)

1. Open the red clip by releasing the upper end of the clip © and then the pin ⑥ of the red clip ④. (Fig. 7) Then install the armrests and close the red clip. (Fig. 6)

Ensure that the red clips ④ engage at two points: first the pin ⑥ and then the upper end of the clip ©. (Fig. 7)

To disassemble the armrests, first release the upper end of the clip © and then the pin ⑥.

## 5 Maintenance

### 5.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

## 5.2 Cleaning and Disinfection

### General Safety Information

### CAUTION!

#### Risk of Contamination

– Take precautions for yourself and use appropriate protective equipment.

### NOTICE!

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

### Cleaning Intervals

### NOTICE!

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product:

- regularly while in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

### Cleaning Instructions

### NOTICE!

Maximum cleaning temperature:

- 60 °C by hand
- 85 °C for 3 minutes by automatic washing systems

1. Clean the product by wiping down all generally accessible surfaces.
2. Dry the parts with a cloth.

### Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

## 6 After Use

### 6.1 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling

in each country. Contact your local waste management company for information.

## 6.2 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *5 Maintenance, page 5*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

## 7 Technical Data

### 7.1 Technical Data (Fig. 9a – c)

Dimensions			
Ⓐ	365 mm	Ⓔ	230 / 270 / 310 mm
Ⓑ	425 mm	Ⓕ	30 / 70 / 110 mm
Ⓒ	20 / 60 / 100 mm	Ⓖ	200 mm
Ⓓ	540 mm	Ⓙ	450 mm
Ⓔ	380 mm	Ⓚ	205 mm
Ⓕ	475 mm	Ⓛ	275 mm
Possible distance between the bolt holes of the toilet bowl:		Ⓜ	135 mm – 175 mm

Weights		with armrests	without armrests
		Ⓒ = 20 mm	3.6 kg
	Ⓒ = 60 mm	4 kg	2.5 kg
	Ⓒ = 100 mm	4.4 kg	2.9 kg
Maximum user weight	150 kg		

Materials			
①	PP	②	PP

All product components are corrosion resistant, are not claimed resistant to ignition and do not contain natural rubber latex.

### 7.2 Environmental Parameters

Do not use the product outside the environmental parameters specified in the table below.

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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Invacare Poirier SAS
Invacare GmbH
Invacare UK Operations Limited