
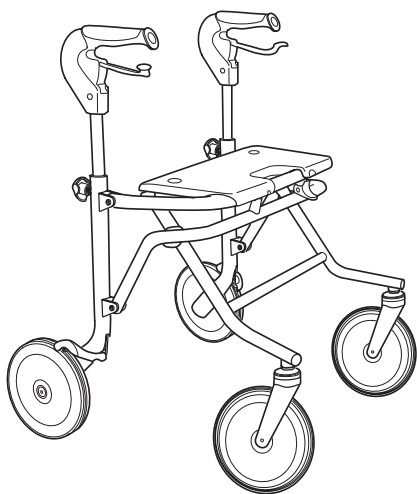
# DOLOMITE MELODY, SOPRANO

  
Do not use as a wheel-  
chair or transport chair.

**DOLOMITE MELODY**  
Maximum user weight 135 kg

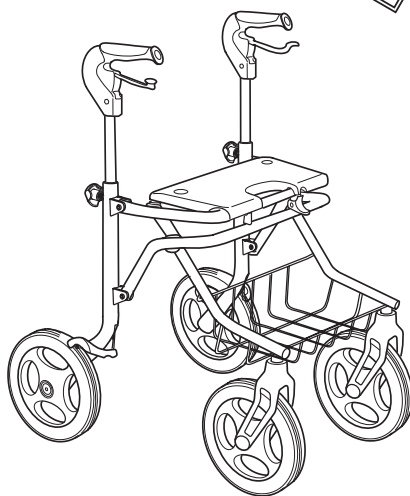**DOLOMITE SOPRANO**  
Maximum user weight 160 kg

## OPERATING INSTRUCTIONS

Study the operating instructions carefully before use. The walker is suitable for use both indoors and outdoors.

## SAFETY

- Do not under any circumstances stand on the walker. The walker must not be used as a wheelchair.
- Both brakes must be in the parking position when the walker is used as a seat.
- The walker must be fully unfolded when used as walking support. N.B. The walker may only be used as a walking support.
- Be extra careful when using the walker on a sloping surface.
- Be extra careful when carrying heavy loads in the basket. The maximum load in the basket is 7 kg for **DOLOMITE MELODY S** 10 kg **DOLOMITE MELODY M, L** and 12 kg for **DOLOMITE SOPRANO**. Loads may only be carried in the basket or on the tray (**DOLOMITE SOPRANO**).

## OPERATING POSITION








When unpacking or unfolding from transport position lift front frame according to fig. C.

1. Unfold the walker by pressing down on the seating tube. Check that the safety catch is in the locked position when the walker is completely unfolded.
2. The seat can be folded up to provide a larger walking space, and in order to increase accessibility.

## FOLDING

If basket is used, it must be dismantled before folding the walker.

1. Press down the safety catch.
  2. Pull the seating tube upwards.
  3. The walker can be folded into two different positions.
- A The walker is to be folded to the first position if it is to be parked when folded.
  - B It is recommended that the walker is folded to a further stage, the inner position, if it is to be transported when folded. Note that the walker cannot stand upright when in this position.
  - C Unfolding from transport position.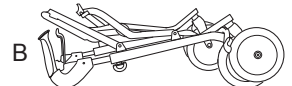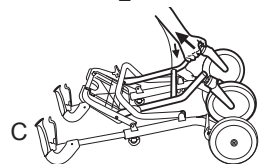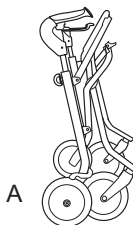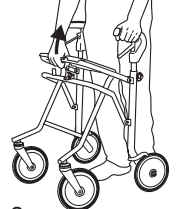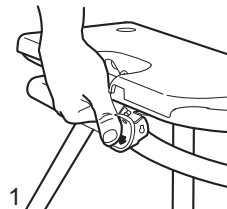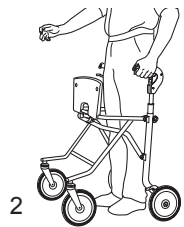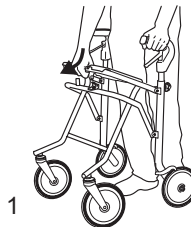



**BRAKE**

Apply the brakes by pulling both brake handles upwards.

**PARKING BRAKE**


1. Press the brake handles carefully downwards to apply the parking brakes. A clear sound is heard when the brakes are applied correctly. It is important that both parking-brakes are applied.
2. Release the parking brakes by pulling the handles upwards.

For users who feel that the walker rolls too quickly and who therefore make frequent use of the brake, we recommend a walker with a slow-down brake (accessory).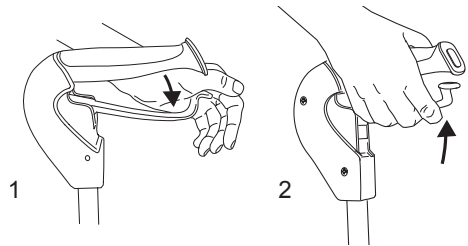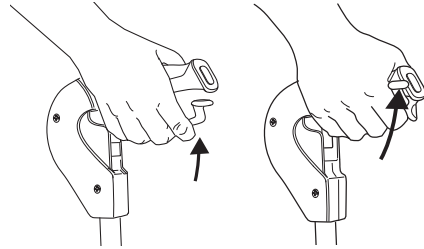

**ASSEMBLING THE TRAY AND THE BASKET**



- A DOLOMITE MELODY
- B DOLOMITE SOPRANO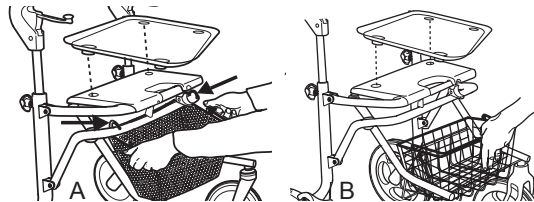

**ADJUSTING THE HANDLEBARS**

NB! The brake must not be in the on position when adjusting the handle.


1. Loosen the height adjustment knobs, but do not remove them completely.
2. Adjust the handlebars to the correct height. The handlebars should normally be placed at the height of the user's wrists.
3. The handles can be adjusted such that they point inwards in order to provide a more comfortable hand position. This may also be required if the user has limited wrist mobility.
4. Tighten the height adjustment knobs.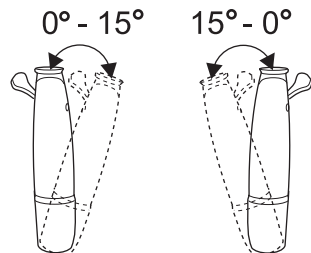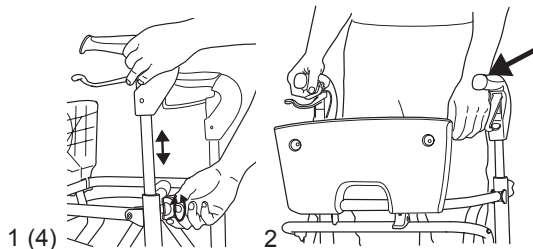

If there is a risk of the user removing the height adjustment knobs, these should be replaced by locking screws (available as an accessory).

## ADJUSTING THE BRAKE EFFECT

Allen key number 3 (not supplied) can be used to increase the braking effect. The distance between the brake head and the wheel should be 0,5 mm.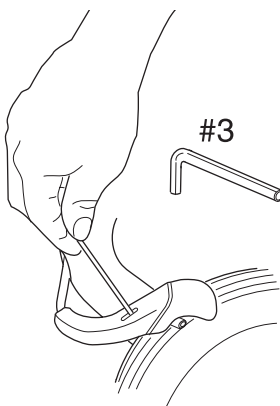

## CARE OF THE WALKER

In order to maintain good performance and safety, you should regularly ensure that the following measures are carried out:

- Clean the walker with a solvent-free household cleaner. Dry thoroughly. Keep the wheels clean.
- Check that screws and adjustments are properly tightened.
- Immediately contact your prescriber if the walker develops a fault. Never use a defective walker.


**DOLOMITE will not accept responsibility for adaptations, faulty repairs, etc., that lie outside of the company's control.** Guarantee period: 5 years. The warranty applies solely to **DOLOMITE's** original parts (not to wear parts). The user's maximum weight for **DOLOMITE MELODY** is 135 kg, and for **DOLOMITE SOPRANO** 160 kg.

### INSTRUCTIONS FOR DESTRUCTION OF PRODUCT AND PACKAGING:

When the product can no longer be used, it and its accessories, together with packaging material, should be sorted properly. The individual parts can be separated and handled according to the materials description.

The label on the **DOLOMITE MELODY** and **SOPRANO** provides all important technical information.

- 1: Article number
- 2: Product name
- 3: Serial number
- 4: Date of manufacture
- 5: Maximum user weight
- 6: NB: read the operating instructions
- 7: Manufacturer's contact information.

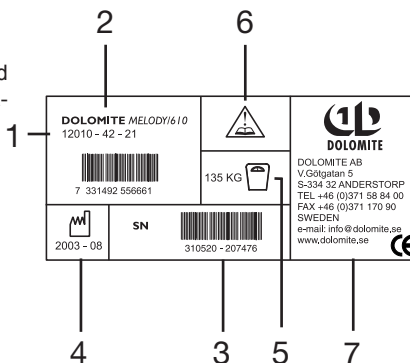DOLOMITE AB, Box 55, 334 21 ANDERSTORP, SWEDEN  
 TEL +46 (0)371 58 84 00, FAX +46(0)371 170 90  
 E-mail: info@dolomite.se, www.dolomite.se

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DOLOMITE AB, Box 55, SE-334 21 ANDERSTORP, SUECIA,