# DOLOMITE *ALPHA ADVANCED*

**SV** BRUKSANVISNING

**NO** BRUKSANVISNING

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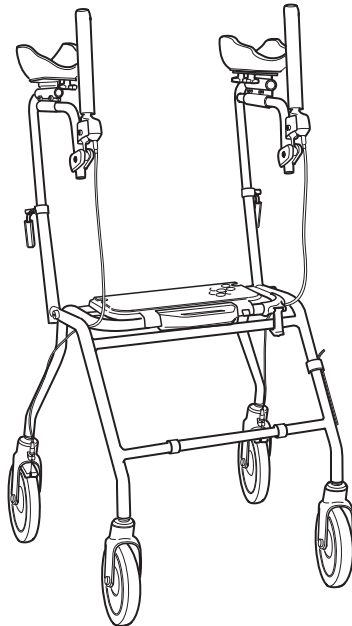
**GB** INSTRUCTIONS

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
# DOLOMITE ALPHA ADVANCED
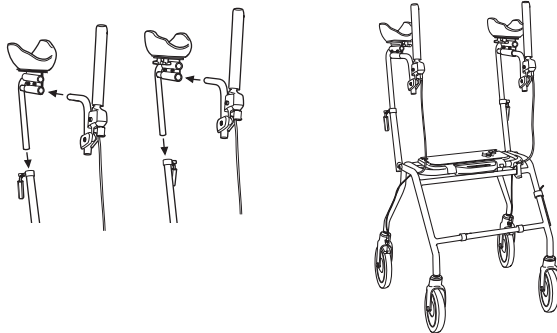
## SV BRUKSANVISNING

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## GB OPERATING INSTRUCTIONS

Study the operating instructions carefully before use. The person who prescribes the product or who is otherwise responsible for the user must ensure that the product is suitable for the user.

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**GB ADJUSTMENT POSSIBILITIES**

- Height adjustment..... 8
- Inwards rotation of arm supports ..... 8
- Depth adjustment ..... 10
- Inwards rotation of arm support plates ..... 10
- Angular setting of handle ..... 10
- Depth adjustment of the arm support plates ..... 10
- Adjustment of distance between arm supports .....8,12
- Inwards rotation of arm support plates ..... 12
- Height adjustment of brake..... 14
- Rotation of brake handle ..... 14

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## **GB** ADJUSTMENT OF BRAKE HANDLE

1. Loosen the screw (G) with the attached key, Important!
2. The fastening device (H) can now be height and sideways-adjusted and the brake can by means of the ball joint be rotated for optional hand location and optional function direction.
3. Tighten the screw (G) again after the brake has been set as desired.
4. The rollator's parking brake is applied by moving the handle away from yourself.  
A clear clicking sound indicates that the brake is in the right position. It is important that both brakes are set in the parking position. To release the brake - pull backwards.  
Note: The brake can be set at a greater angle by twisting the pipe (I) in connection with the depth adjustment

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**GB** The label contains all important technical information regarding DOLOMITE ALPHA ADVANCED:

- 1: Article number
- 2: Product name
- 3: Serial number
- 4: Date of manufacture
- 5: Max. user weight
- 6: NB! See user manual.
- 7: In- and outdoor rollator.
- 8: Max. width of rollator.
- 9: Manufacturer's name.

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**GB**

### INSTRUCTIONS FOR DESTRUCTION OF PRODUCT AND PACKING:

When the product can no longer be used, it and its accessories as well as the packaging material should be assorted in the correct way. The individual parts can be separated and handled according to the list specification.

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**GB MAINTENANCE - SAFETY**

To ensure good performance and safety you should regularly check that the following is carried out:

- Clean the rollator with solvent-free household cleaner. Keep the wheels clean.
- Ensure that screws and adjustments are properly tightened.
- Immediately contact the nearest dealer if a fault should appear. Never use a defective walker. Do not attempt repairs yourself!  
N.B. Your walker should only be used as a walking aid!

User's max. weight: 275 lbs. (125 kg, 20 stones.) DOLOMITES will not accept responsibility for adaptations, faulty repairs, etc, that are outside their control.

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Invacare Poirier SAS
Algot-Trehab Oy