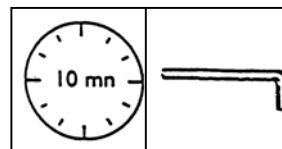
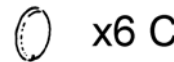
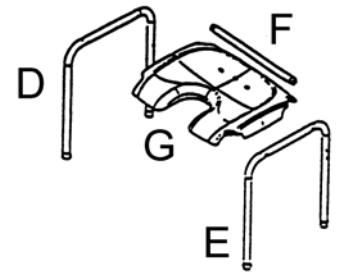
x6 A

## H2120/1

- F 1. Guide utilisateur
- GB 2. User's manual
- D 3. Gebrauchsanweisung
- E 4. Manual del usuario
- I 5. Manuale dell'utente
- NL 6. Gebruiksaanwijzing
- P 7. Guia do utilizador
- SV 8. Bruksanvisning
- DK 9. Brugsanvisning
- SU 10. Käyttäjän käsikirja
- N 11. Bruksanvisning

H2080

H2120/1

H2240/1

H2480/1

H2580/1

H2610

H3000

H3160

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### 2.GB The following operating and maintenance instructions must be observed

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the restrictions on use, the adjustment or maintenance of the product, please contact your dealer.
- Do not exceed the maximum operating load  or the maximum load  shown on the product label.
- Always ensure that the product is suitable for your requirements; in case of doubt, check with a health professional.
- Worn, damaged or missing components must be replaced immediately by an approved repairer.
- Do not leave or store the product out of doors.
- Periodically check the stability and rigidity of the product and, where applicable, that the screws are correctly tightened and the rubber end caps in good condition.
- Take particular care when using the product on damp surfaces.

#### MAINTENANCE

Use a soft, damp cloth to clean away dirt and dust. Use soap and water to remove stubborn stains. Dry with a soft cloth. For disinfection in hospital environments, please follow the hygiene specialist's recommendations.

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