



Invacare® **Sonnet Combi**



Manual GB

*Invacare EC-Høng AIS
Ident. no.: 913988
Version 02 09.2002*

Invacare® Sonnet Combi

Manual for Invacare® Sonnet Combi

This bedside table is developed for the home care sector and for hospitals. The bedside table contains a number of functions and qualities, which are specially designed for these health care sections.



Fig. 1

Position the additional plate by lifting the plate and lead it to the wished height. When the additional plate is lowered more than 45°, the plate will lock in the chosen position. Lower the plate to horizontal. See fig 1.

To lower the additional plate to bottom position: Lift the plate and let it slip down towards the bottom position.

The bedside table can be operated and used from either the right or the left side of the bed, depending of what is most convenient.

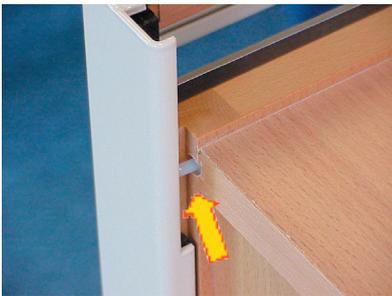


Fig. 2

It is possible to mount the additional plate on both sides of the bedside table.

When it is necessary to change side, press the sliding bolt with one hand and pull the additional plate up vertically with the other, until the bracket is released. The sliding bolt is placed at the top of the back. See fig 2.

Mount the additional plate on the other side of the bedside table, by pressing the sliding bolt, and then guide the bracket to the bottom position.



Fig. 3

When positioning the additional plate: Lift the plate and lead it to the wished height. Then lower the plate to horizontal.

As soon as the additional plate is lowered to horizontal, is the height of the plate locked. See fig 3.

Manual for Invacare® Sonnet Combi



Fig. 4

To ease the cleaning in cases with heavy smudge, is it possible to demount the drawer and clean it in a rinsing room:

Pull out the drawer until it stops, then lift it up and pull out until the drawer is released. Take the insert out. See fig 4.

After cleaning the insert and the drawer, remount the insert and push/lift the drawer back in position in the bedside table.

Use only regular household detergents. Use a sponge or a soft brush to wash the bedside table.

Never use acids, alkalines or solvents such as acetone or cellulose thinner.

The demountable plastic parts can be cleaned under rinsing water and in an automatic dishwasher.

The bedside table does not tolerate cleaning in water-jet based cleaning equipment.

None of the moving parts of the bedside table requires lubricating; they only have to be clean in order to function satisfactorily.

The Invacare® Sonnet Combi must be stored in a room with a humidity of 10 – 80% and with temperature of 0 – 50°.

The storing room must be frost-proof.

During transport must the additional plate be placed in bottom position. The closet door and drawers must be closed.



Customer Sales and Service

INVACARE A/S

Sdr. Ringvej 39
DK-2605 Brøndby
Phone: +45 36 90 00 00
Fax: +45 36 90 00 01

INVACARE AB

Fagerstagatan 9 / Box 66
S-163 91 Spånga
Phone: +46 8 761 70 90
Fax: +46 8 761 81 08

INVACARE Deutschland GmbH

Dehmer Strasse 66
D-32549 Bad Oeynhausen
Phone: +49 57 31 754 0
Fax: +49 57 31 754 150

INVACARE B.V.

Celsiusstraat 46
NL-6716 BZ Ede
Phone: +31 318 69 5 757
Fax: +31 318 69 5 758

INVACARE AS

Grensesvingen 9
P.O. Box 6230 / Etterstad
N-0603 Oslo
Phone: +47 22 57 95 00
Fax: +47 22 57 95 01

INVACARE LTD

South Road
Bridgend Industrial Estate
UK-Bridgend, CF31 3PY
Phone: +44 1 656 664 321
Fax: +44 1 656 667 532

INVACARE N.V.

Autobaan 14
B-8210 Loppem, Brügge
Phone: +32 50 83 10 10
Fax: +32 50 83 10 11

INVACARE Poirier S.A.

Les Roches
F-37230 Fondettes
Phone: +33 2 47 62 64 66
Fax: +33 2 47 42 12 24

Production Company:

INVACARE EC-Høng A/S

Østergade 3
DK-4270 Høng
Phone: +45 58 85 27 22
Fax: +45 58 85 43 86
www.invacare.com

