



Yes, you can.<sup>®</sup>

## *Invacare<sup>®</sup> Sonnet Dual*

---

*User Manual (GB)*



# Manual for Sonnet Dual

This bedside table is developed for the home care sector and for hospitals. The bedside table contains a number of functions and qualities, which are specially designed for these health care sections. Unfold the additional plate by lifting the plate all up and lower it to horizontal.



Fig. 1.

Revert the additional plate by lifting the plate and let it slide towards the bottom. See fig. 1.

The height of the additional plate can be adjusted both in folded and unfolded position. Adjust the height by pressing the locking pins upwards.

One locking pin is placed at the front and one at the backside. Both locking pins must be activated to raise the additional plate. When pressing the locking pins and holding the additional plate at the same time, makes it possible to control the movement.



Fig. 2.

As soon as the locking pins are released, will the plate lock in the new position. See fig. 2.

Oblique the additional plate by activating the nearest locking pin and lead the plate down and backwards.

It is also possible to activate the other locking pin and lift the plate upwards.

The additional plate is now a good support for books and magazines.

NOTE: Remember to place the additional plate horizontal before unfolding it.



Fig. 3.

It is possible to change side of the bedside table. First take out the drawers by pulling it until it stops, then lift it and pull it all out.

Then push in the springdowels of the backplate and demount the backplate. See fig. 3.

# Manual for Sonnet Dual



Fig. 4.

Now demount the cupboard door by unscrewing one of the brackets. See fig. 4.

Remount the backplate at the opposite site of the bedside table. The wooden dowels at the top of the backplate must be placed first, then push up the wooden dowels and place the backplate in the frame.

Place the loose bracket of the cupboard door in the bracket of the bedside table. Place the cupboard door in the opposite bracket of the bedside table and lower the door to horizontal. Remount the loose bracket on the cupboard door.

Replace the drawers.

To ease the cleaning in cases with heavy smudge, is it possible to demount the two drawers and the cover plate, and clean it in a rinsing room. See fig. 5.

Pull out the drawer until it stops, then lift it up and pull out until the drawer is released. Take the insert out.



Fig. 5

The cover plate is mounted with clasp brackets. Lift the cover plate to release the clasp brackets from the screws at the side of the bedside table. See fig. 6.

After cleaning the bedside table, remount the cover plate and push/lift the drawer back in position at the bedside table.

Use only regular household detergents.

Use a sponge or a soft brush to wash the bedside table.

Never use acids, alkalines or solvents such as acetone or cellulose thinner.

The demountable plastic parts can be cleaned under rinsing water and in an automatic dishwasher.

The bedside table does not tolerate cleaning in water-jet based cleaning equipment.

None of the moving parts of the bedside table requires lubricating; they only have to be clean in order to function satisfactorily.



Fig. 6

The Sonnet Dual must be stored in a room with a humidity of 10 – 80% and with temperature of 0 – 50°. The storing room must be frost-proof.

During transport must the additional plate be placed in bottom position. The closet door and drawers must be closed.



**Yes, you can.<sup>®</sup>**

## **Customer Sales and Service**

---

### **Denmark**

#### **INVACARE A/S**

Sdr. Ringvej 37  
DK-2605 Brøndby  
Phone: +45 36 90 00 00  
Fax: +45 36 90 00 01  
[www.invacare.dk](http://www.invacare.dk)  
[denmark@invacare.com](mailto:denmark@invacare.com)

### **Sweden & Finland**

#### **INVACARE AB**

Fagerstagatan 9  
S-163 91 Spånga  
Phone: +46 8 761 70 90  
Fax: +46 8 761 81 08  
[www.invacare.se](http://www.invacare.se)  
[sweden@invacare.com](mailto:sweden@invacare.com)

### **United Kingdom & Ireland**

#### **INVACARE LTD**

Pencoed Technology Park  
Pencoed  
UK-Bridgend, CF35 5AQ  
Phone: +44 1 656 776 200  
Fax: +44 1 656 776 201  
[www.invacare.co.uk](http://www.invacare.co.uk)  
[uk@invacare.com](mailto:uk@invacare.com)  
[ireland@invacare.com](mailto:ireland@invacare.com)

### **Germany**

#### **INVACARE AQUATEC GmbH**

Alemannenstrasse 10  
D-88316 Isny  
Phone: +49 75 62 7 00 0  
Fax: +49 75 62 7 00 66  
[www.invacare-aquatec.com](http://www.invacare-aquatec.com)  
[info@invacare-aquatec.com](mailto:info@invacare-aquatec.com)

### **Netherlands**

#### **INVACARE B.V.**

Celsiusstraat 46  
NL-6716 BZ Ede  
Phone: +31 318 695 757  
Fax: +31 318 695 758  
[www.invacare.com](http://www.invacare.com)  
[nederland@invacare.com](mailto:nederland@invacare.com)  
[csede@invacare.com](mailto:csede@invacare.com)

### **Belgium & Luxemburg**

#### **INVACARE N.V.**

Autobaan 22  
B-8210 Loppem, Brügge  
Phone: +32 50 83 10 10  
Fax: +32 50 83 10 11  
[www.invacare.be](http://www.invacare.be)  
[belgium@invacare.com](mailto:belgium@invacare.com)

### **France**

#### **INVACARE Poirier S.A.S**

Route de St. Roch  
F-37230 Fondettes  
Phone: +33 2 47 62 64 66  
Fax: +33 2 47 42 12 24  
[www.invacare.fr](http://www.invacare.fr)  
[contactfr@invacare.com](mailto:contactfr@invacare.com)

### **Austria**

#### **INVACARE AUSTRIA GmbH**

Herzog Odilostrasse 101  
A-5310 Mondsee  
Phone: +43 6232 5535 0  
Fax: +43 6232 5535 4  
[www.invacare-austria.at](http://www.invacare-austria.at)  
[info-austria@invacare.com](mailto:info-austria@invacare.com)

### **Switzerland**

#### **INVACARE AG**

Benkenstrasse 260  
CH-4108 Witterswil  
Phone: +41 61 487 70 80  
Fax: +41 61 487 70 81  
[www.invacare.ch](http://www.invacare.ch)  
[switzerland@invacare.com](mailto:switzerland@invacare.com)

**Ident. no.: 913912**

**Version: 03**

**Date: 10.2011**

Manufacturer:

**INVACARE Rea AB**

Växjövägen 303, box 200  
S-343 75 Diö