SAVE THESE INSTRUCTIONS

Yes, you can!

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

WARNING

- **DO NOT** use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- Users with limited physical capabilities should be supervised or assisted when using the commode.
- Leg extensions with rubber tips **MUST** be in contact with the floor at all times.
- **ALWAYS** observe the weight limit on the labeling of your commode. Check that all labels are present and legible. Replace if necessary.
- Ensure that the snap buttons fully protrude through the same height adjustment hole of each leg extension. This ensures that the leg extensions are securely locked in position and that an even height adjustment is achieved.
- Make sure that all attaching hardware, screws, nuts and/or bolts are tight at all times.
- Inspect rubber tips on the leg extensions for rips, tears, cracks or wear or if they are missing. Replace them **IMMEDIATELY** if any of these conditions exist.
- If so equipped, the back tube wing nuts must be inspected regularly for tightness - otherwise injury or damage may occur.
- Before removing/installing seat and lid, allow the seat and lid to reach room temperature if exposed to cold. This will help prevent the seat clamps from breaking when being removed or installed.
- The commode seat **MUST** be installed before sitting on the commode.

Installing/Removing the Seat, Lid and Pail

Installing

NOTE: When installing the seat onto the frame, make sure that the seat is evenly spaced between the side frames to ensure proper stability.

NOTE: DO NOT use excessive force when installing the seat and lid.

NOTE: When installing the seat and lid, install the seat first and then the lid.

1. Align the grooves on the underside of the seat with the commode frame.
2. Push down firmly until the seat snaps into place.
3. Put the pail into place.

Removing

NOTE: When removing the seat or lid, remove the pail, lid and then the seat.

NOTE: DO NOT use excessive force when removing the seat and lid.

1. Remove the pail.
1. Grasp the back corner of the lid and lift up firmly.
2. Grasp the back corner of the seat and lift up firmly.

Installing/Removing the Back Rail (if applicable)

NOTE: This procedure applies to Models 6497 and 6500-BHD ONLY.

Installing

1. Depress the snap buttons on each side of the back frame.
2. Put the back rail into the frame. Adjust the back frame until the snap buttons protrude out of the holes on each side of the frame.

NOTE: An audible "click" will be heard when each snap button is properly engaged into the frame holes.

Removing

1. Depress the snap buttons on each side of the back frame.
2. Pull the back frame out of the frame.


Adjusting the Height of the Commode

1. Depress the snap button in the frame and adjust the leg extension to the appropriate height adjustment hole.


NOTE: An audible "click" will be heard.

NOTE: Make sure the snap buttons are fully engaged into the height adjustment holes before use.

2. Repeat STEP 1 to adjust the other leg extensions.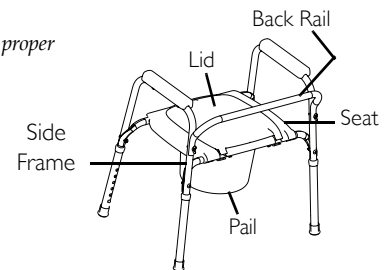

NOTE: Model 9630 shown.

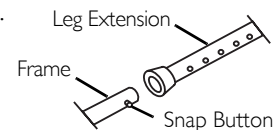Limited Warranty

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for five years for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back page. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts **MUST** be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECT TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS, PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER). SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE. THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER. THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

For more information regarding Invacare products, parts, and services, please visit www.invacare.com

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All rights reserved. Trademarks are identified by the symbols ™ and ®. All trademarks are owned by or licensed to Invacare Corporation unless otherwise noted.
© 2008 Invacare Corporation