Invacare® H335 / H345

R7762-R7768 / R7769 / R7770 / R7772 / R7773 / R7780

en	Support folding handle User Manual	4
de	Stützklappgriff Gebrauchsanweisung	7
fr	Barre d'appui relevable Manuel d'utilisation	10
it	Maniglione di supporto pieghevole Manuale d'uso	14
es	Barra de apoyo abatible Manual del usuario	17
pt	Barra de apoio rebatível Manual de utilização	20
nl	Opvouwbare steungreep Gebruiksaanwijzing	24
da	Støttefoldehåndtag Brugsanvisning	27
no	Støttende foldehåndtak Bruksanvisning	30
sv	Fällbart stödhandtag Bruksanvisning	34
fi	Taittuva tukikahva Käyttöohje	37

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

Images

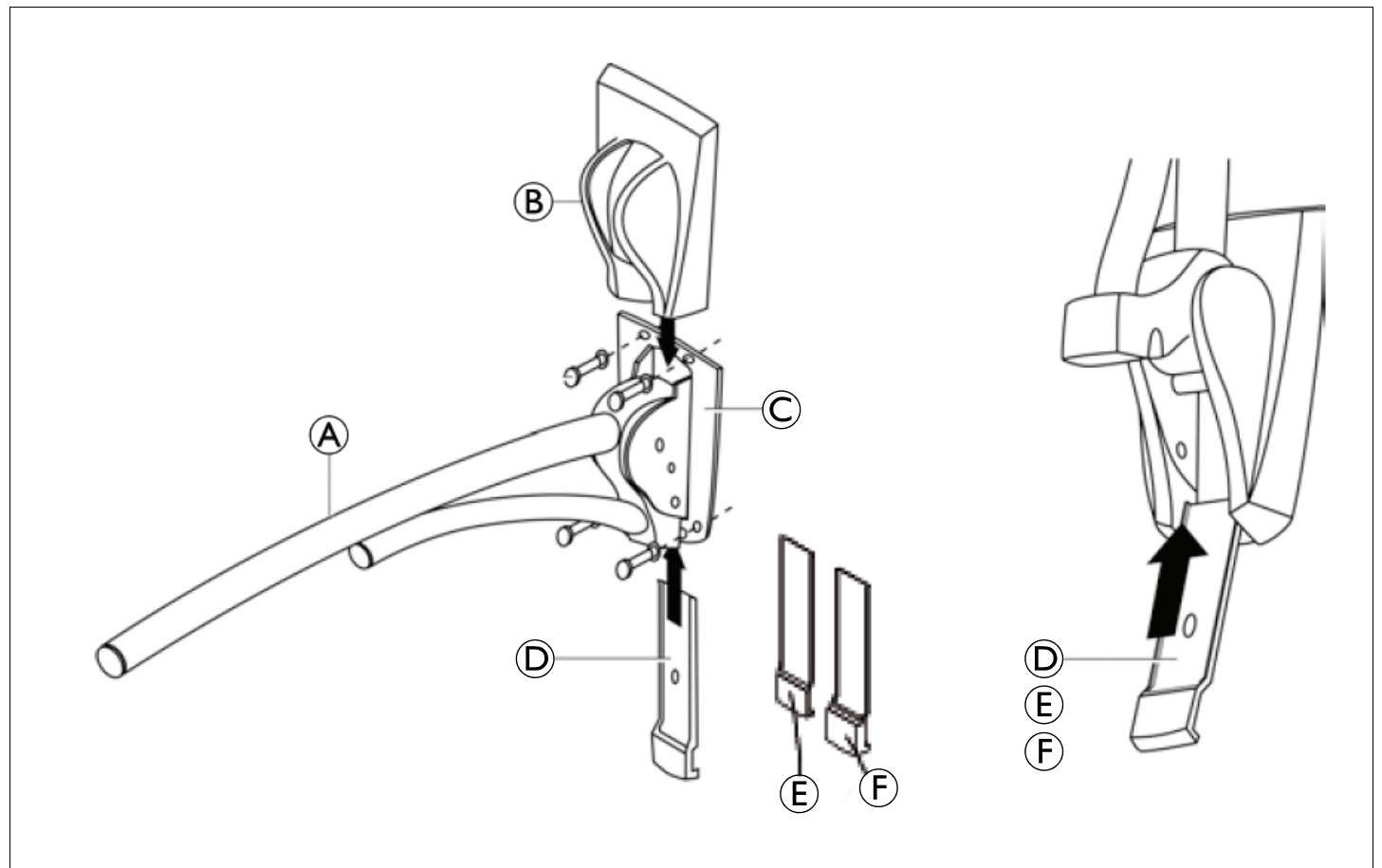
Fig. 1

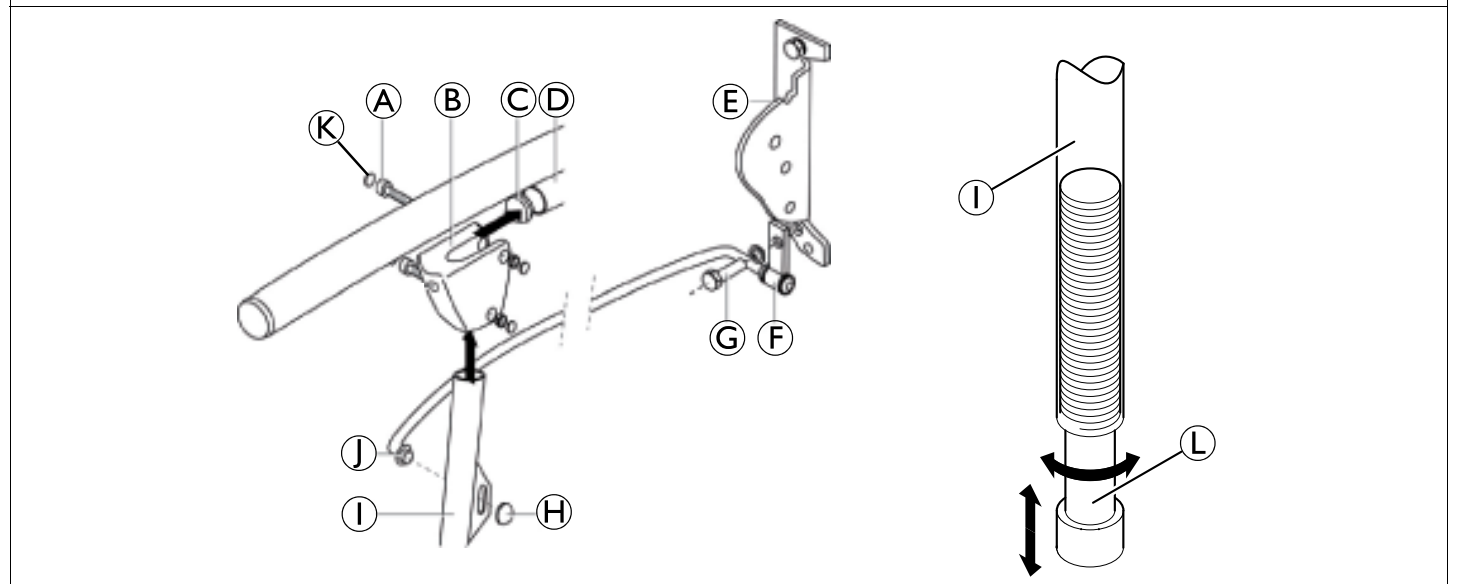
Fig. 2

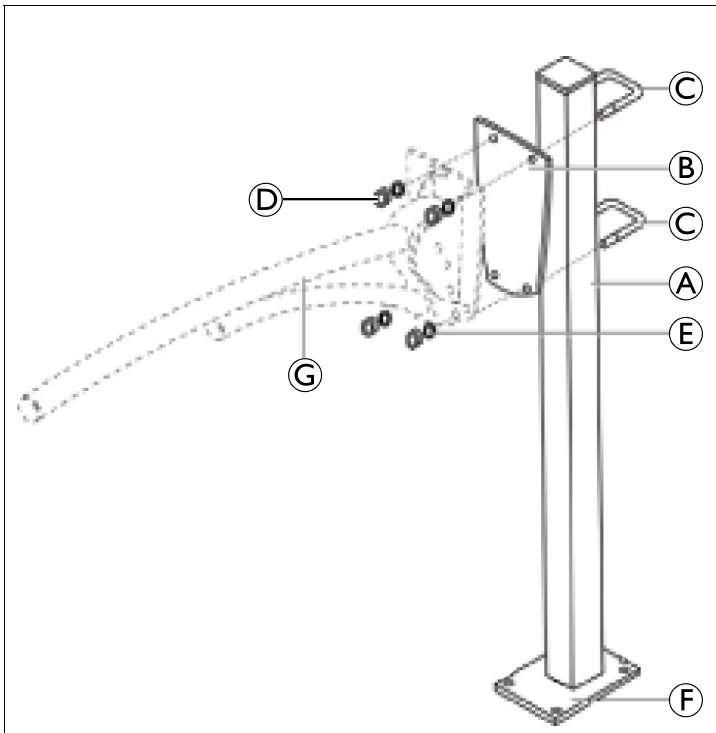
Fig. 3

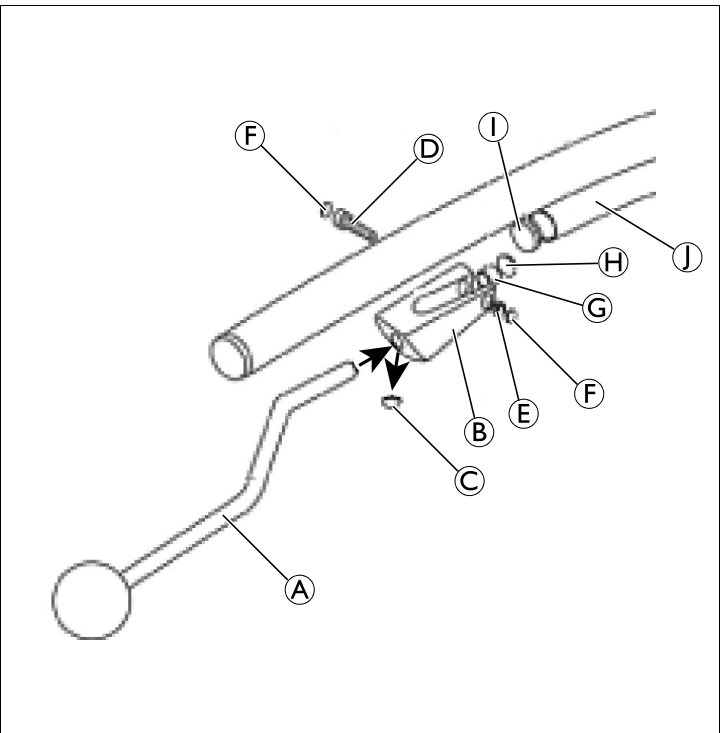
Fig. 4

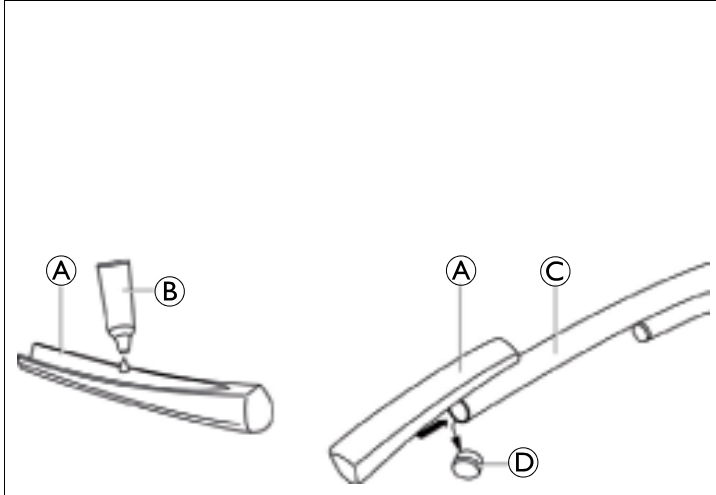
Fig. 5

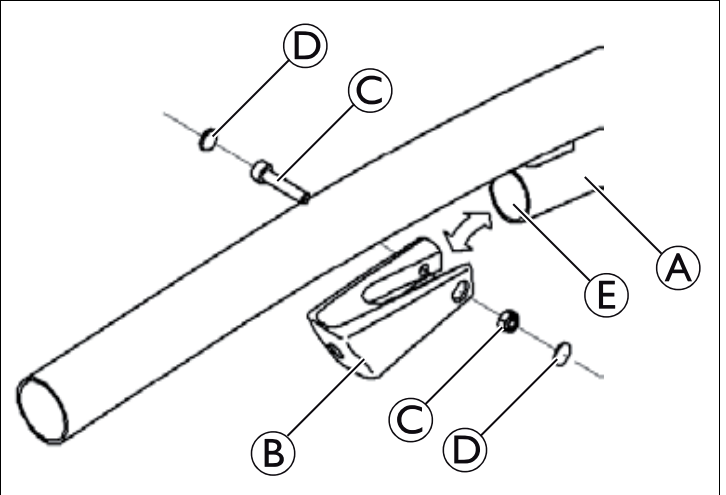
Fig. 6

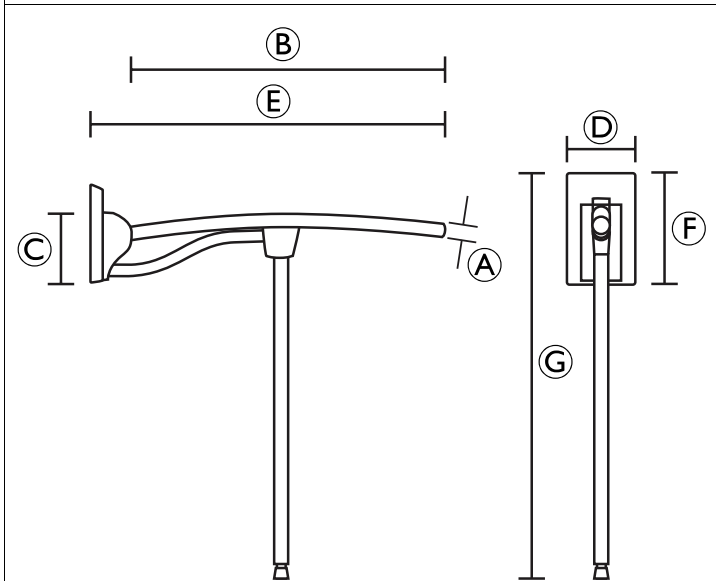
Fig. 7

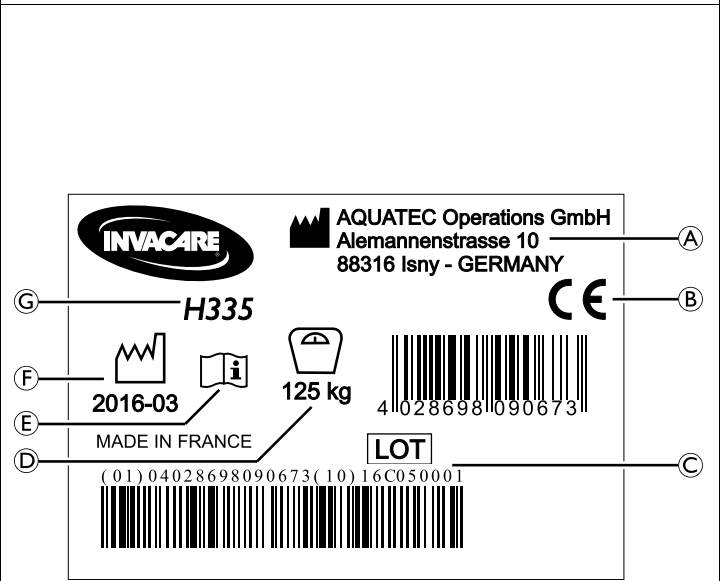
Fig. 8

General

Important note:

This manual MUST be given to the user of the product. BEFORE using this product, read this manual and save for future reference.

This User Manual contains important information about using the product. In order to ensure safety when using the product, read the User Manual carefully and follow the safety information.

If your vision is impaired, you can view the user manual as a PDF file on the Internet and enlarge it on-screen as required. If you cannot enlarge the text and graphics sufficiently, please contact the Invacare distributor for your country. For addresses, see back page of this manual. If necessary, we will provide you with a high resolution PDF file of the user manual.

Intended use

The support folding handles are designed as a support when sitting down, resting and raising from the toilet. Any other use is prohibited. They are not designed to bear the full body weight of the user.

Scope of delivery

Not all products, models or accessories may be available in every country.

Overview — Complete products	
H335	Support folding handle (700 mm)
R7762	Support folding handle (500 – 1000 mm)
R7765	Support folding handle with paper holder (500 – 1000 mm)
R7770	Support folding handle with arm pad and paper holder (500 – 1000 mm)
H345	Support folding handle with support leg (700 mm)
R7772	Support folding handle with support leg (600 – 1000 mm)
R7773	Support folding handle with support leg and paper holder (600 – 1000 mm)
R7780	Support folding handle with support leg, armpad and paper holder (600 – 1000 mm)
Overview — Accessories	
R7763	Floor mounted console
R7764	Support leg
R7765-W	Toilet paper holder
R7766	Armpad
R7767	Aesthetic cover
R7768	Height adjustment plate set (0°, 2°, 4°)
R7769	Support leg with toilet paper holder

The following items are included in the scope of delivery depending on the product combination resp. accessory ordered:

H335 / R7762 – Toilet support bar (Fig. 1)

Pos	Designation	Quantity
Ⓐ	Support bar	1
Ⓑ	Guard	1
Ⓒ	Wall attachment socket (pre-mounted to bracket)	1
Ⓓ	Adjustment plate 2°	1
Ⓔ	Adjustment plate 0°	optional (included in R7768)
Ⓕ	Adjustment plate 4°	optional (included in R7768)

H345 / R7764 – Support leg (Fig. 2)

Pos.	Designation	Quantity
Ⓐ	Set of screw and nut	2
Ⓑ	Hinge piece	1
Ⓒ	End cap	1
Ⓓ	Support bar	1
Ⓔ	Socket	1
Ⓕ	Wall hinge	1
Ⓖ	Fixing screw	1
Ⓗ	Starlock retainer ring	1
Ⓘ	Support leg	1
Ⓢ	Guide	1
Ⓚ	Protective cap	4
Ⓛ	Foot assembly	1

R7763 – Floor mounted console (Fig. 3)

Pos.	Designation	Quantity
Ⓐ	Upright	1
Ⓑ	Attachment plate	1
Ⓒ	Clamp	2
Ⓓ	Nut, DIN 934, M8	4
Ⓔ	Washer, DIN 125, Ø 8,4 x Ø 16	4
Ⓕ	Plate, pre-mounted to upright Ⓐ	1

R7765 – Toilet bracket with paper holder (Fig. 4)

Pos.	Designation	Quantity
Ⓐ	Roll holder	1
Ⓑ	Hinge piece	1
Ⓒ	Protective cap	1
Ⓓ	Screw, DIN 912, M5 x 25	1
Ⓔ	Nut, DIN 934, M5	1
Ⓕ	Protective cap	2
Ⓖ	Spacer Ø 8,2 x Ø 12 x 10	1
Ⓗ	Fastener, Ø 8	1

R7766 – Armpad (Fig. 5)

Pos.	Designation	Quantity
Ⓐ	Armpad, PUR	1
Ⓑ	Glue	1

R7767 – Aesthetic cover (Fig. 6)

Pos.	Designation	Quantity
Ⓑ	Plastic cover	1
Ⓒ	Screw and nut	1
Ⓓ	Protective cap	2

Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the appliance was obtained.

Compliance

Quality is of the utmost importance for our company; all processes are based on the ISO 13485 standard.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class 1.

Invacare is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. We only use REACH compliant materials and components.

For further information please contact Invacare in your country (for addresses, see back page of this manual).

Service life

The expected service life of this product is five years when used daily in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

Safety

Safety information

WARNING!

Risk of falling!

- Do not use if defective.
- Do not use as sports equipment.
- Do not use as a climbing aid.

Labels and symbols on the product

Identification label (Fig. 8)

Ⓐ	Manufacturer address
Ⓑ	Conformity mark
Ⓒ	Lot number
Ⓓ	Maximum user weight
Ⓔ	Read user manual
Ⓕ	Date of manufacture
Ⓖ	Product designation

Setup and usage

Installing the support folding handle

WARNING!

Risk of falling!

- Mount only to walls that have a sufficient load-bearing capacity.

Defining the installation height (Fig. 1)

1. With the user standing upright, without shoes, measure the height of the thumb's upper joint. This height corresponds to the support bar's highest point Ⓐ (middle of support bar).

Installing the support bar (H335 / R7762) (Fig. 1)

1. Insert the 2° adjustment plate Ⓓ.
2. Mark out the location of the holes on the wall (2 above and 2 below outside) taking into account the previously measured height.
3. Remove the 2° adjustment plate.
4. Position the wall attachment socket Ⓒ for the support bar accordingly.
5. Drill and insert 4 dowels appropriate for the type of wall.
6. Fix the socket to the wall using suitable screws, washers.
7. Snap the guard Ⓑ onto the socket.
8. Insert the 2° adjustment plate from below again.

Adjusting the height

The support bar is supplied with a 2° adjustment plate Ⓓ. To adjust the height of the support bar, use option R7768 (3 adjustment plates of 0°, 2° and 4°).

1. Remove the 2° adjustment plate.
2. To lower by 10 mm, insert the 0° adjustment plate Ⓔ.
3. To raise by 10 mm, insert the 4° adjustment plate Ⓕ.

Installing the support leg (H345 / R7764) (Fig. 2)

1. Remove the socket's guard, if fitted.
2. Remove the end cap Ⓒ, if fitted or the cover on the lower tube of the support bar Ⓓ.
3. If not fitted, install the hinge piece Ⓑ by sliding it onto the lower tube. If necessary, use a plastic mallet. Fix with one screw and cover with two protective caps.
4. Install the front end of the guide Ⓘ into the oblong hole in the support leg Ⓙ using a plastic mallet and starlock retainer ring Ⓚ.
5. Remove the lower left-hand fixing screw Ⓜ from the socket Ⓔ.
6. Position the guide Ⓘ on the outside of the support bar, the end with the wall hinge Ⓛ pointing towards the support wall or socket Ⓔ.
7. Fasten the wall hinge Ⓛ using the socket's lower, left-hand side fixing screw Ⓜ.
8. Insert the upper end of the support leg Ⓙ into the hinge piece Ⓑ. Fasten it using the screw and position two protective caps Ⓚ.
9. Install the socket's guard.
10. Check that the whole assembly is firm and operates correctly.

Adjusting the height (Fig. 2)

1. To decrease the height of the support leg turn the foot assembly Ⓙ counterclockwise into the support leg Ⓙ.
2. To increase the height of the support leg turn the foot assembly clockwise out of the support leg.

The height of the support bar can additionally be adjusted with the 0°, 2° and 4° adjustment plates like described above.

Installing the console (R7763) (Fig. 3)

-  This option, when sealed to the floor, enables a support bar (R7762, R7770, R7772 or R7773) to be fastened in place when the wall is unsuitable for this.

1. Position the upright **A**.
2. The plate **F** resting on the floor must have its longest side under the support bar **G**.
3. Mark out the location of the 4 screws.
4. Drill and insert 4 dowels appropriate to the type of floor.
5. Fix the plate to the floor using suitable screws.
6. Position the plate **B** with the rounded section towards the bottom and the support bar **G**.
7. Fasten the plate **B** and the support bar **G** with the two clamps **C**, washers **E** and nuts **D** supplied but without tightening them.
8. With the user standing upright, without shoes, measure the height of the upper joint of the user's thumb and move the support bar to this height.
9. Tighten the nuts **D** on the two clamps **C**.
10. Install the socket's guard.
11. Check that the whole assembly is firm.

Installing options

The basic R7762 / H335 version can be fitted with the following options:

- A toilet paper holder (R7765) mounted on the support bar (Fig. 4).
- A soft, comfortable armpad (R7766) glued to the end of the upper support bar (Fig. 5).
- An aesthetic cover (R7767) designed to hide the lower tube (Fig. 6).

Installing the toilet paper holder (R7765) (Fig. 4)

1. If fitted, remove the end cap **I** from the lower support bar **J**.
2. Remove the protective cap **C** from the hinge piece **B** using a screwdriver
3. Insert the toilet roll support **A** into the hinge piece **B**.
4. Position the spacer **G** and lock the assembly in place with the fastener **H**.
5. Attach the hinge piece **B** by pressing or by tapping with a plastic mallet to the lower support bar **J** and fasten with screw **D** and nut **E**. Attach a protective cap **F** on each side.
6. Check that the whole assembly is firm.

Installing the armpad (R7766) (Fig. 5)

1. If fitted, remove the black end cap **D** from the support bar's upper tube **C**.
2. Clean the parts to be glued together with acetone before applying the glue **B**.
3. Apply the whole tube of glue to the inside face of the armpad **A** and position it onto the support bar **C** being aligned lengthwise, ensure the armpad to point straight upward.
4. Press it down for approximately 30 seconds.

-  Full adhesion is only obtained after approximately 24 hours. Invacare is not responsible for any damage caused by improper fixing or inappropriate use.

Installing the aesthetic cover (R7767) (Fig. 6)

1. If fitted, remove the end cap, position **E**, from the lower support bar **A**.
2. Attach the plastic over **B** over the lower support bar. If necessary, use a plastic mallet to insert the cover completely.
3. Insert the screw and the nut **C** and tighten.
4. Attach a protective cap **D** on each side.

Using the support folding handle

WARNING!

Risk of falling or slipping!

- Before each use, check that the support folding handle is securely fastened and sits firmly.

1. Grab the support folding handle with one hand and carry out the desired body movement.

Raising the support bar

1. Lift the support folding handle from the horizontal position to its maximum vertical position. It locks automatically in this position.

Lowering the support bar

1. Lift the support folding handle approx. 10 mm to unlock it and hold it as it drops down to the horizontal position.

Maintenance

Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined in this user manual are adhered to.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

Cleaning and disinfection

IMPORTANT!

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Health Care Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG139 and your local infection control policy.
- Clean by hand.
 - Clean without scouring agents.
 - Max. cleaning temperature 65 °C for 3 minutes.

The product can be cleaned and disinfected using commercially available agents.

1. Disinfect the product by wiping down all generally accessible surfaces with disinfectant.

Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly.

WARNING!

Risk of injury

- Worn fastening elements can suddenly come loose and cause you to fall.
- Always use new wall plugs and bolts when repositioning the product.

Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

Technical data (Fig. 7)

Product	R7762	R7772
Ⓐ	Æ 32 mm	
Ⓑ	420 – 920 mm ¹⁾	
Ⓒ	135 mm	
Ⓓ	135 mm	
Ⓔ	500 – 1000 mm ¹⁾ / (H335/H345 = 700 mm)	
Ⓕ	220 mm	
Ⓖ	-	750 – 850 mm
Product weight ¹⁾	3.0 kg (500 mm length) 4.0 kg (1000 mm length)	H335: 3.0 kg H345: 3.5 kg
Maximum user weight	125 kg	
Color	White	
Material ²⁾	Steel, Plastic, PU (armpad option)	

¹⁾ Depends on support handle length of the product ordered.

²⁾ All product components are corrosion resistant and do not contain natural rubber latex.

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Environmental parameters

Operating conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Atmospheric pressure	940 - 1060 hPa
Storage and shipping conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing
Atmospheric pressure	795 - 1060 hPa

Invacare distributors:

Belgium & Luxemburg:

Invacare nv
Autobaan 22
B-8210 Loppem
Tel: (32) (0)50 83 10 10
Fax: (32) (0)50 83 10 11
belgium@invacare.com
www.invacare.be

Danmark:

Invacare A/S
Sdr. Ringvej 37
DK-2605 Brøndby
Tel: (45) (0)36 90 00 00
Fax: (45) (0)36 90 00 01
denmark@invacare.com
www.invacare.dk

Deutschland:

Invacare GmbH,
Alemannenstraße 10
D-88316 Isny
Tel: (49) (0)7562 700 0
Fax: (49) (0)7562 700 66
kontakt@invacare.com
www.invacare.de

España:

Invacare SA
c/Areny s/n, Polígono Industrial
de Celrà
E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00
Fax: (34) (0)972 49 32 20
contactsp@invacare.com
www.invacare.es

France:

Invacare Poirier SAS
Route de St Roch
F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66
Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com
www.invacare.fr

Ireland:

Invacare Ireland Ltd,
Unit 5 Seatown Business Campus
Seatown Road, Swords, County Dublin
Tel: (353) 1 810 7084
Fax: (353) 1 810 7085
ireland@invacare.com
www.invacare.ie

Italia:

Invacare Mecc San s.r.l.,
Via dei Pini 62,
I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59
Fax: (39) 0445 38 00 34
italia@invacare.com
www.invacare.it

Nederland:

Invacare BV
Galvanistraat 14-3
NL-6716 AE Ede
Tel: (31) (0)318 695 757
Fax: (31) (0)318 695 758
nederland@invacare.com
www.invacare.nl

Norge:

Invacare AS
Grensesvingen 9, Postboks 6230,
Etterstad
N-0603 Oslo
Tel: (47) (0)22 57 95 00
Fax: (47) (0)22 57 95 01
norway@invacare.com
www.invacare.no

Österreich:

Invacare Austria GmbH
Herzog Odilostrasse 101
A-5310 Mondsee
Tel: (43) 6232 5535 0
Fax: (43) 6232 5535 4
info-austria@invacare.com
www.invacare.at

Portugal:

Invacare Lda
Rua Estrada Velha, 949
P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47
Fax: (351) (0)225 1057 39
portugal@invacare.com
www.invacare.pt

Sverige:

Invacare AB
Fagerstagatan 9
S-163 53 Spånga
Tel: (46) (0)8 761 70 90
Fax: (46) (0)8 761 81 08
sweden@invacare.com
www.invacare.se

Suomi:

Camp Mobility
Patamäenkatu 5, 33900 Tampere
Puhelin 09-35076310
info@campmobility.fi
www.campmobility.fi

Schweiz / Suisse / Svizzera:

Invacare AG
Benkenstrasse 260
CH-4108 Witterswil
Tel: (41) (0)61 487 70 80
Fax: (41) (0)61 487 70 81
switzerland@invacare.com
www.invacare.ch

United Kingdom:

Invacare Limited
Pencoed Technology Park, Pencoed
Bridgend CF35 5AQ
Tel: (44) (0) 1656 776 222
Fax: (44) (0) 1656 776 220
uk@invacare.com
www.invacare.co.uk

USA:

Clarke Health Care Products
7830 Steubenville Pike
Oakdale, PA 15071, USA
Toll Free: (888) 347-4537
Phone: (724) 695-2122
email:
info@clarkehealthcare.com
www.clarkehealthcare.com

Canada:

Invacare Canada LP
570 Matheson Blvd E. Unit 8
Mississauga Ontario
L4Z 4G4, Canada
Phone: (905) 890 8300
Fax: (905) 501 4336

Aquatec Operations GmbH
Alemannenstrasse 10
D-88316 Isny
Germany

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2. Irrota mahdollinen päätysuojus © tai tukitangon alaputken suojus ⑩.