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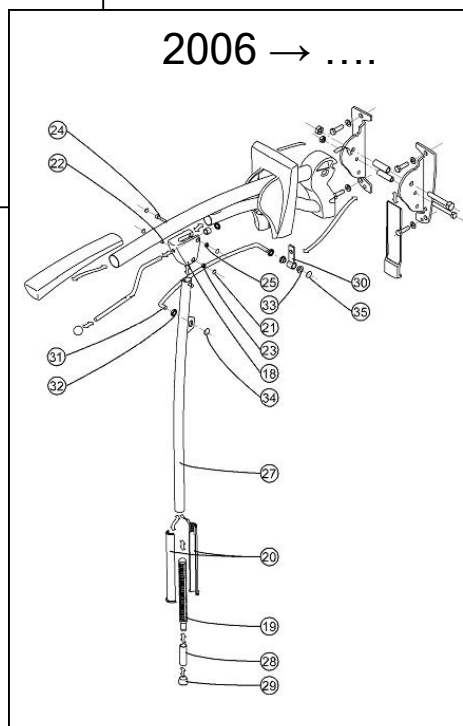
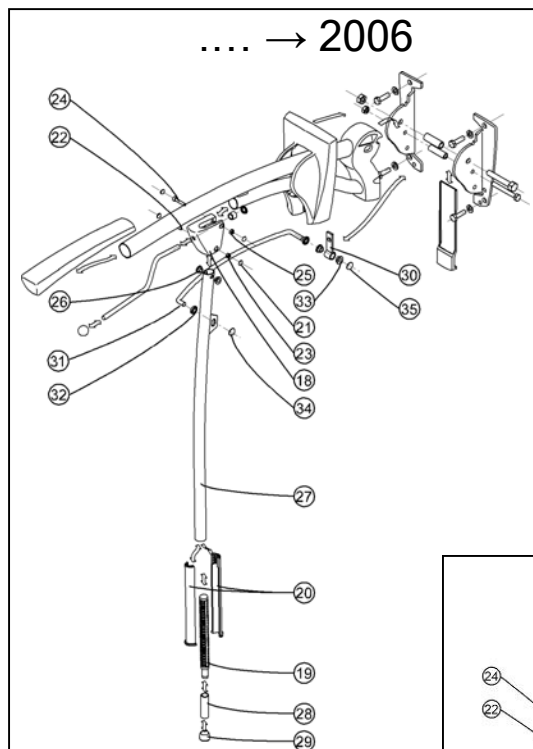
Table of contents	(GB)
General	
Warranty	
Safety – general	
Installation	
Determining the correct height	
Attaching the toilet support to the wall	
Adjusting the height	
Folding uo support rail	
Maintenance	
Optional items	
Attaching the support leg (7764)	
Attaching the toilet roll holder (7765)	
Attaching the armrest (7766)	
Attaching the plastic guard (7767)	
Technical specifications	
List of components and item numbers for 7762 product line	
Exploded view of standard model 7762 and console 7763 (with standard model 7762)	18
Exploded view of comfort toilet bracket 7772	19

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(GB)

Toilet support 7762

The 7762 is a foldable toilet support, available in lengths varying between 50 cm and 100 cm and in different models. The toilet support is intended to function as a universal means of support, also suitable for wet cells.

The standard models come in the following variations:

- Standard model, without the support leg
- Standard model with support leg, without the toilet roll holder
- Standard model with support leg, with toilet roll holder
- 'Comfort' model with armrest and toilet roll holder, available only as a set
- Same as 'Comfort' plus support leg

Introduction

Invacare is delighted that you have chosen to purchase a quality aid.

Through extensive development and manufacture of bathroom, toilet and shower aids, Invacare has become one of the foremost companies in this field. The 7762 range of toilet supports meets CE safety standards which form part of the Medical Devices Directive of 14 June 1993 no. 93/42 EEC.

Warranty

Invacare provides a 2-year manufacturer's warranty on its toilet aids. Normal wear and tear and improper use are not covered under the terms of this warranty. Invacare is not liable for any personal injury as a result of improper use of toilet supports.

Safety - general

Ensure that products in the 7762 range are installed in accordance with the attached drawings and instructions. Toilet supports should preferably be installed by a qualified technician. Always use the means of attachment supplied by Invacare in the R 6670 assembly kit. The 7762 toilet support can only be fixed to a wall with sufficient load-bearing capacity (sand lime brick or similar). In all other cases you should use the accompanying support leg (7764 and 7769), or attach the toilet support to the console mounted on the floor (7763). Please contact Invacare if you have any questions or wish to report any defects.

Installation

Determining the correct height

To maximise ergonomic functioning, the 7762 toilet support must be fitted at the correct height. In order to determine the correct support height for the user, measure the height from the floor to the knuckle on the user's thumb when the user is standing in an upright position with stockinged feet. This is the level of the highest point on the toilet support.

Attaching the toilet support to the wall

- Correctly mark off the holes that need to be drilled on the wall using the wall attachment (use the 2° adjustment plate for the time being).
- Drill the holes in the wall and insert the Rawlplugs.
- Fix the toilet support on the wall, by means of fixations adapted to the nature of the wall (not supplied).
- Click the guard into place on the wall attachment pieces.
- Slide the adjustment plate into place between the wall attachments from underneath.

Adjusting the height

The toilet support is supplied with a standard 2° adjustment plate. If the support has been fixed to the wall and it subsequently turns out that the support height differs somewhat from the required height, the optional 0° and 4° adjustment plates can be used instead to either lower or raise the support height by 1–2 cm (see accompanying drawing).

Folding up support rail

The 7762 toilet support rails can be folded into an upright position without any difficulty. When the support rail is raised from a horizontal position to a vertical position, it will lock into place automatically when it reaches its most upright position, thus securing the rail in a vertical position.

To lower the support rail to its horizontal position, lift the rail slightly to release it and allow it to drop gently into its horizontal position.

Maintenance

The 7762 toilet supports and accessories can be cleaned using water and normal household detergents; do not use a scouring agent or aggressive solvents.

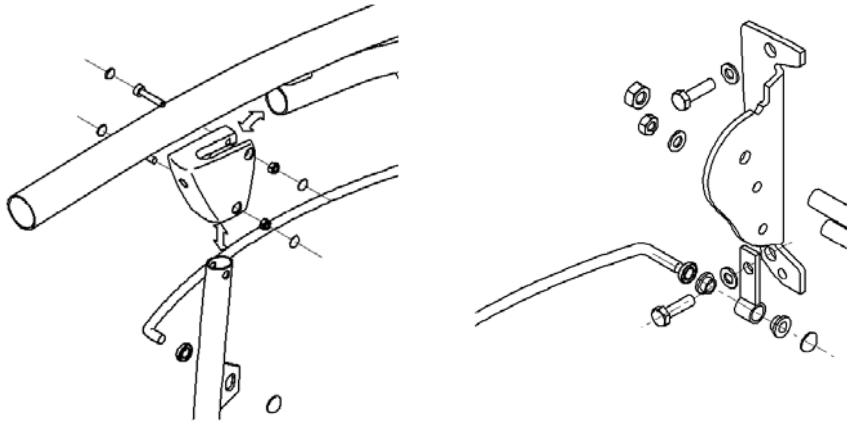
Optional items

General

The 7762 standard model includes the following optional items:



- a support leg (7764)
- a toilet roll holder (7765)
- a comfortable padded armrest (7766)
- a plastic cover to hide the weld (7767)

Attaching the support leg (7764)

- Remove the end cap from the lower support bar.
- Attach the hinge piece, by pressing or by tapping with a plastic hammer.
- Fasten the guide rail with the curve of the rail pointing away from the user.
- Attach the wall hinge to the wall attachment using one of the coach screws.
- Using a knife take off the left or right-hand removable section of the guard.
- Remove the 2° adjustment plate and if necessary replace with the 0° adjustment plate.
- Adjust the height of the support rail by turning the adjustment sleeve in the foot of the support leg so that the leg supports all the vertical load.

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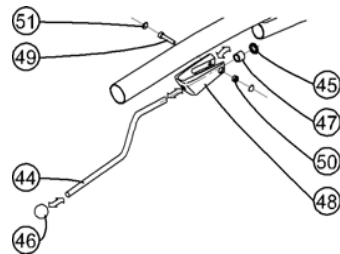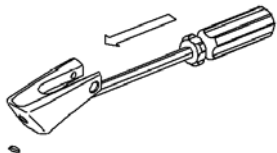
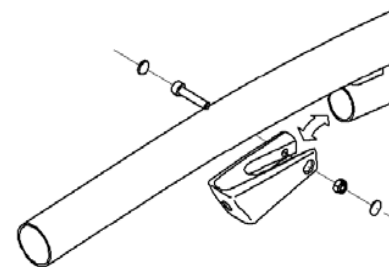
- If necessary remove the standard Ø25, weld cover or hinge piece.
- Using a screwdriver or dowel, take off the removable section from the weld cover or hinge piece, by pressing/tapping outwards.
- Slide the toilet roll holder into the weld cover or hinge piece, place the spacer into position and secure in its entirety using the fastener.
- Fasten the complete roll holder by pressing or tapping with a plastic hammer.

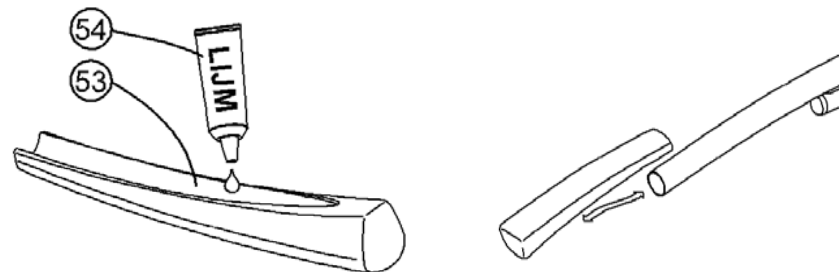
Attaching the armrest (7766)

- Before applying the adhesive glue, clean the adhesive surface of the rail thoroughly using acetone.
- Apply the complete tube of adhesive to the underside of the armrest and place this into position onto the rail ensuring that it is symmetrical.
- Press the armrest down evenly on the rail for 20 - 30 secs. After 24 hours the armrest will be ready to take a full load.

Invacare is not liable for any damage resulting from incorrect attachment/use.

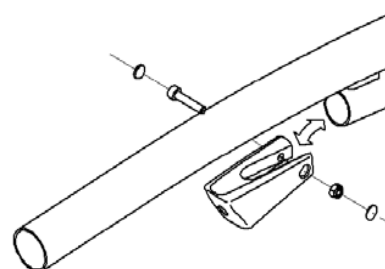Attaching the plastic guard (7767)

- Slide the plastic guard over the lower rail.
- Using a plastic hammer, tap the guard so that it fits snugly against the lower rail.
- Fix the screw, nut and caps.

Technical specifications

- Steel tubing with a shockproof and impact-resistant Rilsan coating
- Hinge pieces manufactured from stainless steel or galvanised steel
- Polyurethane armrest
- Adjustable support height per person
- Available in lengths between 50 cm and 100 cm
- Maximum permissible capacity: 125 kg vertical, 50 kg horizontal
- Weight of standard 7762 model: approx. 4 kg

List of components and item numbers for 7762 product line

Reference	Name	Number
<i>R7762 / 50 - 100</i>	<i>Standard toilet support</i>	
1	Connecting piece (blue/black/white)	1
2	Guard (blue/white)	1
3	End cap Ø 32 (blue/black/white)	1
4	End cap Ø 25 (blue/black/white)	1
5	Adjustment plate 2° (blue/black/white)	1
6	Adjustment plate 0° (blue/black/white)	optional
7	Adjustment plate 4° (blue/black/white)	optional
8	Bracket 50 – 100 cm	1
9	Wall attachment	1 set
10	Knuckle	1
11	Sleeve for locking pin	1
12	Bolt, DIN 931, M 10x60	1
13	Nut, DIN 934, M10	1
14	Bolt, DIN 931, M 8x60	1
15	Nut, DIN 934, M8	1
16	Attachment set (not supplied)	
<i>R7764 / 50 – 100</i>	<i>Support leg</i>	
18	Hinge piece (blue/black/white)	1
19	Spindle	1
20	Screw cover	2
21	Protective cap (blue/black/white)	4
22	Screw, DIN 912, M5 x 30	1
23	Lock nut, DIN 985, M5	1
24	Screw, DIN 912, M5 x 25	1
25	Nut, DIN 934, M5	1
26	Insulation sleeve Ø 5,2 x Ø 6,9	2
27	Support leg	1
28	Adjustment sleeve	1
29	Tube cap Ø 14	1
30	Wall hinge	1
31	Guide rail 60 –100 cm	1
32	Fastener, Ø 8, PA	1
33	Sleeve bearing, Ø 8 x Ø 11 x 7	2
34	Fastener, Ø 8, standard guard	1
35	Fastener, Ø 8, high guard	1
<i>R7763</i>	<i>Console</i>	
36	Upright	1
37	Attachment plate	1
38	Upright cap	1
39	Clamp	2
40	Nut, DIN 934, M8	4
41	Washer, DIN 125, Ø 8,4 x Ø 16	4
42	Attachment set	1
<i>R7765</i>	<i>Toilet roll holder</i>	
44	Roll holder	1
45	Fastener, Ø 8	1
46	Ball knob, Ø 25	1
47	Spacer Ø 8,2 x Ø 12 x 10	1
48	Weld cover (blue/black/white)	1
49	Screw, DIN 912, M5 x25	1
50	Nut, DIN 934, M5	1
51	Guard (blue/black/white)	2
<i>R7766</i>	<i>Armrest</i>	
53	Armrest, PUR	1
54	Glue	1

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