Invacare® H301 Maris

en	Raised toilet seat User Manual	1
de	Toilettensitzerhöhung Gebrauchsanweisung	3
fr	Siège de toilettes rehaussé Manuel d'utilisation	4
it	Tavoletta del WC rialzata Manuale d'uso	6
es	Asiento elevador de inodoro Manual del usuario	7
pt	Elevador de sanita Manual de utilização	8
nl	Verhoogde toiletzitting Gebruiksaanwijzing	10
sv	Upphöjd toalettstols Bruksanvisning	11
no	Hevet toalettsete Bruksanvisning	13
da	Opslået toiletsæde Brugsanvisning	14
fi	Kohotettu wc-jstuiin Käyttöohje	15
lv	Paaugstināta tualetes poda sēdvirsma Instrukciju rokasgrāmata	17

I General

en

I.1 Images

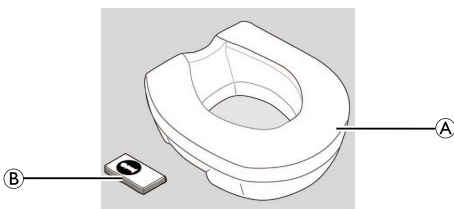
Fig. 1

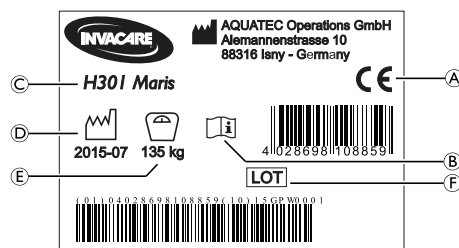
Fig. 2

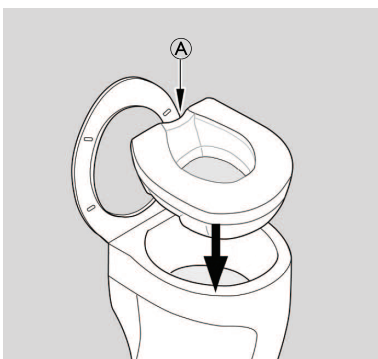
Fig. 3

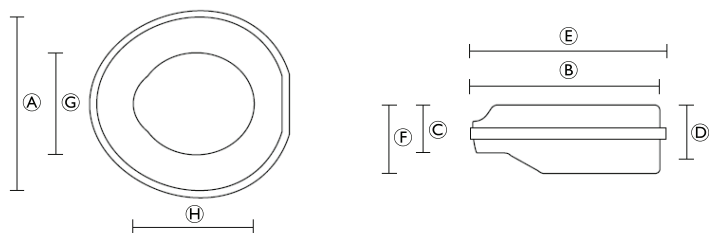
Fig. 4

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

1.2 Symbols in this user manual

In this user manual, warnings are marked with symbols. The title next to the warning symbol indicates the level of danger.

 **WARNING**
Indicates a dangerous situation which could lead to death or serious injury if not avoided.

 **CAUTION**
Indicates a dangerous situation which could lead to minor injuries if not avoided.

 **IMPORTANT**
Indicates a dangerous situation which could lead to damage if not avoided.

 Useful tips, recommendations and information to ensure efficient and smooth operation.

 This product complies with the directive 93/42/EEC for medical products. The launch date for this product is specified in the CE declaration of conformity.

 Manufacturer

1.3 Intended use

The product serves exclusively to raise the toilet seat and is therefore only intended for installation on toilets for people with sufficient torso stability. The user must be able to sit down and get up from the toilet seat safely without help. Any other use is prohibited.

Operating temperature: 10 – 40 °C

1.4 Scope of delivery (Fig. 1)

The following components are included in the scope of delivery:

Ⓐ	Raised toilet seat
Ⓑ	User manual

1.5 Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the appliance was obtained.

1.6 Standards and regulations

Quality is of the utmost importance for our company; all processes are based on the ISO 9001 and ISO 13485 standards.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class I.

The product meets the requirements of the standards DIN EN 12182 (Technical aids for disabled persons - general requirements and test methods), DIN EN ISO 14971 (Medical devices - Application of risk management to medical devices) and DIN EN ISO 10993-1 (Biological evaluation of medical devices - Part 1: Evaluation and testing within a risk management system).

Invacare® is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. We only use REACH compliant materials and components.

For further information please contact Invacare® in your country (for addresses, see back page of this manual).

1.7 Service life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

2 Safety

2.1 Safety information

 **WARNING!**
Risk of falling!

- Do not stand on this product.
- Do not use as a climbing aid.
- Do not use for transfer from the side or partial load.
- Do not tilt.

 **WARNING!**
Risk of injury!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not move or displace the product under load or during use.

2.2 Labels and symbols on the product (Fig. 2)

Ⓐ	Conformity mark
Ⓑ	Note (observe accompanying documents)
Ⓒ	Product designation
Ⓓ	Production date
Ⓔ	Load capacity
Ⓕ	Lot number

 The identification label is attached to the backside of the product.

3 Assembly (Fig. 3)

 **CAUTION!**

- Make sure that the flat side Ⓐ of the raised toilet seat is facing backwards.
- Make sure that the raised toilet seat is placed even and firmly fits on the toilet bowl.

1. Place the raised toilet seat directly on the toilet bowl.

4 Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined in this user manual are adhered to.

 **WARNING!**

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely attached to the toilet bowl.
- Regularly clean and disinfect the product.

4.1 Cleaning and disinfection

 **IMPORTANT!**

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Health Care Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG139 and your local infection control policy.

- Clean by hand.
- Do not use any aggressive (pH value under 4.5 or over 8.5) or abrasive cleaning agents.
- Max. cleaning temperature 60 °C.

 The product can be cleaned and disinfected using commercially available agents.

1. Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

5 After Use

5.1 Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly (® 4.1 Cleaning and disinfection, page 2), check the product for damages and ensure a proper mounting.

5.2 Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

6 Dimensions and weight (Fig. 4)

(A)	365 mm	(E)	400 mm
(B)	385 mm	(F)	185 mm
(C)	125 mm	(G)	190 mm
(D)	135 mm	(H)	250 mm
Product weight	0.9 kg	Color	White
Max. user weight	135 kg	Material*	HDPE
Order number	1522262		

*The material of the product is latex free.

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Il prodotto può essere utilizzato più volte. Il numero di volte dipende dalla frequenza e dal modo in cui viene utilizzato il prodotto. Prima del riutilizzo, pulire e disinfettare accuratamente il prodotto (© 4.1 Pulizia e disinfezione, pagina 7), controllare l'assenza di danni al prodotto e assicurarsi che il montaggio sia corretto.
Invacare AB
Invacare A/S
Aquatec Operations GmbH