
# Dolomite® Jazz

Jazz 510, Jazz 610

en	Rollator User Manual .....	3
de	Rollator Gebrauchsanweisung .....	21
es	Rollator Manual del usuario .....	39
fr	Déambulateur Manuel d'utilisation .....	57
it	Rollator Manuale d'uso .....	75
nl	Rollator Gebruiksaanwijzing .....	93
pt	Andarilho Manual de utilização .....	111

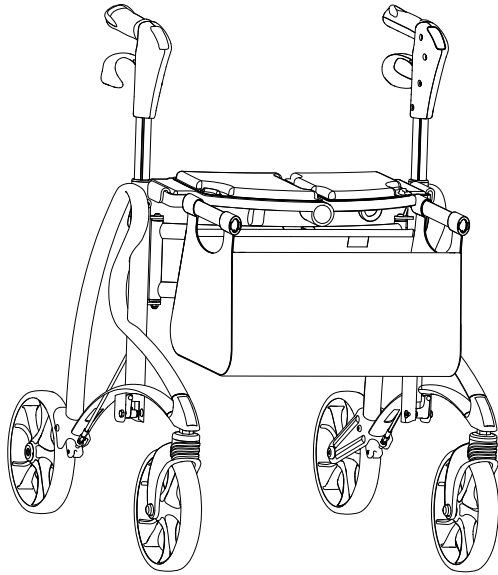This manual **MUST** be given to the user of the product.  
**BEFORE** using this product, this manual **MUST** be read and saved for  
future reference.

**Dolomite**

**Yes, you can.®**

©2019 Invacare® Corporation

All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

Making Life's Experiences Possible is a registered trademark in the U.S.A.

---

# Contents

---

This manual **MUST** be given to the user of the product.  
BEFORE using this product, this manual **MUST** be read and saved  
for future reference.

<b>1 General</b> .....	<b>4</b>
1.1 Introduction .....	4
1.2 Symbols in this Document .....	4
1.3 Compliance .....	4
1.4 Service Life .....	5
1.5 Warranty Information .....	5
1.6 Limitation of Liability .....	5
<b>2 Safety</b> .....	<b>6</b>
2.1 General Safety Information .....	6
2.2 Labels and Symbols on the Product .....	7
<b>3 Product Overview</b> .....	<b>8</b>
3.1 Product Description .....	8
3.2 Intended Use .....	8
3.3 Indications for Use .....	8
3.4 Scope of delivery .....	8
3.5 Accessories .....	8
<b>4 Usage</b> .....	<b>9</b>
4.1 Before Use .....	9
4.2 Unfolding to operating position .....	9
4.3 Folding .....	9
4.4 Walking .....	10
4.5 Sitting down on a chair .....	10
4.6 Rising to a standing position .....	10
4.7 Sitting on the rollator .....	10
4.8 Transporting the rollator .....	11
4.9 Adjusting the handle height .....	11
4.10 Using the Height Adjustment Memory .....	12
4.11 Using the hand brakes .....	13

4.12 Adjusting the brake effect .....	13
4.13 Using the kerb climber .....	13
4.14 Attaching the basket .....	14
4.15 Removing the seat cushions .....	14
<b>5 Maintenance</b> .....	<b>16</b>
5.1 General Maintenance Information .....	16
5.2 Inspection Checklist .....	16
5.3 Cleaning and Disinfection .....	16
5.3.1 General Safety Information .....	16
5.3.2 Cleaning Intervals .....	17
5.3.3 Cleaning Instructions .....	17
<b>6 After Use</b> .....	<b>18</b>
6.1 Disposal .....	18
6.2 Reconditioning .....	18
<b>7 Technical Data</b> .....	<b>19</b>
7.1 Dimensions and weights .....	19
7.2 Material .....	19

# 1 General

## 1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Note that there may be sections in this document, which are not relevant to your product, since this document applies to all available models (on the date of printing). If not otherwise stated, each section in this document refers to all models of the product.

The models and configurations available in your country can be found in the country-specific sales documents.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

For more information about the product, for example product safety notices and product recalls, contact your Invacare distributor. See addresses at the end of this document.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

## 1.2 Symbols in this Document

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

### WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

### CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

### IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

### Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

## 1.3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

## **1.4 Service Life**

The expected service life of this product is five years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

## **1.5 Warranty Information**

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

## **1.6 Limitation of Liability**

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

## 2 Safety

### 2.1 General Safety Information

**WARNING!****Risk of Serious Injury or Damage**

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

**WARNING!****Risk of Serious Injury**

- Incorrect use can lead to hazardous situations.
- Do not use on stairs or on escalators.
  - Do not stand on the product.
  - Be careful when walking on sloping or slippery ground or on uneven surfaces.
  - Avoid walking against obstacles, both forwards and backwards.
  - Do not let children play with the product.
  - Do not use the product to transport a person.
  - Make sure that no clothes, such as long skirts, can get stuck in the wheels.

**CAUTION!****Risk of Injury**

Unbalanced or heavy load may lead to overturning.

- Do not hang any loads, such as bags, to the frame, the push handles or hand knobs.
- Only carry loads with the designated equipment, like basket or tray. Do not exceed the maximum load for the equipment.
- The total load, including the weight of the user and any additional load, must not exceed the maximum user weight.
- Refer to chapter 7 *Technical Data*, page 19 for maximum user weight and load information.

**CAUTION!**  
**Risk of Injury**

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalog or website for available options or contact your Invacare distributor. See addresses at the end of this document.

**CAUTION!**  
**Risk of First Degree Burns**

Parts of the product may heat up, when exposed to high temperatures (above 41° C).

- Be careful if the product is exposed to high temperatures, e.g. direct sunlight.


**IMPORTANT!**

There is a risk of damage due to wear, e.g. corrosion.

- Do not use the product under water, e.g. in pools.
- Do not use the product in hot areas like saunas.
- Do not store the product in wet or humid environments.

**2.2 Labels and Symbols on the Product**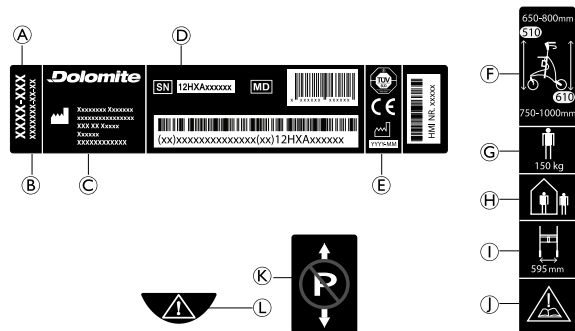

- Ⓐ Part number
- Ⓑ Product name
- Ⓒ Manufacturer address
- Ⓓ Serial number
- Ⓔ Date of manufacture
- Ⓕ Height adjustment range
- Ⓖ Maximum user weight
- ⓓ Indoor / Outdoor use
- Ⓛ Maximum width
- Ⓜ Read user manual
- Ⓚ Do not apply the parking brake during height adjustment
- Ⓛ Height adjustment warning

**Symbols**

	European Conformity		Medical Device
	Manufacturer		Date of manufacture

## 3 Product Overview

### 3.1 Product Description

The four-wheeled walking aid is equipped with a central seat that can be folded up for storage and transportation. Front wheels are steering wheels. Rear wheels are provided with a brake system for slowing down and for locking the wheels. The walking aid is available in different sizes and can be equipped with a wide range of accessories.

### 3.2 Intended Use

The walking aid is intended to support during walking and may also serve as a resting seat.

The walking aid is suitable for indoor and outdoor use on even ground.

The intended user is an adult with a maximum user weight as stated on the product label and in chapter 7 *Technical Data*, page 19.

### 3.3 Indications for Use


#### Indications

Persons with impaired muscle strength and/or balance, who need additional support whilst walking. The remaining walking and cognitive ability must be sufficient for safe use of a walking aid.

#### Contraindications

Persons with severe equilibrium disorders or functional perceptual disorders.

### 3.4 Scope of delivery

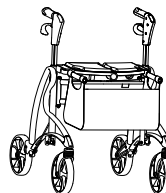- Rollator with standard basket<sup>1</sup>, seat cushion, kerb climber and back strap<sup>2</sup>, fully assembled.
- User manual

<sup>1</sup> not in Denmark, Norway, Finland and Sweden.

<sup>2</sup> only in UK and Canada.

### 3.5 Accessories

Various accessories, like a basket / bag, are available for this product and can be found in the country-specific sales documents. Depending on the configuration accessories may be pre-assembled.

## 4 Usage

### 4.1 Before Use

#### CAUTION!

##### Risk of Injury or Damage to Property

- Before use, verify the general functionality of the product.
- Before use, check the product for any defect.
- Do not use a defective product.


Invacare presume a healthcare professional has made an individual assessment of the user to determine his risk level or therapy requirements. Providers and carers have assessed that the product gives the required support and that its adjustments are appropriate for the user. Carer and user have been appropriately trained by a healthcare professional on how to use the product correctly and safely.

### 4.2 Unfolding to operating position


#### WARNING!

- Always make sure that the rollator is fully unfolded and the lock is engaged before using the rollator.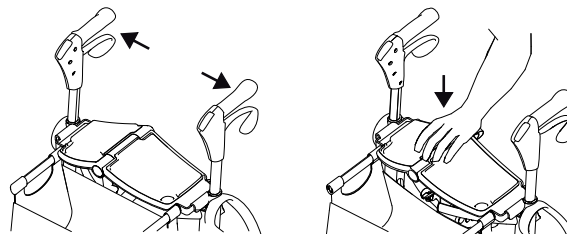

1. Release the rollator by pulling the push handles outwards.
2. Press on the seat in order to lock the rollator in operating position.

### 4.3 Folding

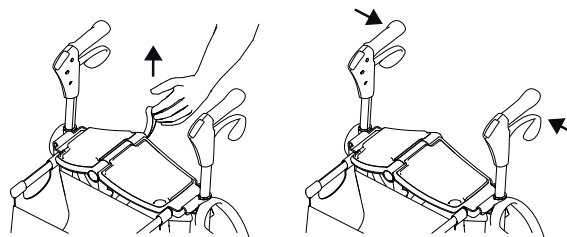1. Stand to the side of the rollator.
2. Lift the folding strap on the seat.
3. Press the push handles inwards to fold the rollator.

#### CAUTION!


- Never lift the rollator with the folding strap when it is in use.

## 4.4 Walking


### **WARNING!** **Risk of falling**

- Be careful if the wheel turns 90 degree and stops against an obstacle.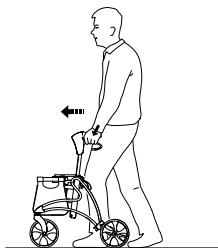

1. Place the rollator a bit in front of you and take a step towards it whilst using the hand grips for support.

## 4.5 Sitting down on a chair

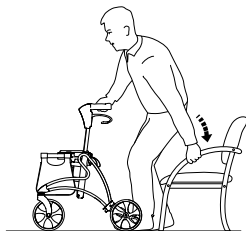
1. Stand in front of the chair with the rollator, the back of the legs should touch the chair.
2. Place the hands on the arms of the chair, lean forward slightly and sit down carefully.

## 4.6 Rising to a standing position

### **WARNING!** **Risk of tipping**

- Never use the rollator to pull yourself up from a sitting position.
- The parking brakes must be in braked position when the rollator is used as a support when getting up, from a chair for example.

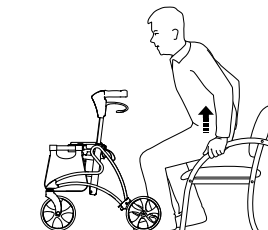1. Place the rollator in front of you, use the armrests of the chair as support and rise carefully to a standing position.
2. Once steady in a standing position, transfer hands to the hand grips of the rollator.

## 4.7 Sitting on the rollator

### **WARNING!** **Risk of falling**

- Use the walking aid for stationary seating only. Both brakes must be in parking position when the walking aid is used as a seat.
- Never sit and push with your feet.
  - Never sit on the walking aid on sloping ground.

**CAUTION!**

There's a risk of getting hurt by the holders for the basket.

- Always sit on the rollator from the correct side, i.e. do not sit down from the frontside of the rollator.


## 4.8 Transporting the rollator

**CAUTION!**

**Risk of pinching**

- Be careful when folding the rollator.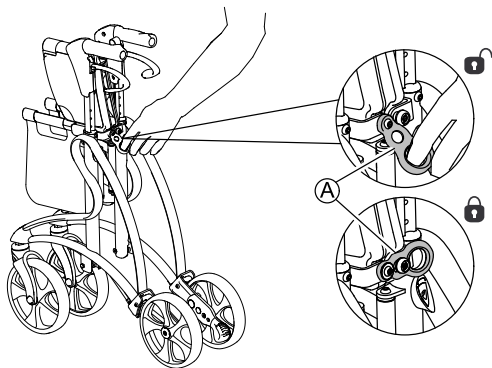

1. Lift the folding strap on the seat and press the push handles inwards to fold the rollator. See also chapter 4.3 *Folding, page 9*
2. Close the locking mechanism **A** to lock the rollator in folded position for transporting.
3. Lift the rollator at the frame and a push handle.

## 4.9 Adjusting the handle height


**CAUTION!**

- Always make sure that the parking brake is released when adjusting the handle height.

**WARNING!**

An incorrect adjustment of the handle height can suspend the brake function.


- After any adjustment of the handle height and before using the rollator, you must always check the brake function.
- Adjust the handle height correctly, if needed.

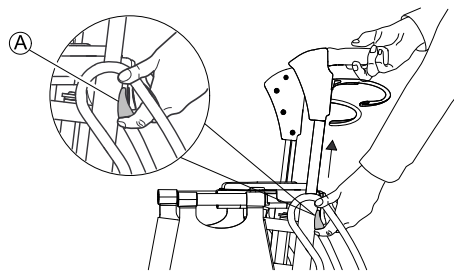1. Stand behind the rollator with flat shoulders.
2. Loosen the handlebar by pushing the locking lever **A** upwards and then adjust the handles to a suitable height.


The handles should usually be positioned at wrist height. Therapist may suggest different heights depending on specific needs of the client.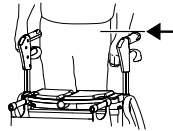

3. Lock the handlebar by pushing the handle slightly downwards, a distinct "click" is heard when the handle is locked in its position.
4. Repeat the procedure on the other side.

## 4.10 Using the Height Adjustment Memory

When adjusting the handle height for the first time, the built-in height adjustment memory can be used.

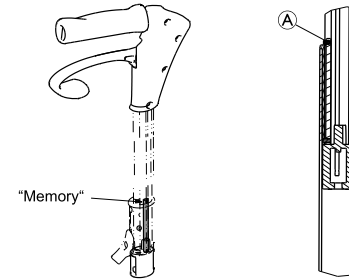1. To access the height memory for the first time, raise the push handles to the highest position to reveal the adjustment screw **A**.
2. Loosen the screw of the height adjustment memory, but do not remove it. Adjust the handles to a suitable height and retighten the screw.
3. Push the handles slightly downwards to fix the height. Suitable tool to use is a 1,5 mm hexagon key (not included).

If the adjustment screw **A** is removed by mistake and the height adjustment memory can not be accessed, raise the handle to the highest position to make the height adjustment memory visible again.

## 4.11 Using the hand brakes


### WARNING!

An incorrect adjustment of the handle height can suspend the brake function.


- After any adjustment of the handle height and before using the rollator, you must always check the brake function.
- Adjust the handle height correctly, if needed.

With the hand brakes, the user can precisely control the brake effect on the rear wheels by pulling the brake handle more or less firmly.

1.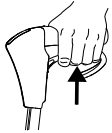

2.

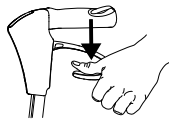1. Apply the brakes by pulling both brake handles slowly and steady upwards.
2. To apply the parking brakes, press the brake handles carefully downwards.


A distinct click indicates that the brake is in the correct position, so that the wheels are locked.

It is important that both brakes are put into parking position.

3. To release the parking brake pull the brake handle upwards.


## 4.12 Adjusting the brake effect

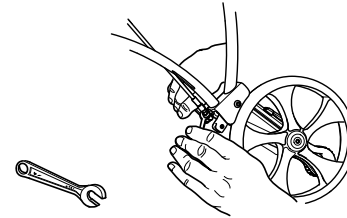1. If braking effect is poor adjust the brake cable. Suitable tools to use are fixed spanners, 8 mm for adjustment and 10 mm for the locking nut (not included).
2. Make sure to tighten the lock nuts after adjustment.

## 4.13 Using the kerb climber

Use the kerb climber to make it easier to mount pavements etc.

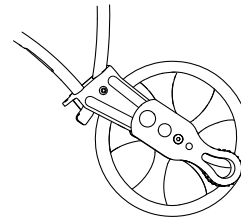
**CAUTION!**

**Risk of falling**

- When using the kerb climber, make sure that the rollator does not roll away uncontrolled after crossing over an obstacle.

**Mount the kerb climber on the opposite side**

The kerb climber can be mounted on both the left and the right side.

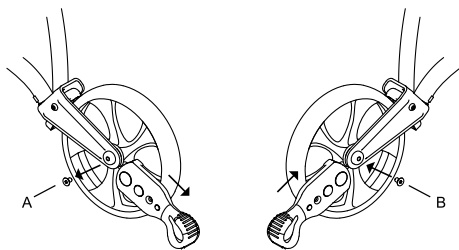1. To mount the kerb climber on the opposite side, loosen the screw and remove the kerb climber (A).
2. Remount the kerb climber on the opposite side (B) and tighten the screw with a 2,5 mm allen key.


**4.14 Attaching the basket**

**WARNING!**

**Risk of imbalance/falling**

- Always lock the brakes before attaching/removing the basket.

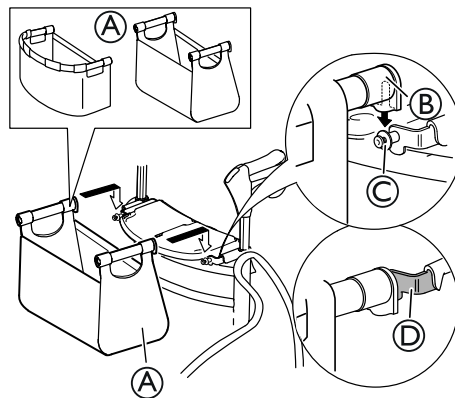1. Attach the basket (A) by pushing the adapters (B) onto the attachment screws (C). Push the basket downwards as much as possible.

Make sure that the top of the adapters (B) are in level with the top of the seat brackets (D).

2. To remove the basket, lift the basket straight upwards.

**4.15 Removing the seat cushions**


The seat cushions on the seat can easily be removed if needed.

**WARNING!**

**Risk of imbalance/falling**

- If the seat cushions are removed, be careful not to get stuck with the clothes on the velcro.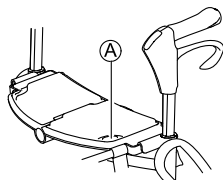

1. To remove the seat cushions, pull the cushion upwards on the marking of the cushion **A**.

## 5 Maintenance

### 5.1 General Maintenance Information

**WARNING!**  
**Risk of Injury**

- Maintenance should be done at least once a year.
- Do not use a defective product.
- Do not attempt repairs yourself. For repairs contact your provider.

### 5.2 Inspection Checklist

To ensure good performance and safety, inspect your product regularly.

- Ensure that all screws and adjustments are properly tightened.
- Ensure that hand grips are tight.  
If hand grips are loose do not use the walking aid. Instructions on how to change hand grips can be obtained from your provider or Invacare distributor.
- Verify brake operation.  
If brake adjustment is required, see chapter 4 *Usage*, page 9.
- Inspect the frame and loaded parts for cracks.
- Inspect the wheels' tightness, wear and damage.
- Make sure that the wheels are free of hair, lint and other debris.

## 5.3 Cleaning and Disinfection

### 5.3.1 General Safety Information

**CAUTION!**  
**Risk of Contamination**

- Take precautions for yourself and use appropriate protective equipment.

**IMPORTANT!**

- Wrong fluids or methods can harm or damage the product.
- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
  - Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
  - Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
  - Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

### 5.3.2 Cleaning Intervals

#### **IMPORTANT!**

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product

- regularly while it is in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

Information on recommended disinfectants and methods can be found on <https://vah-online.de/en/for-users>.

### 5.3.3 Cleaning Instructions

The product can be washed in a washing machine or by hand.

When using a high-pressure cleaner or an ordinary jet of water, do not aim the jet towards bearings, draining holes or labels.

The maximum cleaning temperature is 60 °C. Do not use a steam cleaner.

1. Clean the walking aid with a damp cloth and mild detergent.
2. Clean the wheels with a brush (no wire brush).
3. After washing, wipe the product dry.

#### **Disinfection**

- Wipe down all generally accessible surfaces with disinfectant.

## 6 After Use

---

### 6.1 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

### 6.2 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection


For detailed information, see *5 Maintenance, page 16* and the service manual for this product.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

## 7 Technical Data

### 7.1 Dimensions and weights

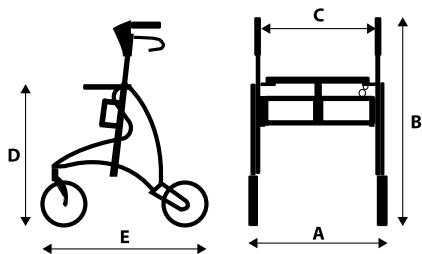[mm]	Jazz 510	Jazz 510H	Jazz 610
Total width (A)	595		
Total height (B)	650 – 800	770 – 900	750 – 1000
Seat width (C)	450		
Seat height (D)	510	510	610
Total length (E)	690		
Width when folded	215		
Front/rear wheel <sup>1</sup> diameter	200		
Front/rear wheel <sup>1</sup> width	33		

<sup>1</sup> Front wheels are steering wheels. Rear wheels are provided with a brake system for dosed breaking and for locking the wheels.

[kg]	Jazz 510	Jazz 510H	Jazz 610
Weight <sup>2</sup>	7.5	7.6	7.8
Max. user weight <sup>3</sup>	150		
Max. load basket	7		
Max. load tray	5		

<sup>2</sup> Total weight excluding equipment.

<sup>3</sup> Weight limitation including the weight of the user (whether standing or sitting) and the content of the basket.

### 7.2 Material

Frame	Polyester powder coated aluminium
Handle bars	Anodised aluminium
Handgrips	TPE
Wheels	Polypropylene with treads of TPE
Plastic parts	Material according to marking (PA, PP, PE)
Screws	Steel, rust protected

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Invacare Poirier SAS
Invacare Austria GmbH
Invacare Rea AB