Invacare® Matrix® PHRK

EN	User Manual	2
FR	Manuel d'utilisation	4
DE	Gebrauchsanweisung	6
ES	Manual de usuario	8
PT	Manual de utilização	10
IT	Manuale Utente	12
DA	Brugsanvisning	14
SV	Bruksanvisning	16
FI	Käyttöopas	18


EN

PHRK Headrest with Multi-Axis Mounting Hardware

The PHRK headrest with multi-axis mounting hardware provides users with a high level of adjustability combined with exceptional comfort and stability. The fully adjustable pivoting hardware allows a wide range of headrest adjustment for the end user. Please refer to the Installation Instructions provided (p.2) to complete the installation of the PHRK headrest.

MAHU MOUNTING HARDWARE KIT:

Figure 1.0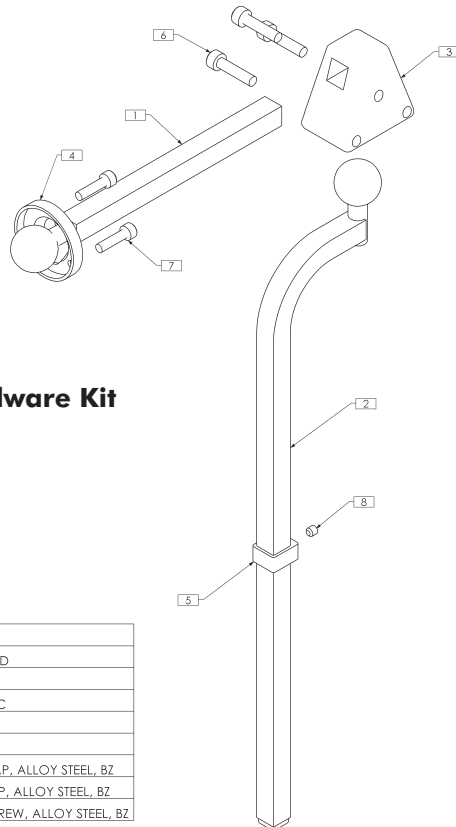

Multi-Axis Mounting Hardware Kit

ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	TRM0037	SMALL HORIZONTAL HEADREST ROD
2	1	TRM0038	LARGE VERTICAL HEADREST ROD
3	1	TRM0039M	TRIANGULAR MULTI-OFFSET, METRIC
4	1	TRM0040	HEADREST MOUNTING RING
5	1	TRM0414M	D-RING, METRIC
6	3	TRH0439	M6-1.0 X 25mm SOCKET HEAD CAP, ALLOY STEEL, BZ
7	3	TRH0704	M5-0.8 X 20mm SOCKET HEAD CAP, ALLOY STEEL, BZ
8	1	TRH0705	M5-0.8 x 5mm HEX SOCKET SET SCREW, ALLOY STEEL, BZ

TOOLS REQUIRED:


Hex Keys: 5mm, 4mm, 2.5mm (provided)
Tape Measure/Ruler

Power Drill
1/4" (6.8mm) Drill Bit

PHRK Headrest with Multi-Axis Mounting Hardware

PHRK HEADREST KIT:

Figure 2.0

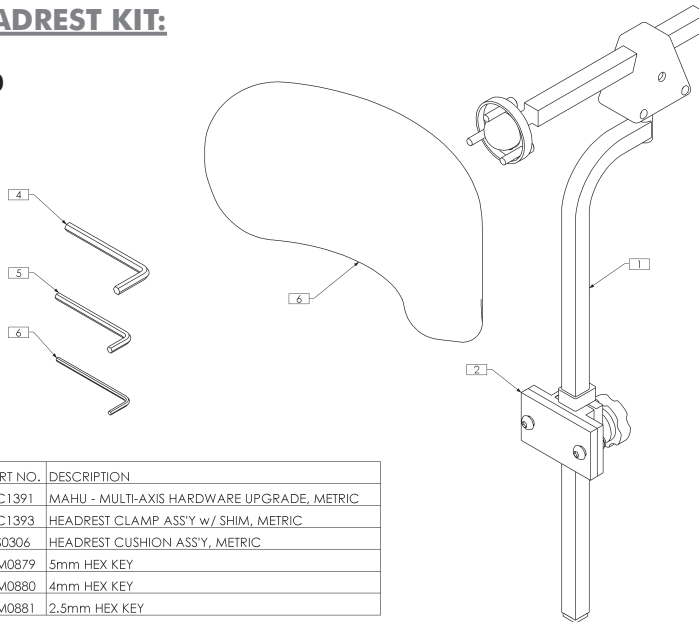ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	TRC1391	MAHU - MULTI-AXIS HARDWARE UPGRADE, METRIC
2	1	TRC1393	HEADREST CLAMP ASS'Y w/ SHIM, METRIC
3	1	TRS0306	HEADREST CUSHION ASS'Y, METRIC
4	1	TRM0879	5mm HEX KEY
5	1	TRM0880	4mm HEX KEY
6	1	TRM0881	2.5mm HEX KEY

INSTALLATION INSTRUCTIONS:

The headrest clamp is secured to the wheelchair back using the hardware provided. Whenever possible, the clamp should be installed into existing mounting holes in the backpan. If no holes are provided, holes may be drilled in the desired location using the mounting plate as a template¹. An optional shim plate may be installed between the clamp ass'y and the back pan to provide additional clearance when needed.

The overall height of the headrest pad/ mounting post can be adjusted via the clamp assembly. (**note:** once the final height position is set, the D-Ring (with set-screw) should be adjusted so that it rests flush with the top of the clamp assembly (to prevent slipping)).

The PHRK headrest pad is secured to the horizontal headrest rod via the mounting ring and hardware provided. (**note:** the headrest pad can be adjusted to any desired angle via the pivot ball at the end of the headrest rod by loosening and tightening the mounting hardware).

The headrest (and headrest rod) can be further adjusted for depth and angle via the triangular multi-offset bracket. To make adjustments, loosen the the hardware in the multi-offset bracket, set headrest to the desired position and re-tighten hardware.

¹ Important! - before drilling/modifying any backpan, be certain to contact the back manufacturer to ensure the back is recommended to support a headrest application.

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Canada • Invacare Canada, Inc., 570 Matheson Blvd. E. Unit 8, Mississauga, Ontario, L4Z 4G4
Danmark • Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby • Tel: (45) (0) 36 90 00 00
Sverige & Suomi • Invacare AB, Box 66, S-163 91, Spånga SWEDEN • Tel: (46) (0) 8 761 7090
Österreich • Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee, Austria