## Invacare® Matrix® SMH

<b>EN</b>	User Manual .....	2
<b>FR</b>	Manuel d'utilisation .....	7
<b>DE</b>	Gebrauchsanweisung .....	12
<b>IT</b>	Manuale Utente.....	17
<b>ES</b>	Manual de usuario .....	22
<b>PT</b>	Manual de utilização .....	27
<b>DA</b>	Brugsanvisning .....	32
<b>SE</b>	Bruksanvisning .....	37
<b>FI</b>	Käyttöopas .....	42

Yes, you can.®

TRD0238-04

## Table of Contents

**EN**

- 1.0 Tools Required
- 2.0 Hardware Kit Contents
- 3.0 Before Installation - Important!
- 4.0 Hardware & MaTRx Back Installation

### 1.0 - Tools Required

Wrench (22mm): (provided)  
Torx Key (T25): (provided)  
Hex Key (2mm): (provided)  
Utility Knife/Scissors  
Phillips Screwdriver  
Tape Measure/Ruler


## 2.0 - Hardware Kit Contents

The **Split Mounting Hardware** has been uniquely designed to allow for mounting of our MaTRx Series wheelchair backs when a rigidizer bar is installed between the wheelchair's back canes. The easy to use split mounting hardware still maintains a high degree of adjustability, while effectively eliminating any interference issues experienced with our standard mounting hardware.

**Note:** the Split Mounting Hardware is **not** recommended for use with MaTRx-PB Heavy Duty backs)

### Split Mounting Hardware Overview Drawing (Right Side)

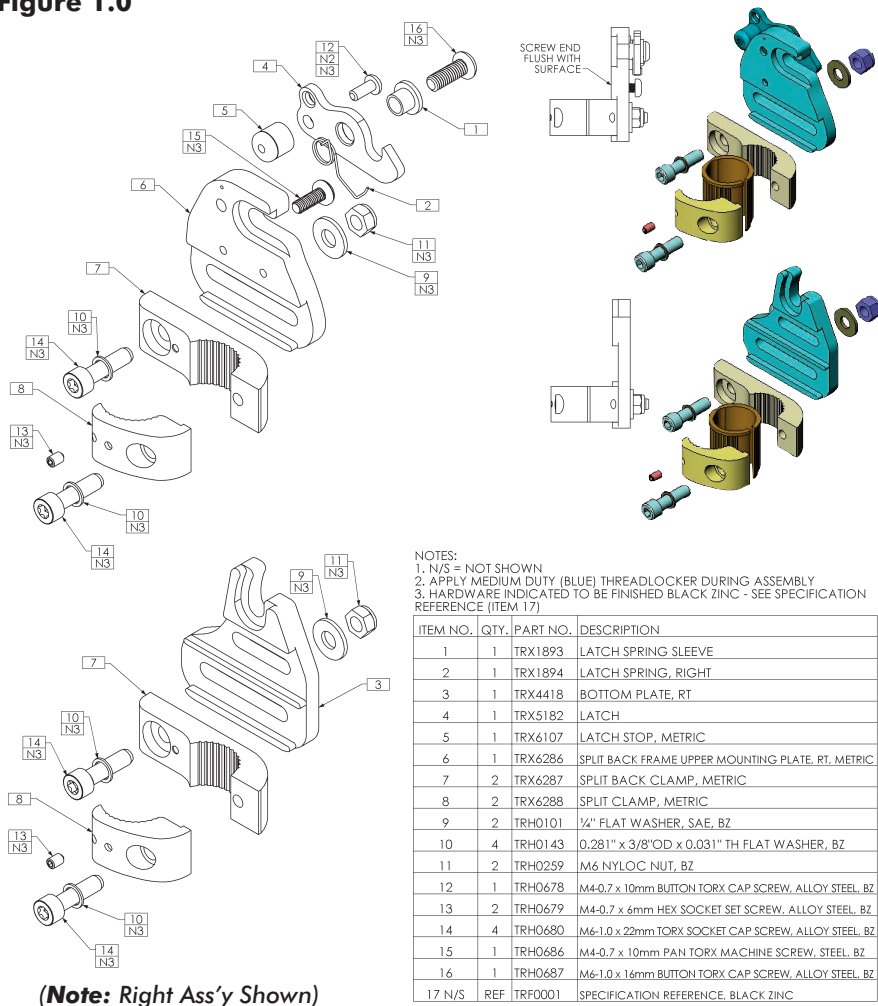**Figure 1.0**

**(Note: Right Ass'y Shown)**

**NOTES:**

1. N/S = NOT SHOWN
2. APPLY MEDIUM DUTY (BLUE) THREADLOCKER DURING ASSEMBLY
3. HARDWARE INDICATED TO BE FINISHED BLACK ZINC - SEE SPECIFICATION REFERENCE (ITEM 17)

ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	TRX1893	LATCH SPRING SLEEVE
2	1	TRX1894	LATCH SPRING, RIGHT
3	1	TRX4418	BOTTOM PLATE, RT
4	1	TRX5182	LATCH
5	1	TRX6107	LATCH STOP, METRIC
6	1	TRX6286	SPLIT BACK FRAME UPPER MOUNTING PLATE, RT, METRIC
7	2	TRX6287	SPLIT BACK CLAMP, METRIC
8	2	TRX6288	SPLIT CLAMP, METRIC
9	2	TRH0101	1/4" FLAT WASHER, SAE, BZ
10	4	TRH0143	0.281" x 3/8" OD x 0.031" TH FLAT WASHER, BZ
11	2	TRH0259	M6 NYLOC NUT, BZ
12	1	TRH0678	M4-0.7 x 10mm BUTTON TORX CAP SCREW, ALLOY STEEL, BZ
13	2	TRH0679	M4-0.7 x 6mm HEX SOCKET SET SCREW, ALLOY STEEL, BZ
14	4	TRH0680	M6-1.0 x 22mm TORX SOCKET CAP SCREW, ALLOY STEEL, BZ
15	1	TRH0686	M4-0.7 x 10mm PAN TORX MACHINE SCREW, STEEL, BZ
16	1	TRH0687	M6-1.0 x 16mm BUTTON TORX CAP SCREW, ALLOY STEEL, BZ
17 N/S	REF	TRF0001	SPECIFICATION REFERENCE, BLACK ZINC

### 3.0 - Before Installation- Important!

#### Mounting Plate Configurations:

##### SEAT DEPTH (Fore/Aft Adjustment):

The **Cane Clamp Kits** may be mounted in a *forward* or *rearward* position on the back canes. The cane clamp assemblies are shipped in the *rearward* mounting configuration- see **Figure 2A** below. This maximizes the seat depth on the wheelchair seat (up to 2.5" (64mm) maximum rearward offset).

By *inverting* the cane clamps on the back canes, the assembly can be reconfigured to give a more *forward* mounting position\* (up to a maximum 2.5" (64mm) forward offset - see **Figure 2B** below).

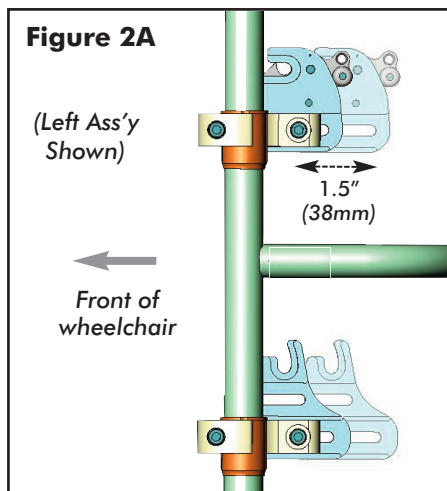
**\*note:** in the forward position, the mounting plates must also be removed and reinstalled in the proper orientation - as indicated below.

The slots in the upper and lower mounting plates, also provide up to 1.5" (38mm) of fore and aft adjustment relative to the cane clamp.


#### ! PLEASE NOTE

When establishing the mounting position for the MaTRx back, ensure the stability of the wheelchair is not compromised.

Adjusting the fore and aft position of the backrest will alter the user's center of gravity within the wheelchair. A recessed back position can significantly reduce the rearward stability of the wheelchair. Similarly, a more forward back position will reduce the wheelchair's forward stability.

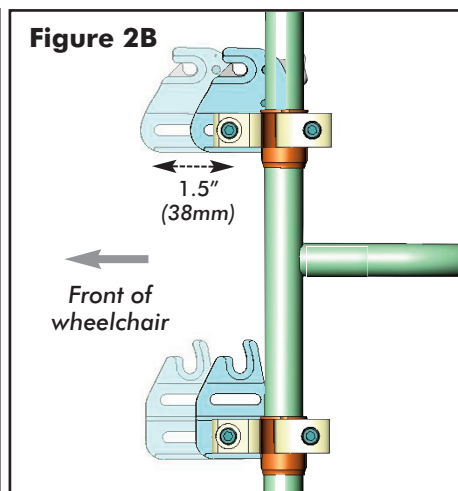Mounting Plates in the **rearmost** back position (maximized seat depth).

Mounting Plates in the **forward most** back position.

## 4.0 Hardware and MaTRx Back Installation

**i Important!** Prior to installation, ensure the **left** and **right** mounting plates are in the proper orientation and secured to the back canes at the same height.

**i Important!** To function effectively and provide proper support, all MaTRx backs should be centered between the back canes.

### Installation Instructions:

- 1 Remove existing wheelchair back upholstery (if applicable).
- 2 Establish the desired cane clamp/mounting plate seat depth configuration for the end user (as per **Section 3.0**).
- 3 Loosen the screws on the **Lower Cane Clamp** ass'y and loosely install the clamps (with mounting brackets) onto each back cane. (*Install plastic inserts if required*).
- 4 Starting with the **left\*** back cane, position/slide the Lower Cane Clamp to the desired height and tighten the clamp screws enough to hold it in place. (*Clamps can be fully tightened after the back is installed and all final adjustments have been made.*)
- 5 From a fixed point on the wheelchair measure the exact height of the *installed* Cane Clamp assembly, and install the second Cane Clamp at the **same height** on the **right\*** back cane. Ensure the Cane Clamps/Mounting Plates are also mounted **parallel** with each other.
- 6 With the lower cane clamp assemblies installed, loosen the screws on the **Upper Cane Clamp** assemblies and install the clamps (with mounting brackets) loosely onto each back cane- approximately 7" (18cm) above the lower clamp ass'y. (*Install plastic inserts if required*). (*Clamps can be fully tightened after the MaTRx back is installed and all final adjustments have been made.*)
- 7 To install the MaTRx-PB elite back onto the mounting plates:
  - Step 1:** Slide the lower mounting pins from the MaTRx back into the notch in the top of the lower mounting plates
  - Step 2:** With the lower pins in place, adjust the height of the upper cane clamp ass'y as needed so that the upper pins (on the MaTRx back) lock/snap into the latch mechanism of the Upper Mounting Plates.
- 8 With the Split Mounting Hardware (upper and lower clamp ass'y) properly adjusted, fully tighten all clamp assembly hardware.
- 9 The mounting pins for the MaTRx back may require adjustment to ensure the backrest is properly fitted/secured to the wheelchair.; For instructions on adjusting the mounting pins, please refer to the installation manual provided with the MaTRx Series back.



**\*note:** refer to **Figure 3.0** to confirm proper mounting plate orientation.

## 4.0 Hardware and MaTRx Back Installation


**i Important!** the left & right mounting plate assemblies shown below are defined from a seated position in the wheelchair (i.e.; forward facing position).

### Split Mounting Hardware Profile (Left & Right Assemblies)

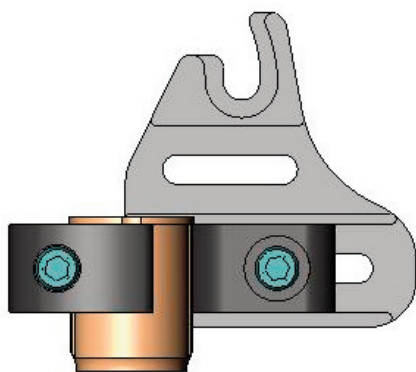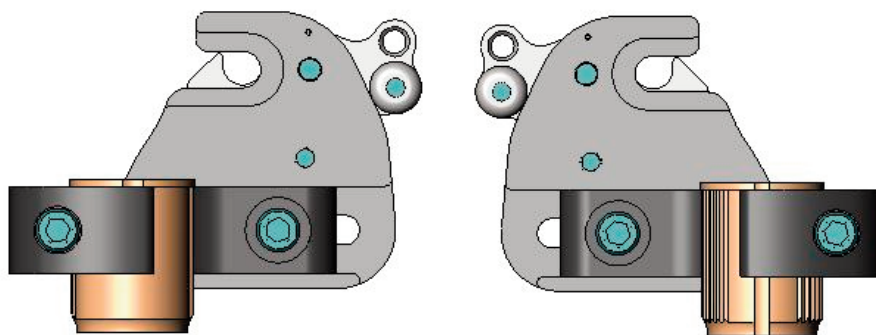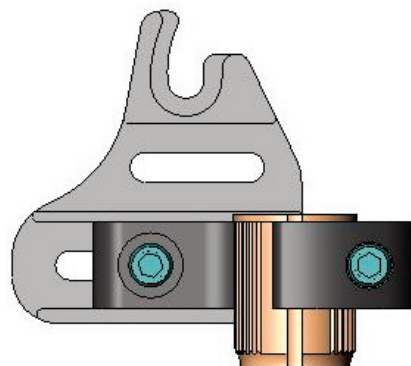Figure 3.0

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**Canada** • Invacare Canada, Inc., 570 Matheson Blvd. E. Unit 8, Mississauga, Ontario, L4Z 4G4
**Danmark** • Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby • Tel: (45) (0) 36 90 00 00
**Sverige & Suomi** • Invacare AB, Box 66, S-163 91, Spånga SWEDEN • Tel: (46) (0) 8 761 7090
**Österreich** • Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee, Austria