

Fixed Lateral Support Installation Instructions: MaTRx-PB Elite

Instructions for Installing Fixed Lateral Supports (Standard and Offset):

- 1 Pull up the bottom of the cover (release velcro fasteners) to expose the mounting slots/channels in the back pan.
- 2 Determine the desired mounting height/orientation for the lateral pads (*upper or lower mounting slot position*).
- 3 If applicable, carefully bend the lateral support(s) to the desired contour (*prior to installation*).
- 4 Loosely install the lateral supports onto the back pan using the hardware provided. (**Note:** the backing plate is installed from the inside of the back pan through the lateral support brackets and secured to the back pan via the washer plates and locking nuts - refer to images below).
- 5 Adjust laterals to the desired position via the mounting slots on the back pan and via the slots in the lateral support bracket. Secure each lateral support into place.
- 6 Re-attach the back cover via the velcro fasteners.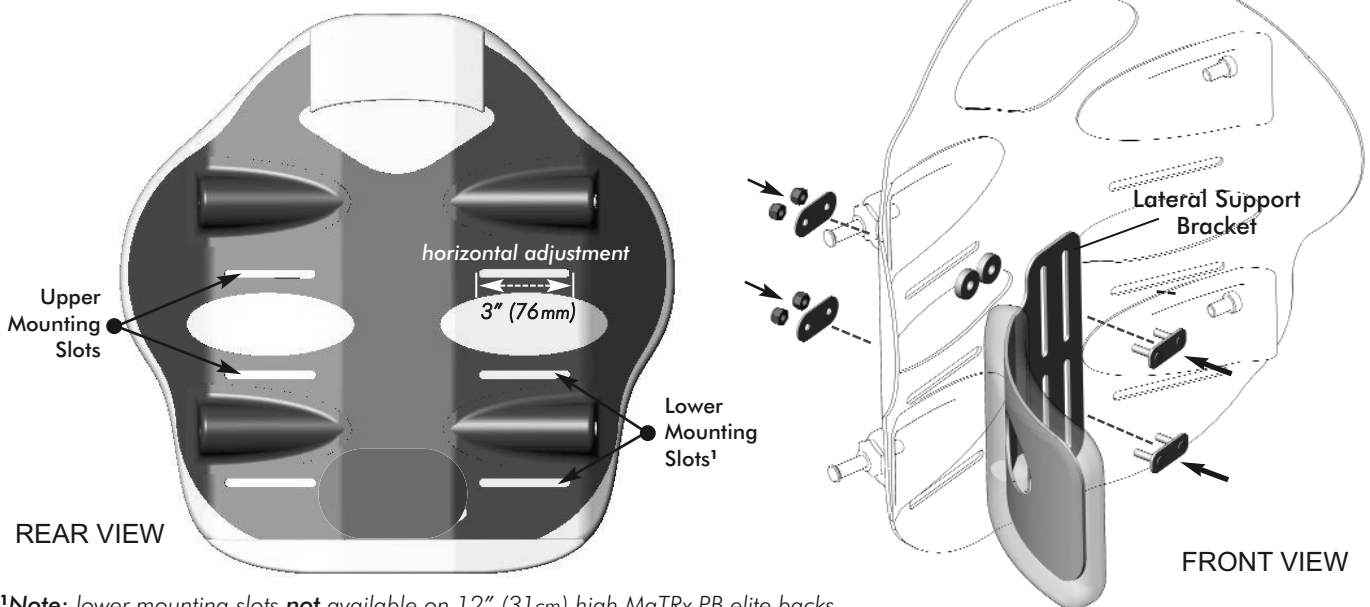



¹Note: lower mounting slots **not** available on 12" (31cm) high MaTRx-PB elite backs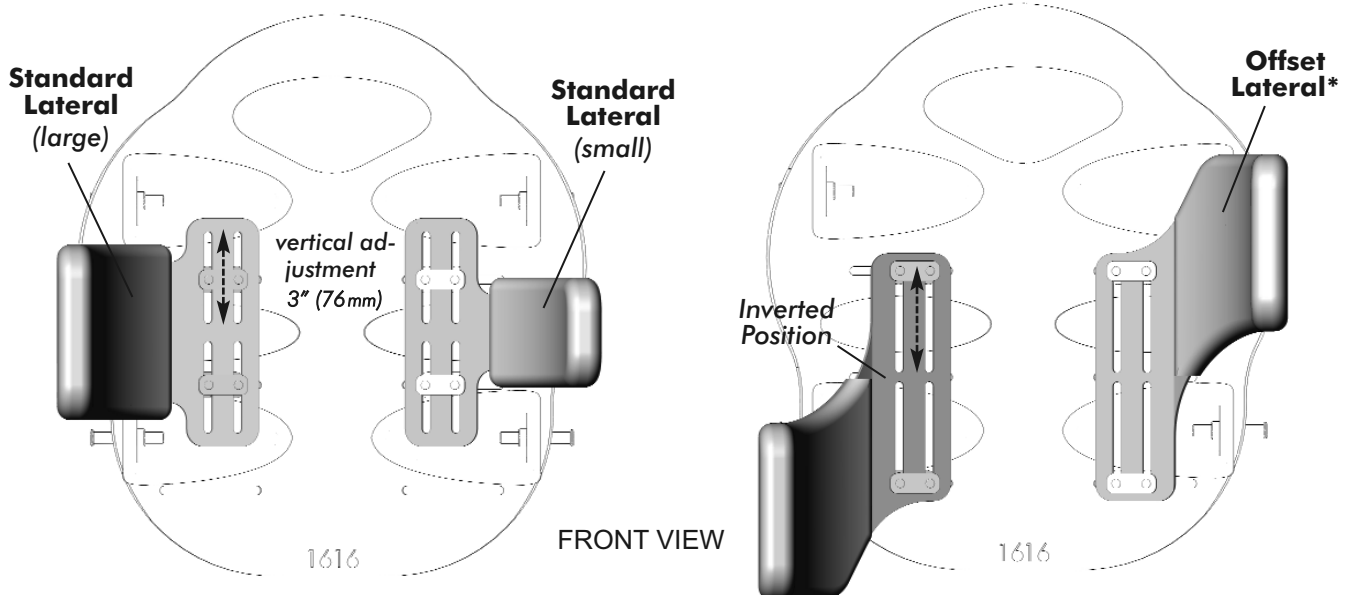

*note: offset laterals may also be mounted in the inverted position (as shown) to provide extended support

Fixed Lateral Support Installation Instructions: MaTRx-PB elite

Standard Lateral Pad Installation (Large and Small Pad)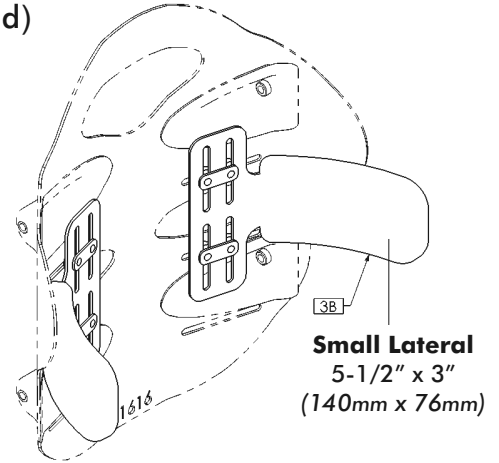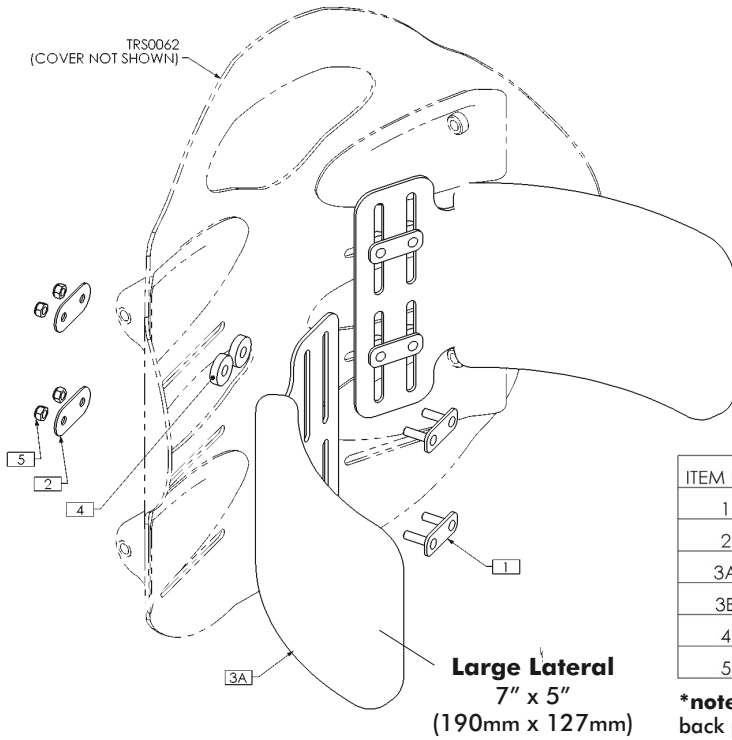

ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	4	TRX6480	BACKING PLATE ASS'Y, METRIC
2	4	TRX6481	WASHER PLATE
3A	2	TRM0757	LATERAL PAD, 7½" x 5" (190mm x 127mm), ELITE
3B	2	TRM0758	LATERAL PAD, 5½" x 3" (140mm x 76mm), ELITE
4*	8	TRM0775	SPACER, ¼" ID x ¾" OD x 0.188" THK
5	8	TRH0697	M5 NYLOC NUT, BZ

*note: spacers are only required when utilizing the upper slot on the back pan

Offset Lateral Pad Installation

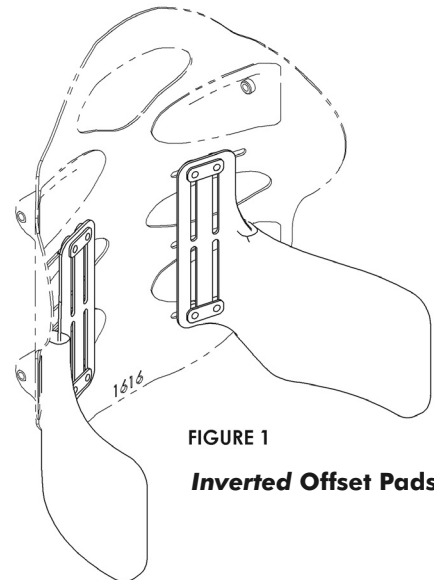
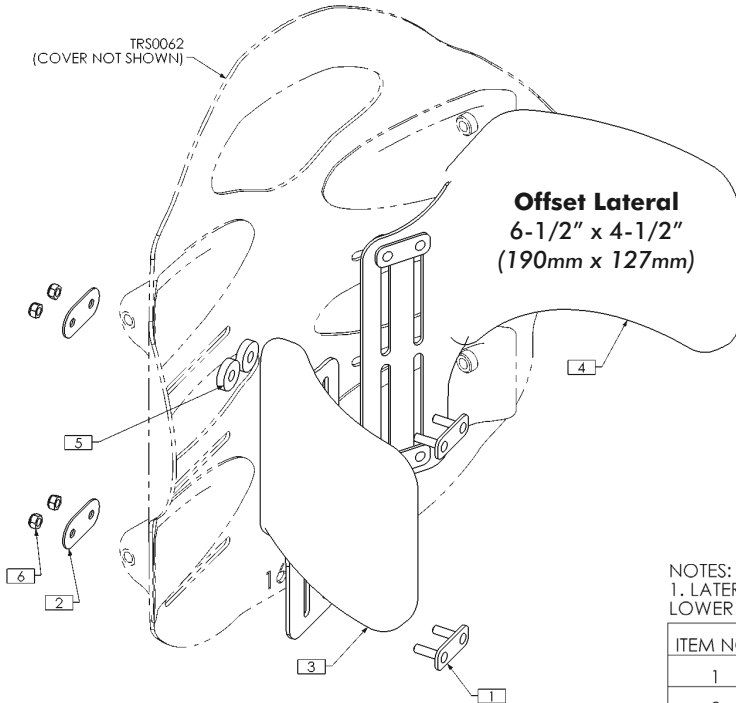
FIGURE 1
Inverted Offset Pads

NOTES:
1. LATERALS CAN BE MOUNTED UPSIDE DOWN ON OPPOSITE SIDES FOR LOWER SUPPORT - SEE FIGURE 1

ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	4	TRX6480	BACKING PLATE ASS'Y, METRIC
2	4	TRX6481	WASHER PLATE
3	1	TRM0759	OFFSET LATERAL PAD, RIGHT, ELITE
4	1	TRM0760	OFFSET LATERAL PAD, LEFT, ELITE
5	8	TRM0775	SPACER, ¼" ID x ¾" OD x 0.188" THK
6	8	TRH0697	M5 NYLOC NUT, BZ

*note: spacers are only required when utilizing the upper slot on the back pan

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Canada • Invacare Canada, Inc., 570 Matheson Blvd. E. Unit 8, Mississauga, Ontario, L4Z 4G4 •
Danmark • Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby • Tel: (45) (0) 36 90 00 00 •
Sverige & Suomi • Invacare AB, Box 66, S-163 91, Spånga SWEDEN • Tel: (46) (0) 8 761 7090 •
Österreich • Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee, Austria •