Dolomite® Futura™ / Dolomite® Symphony™

en	Rollator User Manual	3
de	Rollator Gebrauchsanweisung	21
fr	Rollator Manuel d'utilisation	41
it	Rollator Manuale d'uso	61
nl	Rollator Gebruiksaanwijzing	81

This manual **MUST** be given to the user of the product.
BEFORE using this product, this manual **MUST** be read and saved for
future reference.

Dolomite

Yes, you can.®

©2023 Invacare Corporation

All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

Making Life's Experiences Possible is a registered trademark in the U.S.A.

Contents

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved
for future reference.

1 General	4
1.1 Introduction	4
1.2 Symbols in this Document	4
1.3 Compliance	5
1.4 Service Life	5
1.5 Warranty Information	5
1.6 Limitation of Liability	5
2 Safety	6
2.1 General Safety Information	6
2.2 Labels and Symbols on the Product	7
3 Product Overview	9
3.1 Product Description	9
3.2 Intended Use	9
3.3 Indications for Use	9
3.4 Scope of delivery	9
3.5 Accessories	9
4 Usage	11
4.1 Before Use	11
4.2 Folding and unfolding the walking aid	11
4.3 Adjusting the Handle Height	12
4.4 Walking	13
4.5 Sitting Down on a Chair	13
4.6 Rising to a Standing Position	13
4.7 Sitting on the Walking Aid	14
4.8 Using the hand brakes	14
4.9 Accessories	15
5 Maintenance	16
5.1 General Maintenance Information	16

5.2 Inspection Checklist	16
5.3 Cleaning and Disinfection	16
5.3.1 General Safety Information	16
5.3.2 Cleaning Intervals	17
5.3.3 Cleaning Instructions	17
5.3.4 Disinfection Instructions	17
6 After Use	18
6.1 Disposal	18
6.2 Reconditioning	18
7 Technical Data	19
7.1 Dimensions and Weights	19
7.2 Material	20

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Only use this product if you have read and understood this manual. Seek additional advice from a healthcare professional who is familiar with your medical condition and clarify any questions regarding the correct use and necessary adjustment with the healthcare professional.

Note that there may be sections in this document, which are not relevant to your product, since this document applies to all available models (on the date of printing). If not otherwise stated, each section in this document refers to all models of the product.

The models and configurations available in your country can be found in the country-specific sales documents.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

For more information about the product, for example product safety notices and product recalls, contact your Invacare distributor. See addresses at the end of this document.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.2 Symbols in this Document

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

NOTICE

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

Other Symbols

(Not applicable for all manuals)

UK Responsible Person
Indicates if a product is not manufactured in the UK.

Triman
Indicates recycling and sorting rules (only relevant for France).

1.3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

1.4 Service Life

The expected service life of this product is five years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1.5 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.6 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

2 Safety

2.1 General Safety Information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of Serious Injury

Incorrect use can lead to hazardous situations.

- Do not use on stairs or on escalators.
- Do not stand on the product.
- Be careful when walking on sloping or slippery ground or on uneven surfaces.
- Avoid walking against obstacles, both forwards and backwards.
- Do not let children play with the product.
- Make sure that no clothes, such as long skirts, can get stuck in the wheels.
- Do not use the product to transport a person.

**CAUTION!****Risk of Injury**

Unbalanced or heavy load may lead to overturning.

- Do not hang any loads, such as bags, to the frame, the push handles or hand knobs.
- Only carry loads with the designated equipment, like basket or tray. Do not exceed the maximum load for the equipment.
- The total load, including the weight of the user and any additional load, must not exceed the maximum user weight.
- Refer to chapter 7 *Technical Data, page 19* for maximum user weight and load information.

**CAUTION!****Risk of Injury**

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalogue or website for available options or contact your Invacare distributor. See addresses at the end of this document.

**CAUTION!****Risk of First-Degree Burns**

Parts of the product may heat up, when exposed to high temperatures (above 41° C).

- Be careful if the product is exposed to high temperatures, e.g., direct sunlight.

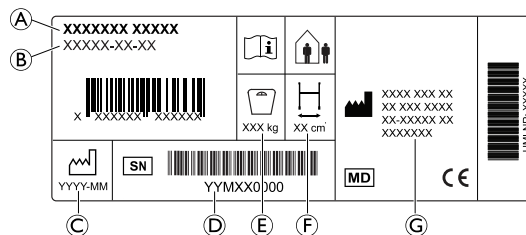
**NOTICE!**

There is a risk of damage due to wear, e.g., corrosion.

- Do not use the product under water, e.g., in pools.
- Do not use the product in hot areas like saunas.
- Do not store the product in wet or humid environments.

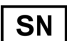
2.2 Labels and Symbols on the Product

Identification label

- (A) Product name
- (B) Part number
- (C) Date of manufacture
- (D) Serial number
- (E) Maximum user weight
- (F) Maximum width
- (G) Manufacturer address

Symbols

	European Conformity		Medical Device
	Manufacturer		Read user manual
	Date of manufacture		Serial number
	Maximum width		Maximum user weight
	Indoor / Outdoor use		

3 Product Overview

3.1 Product Description

The four-wheeled walking aid is equipped with a central seat that can be folded up for storage and transportation. Front wheels are steering wheels. Rear wheels are provided with a brake system for slowing down and for locking the wheels. The walking aid is available in different sizes and can be equipped with a wide range of accessories.

3.2 Intended Use

The walking aid is intended to support during walking and may also serve as a resting seat.

The walking aid is suitable for indoor and outdoor use on even ground.

The intended user is an adult with a maximum user weight as stated on the product label and in chapter 7 *Technical Data*, page 19.

3.3 Indications for Use

Indications

Persons with impaired muscle strength and/or balance, who need additional support while walking. The remaining walking and cognitive ability must be sufficient for safe use of a walking aid.

Contraindications

Persons with severe equilibrium disorders or functional perceptual disorders.

1543649-E

3.4 Scope of delivery

- Walking aid, fully assembled.
- User manual

Accessories, actually assembled to your walking aid, are depending on the configuration and might differ per region.

3.5 Accessories

Various accessories are available for your product and can be found in the country-specific sales documents. Depending on the configuration, accessories may be pre-assembled.

Due to regional differences, refer to your local Invacare website or catalogue for details about available accessories or contact your Invacare provider.

- Basket (several models)

- Bag (under the seat)
- Tray
- Cane holder
- IV-Pole
- Backrest and/or backrest padding
- One-hand brake
- Slow-down brake
- Brake booster
- Hemiplegic handle
- Soft seat
- Crossbar
- Wheel guard
- Kerb climber

4 Usage

4.1 Before Use

CAUTION!

Risk of Injury or Damage to Property

- Before use, verify the general functionality of the product.
- Before use, check the product for any defect.
- Do not use a defective product.

Invacare presumes a healthcare professional has made an individual assessment of the user to determine his risk level or therapy requirements. Providers and carers have assessed that the product gives the required support and that its adjustments are appropriate for the user. Carer and user have been appropriately trained by a healthcare professional on how to use the product correctly and safely.

4.2 Folding and unfolding the walking aid

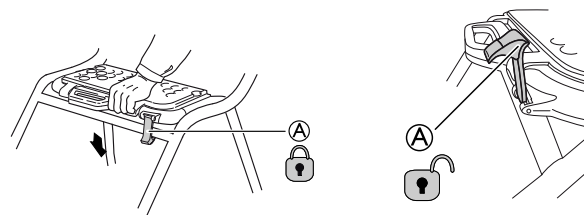
WARNING!

Risk of injury

- Never lift the rollator by the folding handle.

Unfolding to operating position

1. Hold the walking aid with the push handles and make sure that all four wheels are stable on the ground.
2. Press both push handles downwards. Make sure that the seat plate is folded completely down (see picture).
3. Push down the safety latch (A) in the locked position.

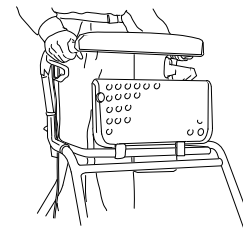
WARNING!

- Before using the product, always check that the safety latch (A) is in locked position indicated by a clear audible “Click”.

Flip up the seat plate


The seat can be folded up to provide a larger walking space, and to increase accessibility.

! **NOTICE!**

The walking aid must be fully unfolded and locked with the folding handle before using the product.
 – The folding handle locks when a clicking sounds is heard.

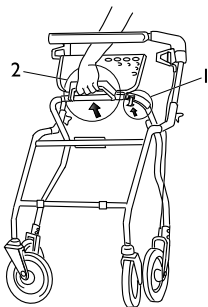Folding

1. Lift up/loosen the safety latch.
2. Lift the handles to fold the walking aid into folded position.

4.3 Adjusting the Handle Height

WARNING!
Risk of falling

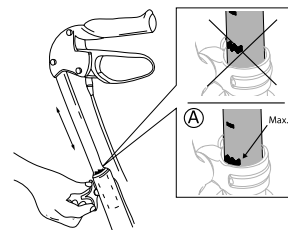
- Always ensure that the hand knobs are tightened securely to prevent the handle from unintended lowering.
- Do not set the handles above their highest allowed position, indicated by the marking on the tubes.

If the user is not supposed to adjust the height of the handles, e.g. for safety reasons, the hand knobs can be replaced by locking screws (available as accessory).

Recommended height: The handles should be positioned at wrist height when standing straight with relaxed arms. Therapist may suggest different heights depending on specific needs of the client.


1. Loosen the hand knob.
2. Adjust the handle to suitable height.
3. Retighten the hand knob. Press on the handle to check for secure fit.
4. Repeat the procedure on the other side.

NOTICE!

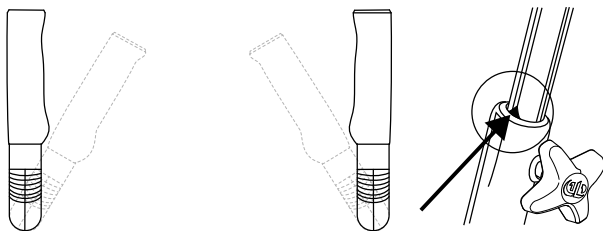– For optimum performance ensure that handles are in the allowed adjustment area at all times. The arrow, which is placed on the tubes on the inside of the walking aid, pointing on the thin inner vertical line indicates maximum allowed outer handle position.


4.4 Walking


WARNING!
Risk of falling

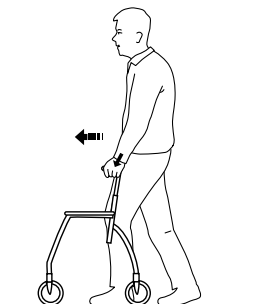– Be careful if the wheels stop against an obstacle, e.g., a kerb.

1. Place the walking aid slightly in front of you and take a step towards it while using the hand grips for support.

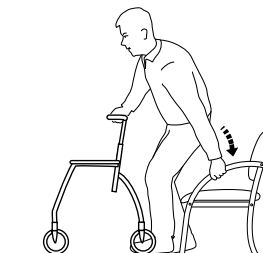4.5 Sitting Down on a Chair


1. Stand in front of the chair with the walking aid, the back of your legs should touch the chair.
2. Lock both hand brakes of the walking aid.
3. Place your hands on the arms of the chair, lean forward slightly and sit down carefully.

4.6 Rising to a Standing Position

WARNING!
Risk of tipping

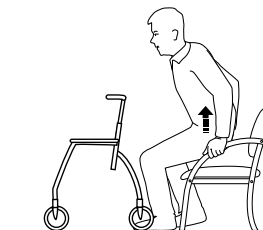– Do not use the walking aid to pull yourself up from a sitting position.

1. Place the walking aid in front of you and lock both hand brakes.
2. Use the armrests of the chair as support and rise carefully to a standing position.
3. Once stable in a standing position, transfer hands to the hand grips of the walking aid.

4.7 Sitting on the Walking Aid

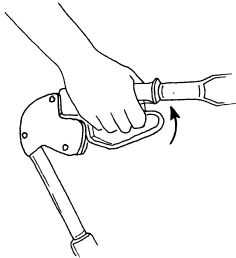
WARNING!

Risk of Falling


- Use the walking aid for stationary seating only.
- Before sitting down, lock both brakes in parking position and make sure the walking aid stands stable.
- Always sit on the walking aid from the rear side. Do not sit down from the front side.
- Do not sit on the walking aid on sloping ground.
- Do not manoeuvre the walking aid with your feet while sitting on it.

4.8 Using the hand brakes

1. Pull both brakes upwards to brake.

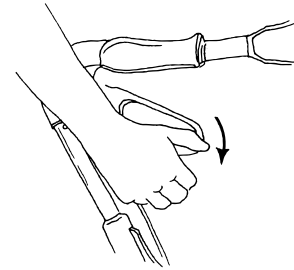Using the parking brake


1. To apply the parking brakes, press the brake handles carefully downwards.
An audible sound indicates that the brake is in the correct position.

NOTICE!

- Make sure that both brakes are put into parking position.
2. To release the brakes, pull the brake handles upwards.

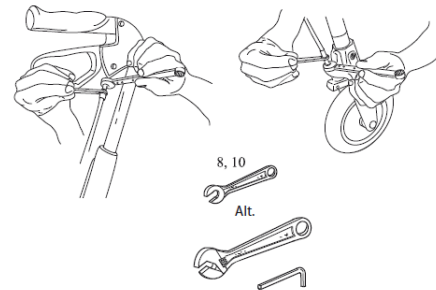Adjusting the braking effect

For desirable braking strength, adjust the brake wire.

Suitable tools: 8 mm and 10 mm spanner (not included).

 The wire can be adjusted in two places - above under the brake handle or below above the brake clamp. The brake handle must be in neutral position.

1. Loosen the counter nut and turn the hexagon nipple counter clockwise in order to increase the braking effect. The distance between the brake clamp and the wheel should be 2 mm.
2. Tighten the counter nut again to fixate the new adjustment.
3. Check the braking effect by engaging the parking brake and check that the wheel can't rotate.

**WARNING!**

– Make sure to tighten the counter nuts after adjustment.

4.9 Accessories

Accessories are not standard on all models. For a list of available accessories, contact your authorized dealer or the Invacare representative (see rear page of this manual).

Basket

Mount the basket by hanging it with the hooks on the front frame.

Tray

Secure the tray by using the plastic plugs.

5 Maintenance

5.1 General Maintenance Information

WARNING!
Risk of Injury

- Maintenance should be done at least once a year.
- Do not use a defective product.
- Do not attempt repairs yourself. For repairs contact your provider.

5.2 Inspection Checklist

To ensure good performance and safety, inspect your product regularly.

- Ensure that all screws and adjustments are properly tightened.
- Ensure that hand grips are tight.
If hand grips are loose do not use the walking aid. Instructions on how to change hand grips can be obtained from your provider or Invacare distributor.
- Verify brake operation.
If brake adjustment is required, see chapter 4 *Usage*, page 11.
- Inspect the frame and loaded parts for cracks.
- Inspect the wheels' tightness, wear and damage.
- Make sure that the wheels are free of hair, lint and other debris.

5.3 Cleaning and Disinfection

5.3.1 General Safety Information

CAUTION!
Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

NOTICE!

- Wrong fluids or methods can harm or damage the product.
- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
 - Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
 - Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
 - Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

5.3.2 Cleaning Intervals

NOTICE!

Regular cleaning and disinfection enhance smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product:

- regularly while in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

5.3.3 Cleaning Instructions

The product can be cleaned in a washing machine or by hand.

When using a high-pressure cleaner or an ordinary jet of water, do not aim the jet towards bearings, draining holes or labels.

The maximum cleaning temperature is 60 °C. Do not use a steam cleaner.

1. Clean the walking aid with a damp cloth and mild detergent.
2. Clean the wheels with a brush (no wire brush).
3. After cleaning, wipe the product dry.

5.3.4 Disinfection Instructions

In Domestic Care

- Disinfectant: We recommend using an alcohol-based surface disinfectant (with 70-90% alcohol).

Read the instructions on the label of your disinfectant. It provides information on the activity spectrum (bacteria, fungi and/or viruses), material compatibility and the correct exposure time.

1. Ensure the surfaces are cleaned before disinfection.
2. Moisten a soft cloth and wipe-disinfect all accessible surfaces and keep them moistened for the exposure time stated on the label of the disinfectant.
3. Allow the product to air-dry.

In Institutional Care

Follow your in-house disinfection procedures and only use the disinfectants and methods specified therein.

6 After Use

6.1 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

6.2 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

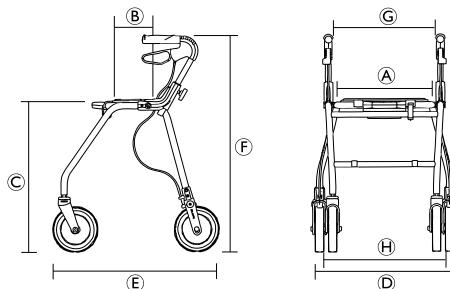
For detailed information, see *5 Maintenance, page 16* and the service manual for this product.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

7 Technical Data

7.1 Dimensions and Weights

[mm]	Symphony 450 Futura 450	Symphony 520 Futura 520	Symphony 600 Futura 600	Futura 680
Seat width ①	360			
Seat depth ②	180			
Seat height ③	475	545	620	705
Total width ④	600	600	600	640
Total depth ⑤	590	660	700	780
Total height (Handle height) ⑥	640 - 790	710 - 850	790 - 930	860 - 1015
Distance between handles ⑦	420			
Distance between rear wheels ⑧	460	470	470	500
Front/rear wheel diameter	200			

All dimensions are subject to certain tolerances.

[kg]	Symphony			Futura			
	450	520	600	450	520	600	680
Weight ¹	6.7	6.8	7.0	7.6	7.8	8.0	8.2
Max. user weight ²	100			150 ³			
Max. load basket	7	9		7	9		
Max. load tray	5						

7.2 Material

	Symphony	Futura
Frame	Polyester powder coated aluminium	Polyester (PES) powder coated steel
Handle bars	Aluminium	
Handgrips	Thermoplastic Elastomers (TPE)	
Wheels	Polypropylene (PP) with treads of Thermoplastic Elastomers (TPE)	
Plastic parts	Material according to marking; Polyamide (PA), Polypropylene (PP), Polyethylene (PE)	
Screws	Steel, rust protected (zinc plated)	

1. Total weight excluding equipment.
2. This weight limitation includes the weight of the user, whether standing or sitting, and any additional load, e.g. the content of the basket.
3. For walking aids produced 2011-02 or earlier, see label on product.

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Invacare GmbH
Invacare Austria GmbH
Invacare Rea AB