
# Invacare® Stand Assist

RPS350-1E, RPS350-1FR

en	Patient Lift User Manual.....	3
de	Aufstehlift Gebrauchsanweisung.....	23
es	Grúa de bipedestación Manual del usuario.....	47
fi	Potilasnostin Käyttöohje .....	67
fr	Lève-personne verticalisateur Manuel d'utilisation .....	87
it	Sollevatore paziente Manuale d'uso .....	109
nl	Stand Assist Patiëntlift Gebruiksaanwijzing .....	133
no	Personløfter Bruksanvisning .....	155
pt	Elevador de transferência Manual de utilização.....	175
sv	Personlyft Bruksanvisning .....	197

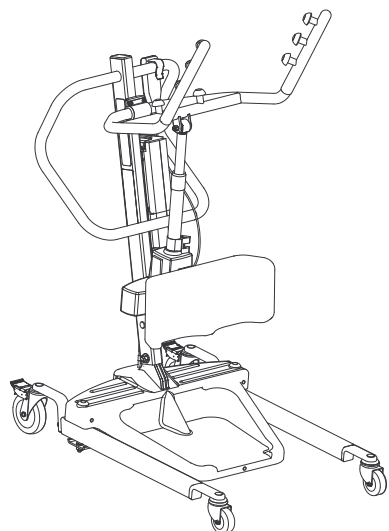This manual MUST be given to the user of the product.  
BEFORE using this product, read this manual and save for future  
reference.

**Yes, you can.®**

© 2018 Invacare Corporation.

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

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# Contents

---

This manual MUST be given to the user of the product.  
BEFORE using this product, read this manual and save for future reference.

<b>1 General</b>	<b>4</b>
1.1 Introduction	4
1.2 Symbols	4
1.3 Intended Use	4
1.3.1 Contraindications	5
1.4 Included Items	5
1.5 Service life	5
1.5.1 Additional information	5
1.6 Warranty information	5
1.7 Compliance	5
1.7.1 Product-specific standards	5
<b>2 Safety</b>	<b>6</b>
2.1 General Guidelines	6
2.2 Operating Information	6
2.2.1 General	6
2.2.2 Positioning	6
2.2.3 Disposal	7
2.3 Radio Frequency Interference	7
2.4 Product Labeling	7
<b>3 Setup</b>	<b>8</b>
3.1 Safe Assembly	8
3.2 Assembling the Mast to the Base	8
3.3 Preparing the Lift for Use	8
3.4 Installing the Shifter Handle	8
3.5 Attaching the Battery Charger Mounting Bracket to the Wall	8
<b>4 Usage</b>	<b>9</b>
4.1 Introduction	9
4.2 Closing/Opening Legs	9
4.2.1 Closing/Opening Manual Legs	9
4.3 Locking/Unlocking the Rear Casters	9
4.4 Raising/Lowering the Lift	9
4.4.1 Raising/Lowering an Electric Lift	9
4.5 Activating a Mechanical Emergency Release	9
4.5.1 Primary Emergency Release	9
4.5.2 Secondary Emergency Release	10
4.6 Performing an Emergency Stop	10
4.7 Charging the Battery	10
4.7.1 Battery Indicator	10
4.7.2 Using a Power Cord to Charge the Battery	10
4.7.3 Using the Battery Charger to Charge the Battery	11
<b>5 Lifting the Patient</b>	<b>12</b>
5.1 Safe Lifting	12
5.2 Positioning the Stand Assist Lift	12
5.3 Lifting the Patient	13
5.4 Moving the Patient	14
<b>6 Transferring the Patient</b>	<b>15</b>
6.1 Safe Transfer	15
6.2 Transferring to a Commode	15
6.3 Transferring to a Wheelchair	16
6.4 Transferring to a Bed	16
<b>7 Troubleshooting</b>	<b>17</b>
7.1 Troubleshooting Table	17
<b>8 Maintenance</b>	<b>18</b>
8.1 Maintenance Safety Inspection Checklist	18
8.2 Cleaning the Sling and the Lift	18
8.3 Reuse	18
8.4 Detecting Wear and Damage	19
8.5 Lubricating the Lift	19
8.6 Adjusting the Base	19
8.7 Replacing the Electric Actuator	19
8.8 Adjusting the Knee Pad Height	19
<b>9 Technical Data</b>	<b>21</b>
9.1 RPS3501E and RPS3501FR Stand Assist Patient Lift	21

# 1 General

## 1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Note that there may be sections in this document, which are not relevant to your product, since this document applies to all available models (on the date of printing). If not otherwise stated, each section in this document refers to all models of the product.

The models and configurations available in your country can be found in the country-specific price lists.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

For more information about the product, for example product safety notices and product recalls, contact your Invacare representative. See addresses at the end of this document.

## 1.2 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

### WARNING!

- Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

### CAUTION!

- Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.

### IMPORTANT

- Indicates a hazardous situation that could result in damage to property if it is not avoided.

Gives useful tips, recommendations and information for efficient, trouble-free use.

Date of manufacture.

This product complies with Directive 93/42/EEC concerning medical devices.

The launch date of this product is stated in the CE declaration of conformity.


Read Manual

Audible tone when battery low. Refer to 4.7 *Charging the Battery, page 10.*

Caster lock.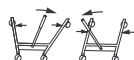

Open/close Legs

Raise/lower the lift arms

Emergency stop

Safe Working Load

Double Insulated, Class II equipment

Type B applied part

Recycle this product. Refer 2.2.3 *Disposal, page 7.*

## 1.3 Intended Use

### WARNING!

#### Risk of Falling

The Invacare stand assist lift is NOT a transport device. It is intended to transfer an individual from one seated surface to another (such as a bed to a wheelchair).

Invacare slings and patient lift accessories are specifically designed to be used in conjunction with Invacare patient lifts.

To avoid injury or falls:

- Patients who are transferred and positioned using the stand assist lift MUST be cooperative, coherent and have head and neck control. Otherwise, injury may occur.
- Patients who are transferred and positioned using the stand assist sling MUST be able to support a majority of their own weight. Otherwise, injury or damage may occur. Refer to the sling user manual for more information.

Stand assist lifts are battery-powered transfer devices, designed to be used in most of the common lifting situations, for example:

- Between the bed and a wheelchair
- To and from the toilet

The stand assist lift is only intended to lift patients up to the maximum weight limit stated in technical data.

The product is intended for indoor use only.

Selecting the appropriate slings and accessories for each individual is important to assure safety when using a patient lift. See Invacare's sling and accessory user manuals for further information on those devices.

Invacare recommends that the patient be transferred to a shower chair or other means for bathing.

The stand assist lift can be turned (rotated) in place for transfers with limited floor space.

### 1.3.1 Contraindications


The stand assist patient lift is contraindicated for patients who:

- are not cooperative
- are not coherent
- do not have head and neck control

Some slings designed for the stand assist patient lift are contraindicated for patients who cannot support the majority of their weight. Refer to the sling user manual for more information.

### 1.4 Included Items

The items listed in the tables are included in your package. Slings are sold separately. The wall charger may be sold separately.

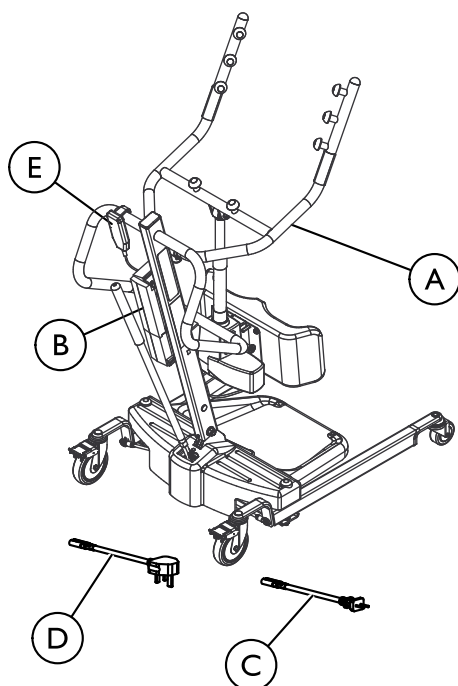Item	Description	Quantity
A	Lifter	1
B	Battery	1
C	Charging Cord, EU	1
D	Charging Cord, UK (RPS350-1E only)	1

E	Hand Pendant	1
F	Lifter User Manual (not shown)	1

### 1.5 Service life

The expected service life of this product is eight years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

#### 1.5.1 Additional information

The expected service life is based on an estimated average of 4 lifting cycles per day.

### 1.6 Warranty information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

### 1.7 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class 1.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

We comply with the current environmental legislations WEEE and RoHS.

#### 1.7.1 Product-specific standards

The product has been tested and conforms to ISO 10535 (Hoists for the transfer of disabled persons) and all related standards.

For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

## 2 Safety

### 2.1 General Guidelines

#### WARNING!

– DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

#### ACCESSORIES WARNING

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories.

Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products. In certain instances, the use of other manufacturer's slings may be possible.

– Contact your local Invacare office for more information about accessories.

#### NOTICE

– The information contained in this document is subject to change without notice.

Check all parts for shipping damage before using. In case of damage, DO NOT use the equipment. Contact the Dealer or Invacare representative for further instructions.

### 2.2 Operating Information

This section of the manual contains general safety information about your product. For specific safety information, refer to the appropriate section of the manual and procedures within that section. For instance, for safety information related to assembling the lift, refer to the Setup section of the manual.

#### 2.2.1 General

#### WARNING!

##### Risk of Falling

DO NOT attempt any transfer without approval of the patient's physician, nurse or medical assistant. Thoroughly read the instructions in this user manual, observe a trained team of experts perform the lifting procedures and then perform the entire lift procedure several times with proper supervision and a capable individual acting as a patient.

- Use common sense in all lifts. Special care MUST BE taken with people with disabilities who cannot cooperate while being lifted.
- Use steering handle on the mast at all times to push or pull the patient lift.
- Be sure to check the sling attachments each time the sling is removed and replaced, to ensure that it is properly attached before the patient is removed from a stationary object (bed, chair or commode).

#### WARNING!

##### Risk of Injury or Damage

Some surfaces may cause the lift to be unstable and cause injury or damage.

The lift can be used in the shower or bath area, but excessive moisture will damage the lift and may cause injury.

- Avoid using the lift on an incline. Invacare recommends that the lift only be used on a flat surface.
- DO NOT roll casterbase over uneven surfaces that may cause the patient lift to tip over.
- DO NOT use the patient lift in the shower or bath or in any prolonged moisture environment.
- Ensure that the patient lift is wiped clean of any moisture after use.
- DO NOT store the lift in a damp area or in a damp condition. Refer to the storage temperature, humidity and pressure ranges listed in *7.1 Troubleshooting Table, page 17*.
- Periodically inspect all components of the patient lift for signs of corrosion. Replace all parts that are corroded or damaged.


#### 2.2.2 Positioning

#### WARNING!

##### Risk of Injury

- ALWAYS be aware of the Lift Arms. Injury to the patient and/or assistant may occur.
- ALWAYS be aware of the Footrest, especially the patient's position on the footrest. Injury to the patient and/or assistant may occur.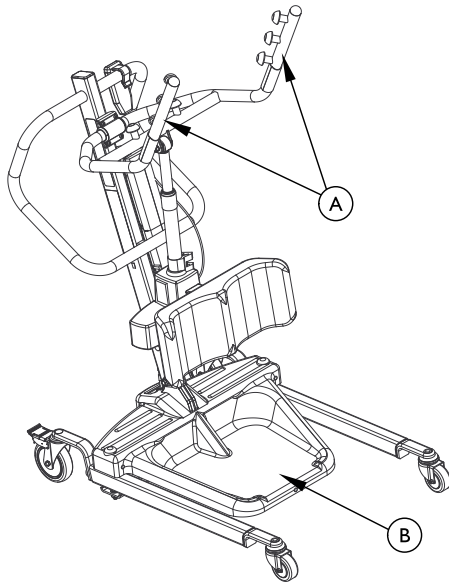

- i (A) = Lift Arms
- (B) = Foot plate

### 2.2.3 Disposal

**WARNING!**  
**Environmental Hazard**

This product has been supplied from an environmentally aware manufacturer that complies with the Waste Electrical and Electronic Equipment (WEEE) Directive 2012/19/EU. Device contains lead acid batteries. This product may contain substances that could be harmful to the environment if disposed of in places (landfills) that are not appropriate according to legislation.

- DO NOT dispose of batteries in normal household waste. They MUST be taken to a proper disposal site. Contact your local waste management company for information.
- Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

### 2.3 Radio Frequency Interference


**WARNING!**  
**Risk of Injury or Damage**



Most electronic equipment is influenced by Radio Frequency Interference (RFI). CAUTION should be exercised with regard to the use of portable communication equipment in the area around such equipment, otherwise injury or damage may occur. If RFI causes erratic behavior:

- PUSH the Red Power Switch OFF IMMEDIATELY.
- DO NOT turn the Power Switch ON while transmission is in progress.

### 2.4 Product Labeling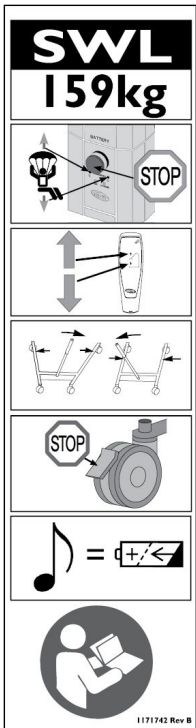

RPS350-IFR

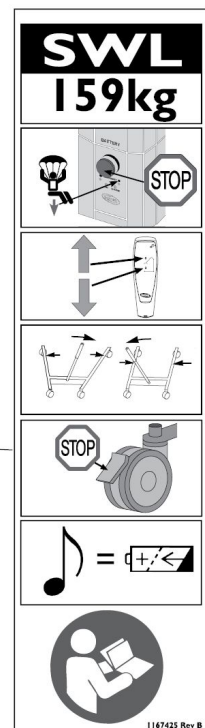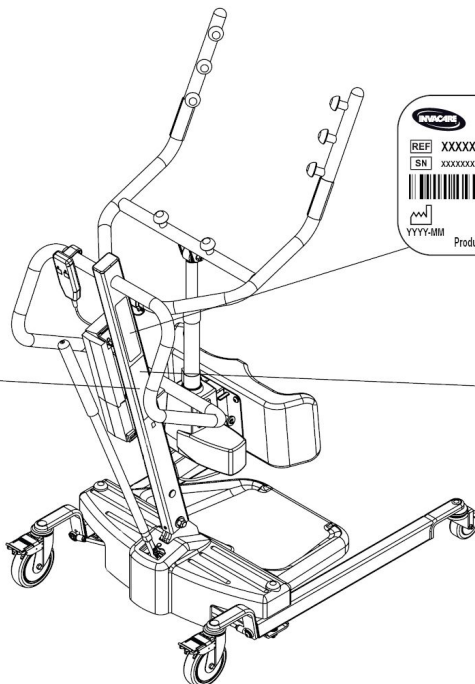RPS350-IE

### 3 Setup

#### 3.1 Safe Assembly

**WARNING!**

**Risk of Injury**

- Improper assembly may cause injury or damage.
- Assembly **MUST** be performed only by qualified personnel.
  - Use only Invacare parts in the assembly of this patient lift. The lift components are manufactured to specifications that assure correct alignment of all parts for safe functional operation.
  - **DO NOT** overtighten the mounting hardware. This will damage the mounting bracket.

There are no tools required to assemble the patient lift.

If there are any issues or questions during assembly, contact a local Invacare representative. Refer to the contact information in the back of this manual.


#### 3.2 Assembling the Mast to the Base


**WARNING!**

- Use only Invacare parts in the assembly of this patient lift. The base legs, the mast, boom, pump assembly and swivel bar are manufactured to specifications that assure correct alignment of all parts for safe functional operation.

**DETAIL A**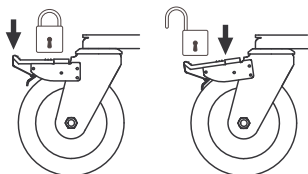

**DETAIL B**

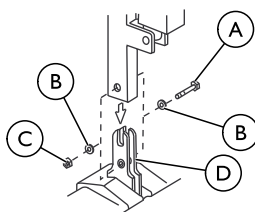Ⓐ Hex Bolt	Ⓒ Nut
Ⓑ Washer	Ⓓ Mounting Bracket

The mast assembly may be removed from the base for storage or transporting. The mast assembly **MUST** be properly secured to the base assembly before use.

1. Put the base on the floor.

Make sure all four casters make contact with the floor.


2. Lock both rear casters Refer to Detail A.

3. Remove the hex bolt, washers and nut that are located in the U-shape cut-out of the base Refer to Detail B.
4. Lift the mast to an upright position.
5. Lower the mast onto the mounting bracket.
6. Attach the mast to the base with the hex bolt, washers and nut. Tighten securely.

#### 3.3 Preparing the Lift for Use

Check and tighten all hardware **BEFORE** use.


#### 3.4 Installing the Shifter Handle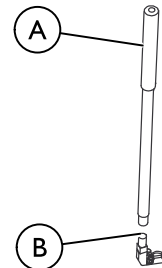

Ⓐ Shifter Handle	Ⓑ Male Adapter in Base
------------------	------------------------

1. Remove the shifter handle from the packaging carton.
2. Thread the shifter handle onto the male adapter on the base.

#### 3.5 Attaching the Battery Charger Mounting Bracket to the Wall

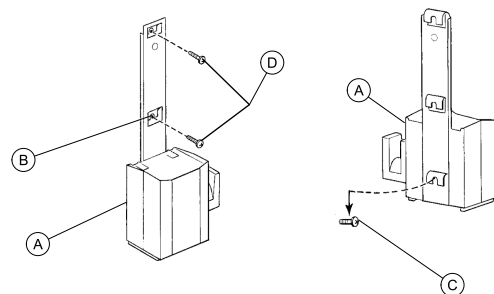Refer to your local regulations concerning proper mounting procedures.

This procedure only applies to RPS350–1E models.

1. Place the battery charger with mounting bracket Ⓐ on the wall at the desired position.
2. With a pencil, mark the middle hole Ⓑ position.
3. Measure down 6½ inches (16.5 cm) from the pencil mark and drill one mounting hole.
4. Install the bottom mounting screw Ⓒ until there is an approximate 1/8-inch (3 mm) gap between the screw head and the wall.
5. Install the battery charger with mounting bracket onto the bottom mounting screw.
6. Drill the remaining two mounting holes.
7. Install the two remaining mounting screws Ⓓ through the mounting bracket and into the wall. Tighten securely.
8. Plug the battery charger into the wall electrical outlet.

ON LED should illuminate.

## 4 Usage

### 4.1 Introduction

The operation of the patient lift is an easy and safe procedure.

**i** Before using the lift with a patient, refer to the following procedures for safety information and instruction:

- 2.2 *Operating Information, page 6*
- 5.3 *Lifting the Patient, page 13*

### 4.2 Closing/Opening Legs

#### **WARNING!** **Risk of Injury**

The lift could tip and endanger the patient and assistants.


- The legs of the lift must be in the maximum open position for optimum stability and safety. If it is necessary to close the legs of the lift to maneuver the lift under a bed, close the legs of the lift only as long as it takes to position the lift over the patient and lift the patient off the surface of the bed. When the legs of the lift are no longer under the bed, return the legs of the lift to the maximum open position.

#### 4.2.1 Closing/Opening Manual Legs

The shifter handle is used to open or close the legs of the base for stability when lifting a patient.

**i** Refer to the safety information in 4.2 *Closing/Opening Legs, page 9* before performing this procedure.

Ⓐ = Shifter Handle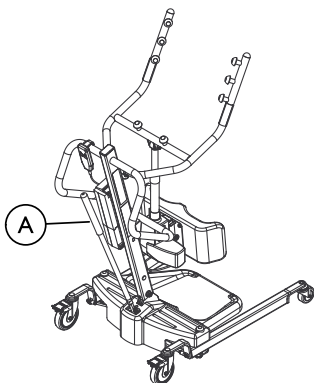


1. To close the legs:  
Pull the shifter handle OUT and away from the stand-up lift and then to your LEFT until it LOCKS in the notch of the bracket.

**i** Left is determined by standing behind the stand-up lift facing towards the front casters.

2. To open the legs:  
Pull the shifter handle OUT and away from the stand-up lift and then to your RIGHT until it LOCKS in the notch of the bracket.

**i** Right is determined by standing behind the stand-up lift facing towards the front casters.

### 4.3 Locking/Unlocking the Rear Casters

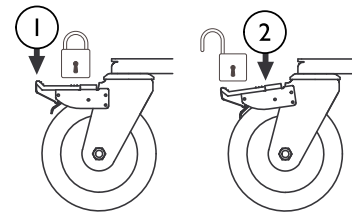1. Step here to lock
2. Step here to unlock

### 4.4 Raising/Lowering the Lift

#### **WARNING!** **Risk of Injury**


The lift could tip and endanger the patient and assistants.

- Invacare does recommend that the rear casters be left unlocked during lifting procedures to allow the patient lift to stabilize itself when the patient is initially lifted from a chair, bed or any stationary object.

#### 4.4.1 Raising/Lowering an Electric Lift

Refer to the safety information in 4.4 *Raising/Lowering the Lift, page 9* before performing this procedure.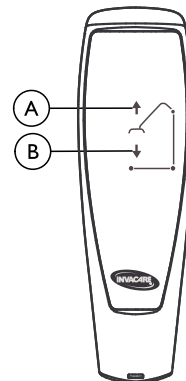

1. To raise the lift — Press and hold the UP Ⓐ button to raise the boom and the patient.
2. To lower the lift — Press and hold the DOWN Ⓑ button to lower the boom and the patient.


Release the button to stop raising or lowering the lift.

### 4.5 Activating a Mechanical Emergency Release


There are two types of mechanical emergency release — primary and secondary.

#### 4.5.1 Primary Emergency Release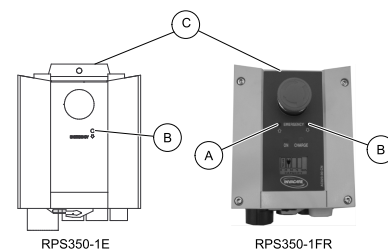

1. Insert a pen into the hole labeled Emergency Up Ⓐ or Emergency Down Ⓑ on the control box Ⓒ.

## 4.5.2 Secondary Emergency Release

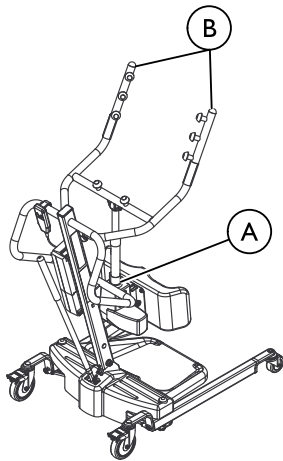
Ⓐ RED Emergency Grip	Ⓑ Lift Arms
----------------------	-------------

**i** It is recommended that the primary emergency release be used. The secondary emergency release is only a back-up to the primary emergency release.

In cases where the primary release is either not functioning or unreachable, a secondary emergency release may be used.

1. Pull up on the EMERGENCY grip Ⓐ and push down on the lift arms Ⓑ at the same time.

## 4.6 Performing an Emergency Stop

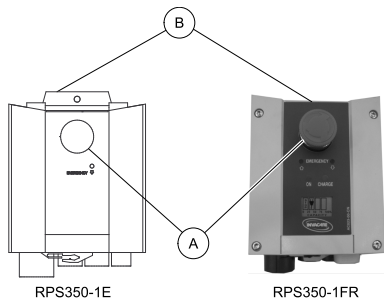1. Press the RED button Ⓐ on the control box Ⓑ in to stop the boom and patient from raising or lowering.
2. To reset, rotate the emergency button clockwise.


## 4.7 Charging the Battery

**i** Invacare recommends the battery be recharged daily to prolong battery life.

There are two different methods to charge the battery. One method uses a power cord that attaches to the control box, and the other requires the battery to be mounted to the battery charger. Follow the appropriate procedure to charge the battery for your patient lift.

### 4.7.1 Battery Indicator

**i** This procedure only applies to the RPS350-1FR models.

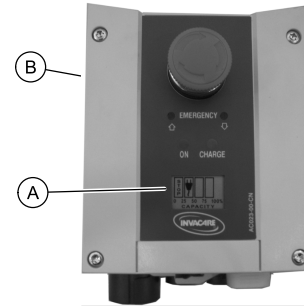The battery indicator Ⓐ is located on the control box Ⓑ. The LEDs indicate the battery state:

Control Box Battery Indicator Ⓐ	Battery State	Description
	Full Charge (100%)	The battery is OK — no need for charging (100%).
	Partial Charge (75%)	The battery is OK — no need for charging (75%).
	Partial Charge (50%)	The battery needs to be charged (50%).
	Low Charge (25%)	The battery needs to be charged (25%). The horn beeps when a button is pressed.
	Low Charge (0%)	The battery needs to be charged.  Some of the functionality of the lift is lost and it is only possible to lower the boom.  An audible alarm will sound (horn will beep) when battery is low. If the audible alarm sounds during a transfer, complete that transfer and then charge the battery.

### 4.7.2 Using a Power Cord to Charge the Battery

**i** This procedure only applies to the RPS350-1FR.


Ⓐ = Power Cord Connects Here

**CAUTION!**

– The emergency stop **MUST NOT** be activated - otherwise it will be impossible to charge the battery. While charging takes place, the patient lift cannot be used. **DO NOT** attempt to move the patient lift without unplugging the power cord from the wall outlet. **DO NOT** attempt to use the patient lift if the battery housing is damaged. Replace a damaged battery housing before any further use.

1. Attach the power cord to the control box.
2. Plug the power cord into a power outlet.
  - The battery will charge in approximately 4 hours. Charging must be done in a room with good air ventilation.
3. Disconnect the power cord from the power outlet after the battery has been fully charged.

### 4.7.3 Using the Battery Charger to Charge the Battery

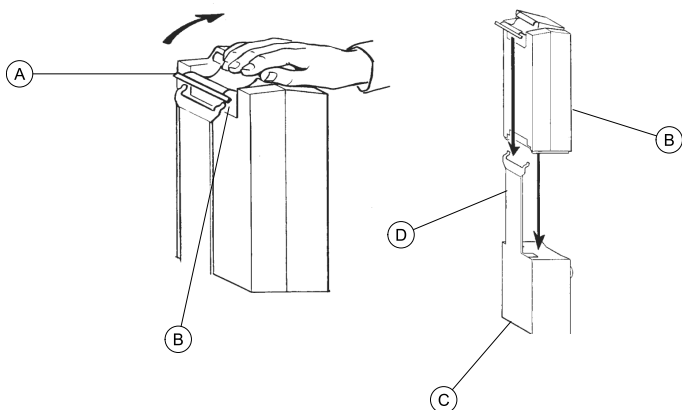This procedure only applies to the RPS350–1E.

1. Lift up on the handle (A) on the back of the battery (B).
2. Lift the battery up and out away from the control box (C).

**CAUTION!**

Mounting the battery improperly may cause injury or damage.

– Make sure there is an audible click when mounting battery on the battery charger to confirm proper mounting.

3. Place the battery on the battery charger (D) as shown. Make sure there is an audible click.

The charge LED will illuminate. When charging is complete, charge LED will stop illuminating. A battery needing to be fully recharged will take approximately four hours.

4. Lift up on the handle on the back of the battery.
5. Lift the battery up and out away from the battery charger.

**CAUTION!**

Mounting the battery improperly may cause injury or damage.

– Make sure there is an audible click when mounting battery on the control box to confirm proper mounting.

6. Reinstall the battery onto the control box as shown. Make sure there is an audible click.

The battery mounts to the control box and battery charger as shown.

## 5 Lifting the Patient

### 5.1 Safe Lifting

#### WARNING!

- DO NOT exceed the maximum weight limitation (SWL) of 159 kg (25 stone).
- DO NOT attempt to any transfer without approval of a Healthcare Professional.
- ALWAYS keep hands and fingers clear of moving parts to avoid injury.
- During transfer, with the patient suspended in the sling, DO NOT roll the base of the lift over any uneven surfaces that would cause the lift to become unstable.
- Use the steering handle on the mast assembly at all times to push or pull the stand assist lift.

#### WARNING!

##### Operating the Lift and Lifting the Patient

- Ensure equipment used with the stand assist lift is of adequate strength to lift the load (e.g. sling). Ensure that should the equipment malfunction that the person being lifted is not exposed to danger.
- Before positioning the legs of the stand assist lift around the patient, make sure that the patient's feet are out of the way of the foot plate, otherwise injury may occur.
- Adjustments for safety and comfort should be made before moving the patient. Patient's arms should be outside of the sling straps.

#### WARNING!

##### Operating the Lift and Lifting the Patient

- Before lifting a patient from a stationary object (wheelchair, commode or bed), slightly raise the patient off the stationary object and check that all sling attachments are secure. If any attachment is not correct, lower the patient and correct the problem, then raise the patient and check again.
- During transfer, with the patient suspended in a sling attached to the lift, DO NOT roll caster base over uneven surfaces that would create an imbalance of the patient lift and could cause the patient lift to tip over. Use steering handle on the mast at ALL times to push or pull the patient lift.
- Invacare recommends locking the rear swivel casters ONLY when positioning or removing the sling (stand assist or transfer) from around the patient.
- Invacare does NOT recommend locking of the rear casters of the patient lift when lifting an individual. Doing so could cause the lift to tip and endanger the patient and assistants. Invacare DOES recommend that the rear casters be left unlocked during lifting procedures to allow the patient lift to stabilize itself when the patient is initially lifted from a chair, bed or any stationary object.

#### WARNING!

##### Using the Sling

- Use the sling that is recommended by a Healthcare Professional for the comfort and safety of the individual that is being lifted.
- Individuals that use the stand assist sling MUST be able to support the majority of their own weight, otherwise injury may occur.
- Bleached, torn, cut, frayed or broken slings are unsafe and their use could result in injury or damage. Discard any sling that meets these criteria IMMEDIATELY.
- DO NOT alter slings.
- Be sure to check the sling attachments each time the sling is removed and replaced, to ensure that it is properly attached before the patient is removed from a stationary object (bed, chair or commode).
- If the patient is in a wheelchair, activate the wheelchair brakes to prevent the chair from moving forwards or backwards.
- Consideration should be taken that prevents exposure to danger to the person being lifted, and that the person can be freed without injury.

#### WARNING!

##### Stand Assist Sling

- DO NOT use the stand assist sling in combination with the patient lift as a transport device. It is intended to transfer an individual from one resting surface to another (such as a bed to a wheelchair).
- Before lifting the patient, make sure the bottom edge of the stand assist sling is positioned on the lower back of the patient and the patient's arms are outside the stand assist sling. The belt MUST be snug, but comfortable on the patient, otherwise the patient can slide out of the sling during transfer possibly causing injury.
- The belt MUST be snug, but comfortable on the patient, otherwise the patient can slide out of the sling during transfer, possibly causing injury.

##### Transfer Stand Assist Sling

- Before lifting the patient, make sure the bottom edge of the transfer sling is at the base of the spine and the patient's arms are outside the transfer sling.
- DO NOT raise the patient to a full standing position while using the transfer sling, otherwise injury may occur.

Refer to the patient sling brochure for more information.

### 5.2 Positioning the Stand Assist Lift

Refer to the Safety section in this manual and review the information in *5.1 Safe Lifting, page 12* before proceeding further and observe all warnings indicated.


Before positioning the legs of the patient lift under a bed, make sure that the area is clear of any obstructions.

**WARNING!**  
**Risk of Injury**

The lift could tip and endanger the patient and assistants.

- The legs of the lift must be in the maximum open position for optimum stability and safety. If it is necessary to close the legs of the lift to maneuver the lift under a bed, close the legs of the lift only as long as it takes to position the lift over the patient and lift the patient off the surface of the bed. When the legs of the lift are no longer under the bed, return the legs of the lift to the maximum open position.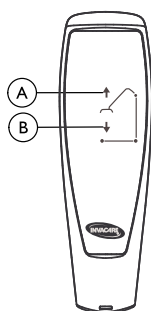

(A) UP Button	(B) DOWN Button
---------------	-----------------

1. Ensure the legs of the stand assist lift are in the maximum open position. If not, use the shifter handle to open the legs.
2. Position the stand assist lift using the mast handle.
3. Press the down arrow button on the hand control to lower the lift arms for easy attachment to the sling.

### 5.3 Lifting the Patient

**WARNING!**


- DO NOT exceed the maximum weight limitation (SWL) of 159 kg (25 stone).
- Individuals that use the stand assist sling **MUST** be able to support the majority of their own weight, otherwise injury may occur.
- DO NOT lock the rear casters of the stand assist lift when lifting an individual. Locking the rear casters could cause the stand assist lift to tip and endanger the patient and assistants. Unlocking the rear casters during lifting procedures allows the lift to stabilize itself while the patient is initially lifted from a wheelchair, bed or any stationary object.
- DO NOT move the patient if the sling is not properly attached to the attachment points on the stand assist lift.
- Make sure that the sling is properly attached **BEFORE** lifting the patient. If any attachments are not properly in place, correct the problem. When the sling is elevated a few inches/cm off the stationary surface and before moving the patient, check again to make sure the sling and all attachments are securely in place. If any problem is found, lower the patient back onto the stationary surface and correct the problem - otherwise, injury or damage may occur.
- Adjustments for safety and comfort should be made before moving the patient.


**WARNING!**

- Use the sling that is recommended by the individual's doctor, nurse or medical assistant for the comfort and safety of the individual that is being lifted. Invacare patient slings are made specifically for use with Invacare patient lifts. It may in certain instances, be possible to use other manufacturer's slings on Invacare lifts. Contact your local Invacare office for more information.


#### Detail A — Stand Assist Lift

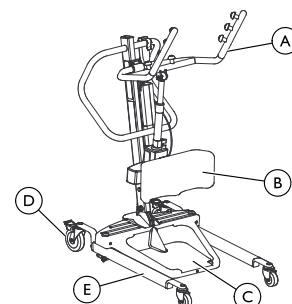#### Detail B — Sling Attachment
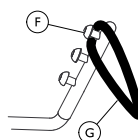
#### Detail C — Brakes

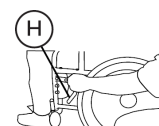(A) Hand Grips	(E) Leg
(B) Knee Pad	(F) Hook
(C) Foot Plate	(G) Loop of sling
(D) Rear Caster	(H) Brake

Refer to the patient sling user manual for more information.

The patient **MUST** be in a seated position first. Use the head section of the bed to get the patient upright then move legs over the side of the bed.

1. Instruct the patient to hold onto the hand grips on both sides of the stand assist lift (Detail A).
2. Instruct the patient to lean back into the standing or transfer sling.

**WARNING!**

- Stand Assist Slings - Before lifting the patient, make sure the bottom edge of the sling is positioned on the lower back of the patient and the patient's arms are outside the sling.
- Transfer Stand Assist Slings - Before lifting the patient, make sure the bottom edge of the sling is at the base of the spine and the patient's arms are outside the sling.

3. Ensure the following:
  - a. The patient's knees are secure against the knee support.
  - b. The patient's feet are properly positioned on the foot plate.
  - c. Slings:
    - the bottom edge of the stand assist sling is positioned on the patient's lower back.
    - the bottom edge of the transfer stand assist sling is at the base of the patient's spine.
    - the patient's arms are outside the sling.
    - the loops of the sling are completely on the hooks of the lift arms (Detail B).
  - d. The rear casters are unlocked.
  - e. The legs are in the maximum open position.

**WARNING!**

- If transferring a patient from a wheelchair, the wheelchair brakes **MUST** be in the locked position before lowering the patient into the wheelchair. Otherwise, injury may occur.

4. If transferring from a wheelchair, engage the brakes on the wheelchair (Detail C).
  5. Press the UP arrow button on the hand control to raise the patient above the surface (bed, wheelchair or commode). The patient's weight will be fully supported by the stand assist lift.
-  The lower center of gravity provides stability making the patient feel more secure and the lift easier to move.

## 5.4 Moving the Patient

**WARNING!**

- The legs of the stand assist lift **MUST** be in the maximum open position for optimum stability and safety. If the patient is in a sling and it becomes necessary to move through a narrow passage, close the legs of the stand assist lift only as long as it takes to move through the passage. When the stand assist lift is through the passage, return the legs to the maximum open position.
- **DO NOT**, during transfer of a patient suspended in the lift sling, roll caster base over uneven surfaces that would create an imbalance of the lift. This could cause the lift to tip over. Use the mast handle at all times to push or pull the lift.

1. Ensure the legs of the stand assist lift are in the maximum open position. If not, press the OPEN LEGS button on the hand control to move the legs to the maximum open position.
2. Move the stand assist lift away from the surface they were lifted from.
3. Slowly move the patient to the desired surface.

## 6 Transferring the Patient

### 6.1 Safe Transfer

#### WARNING!

- DO NOT exceed the maximum weight limitation (SWL) of 159 kg (25 stone).
- DO NOT attempt to transfer a patient without approval of a Healthcare Professional. Thoroughly read the instructions in this owner's manual, observe a trained team of experts performing the lifting procedures and then perform the entire lift procedure several times with proper supervision and a capable individual acting as a patient. Training can be provided. Contact your local Invacare office for more information.
- DO NOT move the patient if the sling is not properly connected to the attachment points of the stand assist lift. Check that the sling is properly connected to the attachment points prior to lifting a patient. If any attachments are not properly in place, correct the problem. When the sling is elevated a few inches/cm off the stationary surface and before moving the patient, check again to make sure that all sling attachments are secure. If any attachments are not properly in place, lower the patient back onto the stationary surface and correct this problem - otherwise, injury or damage may occur.
- Adjustments for safety and comfort should be made before moving the patient. The patient's arms should be outside the straps.


#### WARNING!

- Use the sling that is recommended by a Healthcare Professional for the comfort and safety of the individual that is being lifted.
- DO NOT lock the rear casters of the stand assist lift when lifting an individual. Locking the rear casters could cause the stand assist lift to tip and endanger the patient and assistants.
- The legs of the stand assist lift **MUST** be in the maximum open position for optimum stability and safety. If the patient is in a sling and it becomes necessary to move through a narrow passage, close the legs of the stand assist lift only as long as it takes to move through the passage. When the stand assist lift is through the passage, return the legs to the maximum open position. If it is necessary to close the legs to maneuver the stand assist lift under a bed, close the legs only as long as it takes to position the stand assist lift over the patient and lift the patient off the surface of the bed. When the legs of the stand assist lift are no longer under the bed, return the legs to the maximum open position.
- Be sure to check the sling attachments each time the sling is removed and replaced to ensure that it is properly attached before the patient is removed from a surface.

The use of one assistant is based on the evaluation of the health care professional for each individual case.

### 6.2 Transferring to a Commode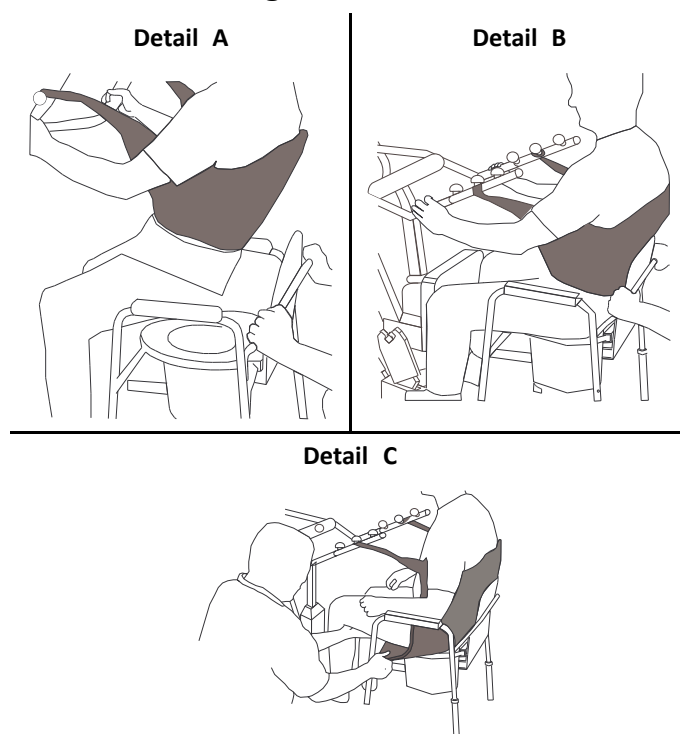

#### WARNING!

- Invacare recommends locking the rear swivel casters only when positioning or removing the sling from around the patient.


1. Lift the patient from the side of the bed.
2. Press the UP button on the hand control to elevate the patient high enough to clear the arms of the commode chair. Their weight will be supported by the stand assist lift.
3. Guide the patient onto the commode chair. This may require two assistants.
4. Press the down arrow button to lower the patient onto the commode chair.
5. Lock the rear swivel casters on the stand assist lift.
6. Depending on the style of sling, perform one of the following:
  - Stand Assist Sling - unhook the stand assist sling from the attachment points on the stand assist lift.
  - Transfer Stand Assist Sling -
    - a. Unhook the transfer stand assist sling from the bottom attachment points on the stand assist lift.
    - b. Lift up on the patient's legs and remove the thigh supports from underneath the patient.
    - c. If desired, unhook the transfer stand assist sling from the top attachment points on the stand assist lift.
7. Instruct or assist the patient in lifting their feet off the foot plate.
8. Remove the sling from around the patient.
9. Unlock the rear casters and pull the stand assist lift away from the commode.
10. When complete, recheck the sling for correct attachments.


The patient can remain in the upper portion of the transfer stand assist sling while using the commode.

## 6.3 Transferring to a Wheelchair

Detail A

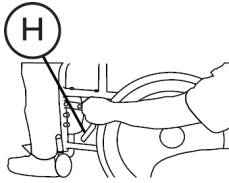Detail B

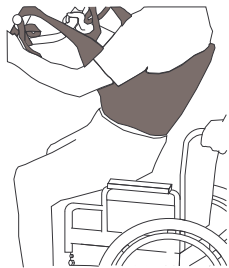Detail C

 H = Wheelchair Brake

1. Ensure the legs of the lift with the patient in the sling are in the open position.
2. Move the wheelchair into position.
3. Engage the brakes of the wheelchair to prevent movement of the chair (Detail A).

**WARNING!**

– DO NOT place the patient in the wheelchair if the brakes are not engaged. The wheelchair brakes MUST be in a locked position before lowering the patient into the wheelchair for transport. Otherwise, injury may result.

4. Position the patient over the wheelchair (Detail “B”).

5. Press the down arrow button and lower the patient into the wheelchair.

**WARNING!**

– Lock the rear swivel casters ONLY when positioning or removing the sling (stand assist or transfer stand assist) from around the patient. Otherwise, injury may occur.

6. Lock the rear swivel casters.
7. Unhook the sling from all attachment points on the lift (Detail C).
8. Instruct patient to lift their feet off the foot plate. Assist the patient if necessary.
9. Remove the sling from around the patient.
10. Unlock the rear casters and pull the stand assist lift away from the wheelchair.

## 6.4 Transferring to a Bed

The lower center of gravity provides stability making the patient feel more secure and the lift easier to move.

1. Position the patient as far over the bed as possible.
  -  If patient is being transferred from a surface that is lower than the bed, press the up arrow button to raise the patient above the surface of the bed. The patient should be elevated just high enough to clear the bed with their weight fully supported by the lift.
2. Press the down arrow button and lower the patient onto the bed.

**WARNING!**

– Invacare recommends locking the rear swivel casters ONLY when positioning or removing the sling from around the patient.

3. Lock the rear swivel casters.
4. Unhook the stand assist or transfer stand assist sling from all attachment points on the stand assist lift.
5. Instruct the patient to lift their feet off of the foot plate.
  -  Assist the patient if necessary.
6. Remove the stand assist or transfer stand assist from around the patient.
7. Unlock the rear casters and pull the stand assist lift away from the bed.

## 7 Troubleshooting

### 7.1 Troubleshooting Table

SYMPTOMS	FAULTS	SOLUTION
Noisy or dry sound from pivots.	Needs lubrication.	Refer to <i>Lubricating the Lift</i> .
Electric actuator fails to lift when button is pressed.	Hand-control or actuator connector loose.	Connect hand control or actuator connector. Ensure connectors are seated properly and fully connected.
	Battery low.	Charge batteries. Refer to 4.7 <i>Charging the Battery, page 10.</i>
	RED emergency stop button pressed IN.	Rotate RED emergency stop button CLOCKWISE until it pops out.
	Battery not connected properly to control box.	Reconnect the battery to the control box. Refer to 4.7 <i>Charging the Battery, page 10.</i>
	The connecting terminals are damaged.	Replace the battery pack. Refer to 4.7 <i>Charging the Battery, page 10.</i>
	Electric actuator in need of service or load is too high.	Refer to 8.7 <i>Replacing the Electric Actuator, page 19.</i> Contact your Dealer or Invacare representative.
Unusual noise from lift arm actuator.	Actuator is worn or damaged or spindle is bent.	Refer to 8.7 <i>Replacing the Electric Actuator, page 19.</i> Contact your Dealer or Invacare representative.
Lift arms will not lower in uppermost position.	Lift arms require a minimum weight load to lower from the uppermost position.	Pull down slightly on the lift arms.
Legs do not open and close properly.	Base may require adjustment.	Refer to 8.6 <i>Adjusting the Base, page 19.</i>

 If problems are not remedied by the suggested means, please contact your dealer or Invacare.

## 8 Maintenance

### 8.1 Maintenance Safety Inspection Checklist

**WARNING!**

- Maintenance **MUST** be performed **ONLY** by qualified personnel.
- A competent person must ensure that the strength and stability of the product remains adequate for all tasks if parts are replaced.
- **DO NOT** overtighten the mounting hardware, this will damage the mounting brackets.

Follow the maintenance procedures described in this manual to keep your patient lift in continuous service.

The Invacare Patient Lift is designed to provide a maximum of safe, efficient and satisfactory service with minimum care and maintenance.

All parts of the Invacare Lift are made of the best grades of steel, but metal to metal contact will wear after considerable use.

There is no adjustment or maintenance of the casters, other than cleaning, lubrication and checking axle and swivel bolts for tightness. Remove all debris, etc. from the wheel and swivel bearings. If any parts are worn, replace these parts IMMEDIATELY.

If you question the safety of any part of the lift, contact your Dealer IMMEDIATELY.

The UK Health and Safety Executive’s Lifting Operations and Lifting Equipment Regulations 1998 require any equipment that is used in the workplace to lift a load be subject to safety inspection on a six monthly basis. This is to be carried out by a suitably trained individual. Please refer to the HSE website for guidance ([www.hse.gov.uk](http://www.hse.gov.uk)).

Regular maintenance of patients lifts and accessories is necessary to assure proper operation. It is the responsibility of the person responsible for the patient lift to ensure that this maintenance is carried out.

After the first 12 months of operation, inspect all pivot points and fasteners for wear. If the metal is worn, the parts **MUST** be replaced. Perform this inspection every six months thereafter.

Date of Inspection:	Initials:
<b>THE CASTER BASE</b>	
<input type="checkbox"/> Inspect for missing hardware. <input type="checkbox"/> Base opens/closes with ease. <input type="checkbox"/> Inspect casters and axle bolts for tightness. <input type="checkbox"/> Inspect casters for smooth swivel and roll. <input type="checkbox"/> Inspect and clear wheels of debris. <input type="checkbox"/> Inspect pivot joints for wear.	
<b>SLINGS AND HARDWARE</b>	
<input type="checkbox"/> Check all sling attachments each time it is used to ensure proper connection and patient safety. <input type="checkbox"/> Inspect sling material for wear. <input type="checkbox"/> Inspect straps for wear. <input type="checkbox"/> Inspect stitching.	

<b>THE ELECTRIC ACTUATOR ASSEMBLY</b>
<input type="checkbox"/> Check for leakage. <input type="checkbox"/> Inspect hardware on mast, boom and base. <input type="checkbox"/> Check for wear or deterioration. If damaged, return to factory. <input type="checkbox"/> Cycle to ensure smooth quiet operation of the electric actuator.
<b>THE LIFT ARMS AND LINKAGE</b>
<input type="checkbox"/> Check all hardware and attachment points. <input type="checkbox"/> Inspect for bends or deflections. <input type="checkbox"/> Inspect bolted joints of the lift arms for wear. <input type="checkbox"/> Inspect to ensure that the lift arms are centered between the base legs. <input type="checkbox"/> Inspect pivot joints for wear. <input type="checkbox"/> Check sling hooks for wear or deflection.
<b>THE MAST</b>
<input type="checkbox"/> Mast must be securely assembled to lift arms. <input type="checkbox"/> Inspect for bends or deflections. <input type="checkbox"/> Inspect pivot joints for wear.
<b>CLEANING</b>
<input type="checkbox"/> Whenever necessary.

### 8.2 Cleaning the Sling and the Lift

**WARNING!**

- After **EACH** laundering (in accordance with laundering instructions on the sling), inspect the sling for wear, tears and loose stitching. Discard any patient sling that meets these criteria IMMEDIATELY.

#### Cleaning the Sling

The sling should be washed regularly in water temperature not exceeding 203°F (95°C) and a biological solution. Refer to the washing instructions in the sling user manual.

#### Cleaning and Disinfecting the Lift

To prevent cross-infection, the hoist must be cleaned and disinfected after each use.

A soft cloth, dampened with water and a small amount of mild detergent, is all that is needed to clean the patient lift. The lift can be cleaned with non-abrasive cleaners.

Never use acids, alkaline or solvents for cleaning the lift. Dry the lift carefully after cleaning.

Motors, control unit and mounting parts can be damaged if the lift is cleaned any other way than stated above.

The lift must be wiped with a moistened, firmly wrung cloth with ordinary household disinfectants. Only use disinfection detergents approved by the facility.

### 8.3 Reuse

This product is suitable for reuse. The maximum number of times it can be reused is dependent upon product condition.

To prevent the transmission of infection, the patient lift and slings must be cleaned after each use. Before reuse, refer to *Cleaning the Sling and the Lift*.

### 8.4 Detecting Wear and Damage

It is important to inspect all stressed parts, such as slings, lifting arm and any pivot for slings for signs of cracking, fraying, deformation or deterioration. Replace any defective parts IMMEDIATELY and ensure that the lift is not used until repairs are made.


### 8.5 Lubricating the Lift

The Invacare lift is designed for minimum maintenance. However, a six month check and lubrication should ensure continued safety and reliability.

Keep lift and slings clean and in good working order. Any defect should be noted and reported to your dealer as soon as possible.

The casters MUST swivel and roll smoothly. A light grease (waterproof auto lubricant) may be applied to the ball bearing swivel of the casters once a year. Apply more frequently if the casters are exposed to extreme moist conditions.

### 8.6 Adjusting the Base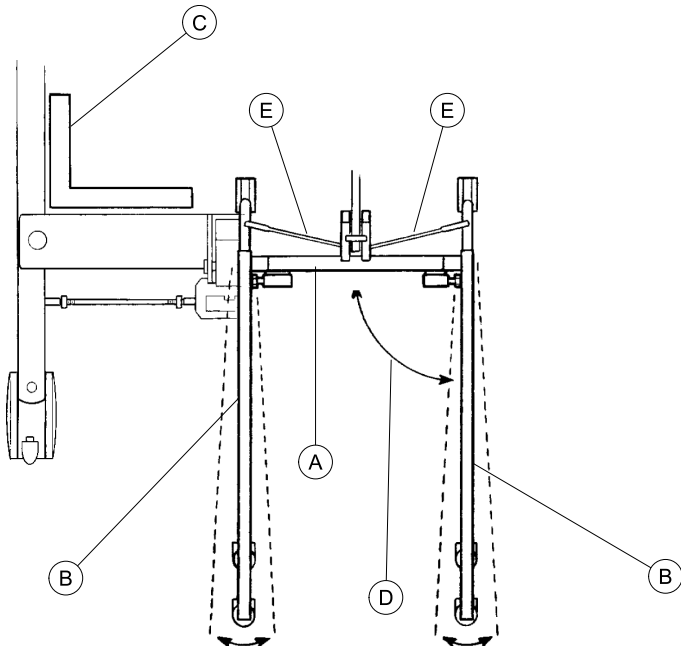


The base (A) adjustment should not require any attention other than adjusting the linkage rods.

**!** Damage to the lift may occur.  
 – The legs must be in the fully closed position before adjusting the linkage rods.

1. Close the legs all the way. Refer to 4.2.1 Closing/Opening Manual Legs, page 9.
2. Check the squareness of the legs (B) when in the closed position.
3. Place a square (C) on the inside of the legs and base to determine the 90° alignment (D).
4. Adjust the linkage rods (E) until 90° alignment is achieved.

**!** Damage to the lift may occur.  
 – DO NOT adjust the linkage rods so the legs are at less than 90° alignment.

### 8.7 Replacing the Electric Actuator

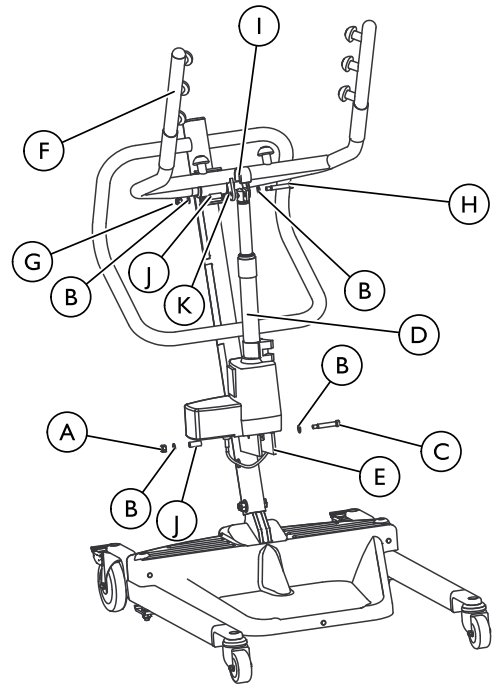(A) Bottom nut	(G) Top nut
(B) Washer	(H) Bolt
(C) Shoulder bolt	(I) Bracket
(D) Electric actuator	(J) Bushing
(E) Mast mounting bracket	(K) Lift arm mounting bracket
(F) Lift arm	


**WARNING!**

– DO NOT overtighten the mounting hardware. This will damage mounting brackets.

1. Remove the bottom nut, washers, shoulder bolt and bushing that secure the electric actuator to the mast mounting bracket.
2. Rest the lift arm on your shoulder and remove the top nut, bolt, bracket, bushing (if applicable) and washers from the lift arm mounting bracket.
3. Remove the electric actuator.
4. Reverse STEPS 1–3 to install the new electric actuator.

### 8.8 Adjusting the Knee Pad Height

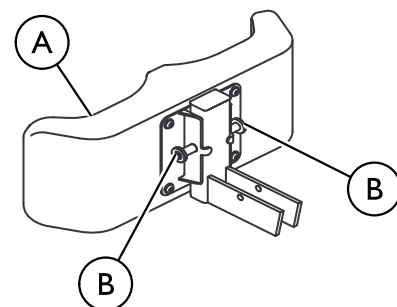(A) Knee pad	(B) Adjustment pins
--------------	---------------------

**WARNING!**

- NEVER adjust the knee pad while patient is in the standing position.
- NEVER try to adjust the knee pad while the lift is moving.
- ALWAYS make sure that the adjustment pins are engaged in corresponding height adjustment holes before use.

1. Pick a height setting that will be comfortable to the patient and provide the necessary support.
  -  The knee pad should be positioned so that the knee portion of the leg is in contact with the pad.
2. Using both hands, pull both adjustment pins outward at the same time.
3. Position the knee pad to the desired height and release adjustment pins into the corresponding alignment holes.
4. Check to make sure that both pins are engaged.

## 9 Technical Data

### 9.1 RPS3501E and RPS3501FR Stand Assist Patient Lift

Height at Sling Hook-up — MAX	168 cm
Height at Sling Hook-up — MIN	102 cm
Base Width (open)	94 cm
Base Width (closed)	66 cm
Base height (clearance)	11,5 cm
Base length	90 cm
Overall height	125 cm
Overall length	99 cm
Overall width	65,5 cm
Castor size (FRONT/REAR)	8 cm / 13 cm
Sling Options	Stand Assist or Transfer Stand Assist
Sling Material	Polyester
Maximum weight limitation (SWL) (patient + sling)	159 kg (25 stone)
Total weight (weight out of carton)	49 kg (8 stone)
Battery (voltage output)	24V DC mb max. 240VA (RCHBL)
Charger Input (voltage supply)	100-240V AC ~ 50/60 Hz
Charger Output/Charging Time	29.5V DC 2.9 Ah Max 6 hours
Audio/Visual Low Battery Alarm	Yes
Motor Safety Devices	Anti-Entrapment

*Approx. Lifts per Charge (working ability)	*100-200 Cycles per charge
Max. current input	Max. 400 mA
Operating temperature	5° to 40° C
Air humidity	20% to 90% at 30 not condensing
Sound pressure	< 50 dB
Atmospheric pressure	700 to 1060 hPa
Storage temperature	above 0° C
Storage air humidity	less than 60%
Storage atmospheric pressure	700 hPa to 1060 hPa
Degree of protection, control unit	IPX4
Degree of protection, hand control	IPX4
Degree of protection, motor	IPX4
Insulation class	Class II equipment, Type B applied part
Intermittens	10%, max, 2 minutes/18 minutes
Battery capacity	2,9 Ah
Manual emergency lowering	Yes
Electric emergency lowering/lifting	Yes/Yes
Expected lifetime	8 years
Lifting Speed	The lift passes the “velocity of lifting and lowering” in the ENISO10535 (<0.15 m/s under maximum load and <0.25 m/s unloaded)

 \* Varies depending on load and stroke.

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Invacare AB