## Invacare® Matrix® accessory

### Elite backrests / PB backrests

en	<b>Support Harness Mounting Bracket</b> Service Manual . . . . .	1
de	<b>Montagehalterung für Hosenträgergurt</b> Service Handbuch . . . . .	6
fr	<b>Support de montage de harnais de soutien</b> Manuel de maintenance . . . . .	10
it	<b>Staffa di montaggio per imbragatura di supporto</b> Manuale per la manutenzione . . . . .	14
es	<b>Soporte de montaje del arnés de apoyo</b> Manual de servicio . . . . .	18
pt	<b>Suporte de montagem do colete de tronco</b> Manual de assistência . . . . .	22
nl	<b>Montagebeugel steunharnas</b> Servicehandleiding . . . . .	26
da	<b>Monteringsbeslag til sele</b> Servicemanual . . . . .	30
nn	<b>Festebrakett for posisjoneringssle</b> Servicehåndbok . . . . .	34
sv	<b>Monteringsfäste för stödsele</b> Servicemanual . . . . .	38
fi	<b>Tukivaljaiden kiinnike</b> Huoltokäsikirja . . . . .	42

DEALER: Keep this manual.  
The procedures in this manual MUST be performed by a qualified technician.

Yes, you can.®

# I General

## I.1 About this manual

This service manual contains information about assembly, adjustment and advanced maintenance of the product. In order to ensure safety when handling the product, read the manual carefully and follow the safety instructions.

For pre-sale and user information, please see the user manual of the support harness. Find the user manual on Invacare's website or contact your Invacare representative (see addresses at the end of this manual).

### Symbols in this manual

In this manual, hazards are indicated by symbols. The symbols are accompanied by a signal word that indicates the severity of the risk.

#### WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

#### IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

#### Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

## I.2 Intended use

The Matr<sup>x</sup>® PB and Matr<sup>x</sup>® Elite Support Harness Mounting Brackets were intended for use with positioning harnesses such as the Invacare Pro-tech series or the Body Point harness series.

Every other use is prohibited.

# 2 Safety

## 2.1 Safety information

#### WARNING!

##### Risk of injury

- Always ensure all harness fittings and hardware are compatible with the support harness mounting brackets provided.
- Do not modify the support harness mounting brackets; Unauthorized modifications to the mounting brackets may result in personal injury and will compromise the limited warranty.

#### WARNING!

##### Risk of injury

Support Harnesses and Positioning Belts are not designed to provide proper restraint during motor vehicle transport and should never be utilized as a Safety Restraint.

- Do not sit in your wheelchair while it is in a moving vehicle. When travelling in a motor vehicle you must be properly secured in a motor vehicle seat with approved safety belts securely fastened.

#### WARNING!

##### Risk of injury

The Support Harness Mounting Bracket Assembly has not been crash tested and is not approved for use as part of a vehicle safety restraint during transport.

- When travelling in a motor vehicle, the user must be properly secured in a motor vehicle seat with approved safety belts securely fastened.

#### WARNING!

– Check your support harness weekly to ensure that it is still in good working condition; to ensure it has no damage or wear, and to ensure it is fixed properly to the wheelchair.

- If the support harness is fastened using a bolted connection, ensure that the connection has not loosened or come undone.
- Be certain to read the service manual provided by the support harness manufacturer for more information about safety and maintenance of your support harness.

#### IMPORTANT!

- Be certain to read the Support Harness User Manual (provided separately) for important safety information and instructions regarding the proper set-up, installation and use of your support harness.


## 3 Setup

### 3.1 Installation Overview (Matrx® Elite Backs)


Pos.	Qty.	Description
(A)	1	Support Harness Mounting Bracket, left
(B)	1	Support Harness Mounting Bracket, right
(C)	2	Support Harness Extension Bracket
(D)	1	Multi wrench, universal
(E)	1	Multi-metric Hex key, 4mm/5mm
(F)	8	4.8mm Flat washer, SAE, BZ
(G)	2	M6 Nyloc nut, BZ
(H)	4	M5 Nyloc nut, BZ
(I)	2	M6 1.0 x 12 mm Button socket cap, alloy steel, BZ
(J)	4	M5 0.8 x 12 mm Socket head cap, alloy steel, BZ

#### 3.1.1 Installing the Support Harness Mounting Bracket (Matrx® Elite)

(Right Extension Bracket assembly shown)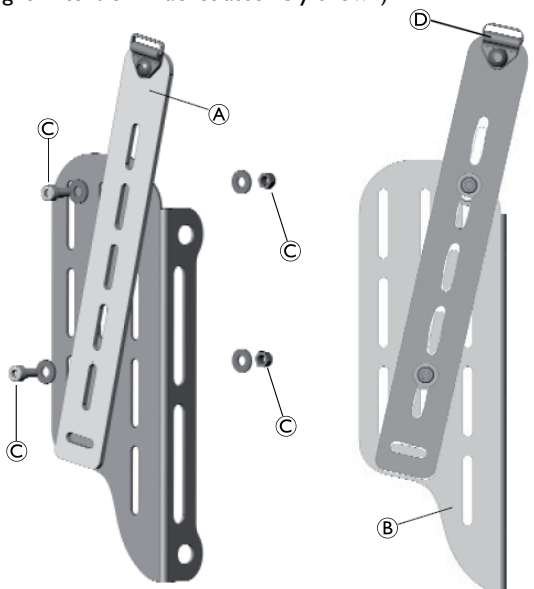

- Loosely install the extension bracket (A) onto the mounting plate (B) in the desired position using bolts, washers, and nuts (C).

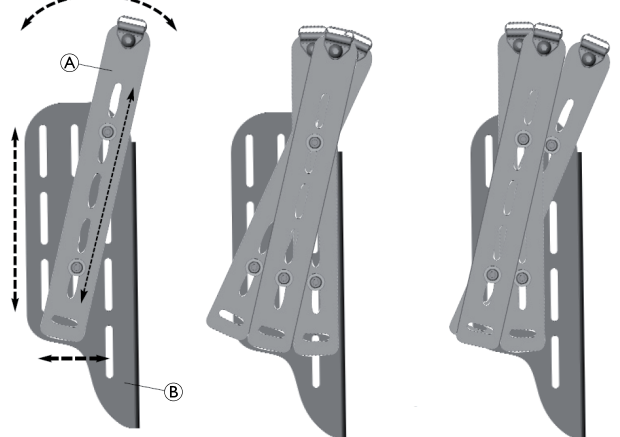
**i** To help pre-determine the mounting position of the extension bracket, align the mounting plate with the back shell and position the extension bracket on the plate to determine which slots/mounting hardware locations will produce the desired position.

- If applicable, remove the existing mounting pins (x4) and flange nuts (x4) from the Matrx Elite Back Shell (retain hardware).

**i** For ease of installation, we recommend installing the extension bracket onto the mounting plate prior to securing the entire assembly onto the back shell.

Use the various mounting slots to achieve the desired angle and height of the extension bracket.

- Adjust the height, width and angle of the extension bracket (A) on the mounting plate (B).

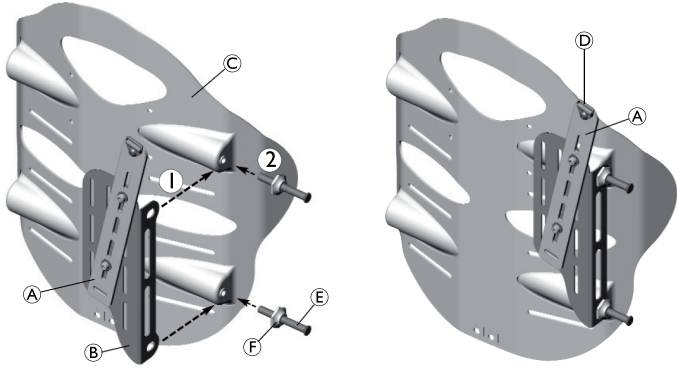
- Working on one side of the back shell (C), align the mounting holes in the mounting plate (B) with the pin hole locations on the elite back shell (1).
- Install/secure the mounting plate using the mounting pins (E) and flange nuts (F) provided with the back shell (2).

 Refer to your Matrix® Elite user manual for instructions on setting the proper mounting pin depth.

**IMPORTANT!**

- Once the pin depth has been properly adjusted, ensure the flange nuts are fully tightened against the mounting plate.

- Repeat steps 3 and 4 for the opposite side of the back shell.

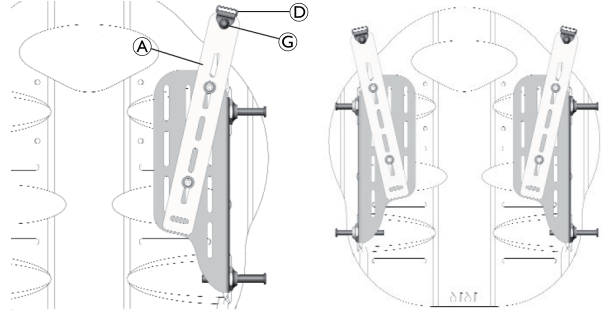
- Install the support clips (D) (included separately with your support harness) onto the extension bracket (A) using the 12 mm button head screws (G) provided.
- Make any final adjustments to the extension bracket position (height/width/angle) and fully tighten hardware.
- Verify that all remaining mounting/installation hardware is secured.

 With the Support Harness Mounting Bracket Assembly fully installed, please refer to your Support Harness User Manual (provided separately) for important safety information and instructions regarding the proper set-up and installation of your support harness.

**3.2 Installation Overview (Matrix® PB Backs)**

Pos.	Qty.	Description
(A)	1	Support Harness Mounting Bracket
(B)	1	Shim
(C)	1	Multi wrench, universal
(D)	1	Multi-metric Hex key, 4mm/5mm
(E)	4	M6 Nyloc nut, BZ
(F)	2	M6 1.0 x 12 mm Button socket cap, alloy steel, BZ
(G)	2	M6 1.0 x 20 mm Button socket cap, alloy steel, BZ
(H)	2	M6 1.0 x 25 mm Button socket cap, alloy steel, BZ
(I)	-	Support harness clips (not included)
(J)	-	Headrest clamps (not included)
1	With headrest clamp	
2	Without headrest clamp	

### 3.2.1 Installing the Support Harness Mounting Bracket (Matrx® PB)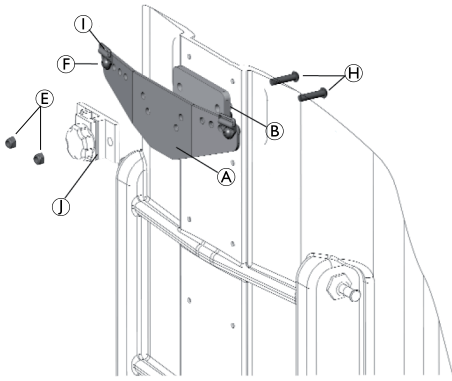

1. Unzip the outer cover along the top of the Matrx® PB back, and peel/pull back the cover and foam to access the pre-drilled mounting holes in the back shell.

 If applicable, remove existing headrest mounting plate and hardware from the back shell.

2. Align the shim plate (B) and harness support bracket (A) with the pre-drilled headrest mounting holes on the back shell.
3. Using the hardware provided, install the mounting screws (C) or (H) from the inside of the back shell so that they protrude through the shim plate and harness support bracket.

 A set of long (25 mm) and short (20 mm) mounting screws is provided in each hardware kit. The short screws (C) allow the support harness bracket/shim to be installed directly onto the back shell; the long screws (H) are intended for use when a headrest clamp (I) (not provided) is installed.

#### **WARNING!**

#### **Risk of injury**

- Failure to ensure the headrest clamp and harness support bracket are properly installed may result in serious personal injury.
- When installing or re-installing a Headrest Clamp/Assembly onto a Matrx® PB back, always ensure the long (25 mm) mounting screws (H) (provided) are used.
- Always ensure there is sufficient screw thread available to fully secure the nyloc nuts (E).


4. Secure the Support Harness Mounting Bracket (and headrest clamp, if applicable) onto the back shell using the nyloc nuts (E).
5. Install the support clips (1) (included separately with your support harness) using the 12 mm clip screws (F) provided.

 Support clips may be installed in 1 of 4 positions (widths) along the harness mounting bracket.

Prior to installing the support clips, please refer to your Support Harness User Manual (provided separately) for important safety information and instructions regarding proper harness set-up.

Support Clip adjustment is available in 12.5 mm increments.

Support Harness installation is shown without a headrest mounting plate.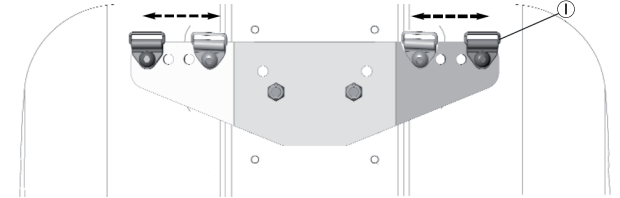

6. Verify that all mounting/installation hardware is fully secured.
7. Re-align the foam and cover along the top of the Matrx® PB back shell and close up the zipper.

 With the Support Harness Mounting Bracket Assembly fully installed, please refer to your Support Harness user manual (provided separately) for important safety information and instructions regarding the proper set-up and installation of your support harness.

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- På den ene siden av ryggrammen © retter du inn monteringshullene i festeplaten ② etter bolthullene på Elite-ryggrammen (1).