Invacare® Pro-tech® accessory

en	Drop Seat Board User Manual	1
de	Sitzbrett Gebrauchsanweisung	4
fr	Plaque d'assise amovible Manuel d'utilisation	8
it	Scocca sedile rimovibile Manuale d'uso	10
es	Plataforma de asiento ajustable Manual del usuario	14
pt	Placa de assento rebatível Manual de utilização	17
nl	Beweegbare zitplank Gebruiksaanwijzing	20
da	Justerbart fast sæde Brugsanvisning	23
no	Drop Seat Board Bruksanvisning	26
sv	Drop Seat Board Bruksanvisning	28

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

About this manual

This manual belongs to an Invacare® accessory and it contains important information about handling and assembly.

! IMPORTANT

- In order to ensure safety when using the product, read the user manual of the related product carefully and follow the safety instructions.

The Drop Seat Board is compatible with a wide range of wheelchair systems. Please contact Invacare® for information regarding suitable wheelchairs.

In keeping with our policy of continuous improvement, we reserve the right to modify designs without prior notice.

Symbols in this manual

Symbols and signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

Manufacturer of the product.

European Authorized Representative


Intended use

This product is intended to be fitted to a rigid frame wheelchair or a wheelchair with a central cross brace in order to provide a proper seat support and increase the sitting tolerance. Any other use is prohibited.

Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Guarantee claims can only be made through the provider from whom the appliance was obtained.

Scope of delivery

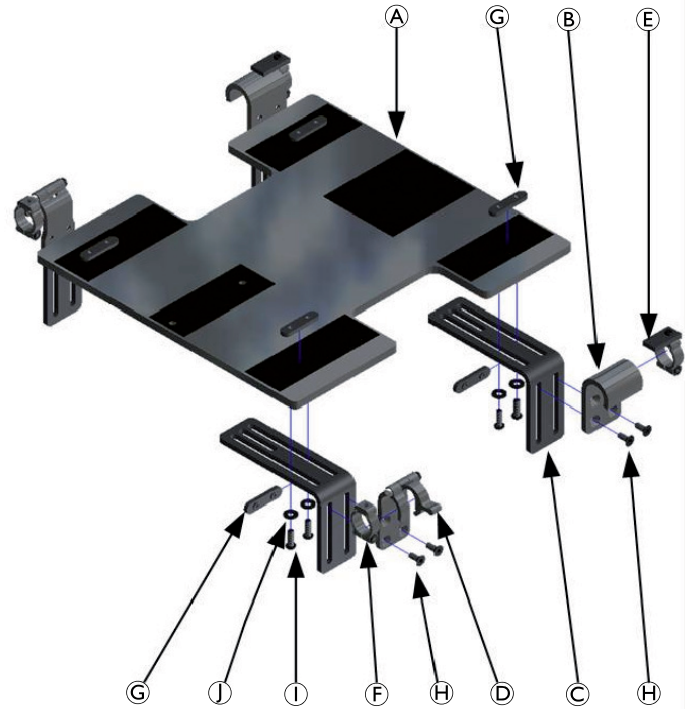	A Seat board (1x)		F Clamp (2x)
	B Tension hook (2x)		G 1" Weld plate (8x)
	C L-Bracket (4x) 4"x3" (for 16"-24" seats) / 4"x2" (for 12"-14" seats)		H 1/4"-20x3/4" Flat head screw (8x)
	D Fast clamp (2x)		I 1/4"-20x3/4" Dome head screw (8x)
	E Retainer clamp (2x)		J 1/4" Washer (8x)

Service life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary depending on frequency and intensity of use.

Setup

Safety information

WARNING!

Risk of injury due to wrong assembly

- This product must be fitted by an experienced clinician or technician.
- Before assessing for the appropriate positioning of the Drop Seat Board, ensure that the user is sitting in an optimum position with their bottom as far back in the seat as possible to assess for appropriate seat depth.

Assembling the product

If for any reason you should require additional training regarding the assembly of this hardware, please contact us at the number listed below. We will be glad to provide it free of charge.

1. Remove all parts from the packaging.

WARNING!

Risk of injury

- Before assembly, inspect the product to ensure that there are no missing or damaged parts.

Fit all screws with a 5/32" (4 mm) Allen key. Tighten all screws to 75 in-lbs (8.5 N).

2. Determine location on seat to attach L-Brackets © avoiding any obstructions / protrusions on chassis tubing.

3. Attach the L-Bracket © to the seat board (A), securing screws (I) and washers (I) through seat board and into weld plate (G).
4. Repeat for the three remaining brackets.

5. Attach the Fast clamps (D) to the front L-Brackets © using flat head screws (H) and weld plate (G) on each side.

6. Attach the Tension Hooks (B) to the rear L-Brackets © using flat head screws (H) and weld plate (G) on each side.

Adjusting the drop seat board

The seat height of the Drop seat board can be adjusted stepless to a dropped or raised seat position.

Dropped seat

1. To achieve a dropped seat position mount the Tension hooks (B) and the Fast clamps (D) high on the L-Brackets (C).

Raised seat

1. To achieve a raised seat position mount the Tension hooks (B) and the Fast clamps (D) low on the L-Brackets (C).

The seat angle of the Drop seat board can be adjusted stepless to a reclined seat position with a front to back difference of up to 3" / 76.2 mm.


Reclined seat


1. To achieve a reclined seat position mount the Tension hooks (B) high and the Fast clamps (D) low on the L-Brackets (C).

Installing the seat onto a wheelchair

Front frame part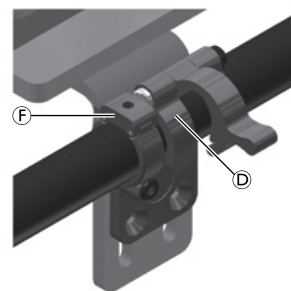

Rear frame part

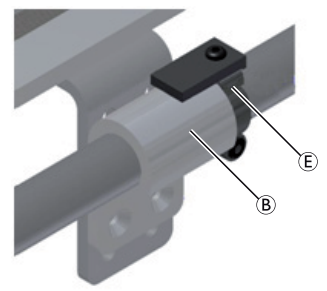1. Place the seat onto the wheelchair by engaging the Fast clamps (D) and the Tension hooks (B) firmly over the frame tubes.

IMPORTANT!

- Make sure that the Fast clamps are placed on the front frame part and the Tension hooks on the rear frame part.

2. Position the seat at the desired location.
3. Remove screws and nuts from the Retainer clamps (E).
4. Spread the Retainer clamps open and slide them over the wheelchair frame tubes, behind each Tension hook, with the tab facing over the Tension Hook.
5. Insert the screws and nuts in the Retaining clamps and tighten.
6. Remove screws and nuts from the Clamps (F).

- Spread the Clamps open and slide them over the wheelchair frame tubes, in front of each Fast clamp ④.

 Due to space limitations, it may be necessary to place the Clamps behind each Fast clamp.

- Insert the screws and nuts in the Clamps and tighten.

**WARNING!**

– Make sure that the Retainer clamps and Clamps are positioned flush with the Tension hooks and Fast clamps. Re-adjust, if necessary.

Removing the drop seat board

- Open the Fast clamps ④.
- Slide the Drop seat board forward until the Tension hooks ⑥ clear the tabs on the Retaining clamps ⑤.
- Lift the drop seat board off of the wheelchair frame.

Placing the drop seat board

- Open the Fast clamps ④.
- Place the Drop seat board on the wheelchair frame tubes.
- Slide the Drop seat board rearward until the Tension hooks ⑥ slide fully under the tabs on the Retaining clamps ⑤.
- Fold down the lever on the Fast clamp until it snaps over the wheelchair frame tube.

Using the Drop Seat Board**WARNING!****Risk of injury**

– Before each use, check the product visually for loose screws or worn screws and fittings.
– Do not use the product if defective.

**WARNING!**

– Always use in conjunction with an appropriate cushion and check that the user does not bottom out on the cushion.
– Ensure that the Drop Seat Board does not protrude forwards beyond the front of the cushion or come into contact with the back of the user's legs in any way. The cushion may overhang the board slightly, i.e. up to half an inch (13 mm).

The Invacare® Drop Seat Board is suitable for use in transport when fitted with heavy duty hardware.

**WARNING!**

– Make sure that the wheelchair, the Drop Seat Board is fitted to, is suitable for use when occupied in transport.

Maintenance and periodic inspection

Ensure the Drop Seat Board is inspected every three months for loose or worn fastenings and that the torque settings are checked by a suitably qualified and competent person, see chapter Assembling the product, page 3.

Cleaning and care

- Clean the mounting hardware and seat board with common household cleaners and a soft cloth.

Should the product become contaminated, ask your infection control officer for advice. Alternatively, contact Invacare® for their information on the decontamination protocol.

Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly (→ Cleaning and care, page 4), check the safety functions and reset if necessary.

Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

Dimensions and weights

Available dimensions Drop Seat Board: ¹⁾	12"/304.8 mm x 12"/304.8 mm 12"/304.8 mm x 14"/355.6 mm 14"/355.6 mm x 14"/355.6 mm 14"/355.6 mm x 16"/406.4 mm 16"/406.4 mm x 14"/406.4 mm 16"/406.4 mm x 16"/406.4 mm 16"/406.4 mm x 18"/457.2 mm 18"/457.2 mm x 16"/406.4 mm 18"/457.2 mm x 18"/457.2 mm 20"/508 mm x 16"/406.4 mm 20"/508 mm x 18"/457.2 mm
Total width x Total depth	
Product weight (fully assembled) ²⁾	1.62 lbs / 0.73 kg (12"x12") 2.72 lbs / 1.23 kg (20"x18")
Maximum user weight (with optional heavy duty hardware)	17.5 Stone / 113 kg (20 Stone / 120 kg)
Material ³⁾	Acrylic/PVC Alloy (Boltaron®, Kydex®)

¹⁾ For example: The actual width of the drop seat board will measure 2 3/4" / 70 mm narrower than the frame width. For example, the 18" / 457.2 mm seat board will be 15 1/4" / 387 mm wide, to fit an 18" / 457.2 mm seat frame.

²⁾ Depends on Drop Seat Board dimensions.

³⁾ This product does not contain natural rubber latex.

Fixation kits are available for 7/8" / 22.2 mm or 1" / 25.4 mm wheelchair frame diameters.

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Invacare AG
Freedom Designs, Inc.