
Invacare Bed Ends

Invacare® Astrid, Invacare® Nora, Invacare® Nanna, Invacare® Una, Invacare® Runa, Invacare® Inga

| | | |
|----|--|---|
| en | Bed Ends for Invacare® NordBed™ User Manual | 1 |
| da | Gavle til Invacare® NordBed™ Brugsanvisning..... | 2 |
| it | Pannelli per Invacare® NordBed™ Manuale d'uso | 3 |
| nl | Beduiteinden voor Invacare® NordBed™ Gebruiksaanwijzing | 5 |
| no | Endegavler for Invacare® NordBed™ Bruksanvisning..... | 6 |
| sv | Sänggavlar för Invacare® NordBed™ Bruksanvisning..... | 7 |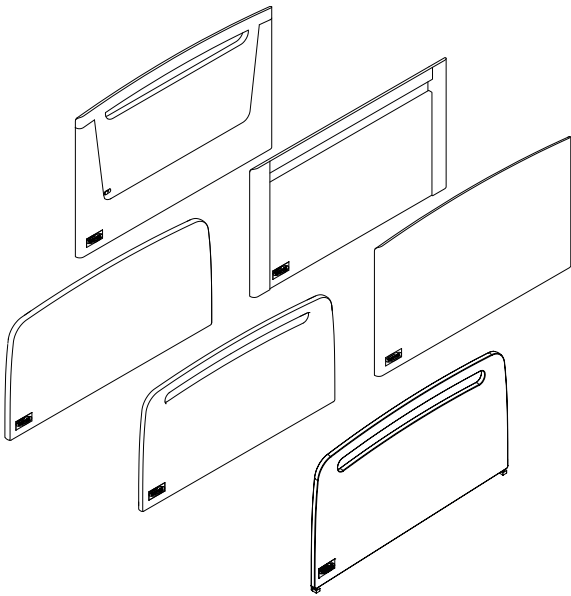

en

1 Introduction

This manual belongs to an Invacare accessory and is an addition to the manuals of the related Invacare product.

To ensure safety when using this accessory, read the user manual of the related product carefully and follow the safety instructions.

For further information see the user manual of the related product.

2 Installing bed ends

2.1 Bed ends for Sanne-bracket

CAUTION!


Risk of squeezing

If the bed end is not correctly locked it can fall down and cause injury to user.

– Make sure the locking pins are engaged.


Mounting

1.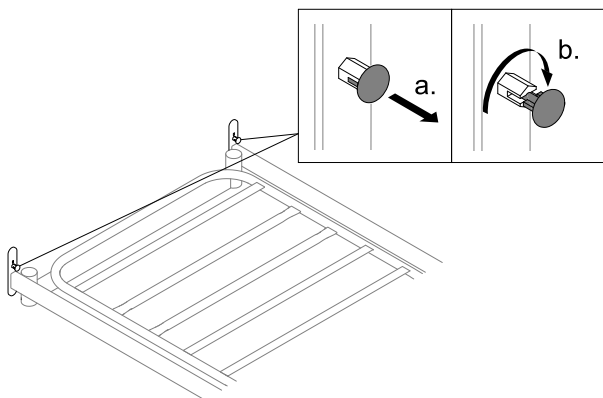

Pull out the locking pin and rotate by 90 degrees to disengage from the bed end on both sides.


2.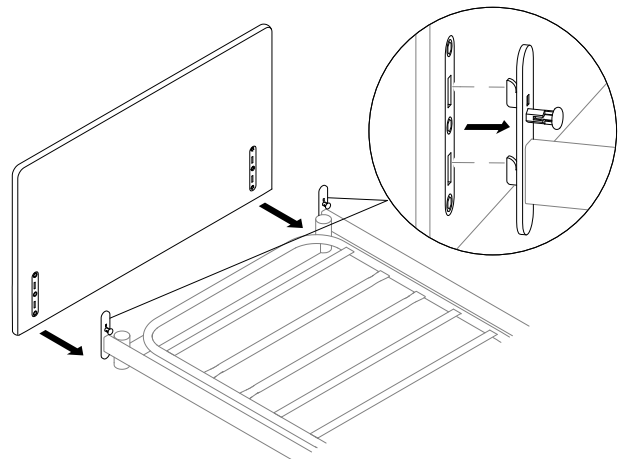

Align the slots in the bed end with the tabs on the Sanne bracket and place the bed end onto the tabs.

3. Push down firmly and ensure that the bed end is fully seated in the tabs.

4.

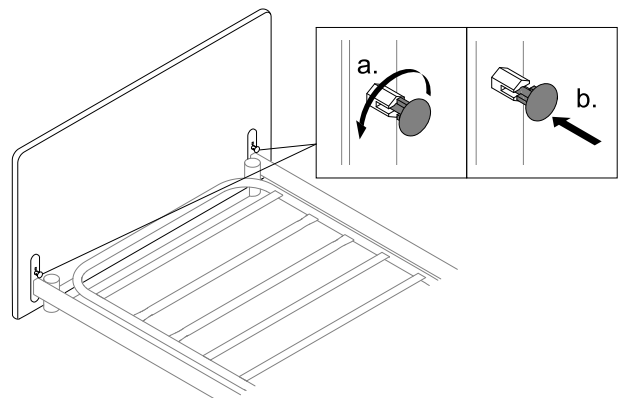Rotate the locking pin and allow it to engage with the bed end on both sides.

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

Yes, you can.®


Dismounting

1. Pull out the locking pin and rotate by 90 degrees to disengage from the bed end on both sides.
2. Hold the bed end firmly on each side and pull up vertically on both sides at the same time.

2.2 Bed ends for Quick-bracket


Mounting

1.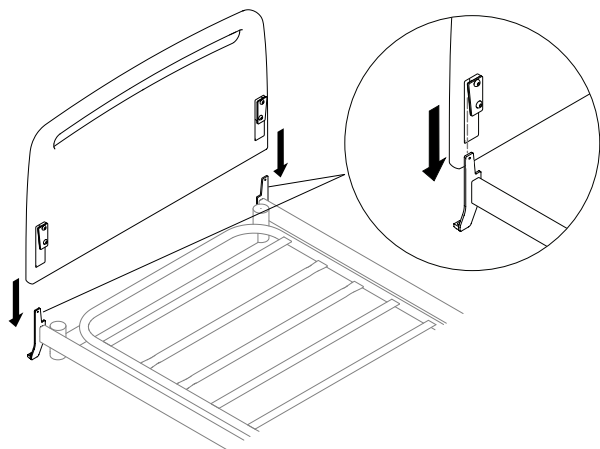

Align the slots on the bed end with the tabs on the Quick bracket and lower the bed end onto the brackets.

2.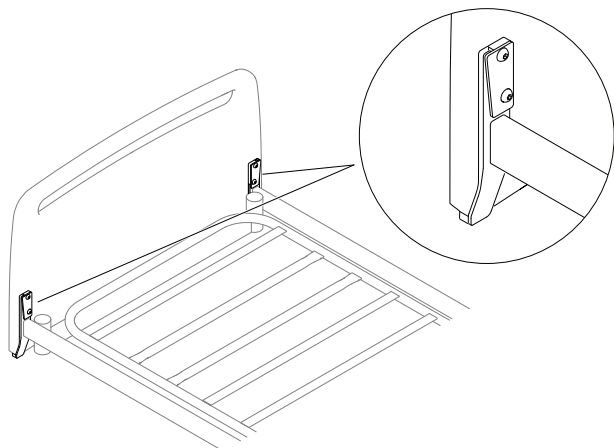

Press down firmly and ensure the bed end is fully seated on the brackets.


Dismounting

1. Hold the bed end firmly on each side and pull up vertically on both sides at the same time.

2.3 Bed end brackets

Depending on the bed end to be mounted, it might be required to pre-mount or change the bed end brackets.

Mounting

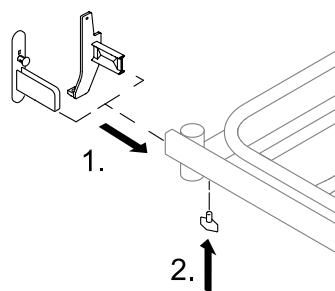1. Insert the bed end brackets into the side tubes of the mattress support platform.
2. Fix the bed end brackets on the side tubes with the screws from below and tighten firmly.

Dismounting

1. Remove the screws and pull the bed end brackets out of the side tubes.

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Invacare AB
Invacare Limited
Invacare Rea AB