# Safety Inspection Checklist

en	Invacare Slings Service Manual .....	2
da	Invacare sejl Servicemanual .....	3
de	Invacare Patientengurte Servicehandbuch .....	4
es	Eslingas Invacare Manual de servicio .....	5
fi	Invacare-nostoliinat Huoltokäsikirja .....	6
fr	Sangles Invacare Manuel de maintenance .....	7
it	Imbracature Invacare Manuale per la manutenzione .....	8
nl	Invacare-draagbanden Servicehandleiding .....	9
no	Invacare løfteseil Servicehåndbok .....	10
pt	Cestas Invacare Manual de assistência .....	11
sv	Invacare-lyftselar Servicemanual .....	12

PROVIDER: Keep this manual.  
The procedures in this manual MUST be performed by a qualified technician.

**Yes, you can.®**

### Safety Inspection Checklist for Invacare Slings

- !** **IMPORTANT!**
- A periodic safety inspection of the sling must be performed at least every 6 months unless otherwise stated in local requirements.
  - Inspections must be performed by a qualified person who is well acquainted with the design, use and care of slings.
  - The inspection record must be retained for examination in the event of an incident.

	Model:	REF / SN number:
---	--------	------------------

Checkpoints for Invacare slings according to ISO 10535:2006 Annex B	✓	✗	Comment
<b>Inspect the product labels</b>			
Check to ensure the following information is legible:			
Max. safe working load and size			
Serial number and reference number			
Date of manufacture			
Laundry instructions			
<b>Inspect the suspension straps</b>			
Check to ensure there are no signs of damage or wear, like holes, tears, loose threads, fraying or changes in color and stiffness on:			
Straps and loops			
Seams at loops and at attachment points to main fabric			
<b>Inspect the fabric and other components</b>			
Check to ensure there are no signs of damage or wear, like holes, tears, loose threads, fraying or changes in color and stiffness on:			
Main fabric and edge band			
Handles and pockets (if present)			
Belts and buckles (if present)			
<b>General Checkpoints</b>			
Check to ensure there are no visible repairs and no use of knots, tapes, needles or other methods to change the shape or size of the sling.			
Make sure the sling is clean.			
Make sure the current version of the user manual is accessible.			
Check to ensure the accessories (if present) have no signs of damage or wear and replace them if deficiencies are detected.			

**Overall assessment**

<input type="checkbox"/> <b>Pass</b> No safety or functional deficiencies were detected. Date of next inspection: _____	<input type="checkbox"/> <b>Fail</b> Safety or functional deficiencies were detected.
---	--

- !** **IMPORTANT!**  
 If safety or functional deficiencies are detected the sling must not be used any more and must be discarded.

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Invacare Australia Pty. Ltd.
Invacare New Zealand Ltd