NOTE: Please read the following instructions carefully prior to assembly.

This item should be fitted by an experienced clinician or technician. This document contains important information that must be passed on to the user at delivery.

Before assessing for the positioning of the harness, ensure that the user is sitting in an optimum position with their bottom as far back in the seat as possible.

TRANSPORTATION

The Invacare Harnesses are designed to provide postural support only and as such should not be used as a passenger restraint for transportation. A separate passenger restraint should always be used, please refer to wheelchair user manual – transportation section.

MAINTENANCE

Before each use:

Undertake a visual check for torn fabric, loose or worn screws.

Bi-Annually:

Ensure each Harness is inspected for loose or worn fastenings and that the torque settings are checked by a suitably qualified and competent person. For heavier and high impact users, the period between inspections should be reduced accordingly.

The Expected Product Life-Cycle for this product is 30 months.

LIMITATIONS OF USE

The Invacare Harnesses are designed to provide postural support only and as such should not be used as a passenger restraint for transportation. Please refer to section 2 above “Transportation”.

TRAINING

If for any reason you should require additional training regarding the assembly of this hardware, please contact us at the number listed below, for free advice and support

COMPATIBILITY

Invacare harnesses are designed to be used in conjunction with the Flo-tech PT back support and Azalea range.

QUALITY

In keeping with our policy of continuous improvement, we reserve the right to modify designs without prior notice. All rights reserved.

CLEANING AND CARE

The Harness range of products may be cleaned with a neutral detergent (e.g. soapy hot water) and / or a single use wipe. The Nylon Covered Neoprene pads can be easily removed for machine washing at 60°C and should be air dried. Not suitable for tumble drying. Refer to washing label

The mounting hardware may be cleaned with common household cleaners and a soft cloth. Should the product become contaminated, please ask your infection control officer for advice.

Store in a dry environment.

CAUTION

- Ensure the central part (collar area) of the harness is not positioned above the suprasternal notch (top of the sternum) as this may pose a choking hazard.

SAFETY

- Anterior harnesses are designed to provide postural support only and as such should not be used as a restraint or as a passenger restraint for transportation. A separate passenger restraint should always be used.
- The harness should be fitted over the sternum and not be positioned around the neck. Allow enough space between the neck and the harness to ensure it does not pose a choking risk.
- Always use in conjunction with an appropriately fitted pelvic belt and backrest, ensuring the pelvis is well supported to minimise the risk of the user slipping down in the seat and choking on the harness.
- Consideration must be given to the routing of Harness straps, ensuring they avoid any moving parts, sharp or abrasive edges.

Flo-tech PT Harness Range Fitting Instructions

A

Sternum Harness
Qty: 1

Size	Part No.
Medium	1525276
Large	1525277

B

Dynamic Harness
Qty: 1

Size	Part No.
Small	1525278
Medium	1525279
Large	1525280

C

Shoulder Harness
Qty: 1

Size	Part No.
Small	1525273
Medium	1525274
Large	1525275

D

Y Harness
Qty: 1

Size	Part No.
Small	1525281
Medium	1525282
Large	1525283

E

Dynamic Chest Support
Qty: 1

Size	Part No.
Medium	1525284
Large	1525285

F

Mounting Clamps
Qty: 2

Size	Part No.
19mm	1525288
22mm	1525288
25mm	1525288

G

Flo-tech PT Interface Screws
Qty: 2

H

Optional Strap Risers
Qty: Pair

INSTRUCTIONS FOR ASSEMBLY

STEP 1—Remove the Harness and components from the packaging

STEP 2—Should this product reach you in an unsatisfactory condition, please contact the supplier.

STEP 3—Align the holes in the top tri slot mounting plates of the Harness (see fig. 1) with screws attaching the L-Bracket to the Back Shell (see fig. 2).

STEP 4 - Attachment can also be made to the wheelchair frame, by means of the Universal Mounting Clamps, which are suitable for round tube frames of 19mm, 22mm, and 25mm (see fig. 3). This is the recommended fixation method for the bottom straps.

It is recommended that all screws be tightened to 8.5Nm.

Optional Strap risers (H) are installed at top of Back Shell L-Bracket using screws (G).

Adjusting the straps

STEP 5 – Pull or loosen strap ends to adjust the harness position, ensuring that the harness is firmly fitted (see fig 4). Always fit in conjunction with a pelvic belt.

Figure 1

Figure 2

Figure 3

INSTRUCTIONS FOR REMOVAL

The harnesses are easily removed and refitted.

Each strap has a quick release buckle to detach the harness from the chair (see fig. 4). To remove the harness, release the top buckles by pressing the buckle with a finger or thumb and then release one of the bottom buckles to allow the user to transfer (see fig 5). To remove the Harness Completely release all four buckles.

To refit carry out above instructions in reverse.

Figure 4

Figure 5

WARRANTY

Flo-tech PT Harnesses are covered by a warranty period of 12 months.

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