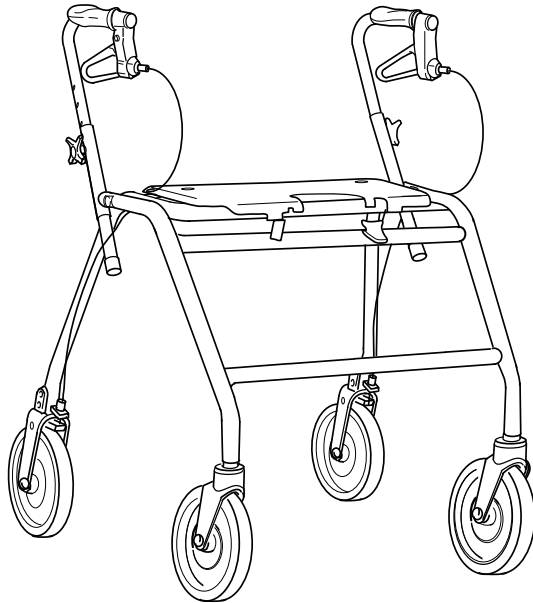
# Dolomite® Maxi+

EN	<b>Rollator</b>	
	User Manual .....	3
DE	<b>Rollator</b>	
	Gebrauchsanweisung.....	15
ES	<b>Rollator</b>	
	Manual del usuario.....	27
FR	<b>Rollator</b>	
	Manuel d'utilisation .....	39
IT	<b>Rollator</b>	
	Manuale d'uso.....	51
NL	<b>Rollator</b>	
	Gebruiksaanwijzing .....	63

This manual **MUST** be given to the user of the product.  
**BEFORE** using this product, read this manual and save for future reference.

**Yes, you can.®**

© 2013 Invacare® Corporation

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

---

# Contents

---

This manual **MUST** be given to the user of the product.  
BEFORE using this product, read this manual and save for future reference.

<b>1</b>	<b>General</b> . . . . .	<b>4</b>
1.1	About this manual . . . . .	4
1.2	Intended use . . . . .	4
1.3	Service Life . . . . .	5
1.4	Scope of delivery . . . . .	5
1.5	Warranty . . . . .	5
<b>2</b>	<b>Safety</b> . . . . .	<b>6</b>
2.1	General Warnings . . . . .	6
2.2	Labels and symbols on the product . . . . .	6
<b>3</b>	<b>Usage</b> . . . . .	<b>7</b>
3.1	Before use. . . . .	7
3.2	Folding and unfolding the rollator . . . . .	7
3.3	Using the hand brakes . . . . .	8
3.4	Adjusting the handle height. . . . .	8
3.5	Walking. . . . .	9
3.6	Sitting down on a chair. . . . .	9
3.7	Rising to a standing position . . . . .	9
3.8	Sitting on the rollator. . . . .	10
3.9	Accessories . . . . .	10
<b>4</b>	<b>Maintenance and care</b> . . . . .	<b>11</b>
4.1	Maintenance . . . . .	11
4.2	Care . . . . .	11
<b>5</b>	<b>After Use</b> . . . . .	<b>12</b>
5.1	Disposal . . . . .	12
5.2	Re-use. . . . .	12

<b>6</b>	<b>Technical data</b> . . . . .	<b>13</b>
6.1	Dimensions and weights . . . . .	13
6.2	Material . . . . .	13

## I General

### I.1 About this manual

This User Manual contains important information about your product. In order to ensure safety when using the product, read the User Manual carefully and follow the safety instructions.

The person who prescribes the product or who is otherwise responsible for the user must ensure that the product is suitable for the user. The user shall be instructed how to use the product. This includes the use of the features and the restrictions in use. The user shall also be notified of the warnings for use given in the user manual.

### Symbols in this manual

In this User Manual warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

#### WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

#### CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

#### IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

#### Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

### I.2 Intended use

#### Indications

The rollator is intended for support and improvement of a restricted walking ability. The user's remaining walking ability needs to be sufficient for leaving the home, for usual daily activities and for participating in social life without endangering themselves or others.

The rollator is intended to support a person during walking and may also serve as a resting seat. Furthermore the rollator is intended to carry a load either in a specially designed basket or on a tray.

The max. user weight is 200 kg. This weight limitation includes the weight of the user whether standing or sitting and the content of the basket.

#### Contraindications

The Rollator is not intended for use by persons with severe equilibrium disorders or functional perceptual disorders.

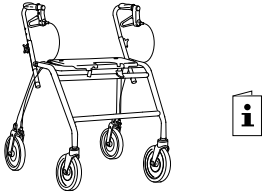
#### Permissible operating conditions and locations

The rollator is only intended for use indoor in living or public areas or outdoor. The rollator is not to be used under water in e.g. pools or open-air bath, neither in hot areas as e.g. saunas.

### 1.3 Service Life

The expected service life of this product is five years when used in accordance with safety instructions, maintenance intervals and correct use, stated in this manual. The effective product life cycle can vary according to frequency and intensity of use.

### 1.4 Scope of delivery

- Rollator, fully assembled.
- User manual

Accessories, actually assembled to your rollator, are depending on the configuration and might differ per region.

### 1.5 Warranty

We supply a two year warranty\* from the date of delivery. Damage caused by wear to items such as wheels, handles and brakes is not covered by the warranty. Damage that arises due to abnormal use is excluded. Damage caused by users who weigh more than max. user weight is excluded. Maintenance and operating instructions must be observed for the warranty to be valid.

\* Deviations may occur due to local agreements.

## 2 Safety

### 2.1 General Warnings

#### WARNING!

##### Risk of falling

- Exercise caution when walking on sloping / slippery ground and on uneven surfaces.
- Do not use the rollator as a wheelchair.
- Be careful when walking rearwards over obstacles.
- Never sit and push with your feet.
- Do not go up or down stairs with the rollator.
- Do not use the rollator on escalators.
- Do not under any circumstances stand on the rollator.
- Be careful when using long skirts, the clothes might get stuck in the wheels.
- Loads must only be transported in the basket or on the tray. Be careful when there is a heavy load in the basket (for the max. load in the basket refer to chapter 6 Technical data, page 13).

#### WARNING!

##### Risk of injury

- Do not let children play with the rollator.
- If the push handles or the seat are exposed to extreme temperature (above 41°C) there might be risk for first degree burns.

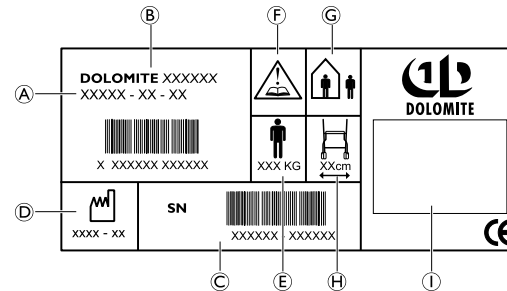
#### IMPORTANT!

##### Risk of corrosion

- The rollator must not be stored outside in wet climate or in humid environments for a long period of time.

### 2.2 Labels and symbols on the product

The label contains all important technical information regarding your rollator.

- |                       |                            |
|-----------------------|----------------------------|
| Ⓐ Article number      | Ⓕ Refer to user manual     |
| Ⓑ Product name        | Ⓖ In- and outdoor rollator |
| Ⓒ Serial number       | Ⓗ Max. width of rollator   |
| Ⓓ Date of manufacture | Ⓘ Manufacturer's name.     |
| Ⓔ Max. user weight    |                            |

## 3 Usage

### 3.1 Before use

We do not recommend that the product is issued to anyone without first carrying out a full assessment of their specific needs. It is important that the user is given the necessary support by the product as prescribed by a qualified professional. This is both for the safety of the user and also their recovery pattern.

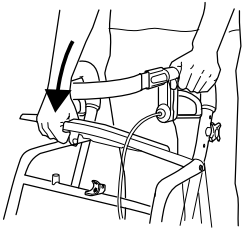
### 3.2 Folding and unfolding the rollator

#### Unfolding to operating position


#### **WARNING!**

– Always make sure that the rollator is fully unfolded and the lock is engaged before using the rollator.

1. Press down the seating tube.
2. Check that the safety catch is in the locked position.


#### Folding

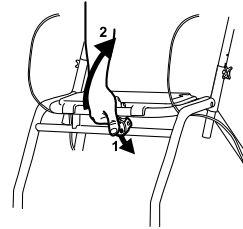1. Press down the safety catch.
2. Pull the seating tube upwards.


#### Flip up the seat plate

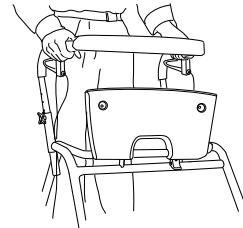You can flip up the seat to provide a larger walking space and to increase accessibility.


### 3.3 Using the hand brakes

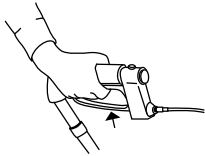1.

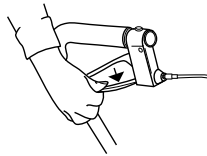2.


1. Apply the brakes by pulling both brake handles upwards.
2. To apply the parking brakes, press the brake handles carefully downwards.

An audible sound indicates that the brake is in the correct position.

 It is important that both brakes are put into parking position.

3. To release the brake, pull the brake handle upwards.


### Adjusting the brake effect

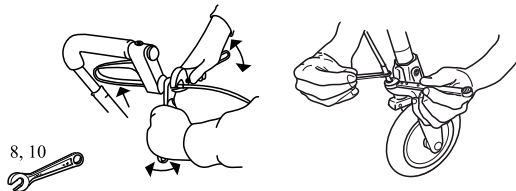For desirable braking strength, adjust the brake wire.


Suitable tools: 8 mm and 10 mm spanner (not included).

### 3.4 Adjusting the handle height

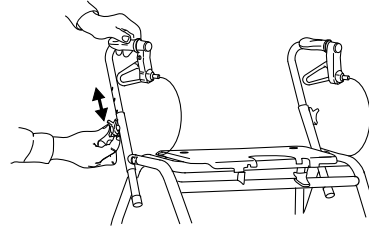1. Stand behind the rollator with flat shoulders.
2. Loosen the knob completely, remove the screw.
3. Adjust the handle to suitable height.

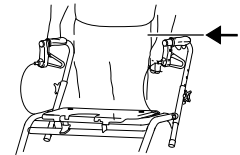 The handles should usually be positioned at wrist height. Therapist may suggest different heights depending on specific needs of the client.

4. Fix the screw into the hexagonal hole.
5. Fasten the knob properly.
6. Repeat the procedure on the other side.


### 3.5 Walking

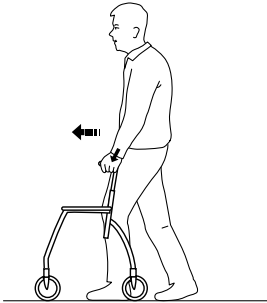
**WARNING!**  
**Risk of falling**

- Be careful if the wheel turns 90 degree and stops against an obstacle.

1. Place the rollator a bit in front of you and take a step towards it whilst using the hand grips for support.

### 3.6 Sitting down on a chair

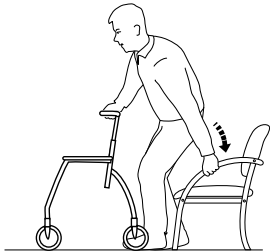
1. Stand in front of the chair with the rollator, the back of the legs should touch the chair.
2. Place the hands on the arms of the chair, lean forward slightly and sit down carefully.

### 3.7 Rising to a standing position

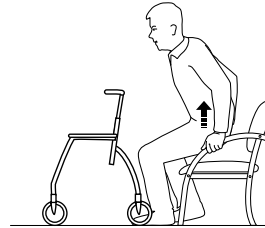**WARNING!**  
**Risk of tipping**

- Never use the rollator to pull yourself up from a sitting position.
- The parking brakes must be in braked position when the rollator is used as a support when getting up, from a chair for example.

1. Place the rollator in front of you, use the armrests of the chair as support and rise carefully to a standing position.
2. Once steady in a standing position, transfer hands to the hand grips of the rollator.

### 3.8 Sitting on the rollator

**WARNING!**

**Risk of falling**

Use the rollator for stationary seating only. Both brakes must be in parking position when the rollator is used as a seat.

- Never sit and push with your feet.
- Never sit on the rollator on sloping ground.

### 3.9 Accessories

Accessories are not standard on all models. For a list of available accessories, contact your authorized dealer or the Invacare representative (see rear page of this manual).

#### **Basket**

Mount the basket by hanging it with the hooks on the front frame.

#### **Tray**

Secure the tray by using the plastic plugs.

## 4 Maintenance and care

### 4.1 Maintenance

Maintenance should be done at least once a year.

To ensure good performance and safety you should regularly check that the following is carried out:

- Ensure that screws and adjustments are properly tightened.
- Never use a defective rollator. Do not attempt repairs yourself!
- If hand grips are loose do not use the rollator. Change hand grips according to the service instructions which can be obtained from your Invacare representative.
- Verify brake operation. If brake adjustment is required, see chapter 3 Usage, page 7 in this user manual.
- Periodically inspect the castors tightness, wear and damage.
- Make sure that the wheels are free of hair, lint and other debris.

### 4.2 Care

The product can be cleaned and disinfected using commercially available agents.

#### Cleaning

#### Important!

- Do not use an abrasive detergent, cleaner or cloth to clean or dry the product.
- Do not use high pressure cleaner or steam cleaner.

- Clean the rollator with solvent-free household cleaner.
- Clean the frame with damp cloth and mild detergent.
- Clean the wheels with a brush (no wire brush!).
- Clean the backrest, seat and basket with mild detergent.

### Disinfection

Disinfect the product by wiping down all generally accessible surfaces with disinfectant.

---

## 5 After Use

---

### 5.1 Disposal

When the product can no longer be used, it and its accessories as well as the packaging material should be sorted in the correct way.

The individual parts can be separated and handled according to their material.

Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

### 5.2 Re-use

This product is suitable for re-use after the following has been performed:

- Inspection
- Cleaning

For details, see chapter 4 Maintenance and care, page 11.

#### **WARNING!**

#### **Risk of injury**

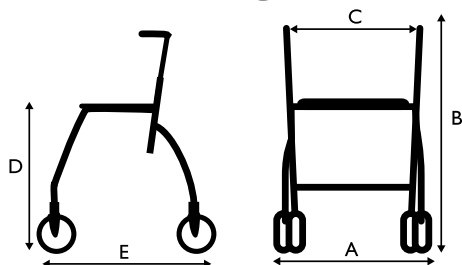
- Always inspect the product according to the service instructions before transferring the product to a different user.
- If any damage is found do not use the product.

The user manual belongs to the product. Make sure that the user manual will be handed over with the product.

## 6 Technical data

### 6.1 Dimensions and weights

	Maxi+ 550	Maxi+ 650
Total width (A)	700 mm	
Total height (B)	725 – 895 mm	815 – 985 mm
Seat width (C)	520 mm	
Seat height (D)	555 mm	650 mm
Total length (E)	640 mm	740 mm
Front/rear wheel <sup>1</sup> diameter	200 mm	
Front/rear wheel <sup>1</sup> width	40 mm	

<sup>1</sup> Front wheels are steering wheels. Rear wheels are provided with a brake system for dosed breaking and for locking the wheel.

	Maxi+ 550	Maxi+ 650
Weight <sup>2</sup>	9,8 kg	10,3 kg
Max. user weight <sup>3</sup>	200 kg	
Max. load basket	9 kg	
Max. load tray	5 kg	

<sup>2</sup> Total weight excluding equipment.

<sup>3</sup> This weight limitation includes the weight of the user whether standing or sitting and the content of the basket.

### 6.2 Material

Frame	Polyester powder coated steel
Handle bars	Chrome plated steel
Handgrips	TPE
Wheels	Polypropylene with treads of TPE
Plastic parts	Material according to marking (PA, PP, PE)
Screws	Steel, rust protected

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Invacare Austria GmbH
Invacare Poirier SAS
Invacare Dolomite AB
c/o Invacare Rea AB