# Addendum

en	Invacare Slings User Manual .....	2
da	Invacare sejl Brugsanvisning .....	3
de	Invacare Patientengurte Gebrauchsanweisung .....	5
es	Eslingas Invacare Manual del usuario .....	6
fi	Invacare-nostoliinat Käyttöohje .....	8
fr	Sangles Invacare Manuel d'utilisation .....	9
it	Imbracature Invacare Manuale d'uso .....	11
nl	Invacare-draagbanden Gebruiksaanwijzing .....	13
no	Invacare løfteseil Bruksanvisning .....	14
pt	Cestas Invacare Manual de utilização .....	16
sv	Invacare-lyftselar Bruksanvisning .....	18

This manual MUST be given to the user of the product.  
BEFORE using this product, this manual MUST be read and saved for future reference.

**Yes, you can.<sup>®</sup>**

## 1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

## 2 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

### 2.1 Product-specific standards

The product has been tested and conforms to ISO 10535 (Hoists for the transfer of disabled persons) and all related standards.

For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

## 3 Safety

### CAUTION!

#### Compatibility of Slings with Attachment System

Invacare uses a common attachment system based on hooks and loops. Loops on the sling are attached to hooks on the lift.

- A risk assessment is always to be carried out by a professional prior to use of Invacare slings with lifts of other manufacturers.
- If you are in doubt about the use of Invacare slings with equipment of other manufacturers, contact your Invacare provider for advice.

### Symbols

<b>MD</b>	Medical device
-----------	----------------

## 4 Intended use

The Invacare slings are non-rigid body-support units to transfer an individual from one surface to another.

Amputee	The sling is intended to be combined with a passive lift using a loop to hook attachment with 2 or 4 connection points.
Comfort Standard	
Comfort	
Comfort Toileting	
Dress Toileting	
Dress Toileting High	
Easy Fit	
Universal Low	
Universal Standard	
Universal High	
Universal High Plus	
Lifting Band	The sling consists of two individual parts and is intended to be combined with a passive lift using a loop to hook attachment with 2 or 4 connection points.
Standing Transfer Vest	The sling is intended to be combined with a ceiling hoist using a loop to hook attachment with 2 connection points.
Stand Assist	The slings are intended to be combined with a active lift using a loop to hook attachment with 2 connections points for Stand Assist slings or 4 connection points for Transfer Stand Assist slings.
Transfer Stand Assist	

The slings are not intended to be left underneath the patient after the transfer.

A healthcare professional or private person who has received proper training is the intended operator of this product.

### Indications

The slings are indicated for the transfer of completely or partially immobile patients with the following minimum degrees of body control:

	Hip control	Trunk control	Head control
Amputee	Limited	Limited	Good
Comfort Standard	Limited	Limited	Good
Comfort	Limited	Limited	Limited
Comfort Toileting	Limited	Limited	Limited
Dress Toileting	Good	Good	Good
Dress Toileting High	Good	Good	Limited
Easy Fit	Limited	Limited	Limited
Universal Low	Limited	Good	Good
Universal Standard	Limited	Limited	Good
Universal High	Limited	Limited	Limited
Universal High Plus	Limited	Limited	Limited
Lifting Band	Good	Good	Good

	Leg control / weight bearing ability	Hip control	Trunk control	Head control
Standing Transfer Vest	Good	Good	Good	Good
Stand Assist	Good	Good	Good	Good
Transfer Stand Assist	Limited	Limited	Good	Good

### Contraindications

For all slings not mentioned in the following table there are no known contraindications.

Dress Toileting	The slings are contraindicated for individuals with high and bilateral amputee legs.
Dress Toileting High	
Lifting Band	The sling is contraindicated for individuals with low tonus, spasticity or amputee legs.
Standing Transfer Vest	The slings are contraindicated for individuals who are not cooperative or are not able to understand and follow instructions.
Stand Assist	
Transfer Stand Assist	

## 5 Accessories

- All Invacare slings
  - Extension loops – loop to extend the suspension straps
- Universal slings only
  - Fleece padding – protective and comfortable padding for the leg straps
- Standing Transfer Vest only
  - Groin Band – additional fixation straps to be attached to the sling and applied around the patient's thighs

## 6 Storage Conditions

The sling must be stored in dry conditions with a maximum relative humidity of 50% and without direct sunlight.

## 7 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

## 8 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

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Invacare Australia Pty. Ltd.
Invacare New Zealand Ltd