Invacare® Dress Toileting High


EN	Sling	User Manual	1
DA	Sejl	Brugsanvisning	3
DE	Patientengurte	Gebrauchsanweisung	4
ES	Eslinga	Manual del usuario	6
FR	Sangles	Manuel d'utilisation	8
IT	Imbracatura	Manuale d'uso	9
NL	Liftmat	Gebruiksaanwijzing	11
NO	Seil	Bruksanvisning	12
PT	Cesta de transferência	Manual de utilização	14
SV	Lyftselar	Bruksanvisning	15

EN

I General

I.1 Figures

Figur
Figur
Figur
Abbildung
Afbeelding
Figures
Figura
Figura
Figura

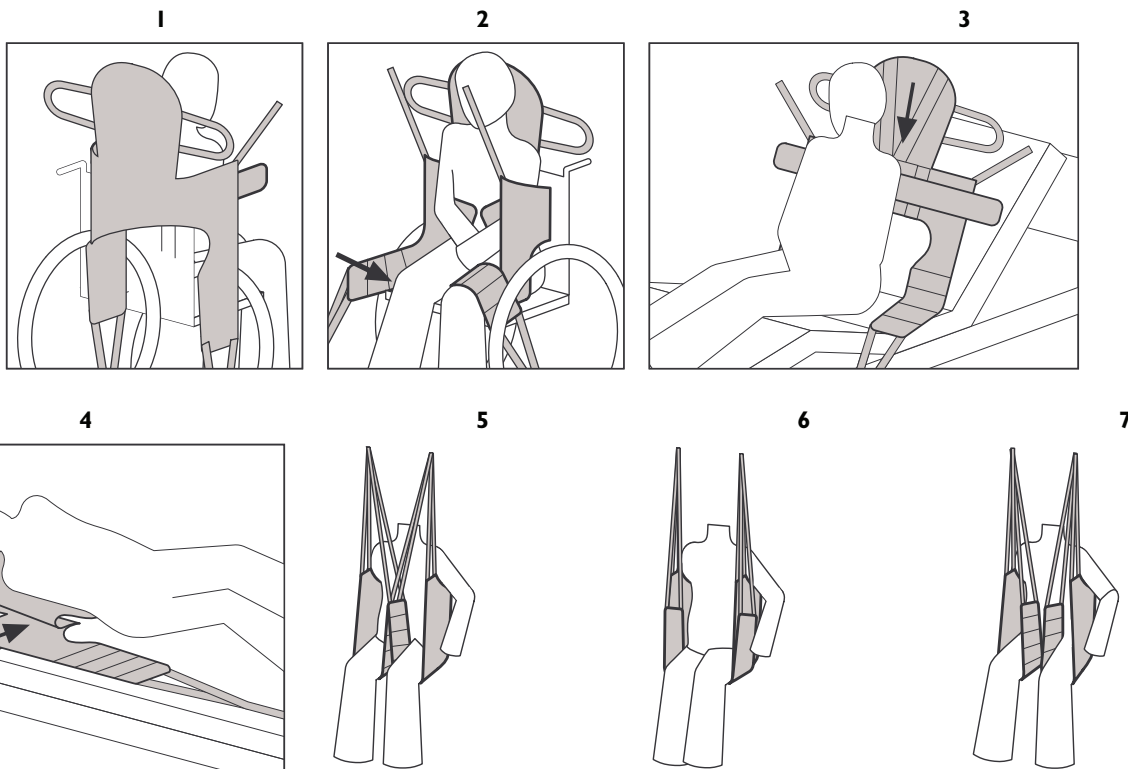This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

1.2 Symbols

	Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words. WARNING! Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury. CAUTION! Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.
	Gives useful tips, recommendations and information for efficient, trouble-free use.
	IMPORTANT indicates a hazardous situation that could result in damage to property if it is not avoided.
	This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.
	Safe Working Load
	Consult the user manual.
	DO NOT exceed 95°C washing temperature.
	DO NOT bleach.
	Tumble Dry, Normal, Low Heat
	DO NOT iron.

1.3 Intended Use

The Invacare® Toileting High sling combines quick and easy application with maximum personal access for dressing and hygiene. It is only suitable for clients with good trunk control but whom may need additional head support; however the sling can promote independence, as the arms remain free for function.

1.4 Service Life

All Invacare slings are guaranteed for one year. Expected service life of the product is 1-5 years. Service life time varies depending on the fabric, frequency of use, laundering practices, and weight carried.

2 Safety

2.1 General Guidelines

WARNING!

- DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- Please check your sling on delivery. If the sling is not in perfect condition, please contact your supplier.

2.2 Safe Working Load

	Solid	
S	1478444	200 kg
M	1478445	200 kg
L	1478446	200 kg
XL	1484780	250 kg

3 Usage

3.1 Sling application

WARNING!

- Slings should be checked regularly for wear and damage (in accordance with each country's regulations). NEVER use a damaged sling.
- If you are in any doubt, please do not attempt to use the sling and seek the advice of a professional or your supplier.
- Invacare recommends that a risk assessment is always carried out by a professional prior to issuing lifting equipment. Always choose the sling design and size according to the client's weight, size and physical ability.
- If you are in doubt about the use of Invacare's slings with other manufacturers' equipment, contact Invacare or your supplier for advice.

Sling application — Sitting in chair or wheelchair

1. Lean the client forward ensuring that he is well supported to reduce risk of falling and to promote feelings of security.
2. Slide the sling down behind the back of the client in a central position with the aperture apex aligned with the base of the spine. The padded upper edge must be right under the arms.

WARNING!

- The arms must be outside the sling.

3. Sit the client back against the sling. **Figure 1.**
4. Pass the leg support towards the front of the chair to each side of the thigh and then pass each leg support under each thigh. **Figure 2.**

Sling application — Sitting in a bed

1. Lean the client forward ensuring that he is well supported to reduce risk of falling and to promote feelings of security.
2. Slide the sling down behind the back of the client in a central position with the aperture apex aligned with the base of the spine. The padded upper edge must be right under the arms.

WARNING!

- The arms must be outside the sling.

3. Sit the client back against the sling. **Figure 3.**
-  Alternatively, if the client does not have sufficient trunk control, roll the sling to mid-way point and roll the client to one side to position the rolled up sling under the lifted side. Ensure the center of the sling (green mark) is placed centrally along the spine. Roll the client to the other side, unroll the sling and roll the client back into position on the sling. **Figure 4.**
4. Bend the knees one at a time to apply the sling under the legs.

3.2 Application of the legs straps

It is crucial for the sitting position that the leg support is placed correctly. Clients with reduced hip control must have the leg supports placed close to the groin.

Crossed legs straps

For maximum security, cross the legs straps. This will reduce the risk of forward pitching and also maintain a better alignment of the hips and lower limbs. **Figure 5.**

Looping the leg sections together

If the client feels this is more comfortable, pass each individual leg strap under both legs. Ensure that the client is not prone to spasm and there has been a risk assessment completed to address the possibility of forward pitching. **Figure 5.**

Uncrossed leg straps

Apply each leg support under each leg, but do not cross the straps. This can be recommended when the client has abdominal sensitivity or requires intimate hygiene. **Figure 7.**

3.3 Attach loops to the spreader bar or lifting strap

WARNING!

- Use only CE marked spreader bars and hoists, preferably with a rated Safe Working Load equal to or higher than that of the sling. If the SWL is lower than that of the sling, the SWL of the spreader bar or hoist is applicable.
- Always check that all the straps are properly attached to the spreader bar or lifting strap before attempting to lift the client.

1. Attach the loops to the spreader bar or lifting straps of the lifting equipment.

Attach the shoulder- and head loops before the leg hoops.

The sling can be used with spreader bars with 2 or 4 attachment points.

For security and comfort reasons, the size (width) of the spreader bar must correspond with the size of the sling.

The position of the client in the sling can be adjusted by using the different adjustment straps. Shorter loops at the shoulders (the blue straps) and longer at the legs (grey) produces a more vertical lift, which will assist with positioning into a chair or wheelchair. By lowering the loops at the shoulders (grey loops) and/or shortening the loops at the legs (blue loops), a more reclined position can be achieved, which is more suitable for a chair to bed transfer.

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© 2014 Invacare® Corporation. All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
Invacare Corporation