
Invacare® HI 13 Marina XL

en	Bath board User Manual	1
de	Badewannenbrett Gebrauchsanweisung	4
fr	Planche de bain Manuel d'utilisation	6
it	Tavola da bagno Manuale d'uso	7
es	Tabla de baño Manual del usuario	9
pt	Prancha de banheira Manual de utilização	11
nl	Badplank Gebruiksaanwijzing	12
sv	Badbräda Bruksanvisning	14
no	Badeplate Bruksanvisning	16
da	Badesæde Brugsanvisning	17
fi	Kylpylevy Käyttöohje	19
lv	Vannas sēdeklis Instrukciju rokasgrāmata	20
lt	Vonios lenta Naudojimosi instrukcija	22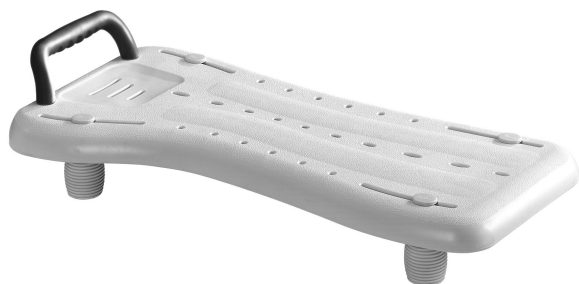

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

I General

I.1 Images

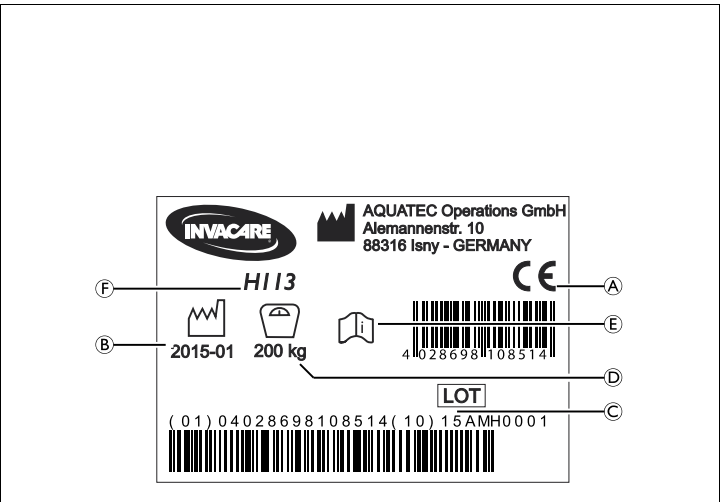
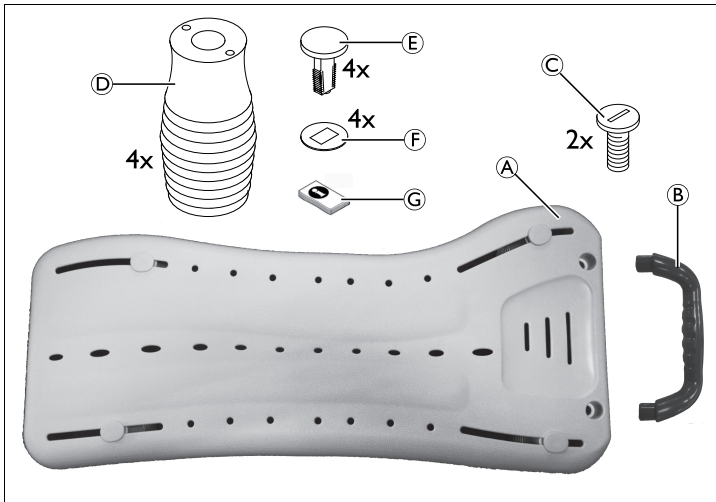
Fig. 1

Fig. 2

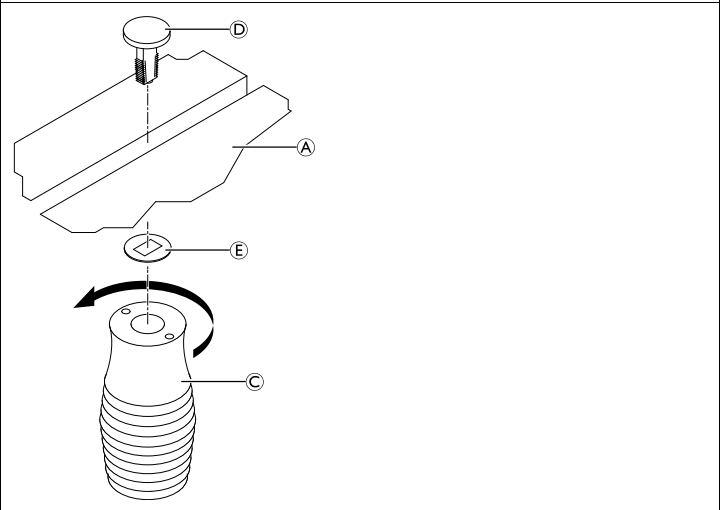
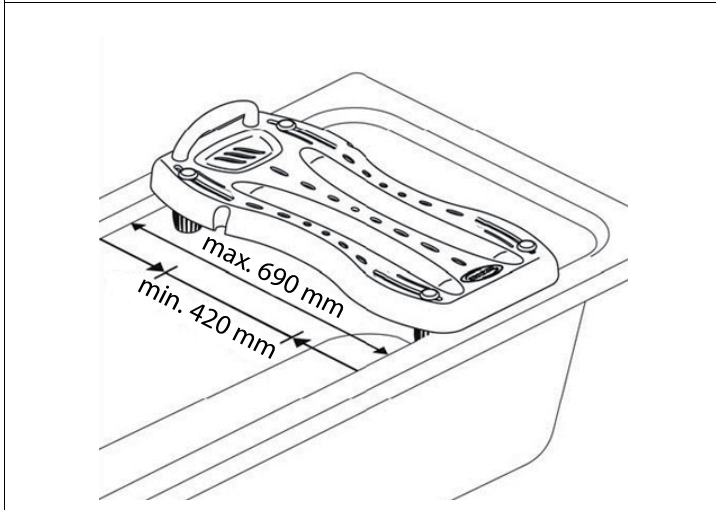
Fig. 3

Fig. 4


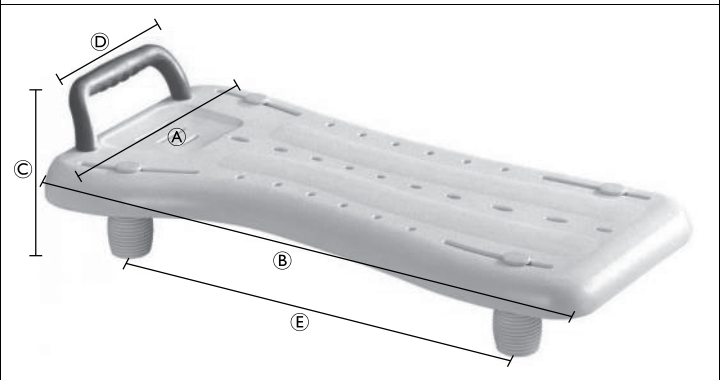
Fig. 5

Fig. 6

1.2 Symbols in this user manual

In this user manual, warnings are marked with symbols. The title next to the warning symbol indicates the level of danger.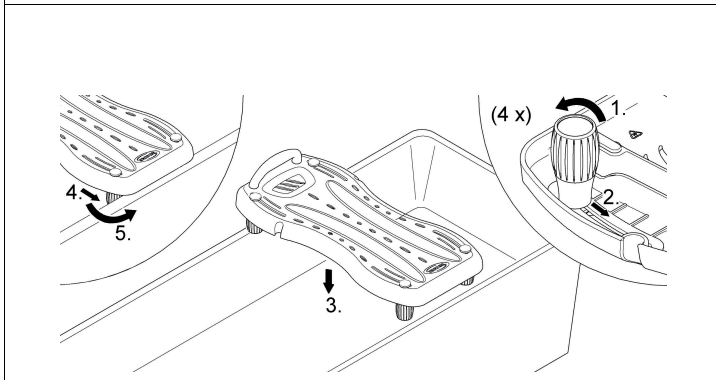

WARNING

Indicates a dangerous situation which could lead to death or serious injury if not avoided.

CAUTION

Indicates a dangerous situation which could lead to minor injuries if not avoided.

IMPORTANT

Indicates a dangerous situation which could lead to damage if not avoided.

Useful tips, recommendations and information to ensure efficient and smooth operation.

This product complies with the directive 93/42/EEC for medical products. The launch date for this product is specified in the CE declaration of conformity.

Manufacturer

1.3 Intended use

The product is intended to be used in domestic areas, nursing homes or clinics as a seating aid for bathtubs for people with sufficient torso stability. Any other use is prohibited.

Operating temperature: 10 – 40 °C

Indications:

- Significant functional impairment or missing functionality of the lower extremities (eg. amputations, paralysis, joint diseases, neuromuscular diseases with impairment of the extremities), when an independent entry and exit from the bathtub or sitting down or standing up is not possible anymore, but the trunk control and usability of the upper extremities are still sufficiently present.

Contraindications:

There are no contraindications associated with proper use.

1.4 Scope of delivery (Fig. 1)

The following components are included in the scope of delivery:

(A)	Seat board	(E)	Screw for foot (4 x)
(B)	Handle	(F)	Washer for foot (4 x)
(C)	Screw for handle (2 x)	(G)	User manual
(D)	Foot (4 x)		

1.5 Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the appliance was obtained.

1.6 Standards and regulations

Quality is of the utmost importance for our company; all processes are based on the ISO 9001 and ISO 13485 standards.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class I.

The product meets the requirements of the standards DIN EN 12182 (Technical aids for disabled persons - general requirements and test methods), DIN EN ISO 14971 (Medical devices - Application of risk management to medical devices) and DIN EN ISO 10993-1 (Biological evaluation of medical devices - Part 1: Evaluation and testing within a risk management system).

Invacare® is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced 1575262-A

to a minimum. We only use REACH compliant materials and components.

For further information please contact Invacare® in your country (for addresses, see back page of this manual).

1.7 Service life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

2 Safety

2.1 Safety information

WARNING!

Risk of falling!

- Do not stand on this product.
- Do not use as a climbing or transfer aid.

WARNING!

Risk of injury!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.

2.2 Labels and symbols on the product (Fig. 2)

(A)	Conformity mark
(B)	Date of manufacture
(C)	Lot number
(D)	Load capacity
(E)	Note (observe accompanying documents)
(F)	Product designation

The identification label is attached to the underside of the bath board.

3 Setup and usage

3.1 Assembling the product

The bath board is suitable for bath tubs with an inner width from 420 – 690 mm (Fig. 3).

- Attach the handle (B) to the seat board (A) in the desired orientation using the 2 screws (C) provided.
- Attach the feet (D) with the screws (E) and washers (F) provided to the seat board (A) (Fig. 4).

3.2 Placing the bath board on the bath tub (Fig. 5)

- Slightly loosen all four feet by turning them counterclockwise.
- Slide the feet inwards.
- Place the bath board with the feet showing down on the bath tub.
- Slide all feet equally to the inner edge of the bath tub.

WARNING!

– Make sure that the bath board legs are mounted tight to the bath tub inner wall.

- Fasten all four feet by screwing them clockwise until they stop.

3.3 Using the product

WARNING!

Risk of injury

– Before use, check that the board lies stable on the bath tub.

1. Place one leg into the bath tub.
2. Move slowly down on the bath board.
3. Move the other leg into the bath tub.

WARNING!
Risk of injury

– Never move the bath board while sitting on it.

4 Maintenance

4.1 Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined in this user manual are adhered to.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

4.2 Cleaning and disinfection

IMPORTANT!

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Health Care Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG139 and your local infection control policy.

- Clean by hand.
- Do not use any aggressive (pH value under 4.5 or over 8.5) or abrasive cleaning agents.
- Max. cleaning temperature 80 °C, for max. 3 minutes.

The product can be cleaned and disinfected using commercially available agents.

1. Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

4.3 Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly (® 4.2 Cleaning and disinfection, page 4), check the safety functions and reset if necessary.

4.4 Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

5 Technical data (Fig. 6)

Ⓐ	350 mm
Ⓑ	740 mm
Ⓒ	160 mm
Ⓓ	180 mm
Ⓔ	420 – 690 mm
Product weight	2.7 kg
Max. user weight	200 kg
Materials*	PP

*All product components are corrosion resistant and Latex free.

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(© 4.2 Reinigung und Desinfektion, Seite 5) und überprüfen Sie die Sicherheitsvorrichtungen und stellen Sie gegebenenfalls die Grundeinstellungen wieder her.
Aquatec Operations GmbH