# Invacare® Comfort, Comfort Standard and Comfort Toileting

EN	<b>Slings</b>	
	User Manual	1
DA	<b>Sejl</b>	
	Brugsanvisning	3
DE	<b>Patientengurte</b>	
	Gebrauchsanweisung	4
ES	<b>Eslinga</b>	
	Manual del usuario	6
FR	<b>Sangles</b>	
	Manuel d'utilisation	7
IT	<b>Imbracatura</b>	
	Manuale d'uso	9
NL	<b>Liftmat</b>	
	Gebruiksaanwijzing	10
NO	<b>Seil</b>	
	Bruksanvisning	12
PT	<b>Cesta de transferência</b>	
	Manual de utilização	13
SV	<b>Lyftselar</b>	
	Bruksanvisning	15

EN

## I General

### I.1 Figures

Figur

Figur

Figur

Abbildung





Afbeelding

Figures

Figura

Figura

Figura

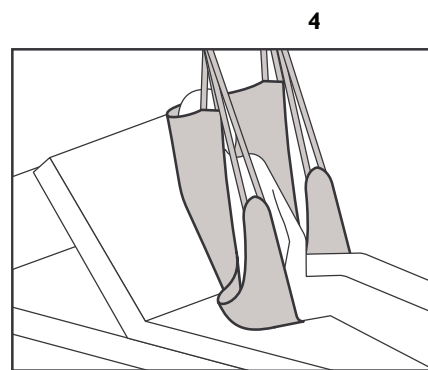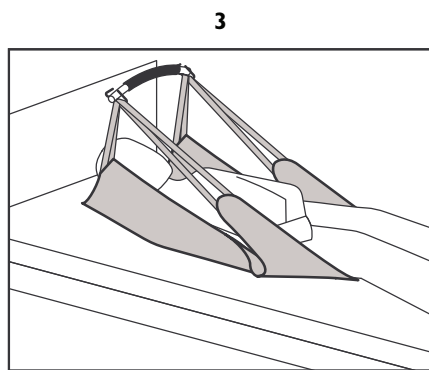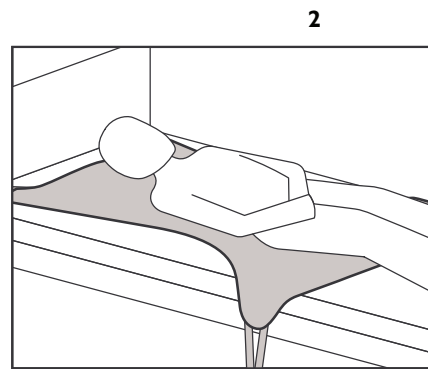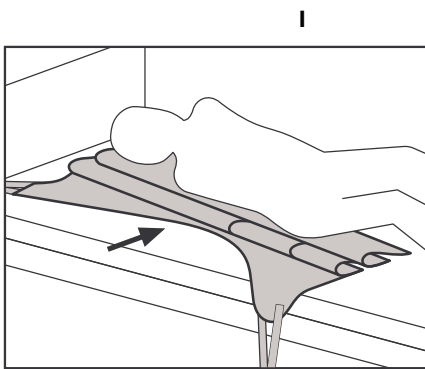This manual **MUST** be given to the user of the product.  
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

## 1.2 Symbols

	Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words. <b>WARNING!</b> Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury. <b>CAUTION!</b> Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.
	Gives useful tips, recommendations and information for efficient, trouble-free use.
	<b>IMPORTANT</b> indicates a hazardous situation that could result in damage to property if it is not avoided.
	This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.
	Safe Working Load
	Consult the user manual.
	DO NOT exceed 95°C washing temperature.
	DO NOT bleach.
	Tumble Dry, Normal, Low Heat
	DO NOT iron.

## 1.3 Intended Use

The Invacare® Comfort, Comfort Standard or Comfort Toileting sling provides maximum support and must be applied in a lying position. It is particularly suitable for either unilateral or bilateral amputees as the whole body is supported during the transfer. Similarly, clients who experience “joint pain” often benefit from the use of a Comfort sling.

## 1.4 Service Life

All Invacare slings are guaranteed for one year. Expected service life of the product is 1-5 years. Service life time varies depending on the fabric, frequency of use, laundering practices, and weight carried.

## 2 Safety

### 2.1 General Guidelines

#### WARNING!

- DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- Please check your sling on delivery. If the sling is not in perfect condition, please contact your supplier.

## 2.2 Safe Working Load

### Comfort Toileting

	Solid	Net	
<b>XS</b>	1451091	1451096	200 kg
<b>S</b>	1451092	1451097	200 kg
<b>M</b>	1451093	1451098	200 kg
<b>L</b>	1451094	1451099	200 kg
<b>XL</b>	1451095	1451100	250 kg

### Comfort Standard

	Net	Spacer	
<b>S</b>	1523070	1523098	200 kg
<b>M</b>	1523096	1523099	200 kg
<b>L</b>	1523097	1523100	200 kg

### Comfort

	Net	Spacer	
<b>XS</b>	1485967	1485948	200 kg
<b>S</b>	1485968	1485554	200 kg
<b>M</b>	1485969	1485777	200 kg
<b>L</b>	1485970	1485778	200 kg
<b>XL</b>	1485947	1451090	250 kg

## 3 Usage

### 3.1 Sling application

#### WARNING!

- Slings should be checked regularly for wear and damage (in accordance with each country’s regulations). NEVER use a damaged sling.
- If you are in any doubt, please do not attempt to use the sling and seek the advice of a professional or your supplier.
- Invacare recommends that a risk assessment is always carried out by a professional prior to issuing lifting equipment. Always choose the sling design and size according to the client’s weight, size and physical ability.
- If you are in doubt about the use of Invacare’s slings with other manufacturers’ equipment, contact Invacare or your supplier for advice.

### 3.2 Sling application — Lying in bed or floor

1. Roll the sling to midway and roll the client to one side to position the rolled up sling under the lifted side. **Figure 1.**
2. Ensure the center of the sling (green mark) is placed centrally along the spine. Roll the client to the other side, unroll the sling and roll the client back into position on the sling. **Figure 2.**
3. Attach the loops. **Figure 3.**
4. If the bed has an adjustable backrest it is more comfortable to elevate the backrest before beginning the lift. **Figure 4.**

### 3.3 Attach loops to the spreader bar or lifting strap

#### WARNING!

- Use only CE marked spreader bars and hoists, preferably with a rated Safe Working Load equal to or higher than that of the sling. If the SWL is lower than that of the sling, the SWL of the spreader bar or hoist is applicable.
- Always check that all the straps are properly attached to the spreader bar or lifting strap before attempting to lift the client.

1. Attach the loops to the spreader bar or lifting straps of the lifting equipment.

Attach the shoulder- and head loops before the leg hoops.

The sling can be used with spreader bars with 2 or 4 attachment points.

For security and comfort reasons, the size (width) of the spreader bar must correspond with the size of the sling.

The position of the client in the sling can be adjusted by using the different adjustment straps. Shorter loops at the shoulders (the blue straps) and longer at the legs (grey) produces a more vertical lift, which will assist with positioning into a chair or wheelchair. By lowering the loops at the shoulders (grey loops) and/or shortening the loops at the legs (blue loops), a more reclined position can be achieved, which is more suitable for a chair to bed transfer.

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© 2014 Invacare® Corporation. All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
#### Invacare Corporation