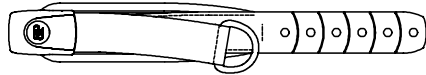
# Evoflex® Pelvic Stabilizer

## INSTALLATION AND USER'S INSTRUCTIONS

ART NOS. EB205, EB225, EB235 & EB275

These instructions provide important information for the safe use and maintenance of the Bodypoint Evoflex® Pelvic Stabilizer. Give these instructions to the user or their caregiver and review them to ensure that they are understood.

**⚠ WARNING!** Product should be installed and fitted by a qualified rehab technician.

### USE:

**⚠ WARNING!** This product should only be used for positioning a person in a wheelchair or work chair. It is NOT intended for use as a transportation safety device, as a personal restraint device, or in any other application where its failure could result in injury. Misuse of this product is unauthorized and unsafe.

**⚠ WARNING!** This pelvic stabilizer must be worn tightly fitted across the lower pelvis or thighs at all times. A loose belt can allow the user to slip down and create a risk of strangulation. Have your seating specialist demonstrate its proper adjustment and use.

**⚠ WARNING!** Accidental release of this pelvic stabilizer can allow the user to slip down or fall from the wheelchair. If the user's movements or cognitive abilities could lead to accidental release, a caregiver must be present at all times during its use. Ensure that all caregivers know how to unfasten the product. Failure to do so may delay release in an emergency.

**⚠ WARNING!** As with any new seating support, this product may change the way a person sits. Users must continue to practice regular pressure relief activities and skin

integrity checks, not only where this product contacts the user, but also in primary pressure-bearing areas such as the sacrum, legs, and buttocks. If increased skin redness or irritation occurs, discontinue use and consult your physician or seating specialist. Failure to do so may result in serious injury, such as pressure ulcers.

**MAINTENANCE:** Check periodically for signs of wear in the stitching, webbing, and pads. If significant wear is found, contact your supplier for qualified repair or replacement by Bodypoint. Under no circumstance should this product be altered or repaired by unqualified persons—health and safety depend on it!

**☰☠☒☒ CLEANING:** Machine wash, warm, 40°C (100°F). Do not bleach. Tumble dry, low temperature, or drip dry. Do not iron. (Placing the product inside a cloth bag during washing helps prevent scratches to the product and the machine.)

**WARRANTY:** This product carries a limited lifetime warranty against defects in workmanship and materials arising under normal use by the original consumer. Contact your supplier or Bodypoint for warranty claims.

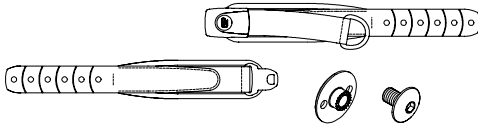
For more information on Bodypoint products, and a list of distributors outside the USA go to [www.bodypoint.com](http://www.bodypoint.com).

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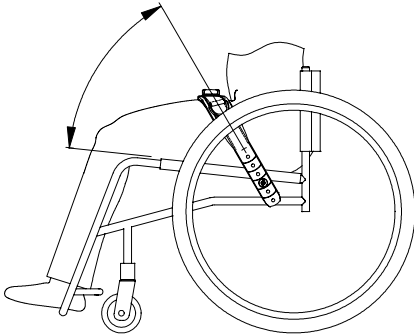
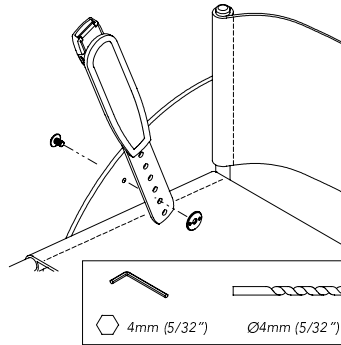
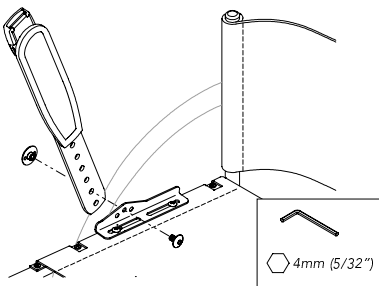
- Evoflex® Pelvic Stabilizer
- M6 x 8 screw
- M6 Low-profile flange nut

**BELT POSITION**

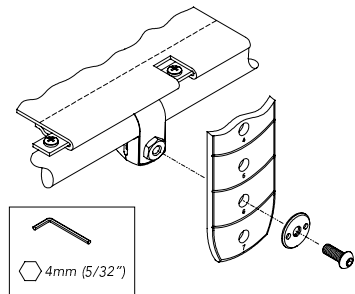
The recommended belt angle is 30°-90° from the seat, according to the positioning needs of the user. Determine the best position for the belt and, with the adjustment strap fully tightened, locate the desired anchor points, then select from the mounting methods shown.

**SIDE GUARD MOUNTING**

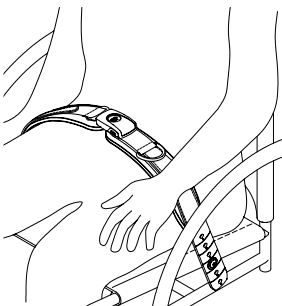
Drill 6mm (1/4") hole or use existing hole/slot to attach belt using included fasteners.

**SEAT TUBE MOUNTING**

(Requires Art. No. HW312-KIT)  
Attach mounting bracket to seat tube. Attach belt using included fasteners through middle hole of HW312 mounting bracket.

**BAND CLAMP MOUNTING**

(Requires Art. No. HW320) Attach band clamp to seat tube. Attach belt using the M6 X 12 screw included with the HW320 band clamp.

**ADJUSTMENT**

When properly adjusted and the belt tightened, it should fit snug so that the user's pelvis is secure. Tighten the D-ring pull until all slack is removed and the pads under the buckle are touching. If length adjustment is necessary to achieve a good fit or to remove the product for cleaning, this can be done at the end straps.

**SAFETY CHECK**

When properly positioned in wheelchair, have user lean forward and side to side to check fit. Check for:

1. Normal operation of buckle and adjustment straps.
2. **Comfort** Look for areas of irritation.
3. **Position** If too high or too low, adjust anchor points.
4. **Interference with other devices** Relocate anchor points as necessary.
5. Move chair through full range of motion, including tilt, folding, and rolling. Check for any interference.

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MDSS GmbH