## Backup Battery for Invacare® NordBed™

en	Backup Battery for Invacare® NordBed™ Service Manual .....	2
da	Backup-batteri til Invacare® NordBed™ Servicemanual .....	3
fi	Vara-akku Invacare® NordBed™ -vuoteelle Huoltokäsikirja .....	4
it	Batteria tampone per Invacare® NordBed™ Manuale per la manutenzione .....	5
nl	Reserveaccu voor Invacare® NordBed™ Servicehandleiding .....	7
nn	Reservebatteri for Invacare® NordBed™ Servicehåndbok .....	8
sv	Batteribackup för Invacare® NordBed™ Servicemanual .....	9

PROVIDER: Keep this manual.  
The procedures in this manual MUST be performed by a qualified technician.

Yes, you can.®

## 1 Introduction

This manual belongs to an Invacare accessory and is an addition to the manuals of the related Invacare product.


To ensure safety when using this accessory, read the user manual of the related product carefully and follow the safety instructions.

For further information see the user manual of the related product.


## 2 Backup battery

The bed can be equipped with a backup battery. The backup ensures that the bed can be adjusted during a power failure or transport.



### Mounting the backup battery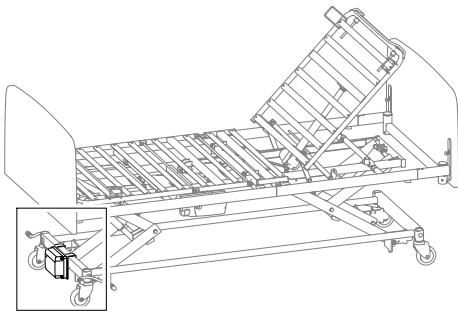


1.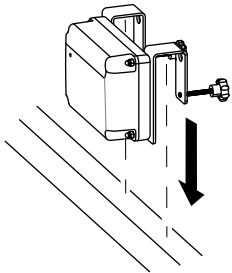


2.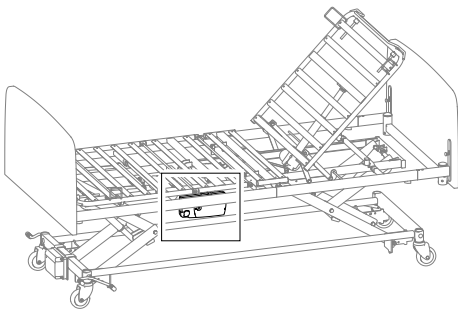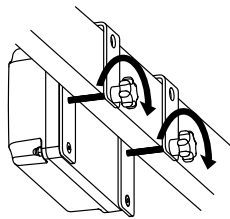


3.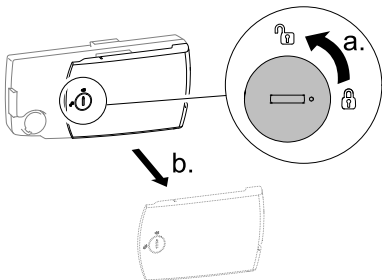

4.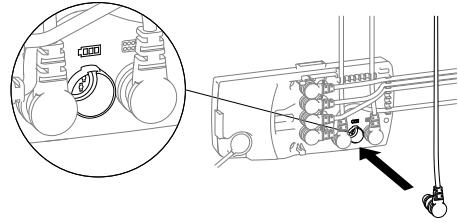

5.

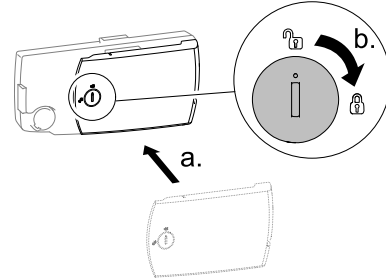1. Place the mounting brackets of the backup battery over the square tube at the foot end of the base frame.
2. Fix the backup battery by tightening the two finger screws.
3. On the control unit turn the lock 90 degrees counter-clockwise to open the lid and remove it.
4. Connect the cable from the backup battery to the control unit in the socket labeled with the battery symbol.
5. Put the lid back on the control unit and close it by turning the lock 90 degrees clockwise.

### Charging the batteries

#### WARNING!

#### Risk of explosion

Old or faulty batteries may generate an explosive gas mixture during charging.

- The battery box is provided with vents to ensure adequate ventilation of the box. Do not block or cover the vents, as this may result in pressure built-up and the risk of explosion.

1. Connect the control unit to the mains.

- The control unit must be connected to the mains for 24 hours prior to first use.
- Batteries must be charged, at least, every 6 months for maintenance purposes.

### Preventive maintenance

Frequent, sudden discharges reduce the lifetime of the battery.

We recommend the batteries to be tested at least once a year.

Batteries are not damaged by continuous connection to the mains.

## Exchanging the batteries

### WARNING!

#### Environmental Hazard

Disposal of batteries.

This product may contain substances that could be harmful to the environment if disposed of in places (landfills) that are not appropriate according to legislation.

- DO NOT dispose of batteries in normal household waste.
- Batteries MUST be taken to a proper disposal site. The return is required by law and free of charge.
- Do only dispose discharged batteries.
- Cover terminals of lithium batteries prior to disposal.
- For information on the battery type see battery label or technical data.

- Batteries have to be exchanged after 4 years. Depending on how the back up is used the exchange may have to take place earlier.
- Batteries must be exchanged as sets.
- Batteries must be replaced with batteries of the same type or mechanically and electrically compatible batteries. See technical data.
- Batteries, in a set, must have identical production codes.

 Prior to insertion make sure that the battery set is correctly connected according to the drawing (in the battery box), and that none of the connections are loose.

### Technical data

- Battery type: Lead-acid
- Capacity: 1.2 Ah
- Input voltage: 29–45 V DC
- Output voltage: 24 V DC, 1.2 A

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©2019 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
Invacare AB
Invacare Limited
Invacare Rea AB