# CBJ Home control unit

en	Replacing the batteries	Service Manual .....	2
da	Udskiftning af batterier	Servicemanual .....	2
de	Austauschen der Akkus	Servicehandbuch .....	3
fi	Akkujen vaihtaminen	Huoltokäsikirja .....	4
fr	Remplacement des batteries	Manuel de maintenance .....	4
no	Bytte av batteri	Servicehåndbok .....	5
sv	Byta batterierna	Servicemanual .....	6

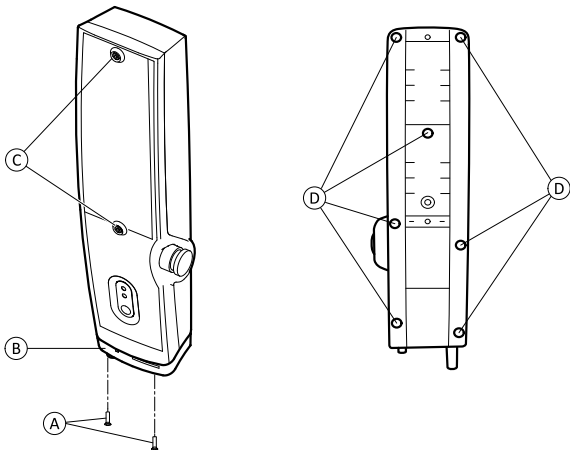
Figure 1

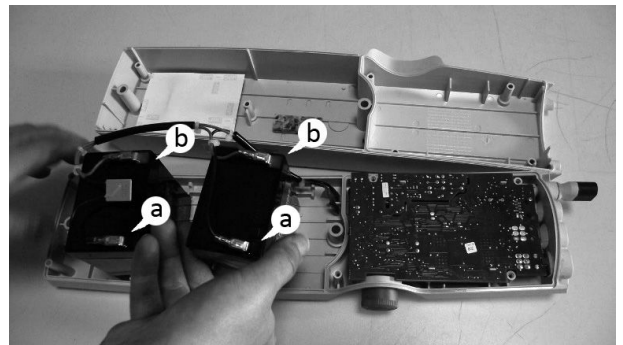
Figure 3

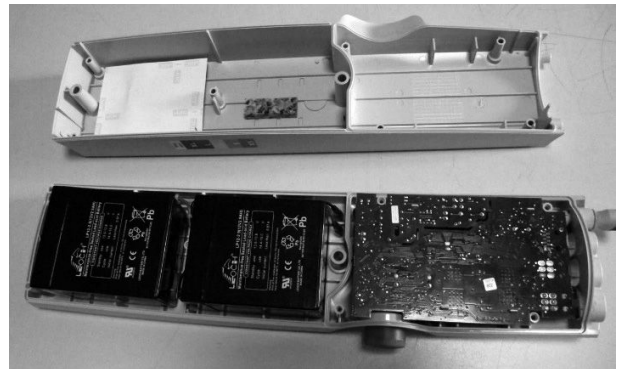
Figure 4

Figure 2

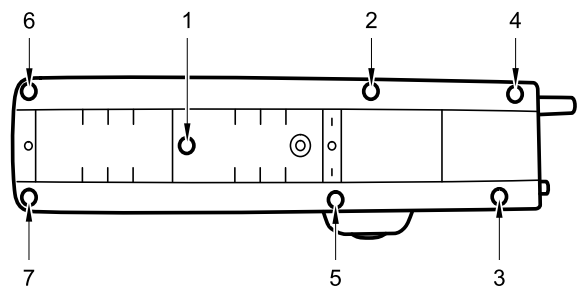
Figure 5

DEALER: Keep this manual.  
The procedures in this manual MUST be performed by a qualified technician.

Yes, you can.®

## 1 Introduction

This document contains important information about assembly, adjustment and advanced maintenance of the product. To ensure safety when handling the product, read this document and the user manual carefully and follow the safety instructions.

Find the user manual on Invacare's website or contact your Invacare representative. See addresses at the end of this document.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

For pre-sale and user information, see the user manual.

For more information about the product, for example product safety notices and product recalls, contact your Invacare representative. See addresses at the end of this document.

## 2 Replacing battery

### CAUTION! Risk of injury

- Activate the emergency stop before removing the control unit.

### IMPORTANT!

- A short circuit can damage electrical components.
- Avoid electrostatic discharge (ESD) when working on electrical parts.

Torx screwdriver (T20) / Phillips screwdriver / Allen key (4 mm)

1. Unscrew two screws **A**, remove plastic cover **B** and remove all cables from the control unit.
2. Unscrew two screws **C** and remove the control unit from the lift.
3. Unscrew seven screws **D** on the rear and remove back cover. See figure 1.
4. Remove both cables from each battery. See figure 2.
5. Replace the batteries.
6. Connect new batteries with the cables. Use the shorter cables for the closest battery. See figure 3.
  - a. black cables on black pin.
  - b. red cables on red pin.
7. Put all cables back into place and make sure no cables are squeezed. See figure 4.
8. Reinstall the back cover. Fix the seven screws in the order as described in figure 5 (Torque 1.5–1.6 Nm).
9. Reinstall the control unit to the lift.

## 3 Technical Data

Battery specifications	2 x 12 V / 2.9 Ah
Battery type	Lead acid (service free, sealed)

## 4 Disposal

### WARNING!

#### Environmental Hazard

Device contains batteries.

This product may contain substances that could be harmful to the environment if disposed of in places (landfills) that are not appropriate according to legislation.

- DO NOT dispose of batteries in normal household waste.
- Batteries MUST be taken to a proper disposal site. The return is required by law and free of charge.
- Do only dispose discharged batteries.
- Cover terminals of lithium batteries prior to disposal.
- For information on the battery type see battery label or chapter *Technical Data*.

Be environmentally responsible and recycle the different materials and components of this product separately through your recycling facility at its end of life.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

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All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
Invacare AB
Invacare Australia Pty. Ltd.