

Dolomite Brass

En Rollator



CE This manual MUST be given to the user of the product. BEFORE using this product, read this manual and save for future reference.



Yes, you can.

Contents

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

1 General.....	4
1.1 About this manual.....	4
1.2 Intended use.....	4
1.3 Service Life.....	4
1.4 Scope of delivery.....	5
1.5 Warranty.....	5
2 Safety.....	6
2.1 General Warnings.....	6
2.2 Labels and symbols on the product.....	6
3 Usage.....	7
3.1 Before use.....	7
3.2 Folding and unfolding the Rollator.....	7
3.3 Using the hand brakes.....	8
3.4 Adjusting the handle height.....	9
3.5 Walking.....	9
3.6 Sitting down on a chair.....	10
3.7 Rising to a standing position.....	10
3.8 Sitting on the walking aid.....	11
3.9 Standard features.....	11
4 Maintenance and care.....	12
4.1 Maintenance.....	12

4.2 Cleaning.....	12
5 After Use.....	13
5.1 Disposal.....	13
5.2 Re-use.....	13
6 Technical data.....	14
6.1 Dimensions and weights.....	14
6.2 Material.....	15

1 General

1.1 About this manual

This User Manual contains important information about your product. In order to ensure safety when using the product, read the User Manual carefully and follow the safety instructions.

The person who prescribes the product or who is otherwise responsible for the user must ensure that the product is suitable for the user. The user shall be instructed how to use the product. This includes the use of the features and the restrictions in use. The user shall also be notified of the warnings for use given in the user manual.

Symbols in this manual

In this User Manual, warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.



WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.



CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.



IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.



Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.



This product complies with Directive 2017/745

concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

1.2 Intended use

The Rollator is intended for support and improvement of a limited walking ability. The user's remaining walking ability needs to be sufficient for leaving the home, for usual daily activities and for participating in social life without endangering themselves or others.

The Rollator is intended to support a person during walking and may also serve as a resting seat. Furthermore, the Rollator is intended to carry a load in a specially designed bag.

The Rollator is not intended for use by persons with severe equilibrium disorders or functional perceptual disorders.

Permissible operating conditions and locations

The Rollator is only intended for use indoors, in living or public areas, or outdoors. The Rollator is not to be used under water in e.g. pools or open-air bath, neither in hot areas as e.g. saunas.

1.3 Service Life

The expected service life of this product is 5 years when used in accordance with safety instructions, maintenance intervals and correct use stated in this manual. The effective product life cycle can vary according to frequency and intensity of use.

1.4 Scope of delivery



- Rollator, fully assembled.
- User manual

1.5 Warranty

We supply a 2-year warranty from the date of delivery. Deviations may occur due to local agreements. Damage caused by normal wear to e.g. wheels, handles and brakes or damage caused by abnormal use (e.g. exceeded max. user weight) is not covered by the warranty. Maintenance and operating instructions must be followed for the warranty to be valid.

2 Safety

2.1 General Warnings



WARNING!

Risk of falling

- Exercise caution when walking on sloping / slippery ground and on uneven surfaces.
- Do not use the rollator as a wheelchair.
- Be careful when walking backwards over obstacles.
- Never sit and push with your feet.
- Do not go up or down stairs with the rollator.
- Do not use the rollator on escalators.
- Do not under any circumstances stand on the rollator.
- Be careful when using long skirts, the clothes might get stuck in the wheels.

Loads must only be transported in the bag. Be careful when there is a heavy load in the bag (for the max. load in the bag refer to chapter “Technical data”).



WARNING!

Risk of injury

- Do not let children play with the rollator.
- If the push handles or the seat are exposed to extreme temperature (above 41°C) there might be risk for first degree burns.




IMPORTANT!

Risk of corrosion

- The rollator must not be stored in wet or humid environments.

2.2 Labels and symbols on the product

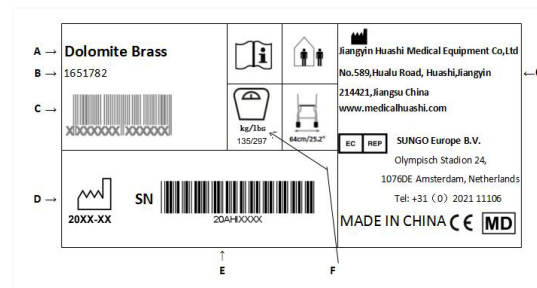

The label contains all important technical information regarding your walking aid.

- A: Product Name
- B: Article Number
- C: EAN Code
- D: Date of Manufacture
- E: Series Number
- F: Maximum user mass
- G: Manufacture


3 Usage

3.1 Before use

We do not recommend that the product is issued to anyone without carrying out a full assessment of their specific need first. It is important that the user is given the necessary support by the product as prescribed by a qualified professional. This is both for the safety of the user and the rehabilitation process.

3.2 Folding and unfolding the Rollator

Unfolding to operating position

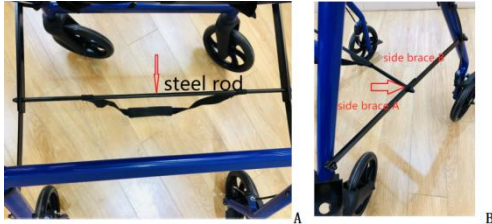

1. Hold the Rollator with the push handles and make sure that all four wheels are stable on the ground.
2. Push the steel rod downwards to unfold the Rollator. (see picture A).
3. When the side braces are in a straight line the Rollator is completely unfolded. (see picture B)



WARNING!


- Before using the product, always check that the Rollator is fully unfolded.

Folding




1. Flip up the seat.
2. Lift up the lifting strap in order to fold the Rollator into folded position.

3.3 Using the hand brakes/running brake

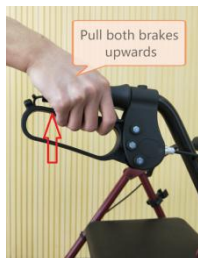

1. Pull both brakes upwards to brake.

Using the parking brake

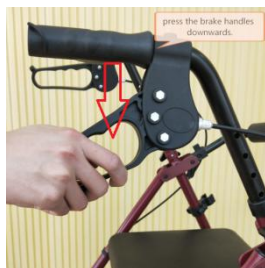

1. To apply the parking brakes, press the brake handles carefully downwards. An audible sound indicates that the brake is in the correct position.





IMPORTANT!

- Make sure that both brakes are put into parking position.

2. To release the brakes, pull the brake handles upwards.

Adjusting the braking effect

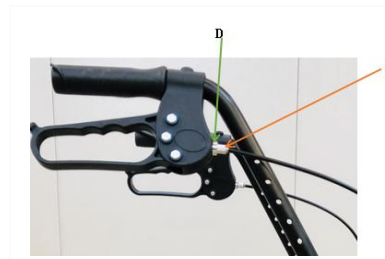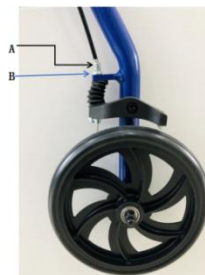

For desirable braking strength, adjust the brake wire.



The wire can be adjusted in two places - above under the brake handle (C/D) or below above the brake clamp (A/B). The brake handle must be in neutral position. Suitable tools: 8mm and 10mm spanner (not included)

Adjusting below above the brake clamp:

1 a. Loosen the locking nut B and turn the hexagon nipple A counter clockwise in order to increase the braking effect. The distance between the brake clamp and the wheel should be 2mm

2 a. Tighten the locking nut B again to fixate the new adjustment.

Adjusting above under the brake handle:

1 b. Loosen the knurled wheel D and turn the knurled wheel C counter clockwise in order to increase the braking effect. If the adjustment at the brake handle is not made easy to operate, a pliers is needed to do this adjustment. (Pliers is not included). The distance between the brake clamp and the wheel should be 2mm.

2 b. Tighten the knurled locking wheel D again to fixate the new adjustment.

3. Check the braking effect by engaging the parking brake and check that the wheel can't rotate.



WARNING!

- Make sure to tighten the lock nuts and the knurled wheel D after adjustment.

3.4 Adjusting the handle height



1. Stand behind the Rollator with flat shoulders.
2. Loosen the knob.
3. Adjust the handle to suitable height.



The handles should usually be positioned at wrist height. Therapist may suggest different heights depending on specific needs of the client.

4. Retighten the knob.



WARNING!
Risk of falling


- Ensure that the handles never been set above their highest position, and tighten the knob firmly after height adjustment.

5. Repeat the procedure on the other side.




If the user unconsciously tends to adjust the height adjustment handles, for safety reason the knobs should be replaced by locking screws (buy it from the dealer).

3.5 Walking

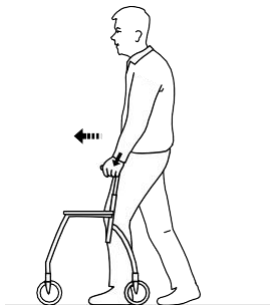

1. Place the Rollator a bit in front of you and take a step towards it whilst using the hand grips for support.

3.6 Sitting down on a chair

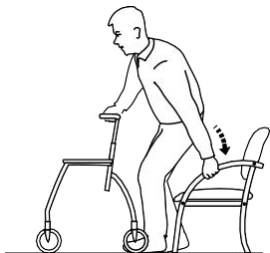


1. Stand in front of the chair with the Rollator, the back of the legs should touch the chair.
2. Place the hands on the arms of the chair, lean forward slightly and sit down carefully.

3.7 Rising to a standing position



WARNING! **Risk of tipping**

- Never use the Rollator to pull yourself up from a sitting position.
- The parking brakes must be in braked position when the Rollator is used as a support when sitting down or getting up, from a chair for example.

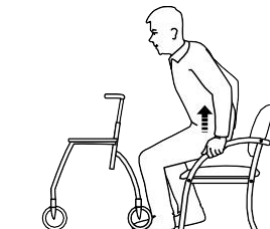

1. Place the Rollator in front of you, use the armrests of the chair as support and rise carefully to a standing position.
2. Once stable in a standing position, transfer hands to the hand grips of the Rollator.

3.8 Sitting on the Rollator

**WARNING!****Risk of falling**

Use the Rollator for stationary seating only. Both brakes must be in parking position when the Rollator is used as a seat.

- Never sit and push with your feet.
- Never sit on the Rollator on sloping ground.



3.9 Standard features

Bag

Preassembled on the frame.

Cane/Crutch Holder

Preassembled on the frame front.

Curb climber

Makes it easier to lift your Rollator over obstacles such as sidewalks. Make sure you are standing stable before pedal the curb climber.

Make sure handling it with gentle.



4 Maintenance and care

4.1 Maintenance



Maintenance should be done at least once a year.

To ensure good performance and safety you should regularly check that the following is carried out:

- Ensure that screws and adjustments are properly tightened.
- Never use a defective walking aid. Do not attempt repairs yourself!
- If hand grips are loose do not use the Rollator. Change hand grips according to the service instructions which can be obtained from your Invacare representative.
- Verify brake operation. If brake adjustment is required, see chapter 3 Usage, page 3.3 in this user manual.
- Periodically inspect the castors tightness, wear and damage.
- Make sure that the wheels are free of hair, lint and other debris.

4.2 Cleaning

The product can be cleaned and disinfected using commercially available agents.



IMPORTANT

- Cleaning must be done at least once a year.
- Do not use an abrasive detergent cleaner.
- Do not use a steam cleaner.
- Never use acids, alkalines or solvents.

- High-pressure cleaner or ordinary jet of water may be used. However, do not aim the jet towards bearings, draining holes or labels.
- Clean with damp cloth and mild detergent.
- Clean the wheels with a brush (no wire brush!).
- After washing, wipe the product dry.

Disinfection

Disinfect the product by wiping down all generally accessible surfaces with disinfectant.

- The product can be washed in a washing machine or by hand.
- The washing and drying temperature may not be higher than 60° C.

5 After Use

5.1 Disposal

When the product can no longer be used, it and its accessories as well as the packaging material should be sorted in the correct way.

The individual parts can be separated and handled according to their material.

Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

5.2 Re-use

This product is suitable for re-use after the following has been performed:

- Inspection
- Cleaning

For details, see chapter 4 Maintenance and care.



WARNING!
Risk of injury


- Always inspect the product according to the service instructions before transferring the product to a different user.
- If any damage is found do not use the product.



The user manual belongs to the product. Make sure that the user manual will be handed over with the product.

6 Technical data

6.1 Dimensions and weights

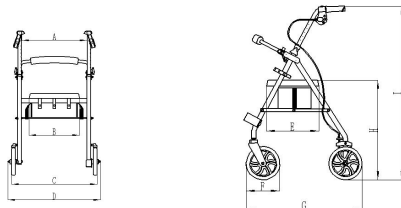

Dolomite Brass	mm
Distance between handlebars / A	486
Width of seat / B	350
Distance between rear wheels / C	580
Overall width / D	640
Depth of seat / E	300
Diameter of front/rear wheel / F	191
Overall depth / G	620
Seat to floor height / H	566
Handgrip to floor height (Max) / I	995

Dolomite Brass	KG
Product Weight ²	7
Max. user weight ³	135
Max. load bag	7

¹ Front wheels are steering wheels. Rear wheels are provided with a brake system for dosed breaking and for locking the wheel.

² Total weight excluding equipment.

³ This weight limitation includes the weight of the user whether standing or sitting and the content of the bag.

6.2 Material

Item	Dolomite Brass
Frame	Polyester powder coated aluminum
Handle bars	Aluminum
Handgrips	PVC
Wheels	PP+TPR
Seat	PU

**United Kingdom:**

Invacare Limited
Pencoed Technology Park, Pencoed
Bridgend CF35 5AQ
Tel:(44) (0) 1656 776 222
Fax:(44) (0) 1656 776 220
uk@invacare.com
www.invacare.co.uk

Schweiz / Suisse / Svizzera:

Invacare AG
Benkenstrasse 260
CH-4108 Witterswil
Tel:(41) (0)61 487 70 80
Fax:(41) (0)61 487 70 81
switzerland@invacare.com
www.invacare.ch

Italia:

Invacare Mecc San s.r.l.,
Via dei Pini 62,
I-36016 Thiene (VI)
Tel:(39) 0445 38 00 59
Fax:(39) 0445 38 00 34
italia@invacare.com
www.invacare.it

Ireland:

Invacare Ireland Ltd,
Unit 5 Seatown Business Campus
Seatown Road, Swords, County Dublin
Tel :(353) 1 810 7084
Fax:(353) 1 810 7085
ireland@invacare.com
www.invacare.ie

Nederland:

Invacare BV
Galvanistraat 14-3
NL-6716 AE Ede
Tel:(31) (0)318 695 757
Fax:(31) (0)318 695 758
nederland@invacare.com
www.invacare.nl

España:

Invacare SA
Avda. Del Oeste n.50, 1a, 1a
Valencia-46001
Tel:(34) (0)972 49 32 14
contactsp@invacare.com
www.invacare.es

Deutschland:

Invacare GmbH,
Am Achener Hof 8
D-88316 Isny
Tel:(49) (0)7562 700 0
Fax:(49) (0)7562 700 66
kontakt@invacare.com
www.invacare.de

Belgium & Luxembourg:

Invacare nv
Autobaan 22
B-8210 Loppem
Tel:(32) (0)50 83 10 10
Fax:(32) (0)50 83 10 11
belgium@invacare.com
www.invacare.be

Portugal:

Invacare Lda
Rua Estrada Velha, 949
P-4465-784 Leça do Balio
Tel:(351) (0)225 105946/47
Fax:(351) (0)225 1057 39
portugal@invacare.com
www.invacare.pt

Österreich:

Invacare Austria GmbH
Herzog Odilostrasse 101
A-5310 Mondsee
Tel:(43) 6232 5535 0
Fax:(43) 6232 5535 4
info-austria@invacare.com
www.invacare.at

France:

Invacare Poirier SAS
Route de St Roch
F-37230 Fondettes
Tel:(33) (0)2 47 62 64 66
Fax:(33) (0)2 47 42 12 24
contactfr@invacare.com
www.invacare.fr



Jiangyin Huashi Medical
Equipment Co.,Ltd
No.589,Huashi Jiangyin
Jiangsu China




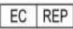




SUNGO Europe B.V
Add: Olympisch Stadion 24, 1076DE Amsterdam,
Netherlands



Invacare Rea AB
Växjövägen 303
SE-343 71 Diö
Sweden

Symbols explanations:

	Date of manufacture
	Manufacturer
	Consult instructions for use
	Authorized representative in the EC
	Indication that the device is a Medical Device
	Product complies with Directive MDR 2017/745

Notice to the user and/or patient:

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Making Life's Experiences Possible™

