Invacare® Adeo

C407A

| | | |
|----|---|----|
| en | Toilet Frame User Manual | 1 |
| fr | Cadre de toilettes Manuel d'utilisation | 3 |
| de | Toilettenstützgestell Gebrauchsanweisung | 5 |
| nl | Toiletframe Gebruiksaanwijzing | 7 |
| it | Telaio per toilette Manuale d'uso | 9 |
| es | Chasis para inodoro Manual del usuario | 11 |
| pt | Chassis de sanita Manual de utilização | 13 |

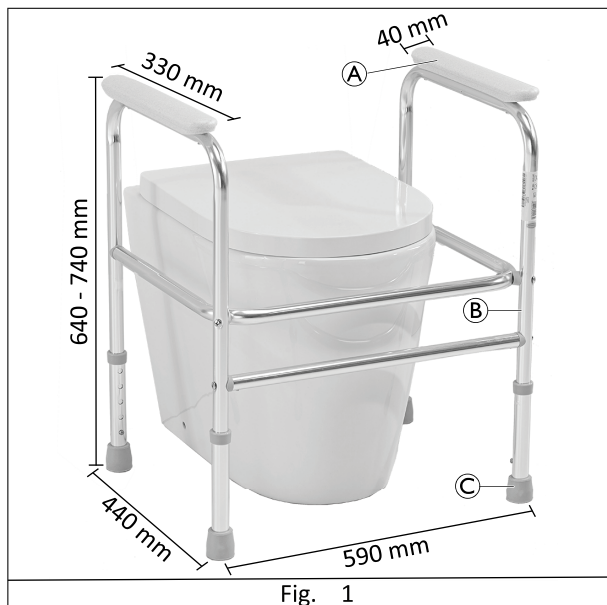
Fig. 1

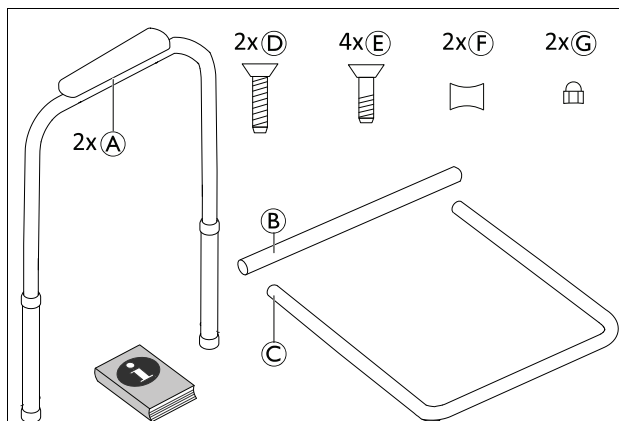
Fig. 2

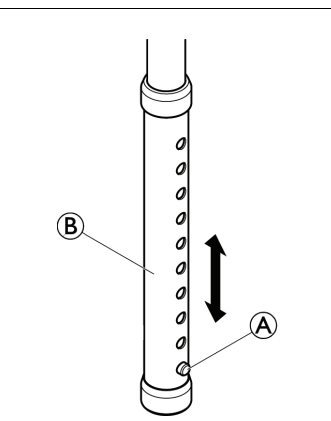
Fig. 3

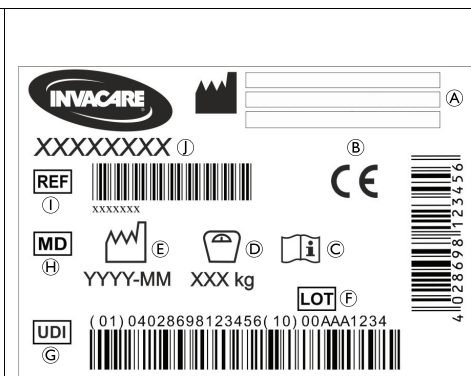
Fig. 4

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions. If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website.

Invacare reserves the right to alter product specifications without further notice.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

en

1.2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.3 Compliance

This CE-marked medical device conforms to: EU Medical Device Regulation, ISO 13485, ISO 14971, REACH, ISO 17966 and related standards.

1.4 Service Life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary depending on frequency and intensity of use.

This manual MUST be given to the user of the product. BEFORE using this product, this manual MUST be read and saved for future reference.

Yes, you can.®

1.5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up
- Unauthorised modifications and/or use of unsuitable spare parts

2 Safety

2.1 Safety Information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material.

WARNING!

Risk of Injury!

Improper use of this product may cause injury.

- Before every use, check the product for damage and ensure that it is securely assembled.
- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not use or store the product outdoors.

WARNING!

Risk of Falling!

Improper use of this product may cause injury.

- Do not move or displace the product when under load or during use.
- All four legs must have full contact to the ground.

2.2 Labels and Symbols on the Product (Fig. 4)

| | | | |
|---|---------------------|---|--------------------------|
| Ⓐ | Manufacturer | Ⓕ | Lot number |
| Ⓑ | European Conformity | Ⓖ | Unique Device Identifier |
| Ⓒ | Read user manual | Ⓗ | Medical Device |
| Ⓓ | Maximum user weight | Ⓘ | Reference number |
| Ⓔ | Production date | ⓵ | Product designation |

3 Product Overview

3.1 Intended Use

Invacare Adeo C407A is a height adjustable toilet frame with armrests to be placed over the toilet. It can be used in combination with a raised toilet seat. This product is intended as support aid for persons with mobility limitations when e.g. sitting down on or standing up from a toilet seat.

Indication

If used without an assistant, the users must be able to keep their own balance. There are no contraindications known

when using the product as intended. For the maximum user weight, see 7 *Technical Data*, page 3.

3.2 Scope of Delivery

The scope of delivery is shown in figure 2.

4 Setup

Assembling the Product (Fig. 2)

1. Fix the u-shaped stabilization tube Ⓒ between the frames with armrest Ⓐ using two screws Ⓔ in the rear holes. Use the screws Ⓓ, saddle spacers Ⓕ and nuts Ⓖ with the front holes.
2. Fix the crossbar Ⓑ using the two lower holes with two screws Ⓔ.

Adjusting the Leg Height (Fig. 3)

1. Press pin Ⓐ and move all adjustment tubes Ⓑ to desired height.
2. Let pin engage at desired height with an audible click.

WARNING!

Risk of Falling

- All four legs must be adjusted to the same height.
- Ensure that the pins are correctly engaged after adjustment.

5 Maintenance

5.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

5.2 Cleaning and Disinfection

General Safety Information

CAUTION!

Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

NOTICE!

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

Cleaning Intervals

! **NOTICE!**
Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.
Clean and disinfect the product:
– before first use,
– regularly while it is in use,
– when it has been in contact with any body fluids.

Cleaning

! **NOTICE!**
Maximum cleaning temperature:
– 60 °C by hand

1. Clean the product by wiping down all generally accessible surfaces.
2. Dry the parts with a cloth.

Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

6 After Use

6.1 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *5 Maintenance, page 2*.

Make sure that the user manual is handed over with the product.

6.2 Disposal

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

7 Technical Data

7.1 Weights and Materials (Fig. 1)

| Weights | |
|---------------------|--------|
| Weight | 2.0 kg |
| Maximum user weight | 100 kg |

| Materials | | | |
|-----------|-----|---|----------|
| Ⓐ | PE | Ⓑ | Aluminum |
| Ⓒ | TPE | | |

All product components are corrosion resistant and do not contain natural rubber latex.

7.2 Environmental Parameters

| Operating Conditions | |
|----------------------|------------|
| Ambient temperature | 10 - 40 °C |

| Relative humidity | 15% - 93%, non-condensing |
|---------------------------------|---------------------------|
| Storage and Shipping Conditions | |
| Ambient temperature | 0 - 40 °C |
| Relative humidity | 30% - 75%, non-condensing |

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Invacare Poirier SAS
Invacare GmbH