twion[®]
ENERGIZE YOUR WHEELCHAIR

MOBILITY PLUS PAKET
BEDIENUNGSANLEITUNG
USER MANUAL
MANUEL D'UTILISATION
GEBRUIKSAANWIJZING

Lizenzcode-Karte innen
License code card inside
Carte du code de licence
à l'intérieur
Bijgesloten: kaart met
licentiecode

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1. General information	5
1.1 The Mobility Plus package allows you to:	5
1.2 Requirements	5
1.3 Important safety instructions	6
1.4 Sources of interference for Bluetooth connection	10
2. Connecting to the twion wheels	13
2.1 Initial installation and registration	13
2.2 Actively connecting with your twion wheels	15
2.3 If servicing is required	15
3. Applications	16
3.1 Applications at a glance	16
3.2 Cruise mode	20
3.3 Remote	22
4. Standby mode	24
5. Replacing the outer ring	25

1.1 The Mobility Plus package allows you to:

- Increase the assistance speed of your twion wheels from 6 km/h to 10 km/h.
- Travel with power assistance in cruise mode without having to repeatedly move the push rims (similar to cruise control in a car).
- Remotely control your twion wheelchair via smartphone. This type of control is only permitted when the wheelchair is empty.
- Access product information, for example the battery status of your wheels.

1.2 Requirements

The Mobility Plus package can only be used with a smartphone with Internet access. Before installing the twion app, make sure it is compatible with the operating system version on your smartphone (Android or iOS). You can find out which version is required for smooth operation of the twion app in the Google Play Store or the App Store.

1. GENERAL INFORMATION

1.3 Important safety instructions

Please read the operating instructions of your twion before using these functions for the first time and pay special attention to the safety and hazard information contained in the operating instructions. For the use of the Mobility Plus package, the following information is to be observed in addition to the general safety information.

Please observe the country-specific road traffic regulations if you increase the assistance speed from 6 km/h to 10 km/h.

Remote

The wheelchair may only be remotely controlled with the remote function when it is empty.

Cruise mode

Before using cruise mode, you should first familiarise yourself with the handling characteristics in a safe and danger-free environment. Develop a feeling for using continuous drive, the initiation of changes in direction during continuous drive, accelerating to a higher sustained speed, braking and behaviour during an emergency stop. You should be well practised in these basic situations before travelling on public roads.

When cruise mode is activated, your hands must always be positioned near the push rims in order to make prompt changes in direction and to safely stop the wheelchair.

Always deactivate cruise mode when stopped or during breaks in driving to avoid involuntary use of the function. The function should be deactivated via the cruise mode function in the Mobility Plus app. Therefore, please familiarise yourself with the app.

Cruise mode may only be activated by the person sitting in the wheelchair. Unauthorised persons with no knowledge of the cruise mode function, may neither activate the function nor use it.

So that the battery display of the left and right drive wheel batteries corresponds to the display in the twion app on your smartphone, and the propelling movement via the push rims can be correctly interpreted in regards to direction, the wheels must be fitted correctly (right and left). Use the stickers (L/R) provided to label the wheels accordingly (from the point of view of the wheelchair driver seated in the driving direction).

1. GENERAL INFORMATION

Never use the the wheelchair's parking brakes to change direction while driving in cruise mode. Your wheelchair brakes are parking brakes which are designed to keep the wheelchair in its parking position, not service brakes. Other than for parking, the brakes may only be used in an emergency.

Always keep an eye on the road ahead of you since the wheelchair's wheels can become blocked in the case of obstacles and bumps in the road and this can lead to crashes with severe injuries. To avoid dangerous situations, only climb or descend kerbs with cruise mode deactivated. Using cruise mode at hazard points, especially the following listed points, is not allowed:

- Quay walls, landing stages and docks, paths and locations close to water, unsecured bridges and embankments.
- Narrow paths, inclines (e.g. ramps and driveways), narrow paths on a slope, mountain paths.
- Narrow and/or steep downward/sloping paths on main roads and side streets or near precipices.
- Routes covered in leaves or snow or that are icy.
- Ramps and lifting devices on vehicles.

If there is impermissible use, this is considered to be misuse of the device. Alber declines all liability for resulting damages.

1. GENERAL INFORMATION

1.4 Sources of interference for Bluetooth connection

If a Bluetooth connection cannot be established between your smart-phone and your wheels, or if the connection is disrupted, this may be due to the following sources of interference.

Microwave ovens

The use of a microwave near your Bluetooth device can cause interference.

Direct Satellite Service (DSS)

The coaxial cable used with some types of satellite dish and the connections can cause interference. Check cables for damage that could cause radio interference (loss of HF). If you suspect interference, try replacing the cable.

2.4 GHz or 5 GHz telephones

A cordless telephone operating at 2.4 GHz or 5 GHz can cause interference in wireless devices or networks during calls.

Wireless video transmitters

Wireless video transmitters operating in a bandwidth of 2.4 GHz or 5 GHz can cause interference in wireless devices or networks.

Wireless loudspeakers

Wireless audio devices operating in a bandwidth of 2.4 GHz or 5 GHz can cause interference in other wireless devices or networks.

Other wireless devices

Other wireless devices operating in a bandwidth of 2.4 GHz or 5 GHz (microwave transmitters, wireless cameras, baby monitors, neighbours' WLAN devices etc.), may cause interference in Bluetooth connections.

1. GENERAL INFORMATION

2. CONNECTING TO THE TWION WHEELS

2.1 Initial installation and registration

1. Install the twion app.
2. Scratch to reveal the licence code on the code card supplied
3. In the app, in the *Connection Manager* menu (→ symbols in Chapter 3 – for iOS these can be found under *More*).
→ *Search for devices in Create new wheel pair* and select two Bluetooth modules.
4. Accept online activation.
5. Enter licence code.
6. Check submitted wheel codes and press OK.
7. Select the corresponding twion wheel based on acoustic signal.
8. Enter your name and personal password. The password may have a maximum of 4 digits.

If you cannot connect with the Alber server, follow the instructions at www.twion.de.

If registration was successful, the following symbol appears in the top menu bar:  **connected**

2. CONNECTING TO THE TWION WHEELS

The current battery charge levels are displayed in the cockpit. If this is not the case, go back to step 2 and repeat the process.

Your password must be entered in different situations. For this reason, you should use a password you can remember easily and keep it in a safe place.

2.2 Actively connecting with your twion wheels

Switch on the twion wheels using the On/Off button on the wheel.

Enter the password under *Connection Manager** → *Active Connection* → *Connect*.

Connection to the twion wheels can be terminated under *Active Connection* → *Disconnect*.

If your wheelchair moves in the opposite direction to that desired when using the cruise or remote function, the direction can be changed under *Connection Manager** → *Manage Wheel Pair* → *Swap Wheels*. Alternatively, you can swap over the left and right twion wheels on the wheelchair.

2.3 If servicing is required

If a wheel requires servicing, proceed as follows:

1. Delete the wheel pair under *Manage Wheel Pair* → *Delete Wheel Pair*.
2. Repeat initial installation as described in section 2.1. The licence code can still be used.

3. APPLICATIONS

3.1 Applications at a glance

Connect

- Establish the connection between the smartphone and the twion wheels.
- Manage initial installation and wheel pairs.
- Reliable connection established quickly at any time using your own password.

Cockpit

- Find out driving parameters such as battery status and speed.
- Easy changing between ECO, SPORT and AUTO driving modes:

SPORT for dynamic driving:

- Full power assistance for motors
- High sensitivity of force sensor on push rim
- Short motor response time (delay 0.7 seconds)
- Long after-running of motors means less pushing is required over longer distances (1 second)

ECO for energy-efficient driving:

- Reduced power assistance for motors (90 % instead of 100 %)
- High sensitivity of force sensor on push rim
- Longer motor response time (delay 1 second)
- Shorter after-running of motors (0.8 seconds)

AUTO:

- In this setting, SPORT driving mode remains active until the batteries fall below a certain charge level, then the app automatically switches to ECO to save energy.

Cruise

Cruise control. Once activated, cruise mode maintains the introduced speed as long as it is at least 1 km/h. If necessary the speed can be gradually increased with further propelling movements.

3. APPLICATIONS

Mute

- Mute mode switches off the charge display and the acoustic signals from the wheel at the touch of a button.
- An individual profile can be configured.

Speed

Increases the assistance speed from 6 km/h to 10 km/h.

Info

Service area with quick start guide and contact details should servicing be required.

Settings

- Set the language (on iOS devices, the language is controlled by the language setting of the iOS device and not by the twion app), unit of measurement and personal data storage.
- Information about the app.

Remote

The empty wheelchair can be controlled either via the integrated tilt sensor (by gently tilting the smartphone) or using the virtual joystick.

Standby

Activate and deactivate the twion wheels via smartphone.

Tour

- Recording and display of routes via smartphone GPS signal.
- Detailed display and storing of TOUR data.

3. APPLICATIONS

3.2 Cruise mode

Cruise mode allows twion users to use power-assisted driving without having to repeatedly move the push rims (similar to cruise control in a car).

Activation

- Via the 'Cruise' application in the app's toolbar.
- Activate by selecting → 'On'.

Moving around

Display in cockpit shows that cruise mode is activated.

- When the speed exceeds 1 km/h, cruise mode maintains the speed set by the propelling movements of the wheelchair driver.
- The maintained speed can be increased by moving the push rims again.
- You can halt cruise mode by braking simultaneously with both push rims.

Deactivation

- Via the 'Cruise' application in the app's toolbar.
- Deactivate by selecting → 'Off'.

3. APPLICATIONS

3.3 Remote

With the help of the Remote function, you can control the empty twion wheelchair either with the virtual joystick or with the integrated tilt sensor (by gently tilting the smartphone).

Joystick control

Press 'Start' → change direction using the virtual joystick

Tilt sensor control

Press both 'Press' buttons at the same time → change direction by gently tilting the smartphone forward/back/left/right

4. STANDBY MODE


If the system is switched off using the On/Off switch on the wheel

- the connection to the smartphone is disconnected and the wheel cannot be switched on via the smartphone.
- the system does not consume any power.


If the system is switched off via the smartphone

- the system goes into standby mode.
- the connection to the smartphone is disconnected, but can be re-established at any time.
- the system consumes a small amount of power. The maximum standby time is displayed, depending on the available battery capacity. If this time is exceeded, the system switches off and can only be switched on again via the On/Off switch on the wheel.

5. REPLACING THE OUTER RING

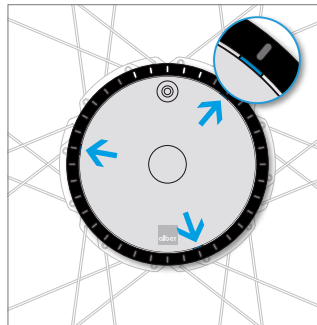
Find one of the three recesses at the outer edge of the cover (between ring and cover).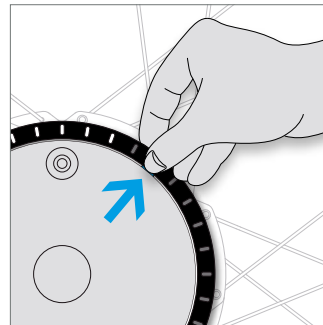


Grip the edge of the ring and press your thumbnail into the notch. Pull outwards at the same time.


5. REPLACING THE OUTER RING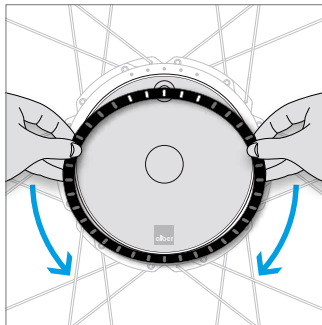

Move your fingers around the ring to release the other two fastenings.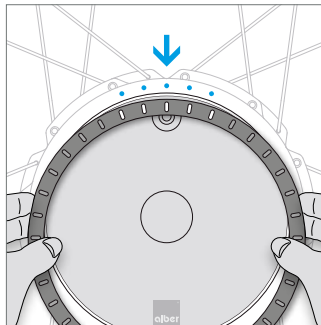

Align the new ring so that the 5 LED recesses correspond to the 5 LEDs on the drive unit.

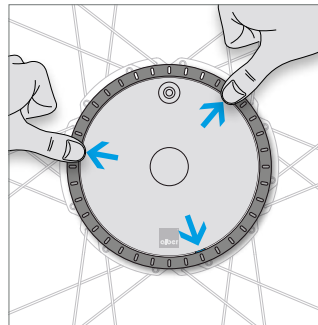Insert the pins on the underside of the ring into the openings provided and push in the 3 locking points until they engage.

Do you require further tips, interactive demonstrations or help? Further information can be found at www.twion.de.

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Alber GmbH