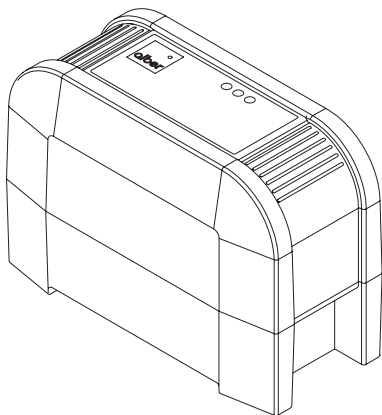
BC3615LA

Gebrauchsanweisung	DE
Instructions for Use	EN
Istruzioni per l'uso	IT
Manual de instrucciones	ES
Instruções de uso	PT
Instructions d'utilisation	FR

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Safety instructions

- Before starting your device, please read the operating instructions carefully.
- The BC3615LA charger must only be used for the charging of the batteries of in the Esprit and e-fix eco model. Any other use is prohibited and could result in serious damage to the equipment.
- The charger may only be used to charge sealed lead-acid batteries with a rated voltage of 36V and a rated capacity of between 5Ah and 12Ah.
- Never use the charger to try to charge non-rechargeable batteries.
- Use the device only, if the connecting cables and the housing are undamaged. If the mains connecting cable of the charger is damaged, it must be replaced with a special connecting cable available only from the manufacturer or its service agents.
- Do not stretch the connecting cables to prevent damage from tensile stress.
- The device is designed for operation in dry rooms and must under no circumstances be exposed to rain, spray water or steam.
- Do not carry the device with you on the wheelchair.
- After the charging process is completed, disconnect the mains plug from the socket and then disconnect the charger cable from the battery.
- Do not cover the device while in use.
- Do not open the device.
- Do not allow children to play with the device.
- Children must be supervised by a responsible person to ensure that they do not play with the device.
- This device is not intended for children and other persons whose physical, sensory or mental capabilities, or whose lack of experience and knowledge, prevents them from using it safely without supervision or instruction from a responsible person.
- Caution! Risk of explosive gas production during charging! Keep device away from naked flames and sparks. Ensure adequate ventilation during the charging process.

Intended use

The BC3615LA charger has been specifically designed for the charging of lead batteries in Esprit and e-fix eco models and must not be used for any other purpose. The charger is designed for stationary use and must not be carried on the wheelchair. The device can be used anywhere in the world, as it adjusts itself automatically to the respective mains voltage. If necessary, connect a suitable mains adapter to the charger plug.

Start-up

- Connect the charger adapter to the charger socket of the battery.
- Connect the mains plug to the socket.
- The charger is automatically started.
- Observe operating mode indicator.
- When the operating mode indicator indicates that the battery is charged, disconnect the on-board power plug from the socket. If necessary, the charging process can be interrupted at any time.
- Disconnect the charger adapter from the battery.

2

Operating mode indicator

The BC3615LA charger is equipped with an operating mode indicator showing the progress of the charging process.

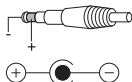
- Indicator is red: The battery is deep-discharged or connected with reversed polarity. Contact your competent service partner.
- Indicator is yellow: The charger is ready for use.
- Indicator is green and flashing: Charging is in progress.
- Indicator is green: Charging is complete.
-

Care and maintenance

The device is maintenance-free. If necessary, clean the housing with a moist cloth. Do not use aggressive detergents. Protect the charger adapter and the mains plug from dirt.

Technical data BC3615LA

OEM-Model:	WS60-1-36V
Mains voltage:	100-240VAC, 50/60Hz, 1.3-0.64A
Output:	54W
Output voltage:	36VDC
Output current:	1.5A
Protection rating:	Ip31
Protection class:	II
Ambient temperature:	Operation 0°C...50°C Storage -40...+65°C
Air humidity:	Operation 10...80% Storage 10...95%
Air pressure:	Operation 700...1060hPa Storage 500...1060hPa
Charger adapter polarity:	

During charging, the battery must be placed in a well ventilated area.

Signs and symbols

Protective insulated /device class 2

Do not use outside

Compliant with European directives

Please act in an environmentally-conscious manner and bring this product to your regional recycling centre at the end of its service life. Please familiarise yourself with the applicable legislation in your country regarding disposal, because the WEEE Directive does not apply in all European States. These components are also alternatively taken back by Alber or Alber dealers for proper, environmentally sound disposal.

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Alber GmbH