5 A charger

en	Lead Acid Battery Charger User Manual	3
cs	Nabíječka olovené baterie Návod k obsluze.....	7
et	Pliiaku laadija Kasutusjuhend	11
it	Švino rüģšties akumulatoriaus įkroviklis Naudojimosi instrukcija	15
pl	Ładowarka akumulatorów kwasowo-ołowiowych Instrukcja obsługi.....	19
bg	Зарядно устройство за оловно-киселинен акумулатор Ръководство	23
hr	Punjač olovnih akumulatora s kiselinom Upute za uporabu	29
hu	Akkumulátortöltő ólmos-savas akkumulátorokhoz Használati utasítás	33
ro	Încărcător de acumulatori cu acid și plumb Instrucțiuni de folosire	37
sl	Polnilnik svinčevokislinskega akumulatorja Navodilo za uporabo	41
uk	Зарядний пристрій для свинцево-кислотних акумуляторів Посібник користувача	45

This manual **MUST** be given to the user of the product.
BEFORE using this product, this manual **MUST** be read and saved for
future reference.

Yes, you can.®

© 2019 Invacare Corporation

All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

Invacare reserves the right to alter product specifications without further notice.

Contents

This manual **MUST** be given to the user of the product.
BEFORE using this product, this manual **MUST** be read and saved
for future reference.

1 5 A battery charger	4
1.1 Symbols found on the product	4
1.2 Features	4
1.3 LED display	4
1.4 Caution.	4
1.5 Operating instructions.	5
1.6 Troubleshooting	5
1.7 Technical specifications	6

1 5 A battery charger

1.1 Symbols found on the product

	This product bears the CE declaration of conformity. The launch date of this product is stated in the CE declaration of conformity.
	C-Tick (Australian EMC)
	This product complies with European safety requirements.
	
	Insulation class: Class II

1.2 Features

- (A) Battery charging plug
- (B) POWER LED indication
- (C) CHARGING LED indication

1.3 LED display

Green flashes	Waiting for connection to battery
Orange flashes	Pre-charge
Orange	Charging
Green and orange flashes	Charged 85 %
Green	Fully charged
Red flashes	Defect

1.4 Caution

- Before using the battery charger, read all instructions and cautionary markings.
- In order to extend the service life of the charger, you should not leave the mobility device connected to it for prolonged periods. Turn of the power after charging.
- Do not use the charger for totally discharged batteries or when the batteries are faulty.
- Use the charger in a well ventilated room.
- Only use the charger for gel or AGM batteries (16-60 Ah).
- Do not use for a voltage input other than that specified.
- The temperature of case will rise when charging. Avoid to touch the case directly.

- "Output Connector Not For Current Interrupting".
- For continued protection against risk of fire, replace only with same type and ratings of fuse.
- To reduce the risk of fire and electric shock, install in a temperature- and humidity-controlled indoor area relatively free of conductive contaminants.
- Power Supply Cord: Use UL Listed detachable power supply cord-No. 18 AWG, 2 conductors, flexible cord, rated 10 A, VW - 1, 105 °C, minimum 1.8 m, maximum 3 m long. Provided with a molded-on, non-polarized attachment plug with a 15 A, 125 V (NEMA1-15P) configuration and a molded-on connector which mates with the power inlet.

1.5 Operating instructions

1. Make sure the battery charger output voltage is the same as the output voltage of the connecting battery.
2. Connect the battery charger to the charging socket of the mobility device.
3. Connect the power cord. The LED indicates red and green flash when AC power is on.
4. Start charging. Refer to *1.3 LED display, page 4*.

1.6 Troubleshooting

- If the POWER LED (red) is off:
 - Check that the charging cable is correctly connected.
 - If the LED still does not light, the battery charger may be faulty. Contact your provider.
- If the CHARGING LED is off:
 - Check that the charging cable is correctly connected.
 - If the battery is fully charged, the charger will switch to trickle charge mode and the CHARGING LED light goes out.
 - If the charging process was not started (orange LED), the battery could be faulty. Contact your provider.
- If the green CHARGING LED keeps flashing, it cannot turn to indicate charging:
 - Check if the battery is connected successfully.
 - Check if the output connection is short or open.
 - If the battery connection is ok, the battery charger may be faulty.
- If the POWER LED (red) keeps flashing:
 - Check if the battery connection is reversed.
 - Check if the output connection is short or open.
 - Check if the environment temperature is too low (< 0 °C)
 - If the POWER LED keeps flashing, the battery charger may be faulty.
- If the CHARGING LED does not change from orange to green:
 - The battery cannot be charged correctly. It may be faulty. Stop charging and contact your provider.
- If the CHARGING LED changes from orange to green immediately:
 - The battery is either fully charged or faulty. Contact your provider.

1.7 Technical specifications

Item	Battery charger
Model	4C24050A
Output current (DC)	5 A \pm 5 %
Floating voltage (DC)	27.6 V
Input current (AC)	2.5 A max.
Input voltage (AC)	100 - 240 V, 50 Hz
Operating temperature	0 °C – 40 °C
Battery application	24 V gel or AGM batteries (16 Ah – 60 Ah)
Output detection	<ol style="list-style-type: none"> 1. Short-circuit protection 2. Reverse power protection 3. Overheat protection 4. Charging plug protection 5. Automatic cut off when reaching 12 hours constantly charging
Dimensions L x W x H	190 mm x 100 mm x 55 mm
Weight	965 g

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Invacare AG
Invacare Limited
CHIEN TI ENTERPRISE CO. LTD.