


## 1. Vorderes Saugerpaar / Front suction cups

### Selbstbelüftende Knebelsauger / self-releasing suction cups with toggle



- ▶ Knebel durch die Öffnung führen und in die Aussparung einlegen.
- ▶ *Put the toggle through the keyhole bore and place it in the corresponding recess.*

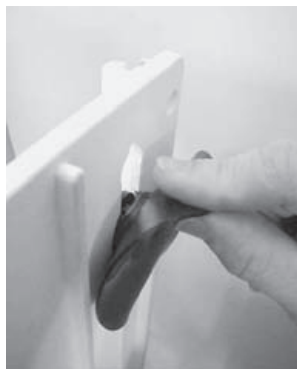

- ▶ Saugerkopf durch die Öffnung führen und nach oben ziehen.
- ▶ *Feed the head of the suction cup through the bore and pull the suction cup upwards.*



- ▶ Saugerdeckel von innen anbringen – fertig!



- ▶ *Fix the suction cup with the red lid – ready!*



## 2. Hinteres Saugerpaar / rear suction cups



- ▶ Laschensauger mit Gewindebolzen am Gewinde ansetzen und anschrauben – fertig!
- ▶ *Put the suction cup with the threaded bolt on the sleeve nut and screw it in – ready!*