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Head support Aquatec® ORCA, white (Order-no. complete set: 15461)

Head support Aquatec® ORCA, blue (Order-no. complete set: 15462)

Pos.	Designation	Qty.
④	Mountings (plastic part)	4
⑥	Countersunk head screw Ø 4x20 (VA)	4
③	Round steel bracket Ø 8 mm	1
②	Cover - woven mat with Velcro fastener	1

Safety

CAUTION Risk of injury!

- Follow the accident prevention regulations for the tools and machines used.
- Pay attention to the safety of your fingers when drilling through the backrest.
- Use an even surface when drilling (e.g. an old wooden board).
- Always drill orthogonally to the surface.

Preparation

1. Unbutton the backrest cover.

For backrests without existing headrest assembly drill holes carry out following steps:

Tools: Battery powered drill; Drill bits: Countersink bit 10 x 90°, twist drills Ø 4.2 mm

1. In accordance with Fig. 2 drill the four holes ⑤ with a Ø 4.2 mm twist drill (Drilling positions are marked on the backrest).
2. Countersink the 4 holes with Ø 4.2 mm ⑤ with the countersink bit on the front side (cover side) to Ø 8.5 mm.

Assembly (Fig. 3)

Tools: Phillips type screwdriver size 2

1. Screw the mountings ④ through holes ⑤ to the back of the backrest ① with countersunk head screws ⑥.

! IMPORTANT

▸ Make sure that the pins ③ of the mountings ④ engage in the holes ⑤.

2. Re-attach the backrest cover.
3. If not already pre-assembled, pull the headrest cover ② over the round steel bracket ③ and fasten with the Velcro fastener.
4. Insert the round steel bracket into the screwed on mountings.

Adjusting the height of the Head support (Fig. 4)

1. Hold the round steel bracket with cover on both sides and move **simultaneously** in the mountings to the desired position.

© 2021, Invacare GmbH, D-88316 Isny, all rights reserved.
Part No. 1471553-B / 2021-10

AQUATEC®

Yes, you can.